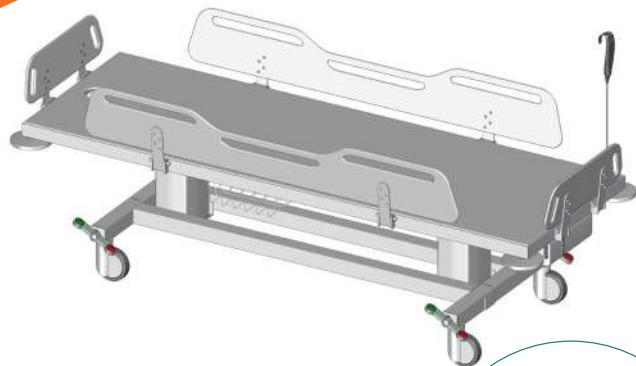
**MSCT 1:**

R9414717 - R9414721
R9434717 - R9434721
R9454717 - R9454721
R9464717 - R9464721

MCT 2:

R9412717 - R9412721
R9432717 - R9432721
R9452717 - R9452721
R9462717 - R9462721

MSCT 1 / MCT 2

Change trolley (MSCT 1 shower) – Operation and maintenance manual	en
Mobilt skifteleje (MSCT 1 bad) – Brugs- og vedligeholdelsesvejledning	dk
Mobile Ankleideliege (MSCT 1 Dusche) – Gebrauchs- und Pflegeanleitung	de
Table d'habillage mobil (MSCT 1 douche) – Manuel d'utilisation et d'entretien	fr
Mobiel aankleedtafel (MSCT 1 douche) – Gebruiks- en onderhoudshandleiding	nl
Mobil vårdbräns (MSCT 1 dusch) – Bruks- och underhållsanvisning	sv
Mobilt stellebord (MSCT 1 dusj) – Bruks- og vedlikeholdsanvisning	no
Camilla para cambio (MSCT 1 ducha) – Instrucciones de uso y mantenimiento	es
Barella per cambio (MSCT 1 doccia) – Manuale d'uso e manutenzione	it
Vaatteiden vaihtovaunu (MSCT 1 suihku) – käyttö- ja huolto-opas	fi
Wózek toaletowy (MSCT 1 prysznic) – Instrukcja obsługi i konserwacji	pl

English	12
Dansk	38
Deutsch	64
Français	90
Nederlands	116
Svenska	142
Norsk	168
Español	194
Italiano	220
Suomi	246
Jzyk polski	272



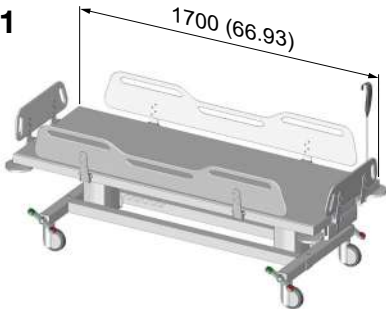
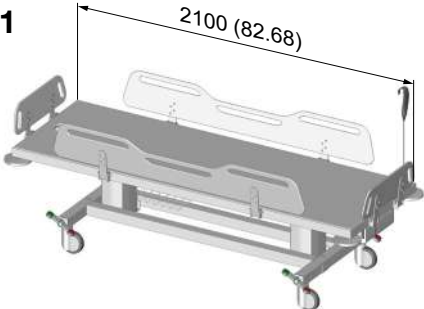
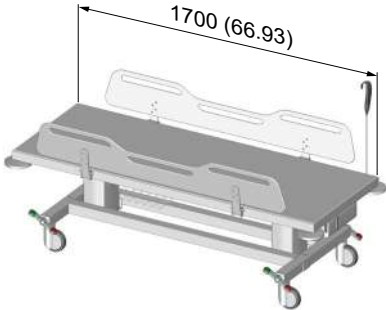
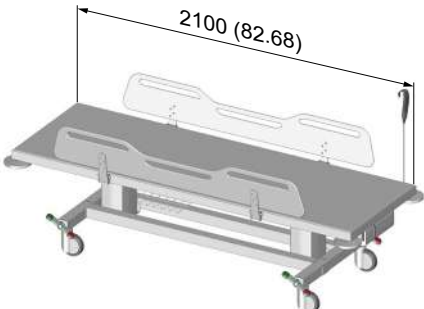
Pressalit's Environmental and Quality Management Systems are certified by LRQA

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REF

mm (Inch)

MSCT 1	 Diagram of the MSCT 1 stretcher. The length is indicated as 1700 (66.93) mm. The stretcher is shown in a side view, highlighting its adjustable backrest and side rails.	R9414717 R9434717 R9454717 R9464717
MSCT 1	 Diagram of the MSCT 1 stretcher. The length is indicated as 2100 (82.68) mm. The stretcher is shown in a side view, highlighting its adjustable backrest and side rails.	R9414721 R9434721 R9454721 R9464721
MCT 2	 Diagram of the MCT 2 stretcher. The length is indicated as 1700 (66.93) mm. The stretcher is shown in a side view, highlighting its adjustable backrest and side rails.	R9412717 R9432717 R9452717 R9462717
MCT 2	 Diagram of the MCT 2 stretcher. The length is indicated as 2100 (82.68) mm. The stretcher is shown in a side view, highlighting its adjustable backrest and side rails.	R9412721 R9432721 R9452721 R9462721

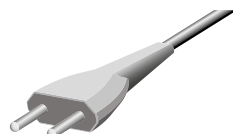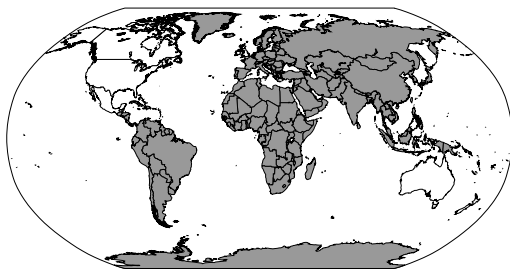
REF

R9432717
R9432721
R9434717
R9434721


REF

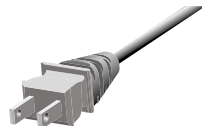R9452717
R9452721
R9454717
R9454721

REF



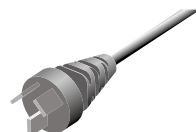
R9462717
R9462721
R9464717
R9464721

REF

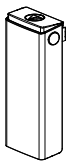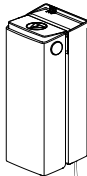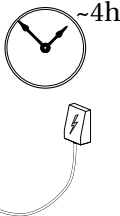R9412717
R9412721
R9414717
R9414721

		Max P	U	Max I	IPXX
 	-	-	25.2- 25.7 V DC	2,85Ah	IPX6
 	R9432717 R9432721 R9452717 R9452721 R9412717 R9412721 R9434717 R9434721 R9454717 R9454721 R9414717 R9414721	80.5 W	230 V~ 50/60 Hz	350 mA	IPX4
	R9462717 R9462721 R9464717 R9464721	42 W	120 V~ 50/60 Hz	350 mA	


MSCT 1	 	  200 kg / 441 lbs
R9414717, R9434717, R9454717, R9464717	107 kg / 236 lbs	  50kg / 111 lbs
R9414721, R9434721, R9454721, R9464721	111 kg / 245 lbs	  250kg / 552 lbs

MCT 2	 	  200 kg / 441 lbs
R9412717, R9432717 R9452717, R9462717	102 kg / 225 lbs	  250kg / 552 lbs
R9412721, R9432721 R9452721, R9462721	106 kg / 234 lbs	  250kg / 552 lbs


mm

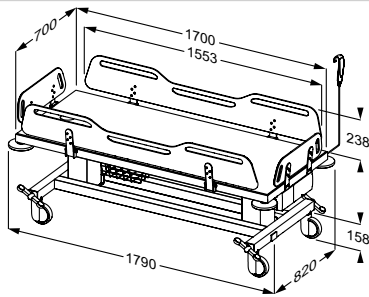MSCT 1


R9414717
R9434717
R9454717
R9464717

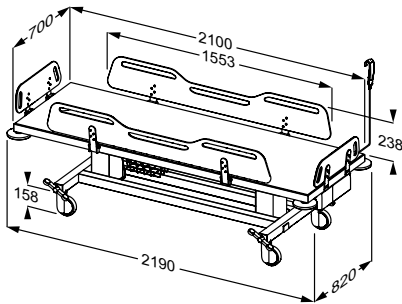MSCT 1


R9414721
R9434721
R9454721
R9464721

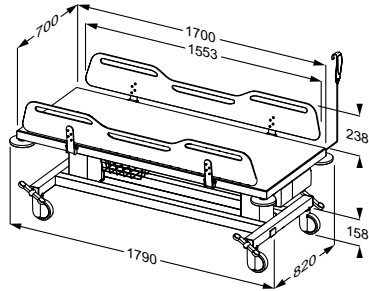MCT 2

R9412717
R9432717
R9452717
R9462717

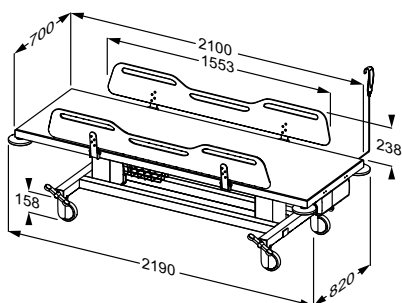
MCT 2

R9412721
R9432721
R9452721
R9462721


inch

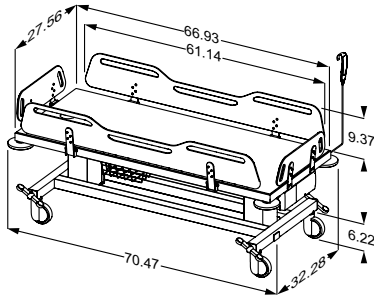MSCT 1


R9414717
R9434717
R9454717
R9464717

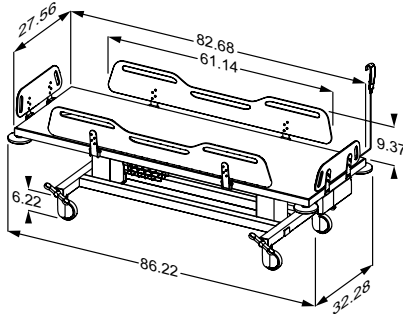MSCT 1


R9414721
R9434721
R9454721
R9464721

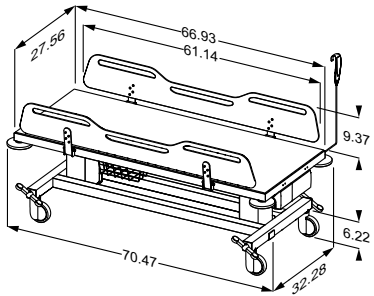MCT 2


R9412717
R9432717
R9452717
R9462717

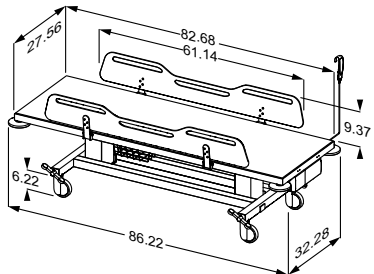
MCT 2







R9412721
R9432721
R9452721
R9462721

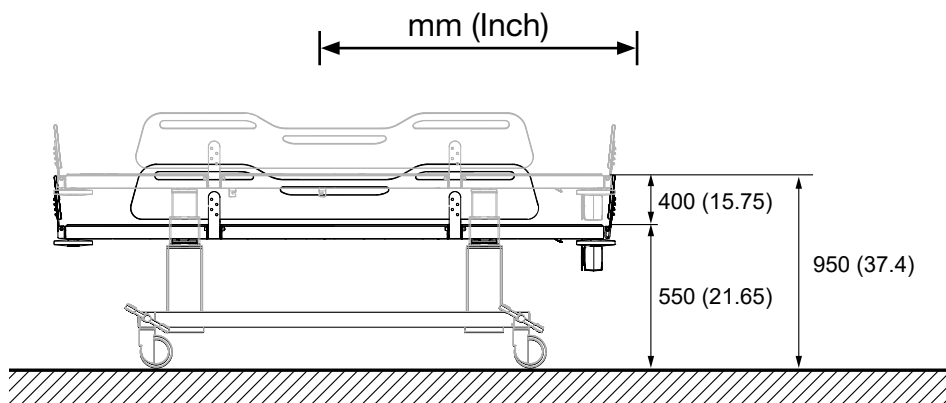
(MSCT 1)

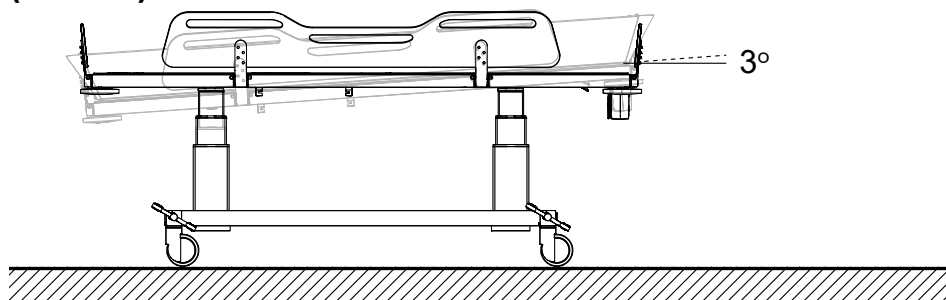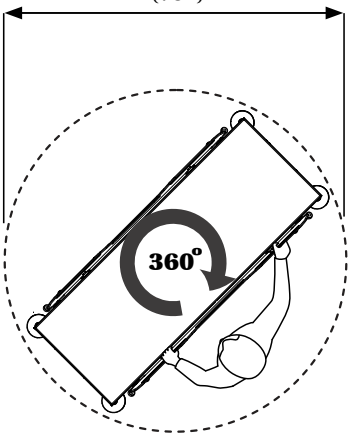
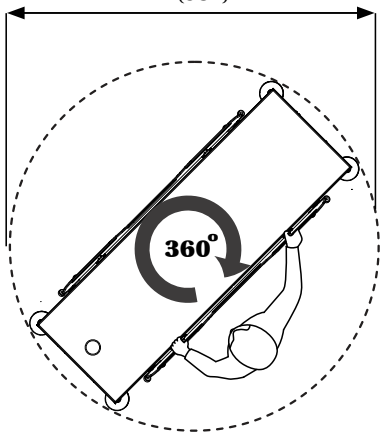
2000 mm (79")  R9412717, R9432717 R9452717, R9462717 R9414717, R9434717 R9454717, R9464717	2400 mm (95")  R9412721, R9432721 R9452721, R9462721 R9414721, R9434721 R9454721, R9464721
--	--

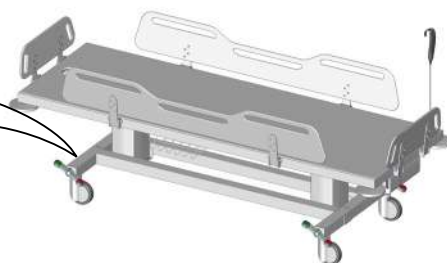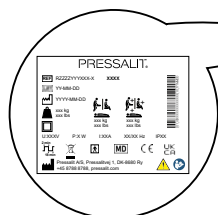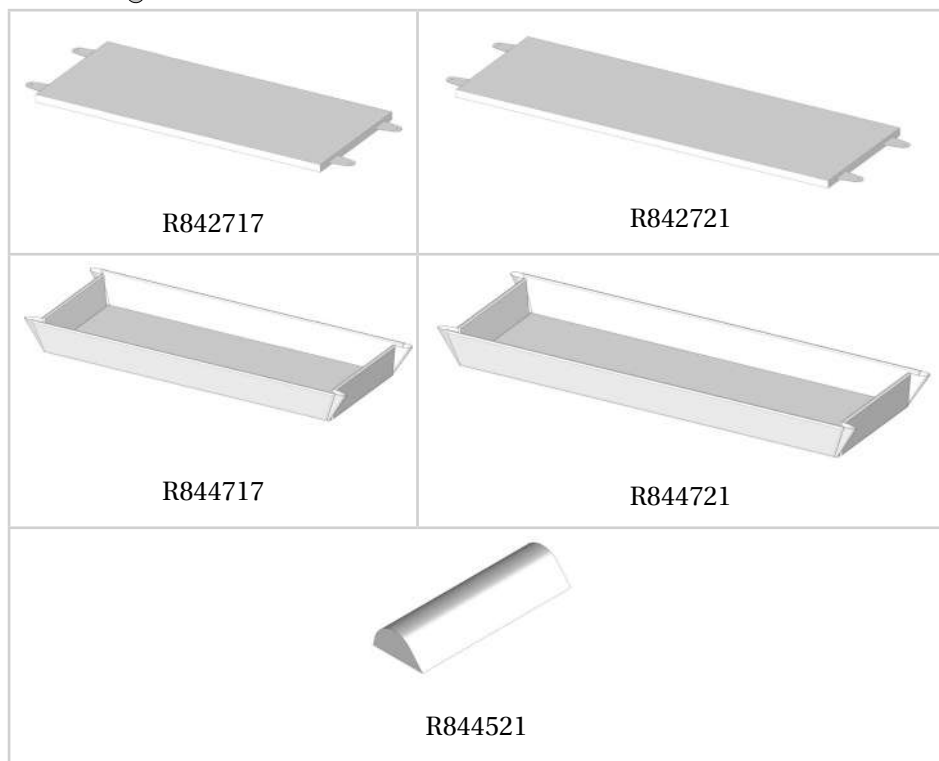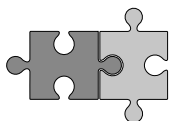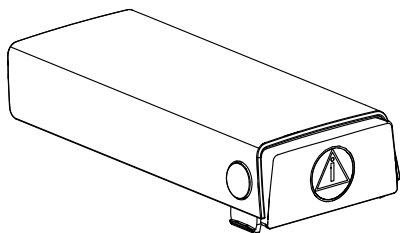V6361

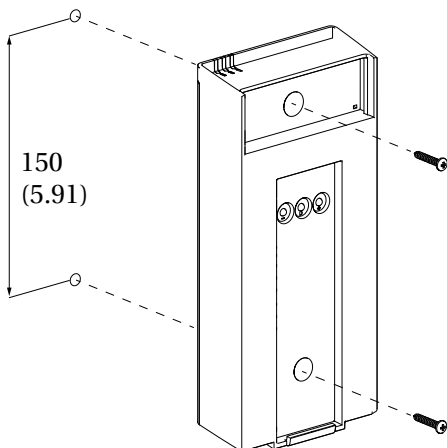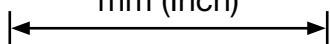mm (Inch)

V6362

EU/world

V6366

UK/IRL

V6367

US/CAN


V6368

AUS/NZ


V6369

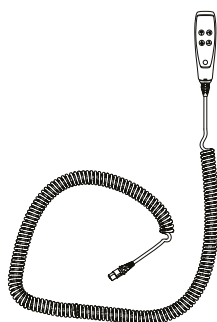MSCT 1

V6325
~5 m / 20 ft

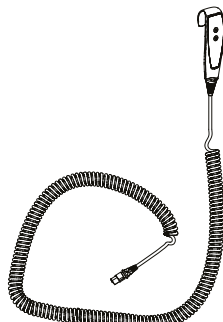MCT 2

T4361
~5 m / 20 ft

Content

Symbols used in this manual.....	13
General safety.....	14
Safety distances.....	15
Connection to main supply.....	16
Intended use.....	17
Intended operator profile.....	17
Intended user profile.....	17
Operating environment.....	17
Contraindications for use.....	18
Biocompatibility.....	18
Flammability.....	18
Operating instructions.....	19
Safety brakes.....	19
Safety rails.....	20
Height adjustment.....	21
Moving.....	22
Tilt function (MSCT 1 only).....	22
Emergency stop.....	23
Releasing the emergency stop.....	23
Charging the battery.....	24
Charging LED indicators.....	25
Accessories.....	26
Mattress.....	26
Shower mattress (MSCT 1 only).....	26
Head rest.....	28
Cleaning.....	29
Disinfection.....	29
Maintenance.....	30
Product label.....	30
Resetting.....	33
Troubleshooting.....	33
System LED indicators.....	34
Electromagnetic compatibility.....	34
Electromagnetic emissions.....	35
Electromagnetic immunity.....	35
Storage and transportation.....	36
Technical data.....	36
Disposal and recycling.....	37

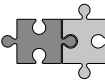
Operational signage

A Quick-guide is included with the product. It can also be downloaded from pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

 WARNING				WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
 CAUTION				CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.
IMPORTANT				IMPORTANT indicates important information about handling and use of the product.
NOTICE				NOTICE is used to address practices not related to personal safety.
 Hand injury warning	 Head injury warning	 Genitalia injury warning	 Feet injury warning	
 Electricity / voltage warning	 Power specifications	 Compatible products	 Spare parts	
 Product weight	 Max. user weight	 Max. load	 Article number / versions	
 Do not leave patient unattended	 Children and pets warning			

en

General safety

Read and understand the manual completely before use. Comply with this manual to avoid accidents involving serious personal injury and the product becoming damaged or destroyed. Save and keep the manual close to the unit.

WARNING

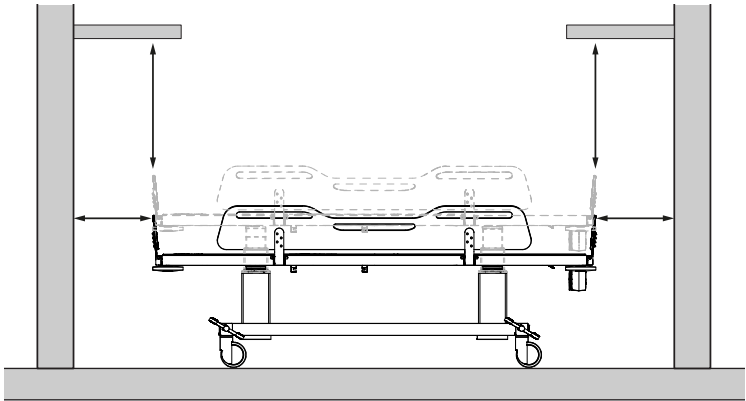
- To reduce the risk of electric shock always unplug the unit from the electrical outlet before cleaning or servicing.
- Never use or handle this product in other ways than specified in this manual or interconnect this equipment with other equipment not described in this manual, as this can put your safety at risk and cause damage to the product.
- Never allow children, invalids, or disabled persons to operate this product unless under supervision. The product is not intended for play.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.
- If you suspect that any individual parts may be damaged, contact your Pressalit dealer or service provider. Do not perform repairs, disassembly, or assembly operations, readjustments or modifications to this product by yourself. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.
- Do not use any spare parts, extensions (e.g. extension cord) or accessories other than those specified for the product.
- Keep the cords away from heated surfaces.
- Never drop or insert any object into any openings in the product.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- This product is intended for indoor use only.

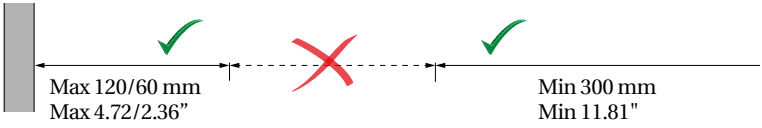
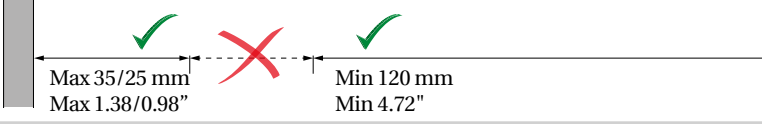
NOTICE

- Pressalit cannot be held liable if the product is used in any way which differs from that described in this manual.
- We reserve the right to amend this manual without prior notice.

Safety distances

Make sure that the unit is at safe distance from surrounding objects. Check the distances before any use.

ISO 17966:2016			
		Max 120/60 mm Max 4.72/2.36"	Min 300 mm Min 11.81"
		Max 8/4 mm Max 0.31/0.16"	Min 25 mm Min 0.98"
		Max 8 mm Max 0.31"	Min 75 mm Min 2.95"
		Max 35/25 mm Max 1.38/0.98"	Min 120 mm Min 4.72"

WARNING

If the unit moves within the safety distances, you must keep a close eye on especially surrounding people and objects to avoid crushing hazards.

Connection to main supply

WARNING

- Connect the product to a mains supply in compliance with IEC 60601-1 and IEC 60364-7, as well as any locally applicable standards.
- Ensure that the voltage of the local mains supply to be connected is compliant with that stated on the label on the product.
- To ensure protection by electrical separation, the product must be supplied by a mains supply circuit with incorporated a residual current protective device (RCD) with a rated residual operating current not exceeding 30 mA in compliance with IEC 60601-1 and IEC 60364-7, as well as any locally applicable standards.
- Ensure that location of the mains plug is easy to access and use if separation from mains is needed.
- External cables (mains plug) need to be fixed in such a way that users cannot stumble and injure themselves.

en

Intended use

The MSCT 1 and MCT 2 are intended for changing a person who is unable to stand or sit. The person lies horizontally on the change trolley.

Note: The MSCT 1 model is particularly suitable for showering the person, as it is equipped with a drain, tilt function, and end rails for use with a shower mattress.

Intended operator profile

Only personnel who are trained in healthcare and/or in changing someone under their care should use this product.

Intended user profile

The product can be used by all – adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 441 lbs.**

Operating environment

The change trolley is intended to be used at room temperature in public changing places, schools, private homes, day/residential/nursing facilities and hospitals under normal working conditions.

The change trolley must not be exposed to chlorine-containing fumes.

When stored outside operating temperature wait minimum 3 hours at 20°C before use. This allows the unit to reach operating conditions.

Environmental factor	Operating range
Temperature	5°C to 40°C (41°F to 104°F)
Relative humidity	20% to 80% at 30°C (86°F)
Atmospheric pressure	700 to 1060 hPa
Meters above sea level	Max. 3000 m / 9,800 feet
Duty cycle	2 minutes of continuous use, followed by an 18-minute pause.

CAUTION

The product surface may become hot or cold if exposed to external sources of heat or cold, such as sunlight or frost.

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities. A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

Contraindications for use

The following only applies if the product is used for showering:

It is not recommended that patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions are bathed on the change trolley. Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feeding or drainage lines (e.g. catheter, drainage tube) may be bathed on the change trolley only after consulting with the attending physician.

Biocompatibility

The biocompatibility of the materials that come in contact with the user's skin during the intended use has been evaluated according to the procedures outlined in DS/EN ISO 10993-1:2020, which involves assessing the materials for harmful substances. Based on this evaluation, the materials are considered suitable for their intended use.

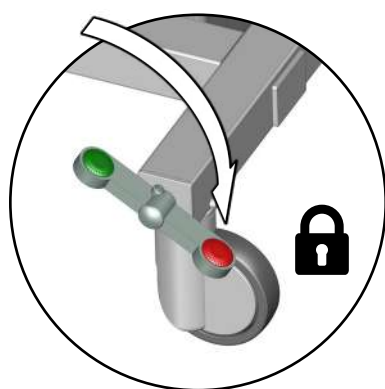
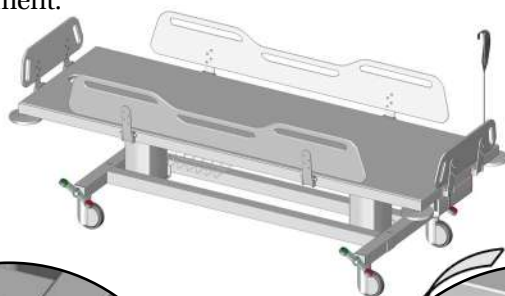
Flammability

The product materials have undergone flammability testing in accordance with ISO 8191-1:1987 and ISO 8191-2:1988.


Operating instructions

Safety brakes

The change trolley is equipped with brakes on all four wheels to prevent accidental movement.

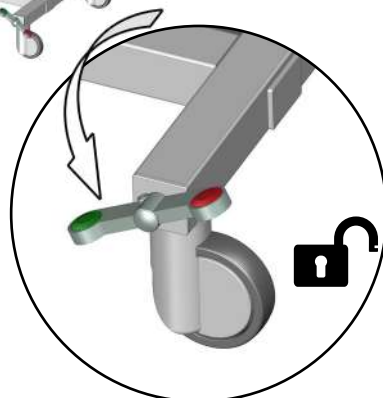Lock the brakes: Step on the red button on one of the four rocker pedals to lock the brakes on all four wheels.

Unlock the brakes: Step on the green button on one of the four rocker pedals to unlock the brakes on all four wheels.

Always use your foot to lock/unlock the wheels. Wear closed-toe shoes and avoid using your hands. Be mindful of the brakes to avoid tripping or bumping into them.

Ensure all four wheels are locked before transferring a patient to/from the mobile change trolley.

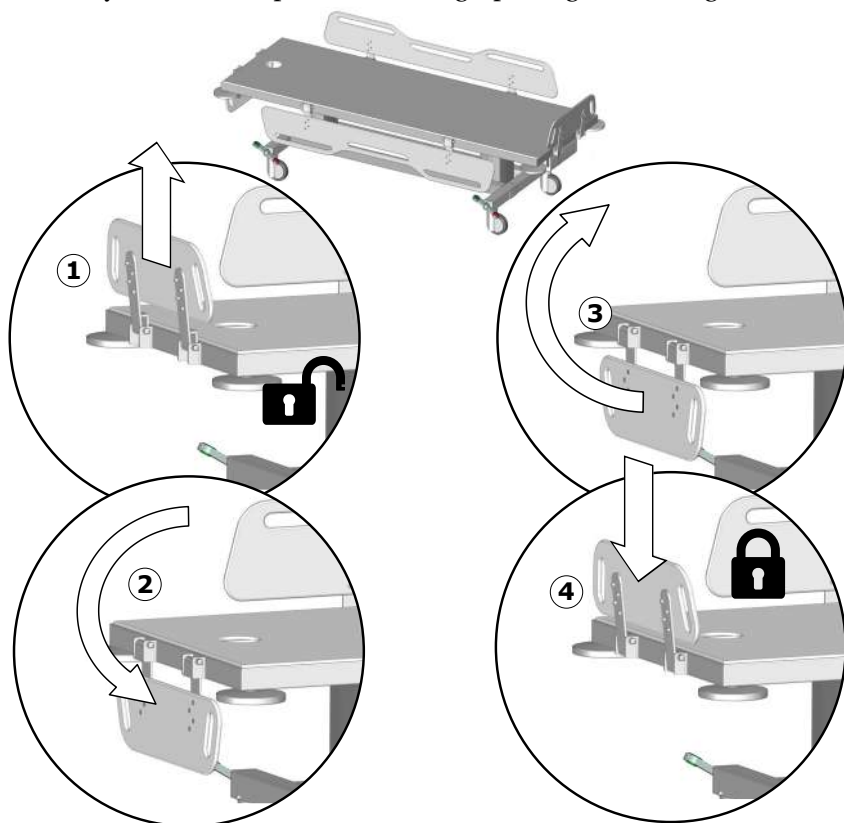
Never leave a patient unattended on the trolley.

Safety rails

The safety rails can be folded up and down individually. All safety rails function in the same way.

Note: The MSCT 1 model is equipped with end rails that can be folded up and down for easy access to the patient's feet/legs (putting on stockings, etc).

Fold down: Lift the safety rail upwards to unlock it①, and swing it down②.

Fold up: Swing the safety rail up③ and push it down to lock it in place④.

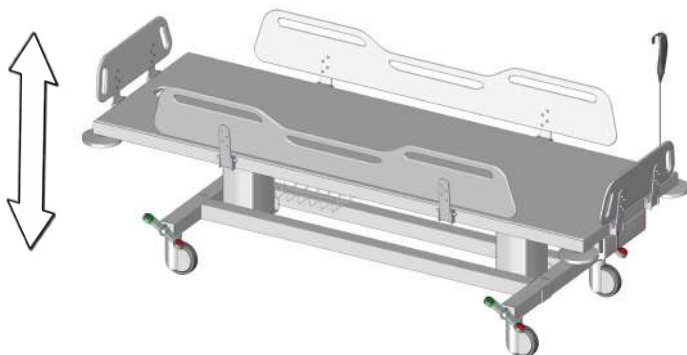
CAUTION

To protect the patient, it is recommended that the side rails are upright and locked when using the mobile change trolley.

Be careful when adjusting the side rails to avoid pinching hands, feet, or other body parts.

Height adjustment

200 kg / 441 lbs

Max load:

250 kg / 552 lbs

The change trolley can be adjusted 400 mm / 15.75" up and down.
The height adjustment is controlled by the wired remote control unit.

Press and hold the Up button to raise the unit.
When the button is released, the motion stops.

Press and hold the Down button to lower the unit. When
the button is released, the motion stops.

Do not place your foot on the bearing frame and be especially cautious of the risk of pinching hands, feet, or objects when adjusting the height of the unit.

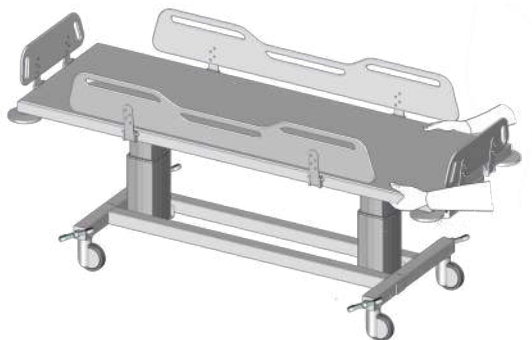
Never let children or pets be near the unit during height adjustment.

**Ensure that the remote control cord is not wrapped around the patient's neck, as it poses a strangulation risk.
When not in use, place the remote control on the side rail.**


Moving

To move the trolley, unlock all four wheels. Grab the lying surface itself, not the safety rails, and proceed with caution while moving the unit.

To prevent the trolley from tipping over, avoid using it on floors with lowered drains, holes, or significant slopes.

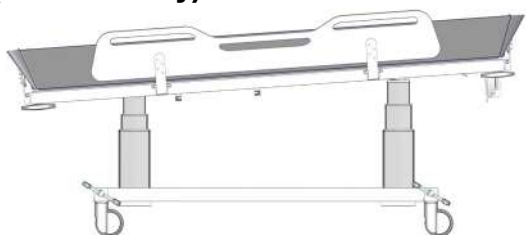en Tilt function (MSCT 1 only)

Note: The tilt function is only for the MSCT 1 model. The shower change trolley can be tilted approximately 3 degrees to allow the water to drain when using the optional shower mattress. The tilt function is controlled by the wired remote control unit.

Press and hold the Tilt up button to tilt the unit. The motion stops when the unit is tilted approx. 3 degrees, or when the tilt up button is released.

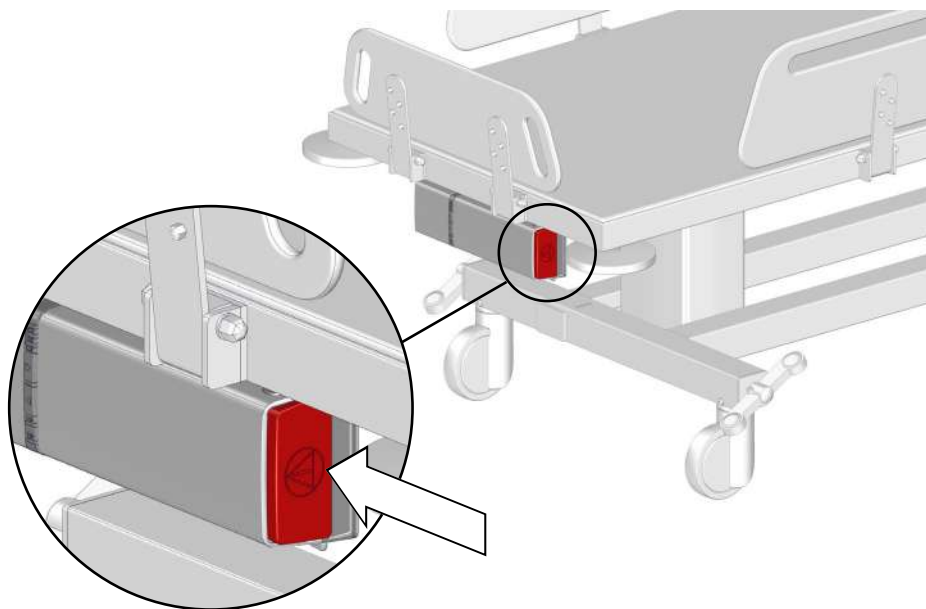
Press and hold the Tilt down button to level the unit. The motion stops when the unit is level, or when the tilt down button is released.

It is still possible to raise and lower the unit when it is in tilted position.

Emergency stop

In case of an emergency, push the emergency stop on the battery. The upward or downward movement will stop immediately.


en

Releasing the emergency stop

Push the two release buttons to remove the battery.




Reattach the battery again. Make sure the battery lock clicks into place.

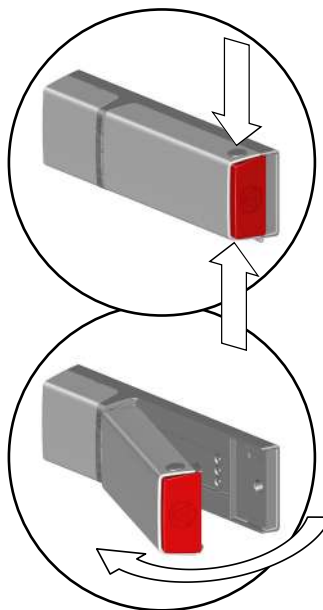The emergency stop is now released.

Charging the battery

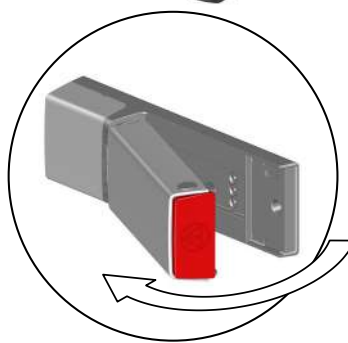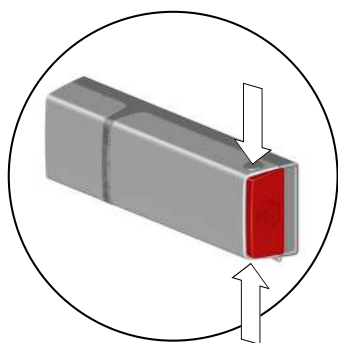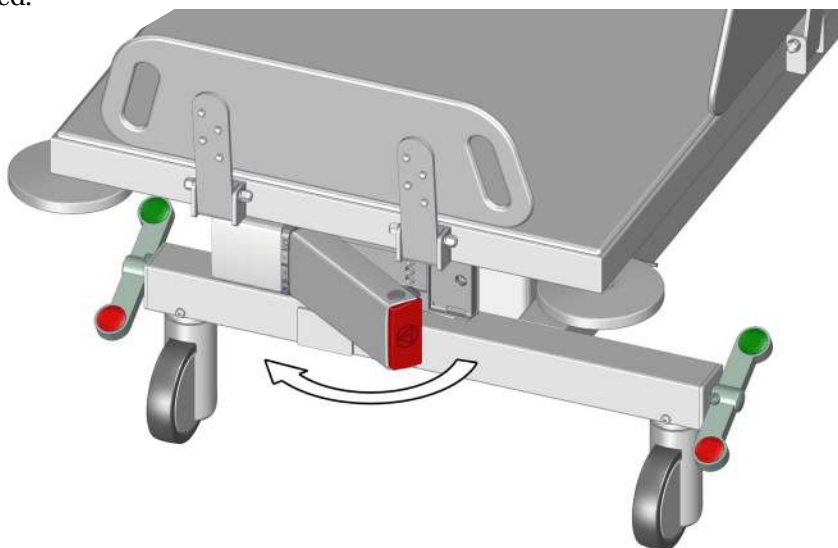
The battery can height adjust the change trolley up to 30 times when fully charged.

Push the two release buttons to remove the battery for charging in the external charger.

WARNING

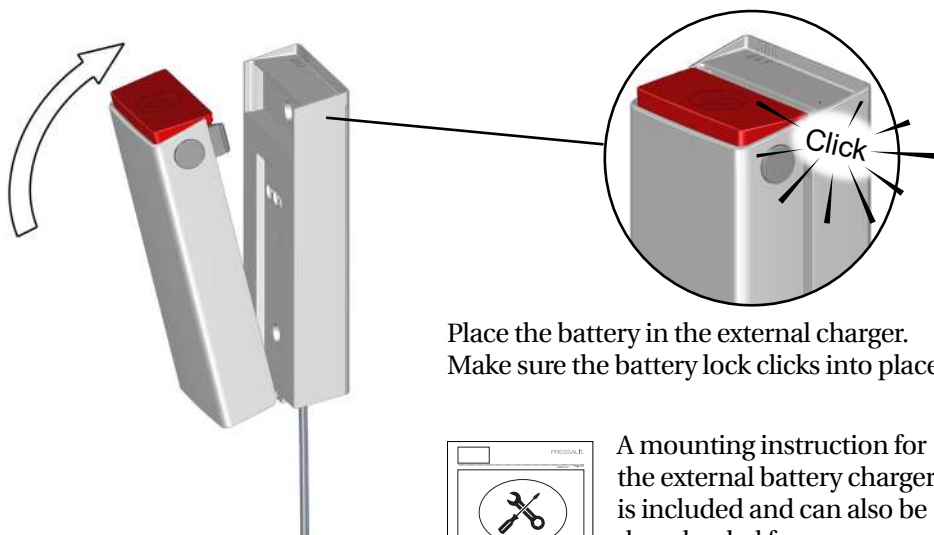- Never use a battery if visible damage to the housing or battery itself are observed due to risk of short circuits, fire hazard or explosion.
- Do not drop or strike the battery and avoid overcharging and heating the battery, as this could lead to thermal runaway.
- Charge batteries in a well-ventilated space under supervision. Always use the original charger.

Place the battery in the external charger.
Make sure the battery lock clicks into place.

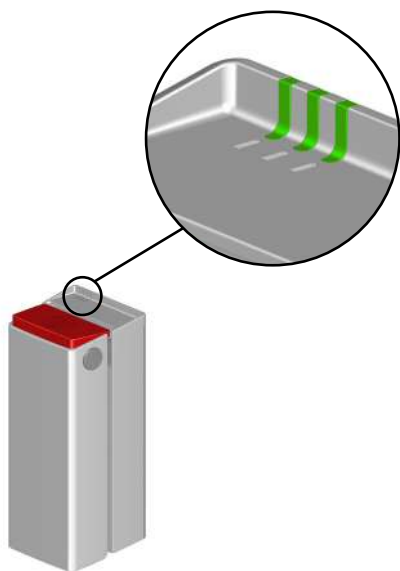
A mounting instruction for the external battery charger is included and can also be downloaded from pressalit.com.

en

Charging LED indicators

The three LEDs on the external charger show the battery charging status.



	Charging status
	90-100%
	65-90%
	40-65%
	0-40%
	Charging stopped due to low battery temperature, high battery temperature or other error conditions
	Charging stopped due to lost communication with battery

Accessories

Mattress

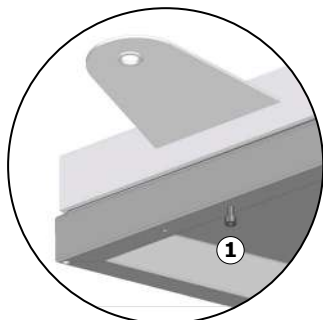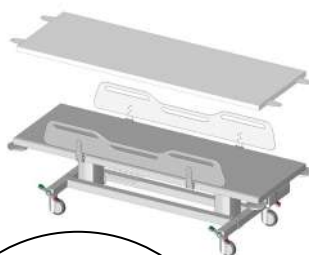The optional mattress can be placed on the change trolley for optimal comfort. Both sides of the mattress can be used, as it is reversible.


Unscrew the locking screw for the mattress ① approximately one centimeter.

Pull the mattress flap down over the screw. Do the same with all four flaps.

en Shower mattress (MSCT 1 only)

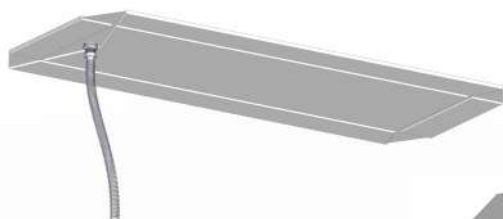
The MSCT 1 model can be used as a bathtub with the optional shower mattress (R844721 and R844717).


Attach the drain hose to the mattress drain, as per the included mounting instruction.

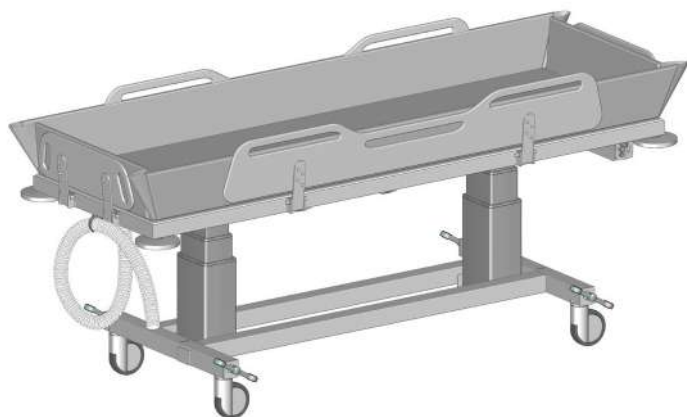Pull the drain hose through the shower trolley's drain hole, fold the mattress and place it on the trolley. Make sure that all four side supports on the shower trolley are folded up and locked.

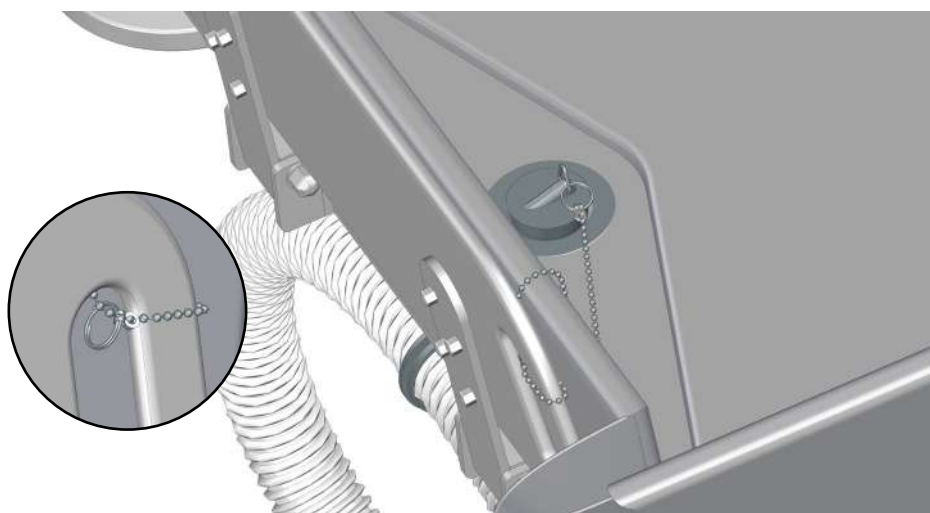Coil up the hose and attach it to the hose hanger.

Attach one key ring of the chain to the plug. Make a loop around one of the footboard handles with the other ring.

IMPORTANT

- Do not move the trolley when there is water in the shower mattress.
- Use gloves when handling the waste hose, as it may have been in contact with human waste.

Mounting instructions for both mattress types are included with the products.

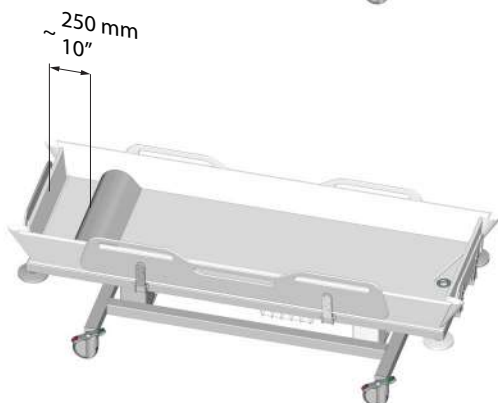
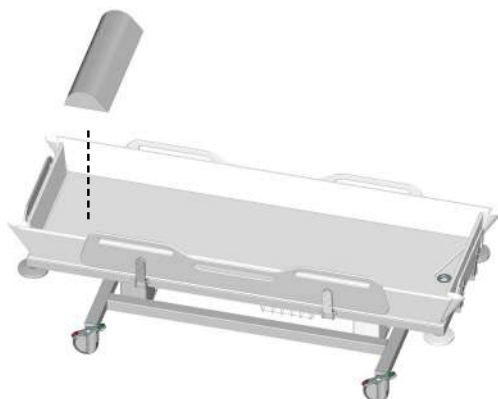
All instructions can also be downloaded from pressalit.com.

(MSCT 1 only) The instruction for the shower mattress provides instructions on how to attach the drain hose.

Head rest

The optional head rest (R844521) can be used with the mattress for added comfort during use of the change trolley.

Place the head rest at the head end of the trolley, approx. 250 mm (10") from the headboard.

The mattress and the head rest must be cleaned with hot water or a mild soapy cleaning agent.

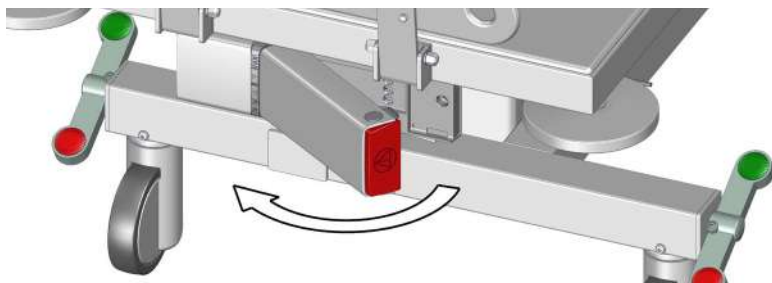
A list of suitable cleaning agents/disinfectants tested by Pressalit is available at pressalit.com.

IMPORTANT

To maintain hygiene, it is recommended to regularly check the mattresses and the head rest for visible damage or signs of wear and tear. If any damage is found, the mattresses and the head rest should be replaced immediately to prevent any risk of contamination or infection.

Cleaning

Remove the battery before cleaning the shower change trolley.

Clean the product regularly and between uses to maintain general hygiene and functionality. The product is to be cleaned with hot water or a mild soapy cleaning agent. Pay special attention to the user contact surfaces. A list of suitable cleaning agents/disinfectants tested by Pressalit can be found in the technical datasheet or on pressalit.com under the specific product.

CAUTION

- Always read the supplied instructions of the cleaning agent and comply with these instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.
- Do not perform any type of cleaning or disinfection of the change trolley while the user is lying on it.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acid or base.
- Do not spray the product directly with a high-pressure cleaner.
- Avoid use of chlorine-based cleaning or disinfecting products unless specifically permitted in the product specification.
- For surface disinfection, chlorine-based disinfectants may be used only in accordance with the product specification and safety precautions.

Disinfection

When using the change trolley for multiple patients, it is necessary to both clean and disinfect the trolley between uses on different patients to prevent the risk of cross infection. Disinfection and cleaning is also required after storage or maintenance.

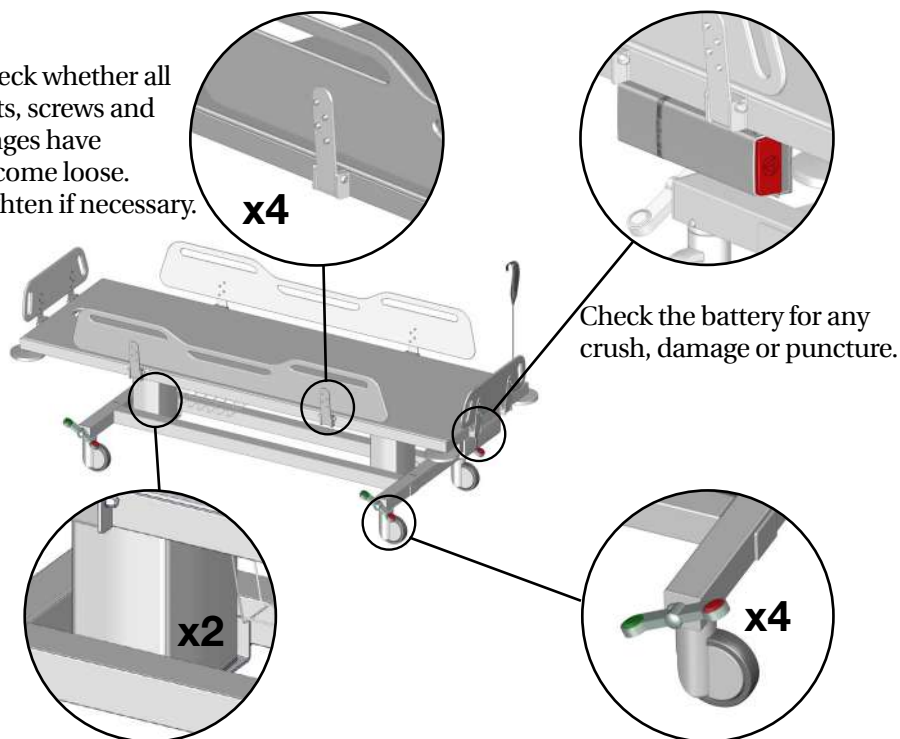
Maintenance

Maintenance inspection should be carried out once a year. The inspection should be recorded in a logbook describing what has been checked and replaced. Spare parts should always be obtained from your supplier or Pressalit in order to ensure a continuously safe product and the guarantee.

Contact your supplier or Pressalit for purchase of spare parts and drawings.

Perform a visual inspection of steel parts for penetrating rust.

Check whether all nuts, screws and hinges have become loose. Tighten if necessary.


Check lifting columns for wear and correct function, including wires, plugs and piston rod.

Check the wheels for corrosion. Replace them if necessary.

NOTICE

If the product is used in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.

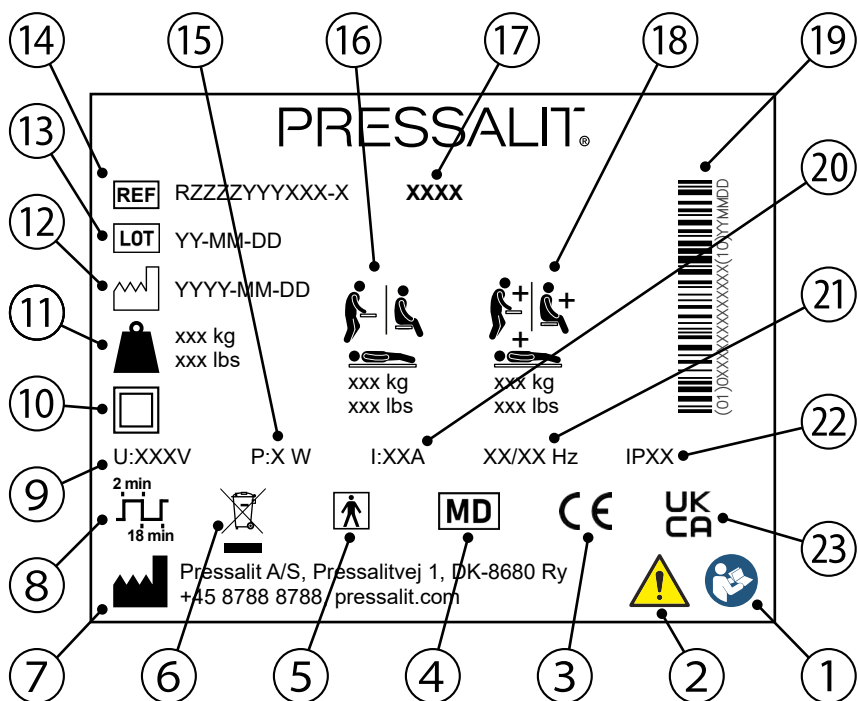Product label

- 1 Read the manual before use
- 2 Consult the manual for important cautionary information such as warnings and precautions
- 3 These products are CE-marked in accordance with relevant requirements in applicable legislation.
A declaration of conformity can be found at pressalit.com
- 4 Medical device
- 5 Type BF applied part. Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock
- 6 Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑦	Name and address of manufacturer
⑧	Duty cycle
⑨	Mains voltage
⑩	Class II Product, double insulated
⑪	Product weight
⑫	Production date
⑬	Batch code
⑭	Article number
⑮	Maximum power consumption
⑯	Maximum user weight
⑰	Model name
⑱	Max load
⑲	GS1-128 barcode
⑳	Electric Current
㉑	Frequency
㉒	IP class
㉓	UK Conformity Assessed. UK responsible representative: Pressalit Ltd., 100, Longwater Avenue, Green Park, Reading, Berkshire, RG2 6GP, United Kingdom.

Resetting

If the lifting function does not work properly or if the two lifting columns are out of sync, try resetting the lifting system.

Press and hold the up button and the down button simultaneously.

The columns will move down after a few seconds while a beep tone is heard.

Keep holding both buttons until the beep tone disappears.

The lifting columns are now reset.

Troubleshooting


Symptom	Cause	Action
No power.	Battery depleted.	Recharge the battery. See "Charging the battery" .
The change trolley appears skewed even though the tilt function (MSCT 1 only) is not activated.	The lifting columns are out of sync.	Reset the lifting columns. See "Resetting" .

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the warranty. Expected service life of the change trolley is 10 years. Expected service life of all spare parts are 10 years, but depending on the amount of use, the service life can be reduced.

en

System LED indicators

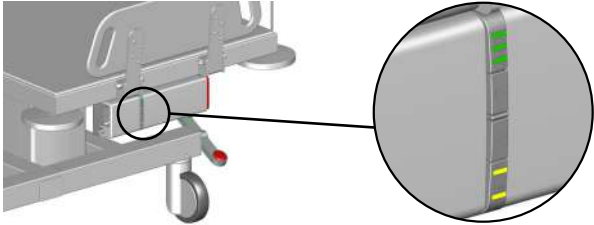The two yellow LEDs on the internal charger show the system status.

LEDs	Cause	Status	Action
 	Emergency stop activated	The unit cannot be raised or lowered	Release the emergency stop. See " Releasing the emergency stop "
 	System error	The unit cannot be raised or lowered	Reset the lifting columns. See " Resetting "
 	Unit overloaded	The unit cannot be raised or lowered	Reduce the load
 	Duty cycle guard	The unit cannot be raised or lowered	Wait until the unit is ready
 	Service needed	The unit is still functioning	Contact Pressalit A/S or your local dealer

Electromagnetic compatibility

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Electromagnetic emissions




Test standard	Test level / details	Test results
60601-1-2 ch. 7.1.7 - CISPR 14	30MHz – 1GHz	Passed

Electromagnetic immunity

Test standard	Compliance level	Test level / details	Test results
EN60601-1-2 and EN17966 (Home healthcare environment)	=> 10V/m 2,7GHz	15V/m Frequency range: 80MHz- 6GHz 1% step size	Passed
EN60601-1-2 (Proximity fields from RF wireless communications equipment)	=> 10V/m 2,7GHz	15V/m Frequency range: 385 - 5785 MHz	Passed
IEC 61000-4-4 (Electrical fast transients / bursts)		Quick pulse simulation	Passed
Conducted disturbances induced by RF	Pulse subjection, 6V	Pulse subjection, 10V	Passed
Electrostatic discharge		Statically charged person discharge on unit, 8kV and 15kV	Passed (*)

* If a statically charged person discharges on or near the shower trolley control box, a fault may occur in the lifting columns, and a reset must be performed. See "**Resetting**".

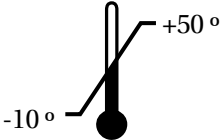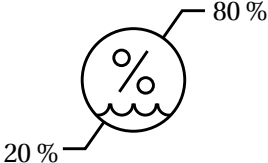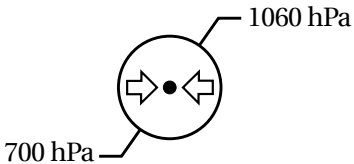Storage and transportation

	The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.
	The relative humidity for transportation and storage of the unit must be between 20 and 80% (RH).
	The air pressure for transportation and storage of the unit must be between 700 to 1060 hPa.

IMPORTANT

- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Technical data

Max. user weight	200 kg / 441 lbs
Max load	250 kg / 552 lbs
Materials	Powder-lacquered stainless steel Plastics Compact Density Fiberboard (CDF)

Weight	(MCT 2) R9414717, R9434717, R9454717, R9464717	102 kg / 225 lbs
	(MCT 2) R9414721, R9434721, R9454721, R9464721	106 kg / 234 lbs
	(MSCT 1) R9414717, R9434717, R9454717, R9464717	107 kg / 236 lbs
	(MSCT 1) R9414721, R9434721, R9454721, R9464721	111 kg / 245 lbs
IP class	The product has an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.	
Noise level	≤ 50 dB (A)	

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected. When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion. The following table provides information on the recycling and handling of the product. If in doubt contact your local authorities to determine the proper method of disposal.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available. If such systems are not available, incineration of plastic waste can be an alternative, as it can generate energy that can be used for power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.
Batteries	Disposal of the battery must be in accordance with local regulations.

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