




<p>Product Item number</p>	<p>PLUS backrest, vertical and horizontal adjustable R6444035</p>
<p>Color Product description</p>	<p>NRF:6317051 EAN:5708590365527 HMI: 124205 FDA:INN/D135865</p> <p>Red (RAL 3003)</p>  <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 15.2" x 8.2", soft feel. Height adjustable 150 mm and sideways adjustable, for mounting on a horizontal walltrack behind a wall-mounted toilet without cistern. Can be removed easily when not needed and reattached again without the use of tools. Materials: Aluminium and polyurethane foam with a plywood core. For fitting to horizontal PLUS wall track (not included).</p>
<p>Adjustability Adjustment height Product capacity</p>	<p>Vertical and horizontal adjustable</p> <p>5,91 inch</p> <p>Maximum load: 110 lbs. Maximum user weight: 220 lbs. When the backrest is loaded with 110 lbs, each wall fastener will be carrying 73 lbs. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 110 lbs, it is – according to ISO 17966 – the equivalent of a user weighing 220 lbs leaning on the backrest.</i></p>
<p>Materials Ergonomy</p>	<p>The backrest is made of polyurethane (PU) foam which is covered with a moulded coated surface that is protected using a clear PU lacquer.</p> <p>Soft and slip-resistant material that has a warm feel.</p>

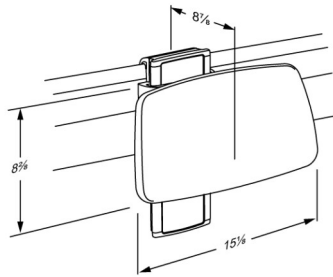

Product	PLUS backrest, vertical and horizontal adjustable
Item number	R6444035
Features	For tankless toilet
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p>NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>



PRESSALIT®

Product	PLUS backrest, vertical and horizontal adjustable
Item number	R6444035

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®