R2122, R2123
R2150, R2151
R2152, R2153
R2170, R2172

R4822, R482219
R4827, R482719
R4828, R4850
R485019, R4852
R485219, R4855
R485519, R4857
R485719, R4870
R4875

MATRIX

Wash basin lifter – Operation and maintenance manual	en
Vaskeløfter – Brugs- og vedligeholdelsesvejledning	dk
Waschtisch-Lifter – Gebrauchs- und Pflegeanleitung	de
Support de lavabo – Manuel d'utilisation et d'entretien	fr
Muurframe voor wastafel – Gebruiks- en onderhoudshandleiding	nl
Upphängning till tvättställ – Bruks- och underhållsanvisning	sv
Oppheng til servant – Bruks- og vedlikeholdsveiledning	no
Soporte de lavabo - Instrucciones de uso y mantenimiento	es
Supporto per lavabo - Manuale d'uso e manutenzione	it
Wspornik umywalki – Instrukcja obsługi i konserwacji	pl
洗面盆支架 - 操作和维护手册	中文

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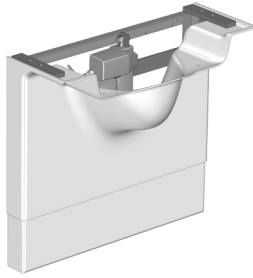
Pressalit's Environmental and Quality Management Systems are certified by LRQA

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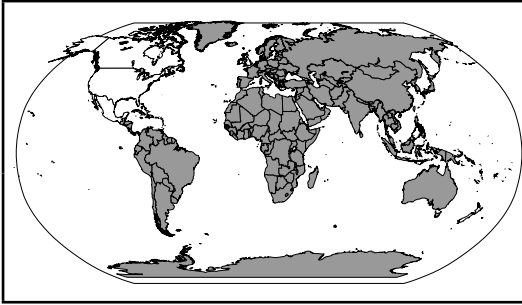
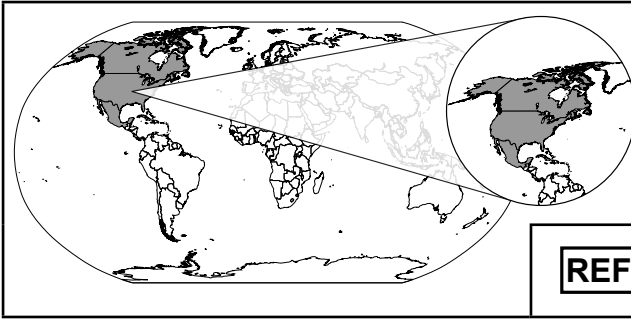
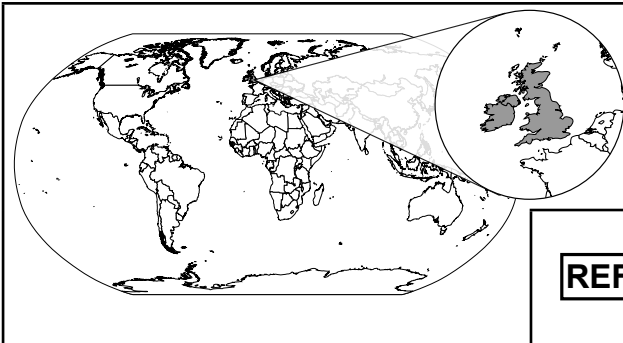
REF


R4827, R4855, R4857
R4875, R482719
R485519, R485719

	Max P	U	Max I
R4822, R4850, R4852, R4870, R482219, R485019, R485219, R4827, R4855, R4857, R4875, R482719, R485519, R485719	330W	230V	1,5A
R4828	330W	120V	2,75A

		
R4822, R4827, R4828 R4850, R4855, R4852 R4857, R4870, R4875	18,8 kg / 41,5 lbs	150 kg / 330 lbs
R482219, R482719 R485019, R485219 R485519, R485719	22,5 kg / 50 lbs	

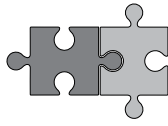R2122 ▶ R4822, R4827, R4828
R482219, R482719

R2123 ▶ R4822, R4827, R4828
R482219, R482719

R2150 ▶ R4850, R4855, R485019,
R485519

R2151 ▶ R4850, R4855, R485019,
R485519

R2152 ▶ R4852, R4857, R485219,
R485719

R2153 ▶ R4852, R4857, R485219,
R485719

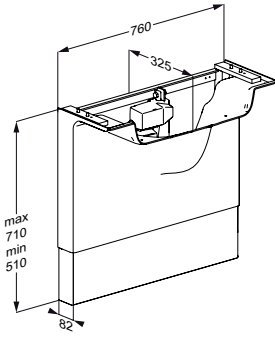
R2170 ▶ R4870, R4875


R2172 ▶ R4870, R4875


mm


R4822, R4827, R4828

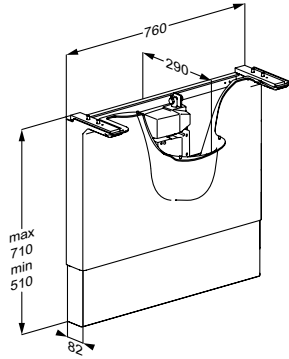
R4850, R4852, R4855, R4857

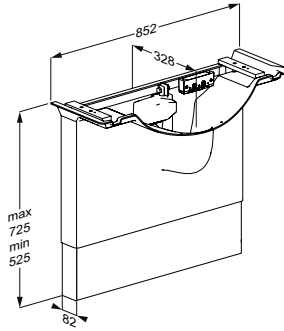
R4870, R4875

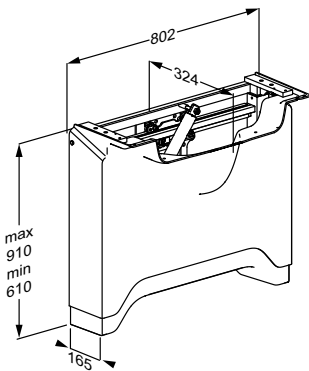
R482219, R482719

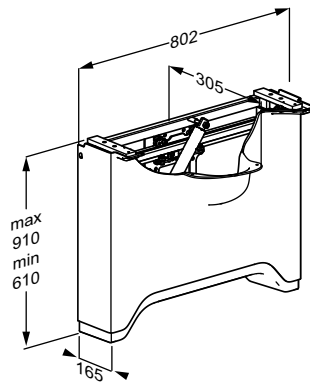
**R485019, R485519, R485219,
R485719**

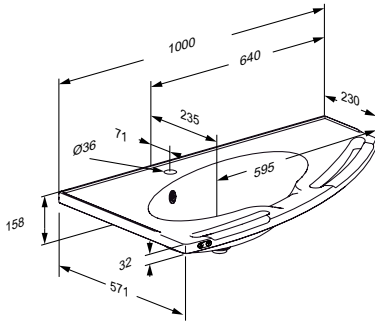
R2150, R2151

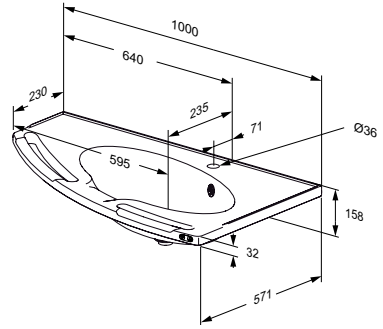
R2152, R2153

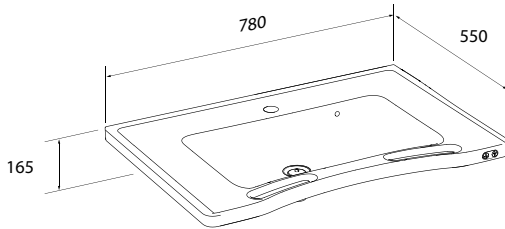R2122, R2123

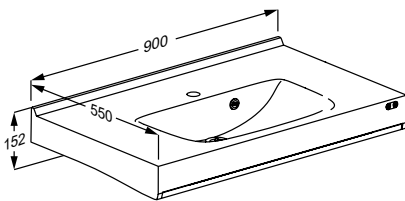
R2170

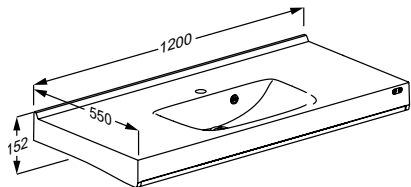
R2172

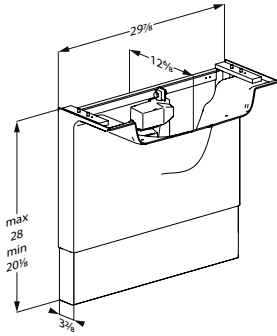
inch


R4822, R4827, R4828

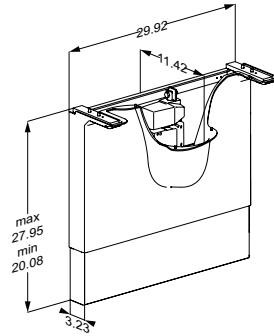
R4850, R4852, R4855, R4857

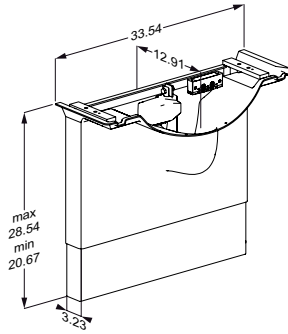
R4870, R4875

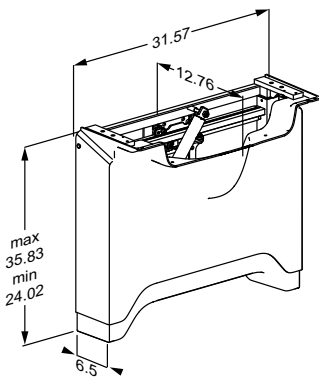
R482219, R482719

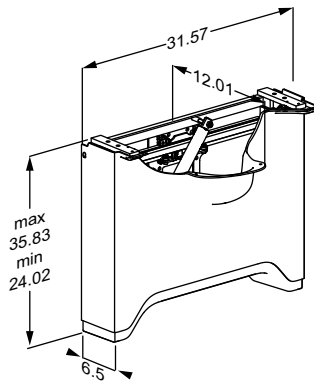
**R485019, R485519, R485219,
R485719**

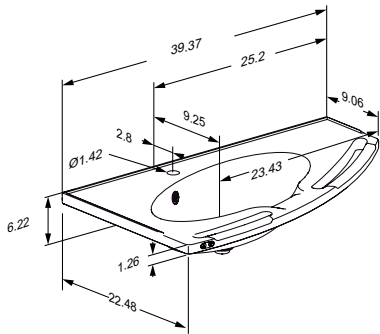
R2150, R2151

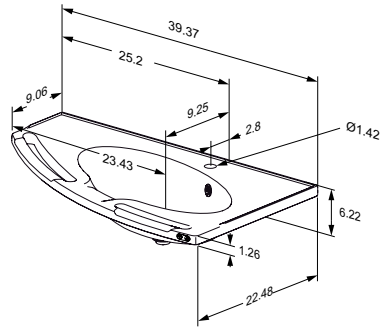
R2152, R2153

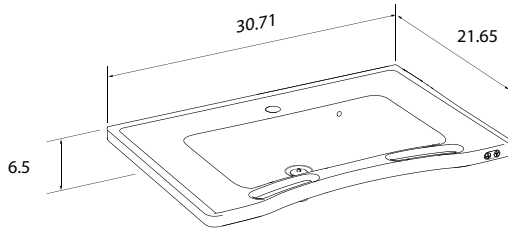R2122, R2123

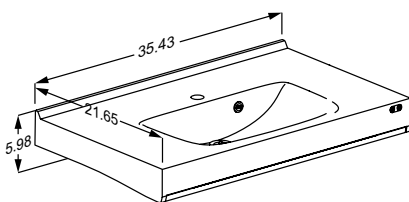R2170

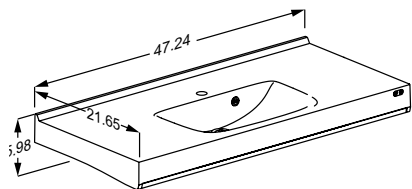R2172

Content

Mounting instruction	12
General safety	13
Symbols used in this manual	14
General safety	15
Safety distances	16
Intended use	18
Intended operator profile	18
Intended user profile	18
Operating environment	18
Contraindications for use	18
Operating instructions	19
Sideways adjustment	20
Safety rail	21
Resetting the wash basin lifter	21
Cleaning	22
Maintenance	23
Maintenance inspection	23
Product label	28
Trouble-shooting	30
Electromagnetic compatibility	31
Electromagnetic immunity	32
Electromagnetic emission	36
Technical data	37
Storage and transportation	37
Disposal and recycling	38

en

Mounting instruction

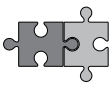
A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

				<p>WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>
				<p>CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p>
				<p>IMPORTANT indicates important information about handling and use of the product.</p>
				<p>NOTICE is used to address practices not related to personal safety.</p>
 <p>Head injury warning</p>	 <p>Hands injury warning</p>	 <p>Genitalia injury warning</p>	 <p>Feet injury warning</p>	
 <p>Electricity / voltage warning</p>	 <p>Compatible products</p>	 <p>Article number / versions</p>	 <p>Power specifications</p>	
 <p>Product weight</p>	 <p>Max. user weight</p>			

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

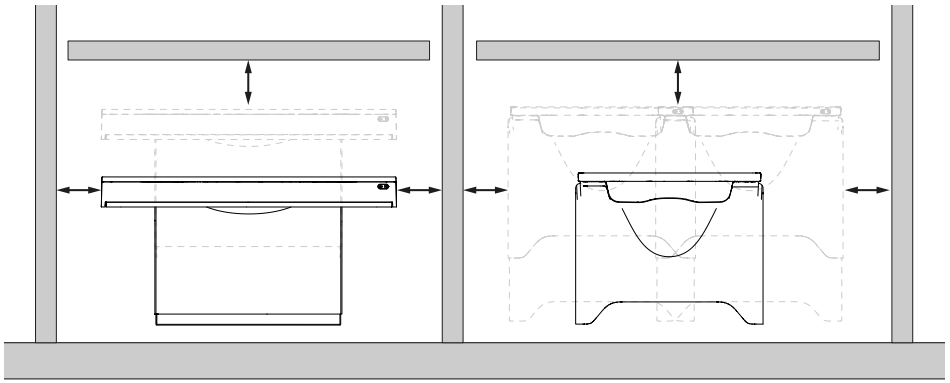
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.






Safety distances

Make sure that the wash basin lifter is at safe distance from surrounding objects. Check the distances before any use.

ISO 17966:2016		
	Max 120 mm (4.72") Min 300 mm (11.81")	Max 60 mm (2.36") Min 300 mm (11.81")
	Max 8 mm (0.31") Min 25 mm (0.98")	Max 4 mm (0.16") Min 25 mm (0.98")
	Max 35 mm (1.38") Min 120 mm (4.72")	Max 25 mm (0.98") Min 120 mm (4.72")
	Max 8 mm (0.31") Min 75 mm (2.95")	Max 8 mm (0.31") Min 75 mm (2.95")

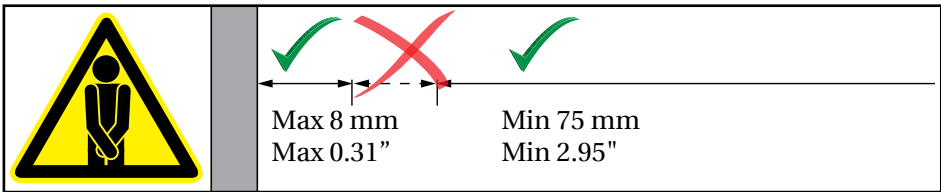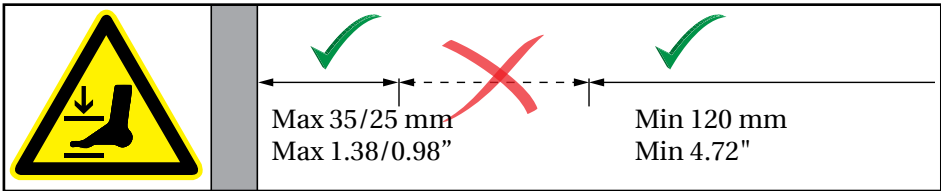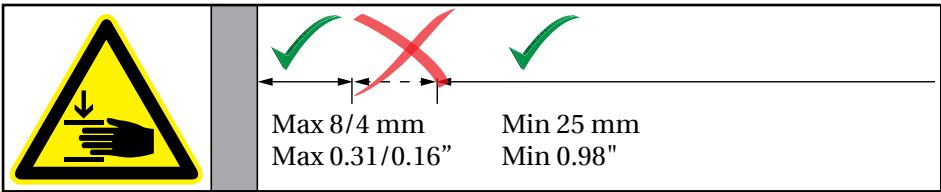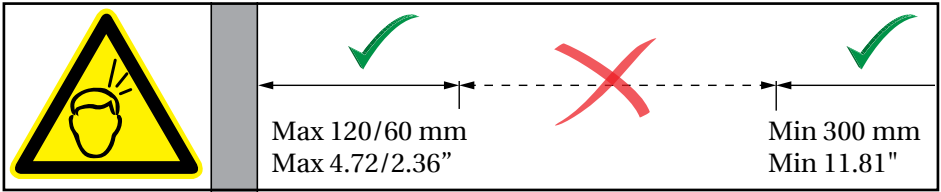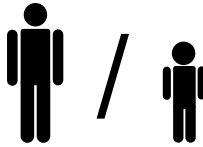en

 **WARNING**

If the unit moves within the safety distances, you must keep an especially close eye on surrounding people and objects to avoid crushing hazards.

Intended use

This product is intended for optimum ergonomics at a wash basin, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Intended user profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature, in private homes, nursing homes and hospitals under normal working conditions.

Contraindications for use

People who are not capable of keeping their balance in an upright position are not recommended to use this wash basin lifter.

en

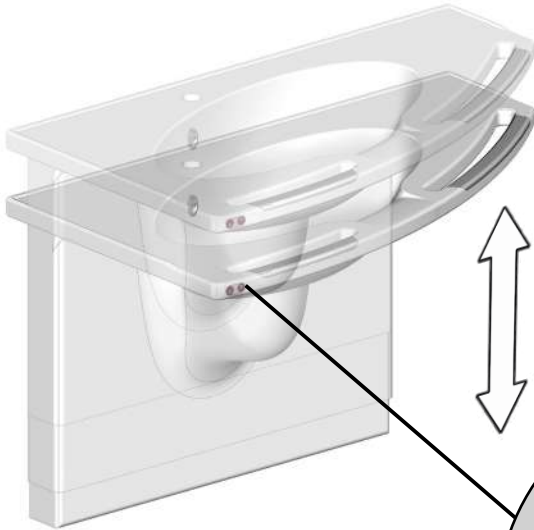
The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

NOTICE


Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

Operating instructions

The wash basin lifter models R4822, R4827, R4828, R4850, R4855, R4852, R4857, R4870, and R4875 can be adjusted 200 mm (7.87") up and down.
Max load is 150 kg/330 lbs.

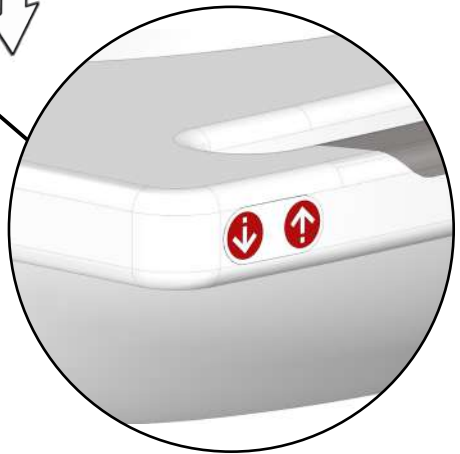The height adjustment is controlled by operating buttons in the front edge of the wash basin

Press and hold the up button to raise the wash basin, and press and hold the down button to lower it. When the button is released, the motion stops.

 **CAUTION**

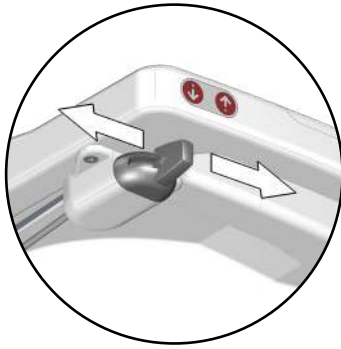
Never place objects underneath the wash basin.

 **Take care to avoid injury to hands when in the vicinity of moving mechanical parts.**

Sideways adjustment

The wash basin lifter models R482219, R482719, R485019, R485519, R485219 and R485719 can be adjusted 300 mm (11.81") up and down, and also 500 mm in total (19.69") to either the left or right.

Max load is 150 kg/330 lbs.

Unlock the sliding mechanism by pressing and holding the side adjustment button to either the left or right.

Grasp the handles of the sink and push the

wash basin lifter to either the left or right. The lifter slides quite easily, so you don't have to use much effort.

When the wash basin lifter is in the desired position, release the side adjustment button and the lifter will lock in place with a click.

CAUTION

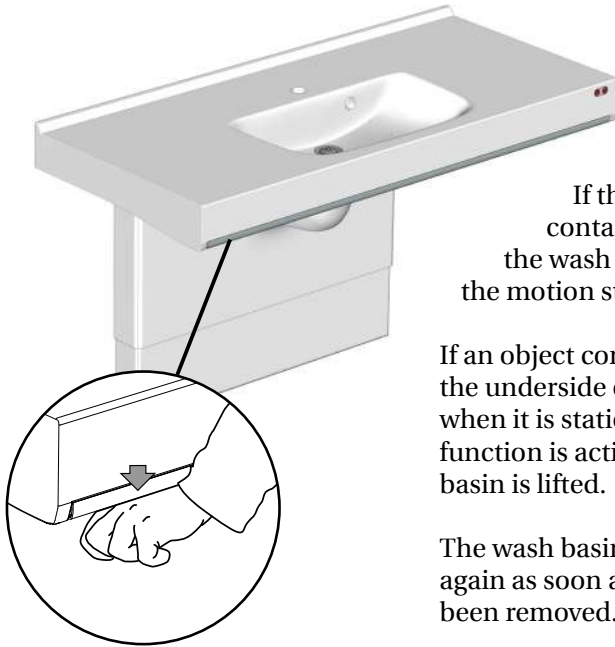
Make sure the locking mechanism is properly engaged before any use or height adjustment. This is ensured by pushing the wash basin lifter a little to the right or left until a click is heard.

Be careful not to injure your feet when moving the wash basin lifter sideways

Safety rail

The wash basin lifters **R4870** and **R4875** are equipped with a safety rail.

If the safety rail comes in contact with an object when the wash basin is being lowered, the motion stops immediately.

If an object comes in contact with the underside of the wash basin when it is stationary, the safety function is activated and the wash basin is lifted.

The wash basin lifter can be used again as soon as the obstruction has been removed.

Resetting the wash basin lifter

If the wash basin lifter is jammed, try to reset it.

- ① Make sure that the safety rail is not activated. Lower the wash basin lifter all the way down to its starting position.
- ② Release the lowering button, and after 3-4 sec. press it in again until you hear a slight click after approx. 10 sec.
- ③ Release the lowering button and then raise the wash basin lifter to its highest position. You will hear another click.
- ④ Lower the wash basin lifter a couple of centimeters.
- ⑤ Raise it again to its highest position. You will hear another click.
The wash basin lifter is now reset. The wash basin lifter should also be reset if there has been an interruption of the power supply.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

en

Maintenance

Expected service life of the wash basin lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that maintenance inspection is done once a year.

Maintenance inspection

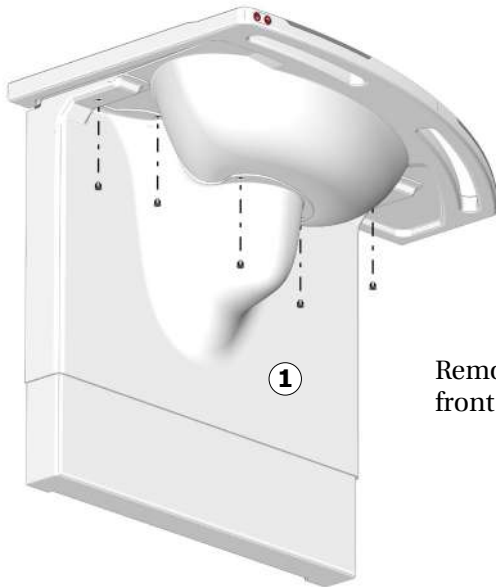
Take the wash basin to its highest position.


Disconnect mains before removing the covers.

For R4822, R4827, R4828, R4850, R4855, R4852 and R4857


en

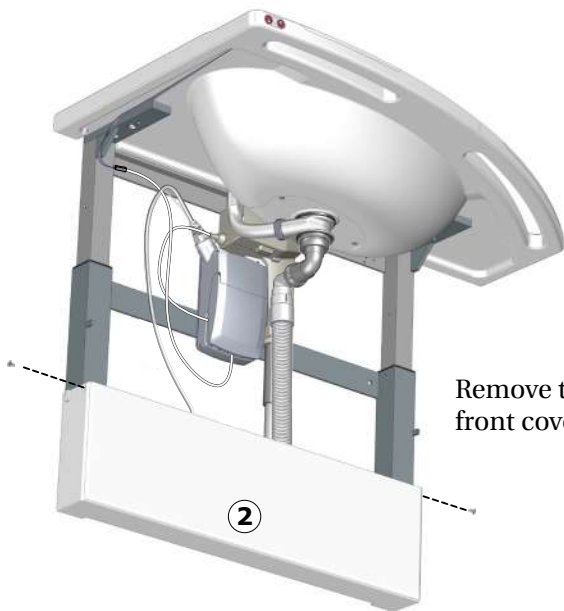Remove the screws and the upper front cover ①.

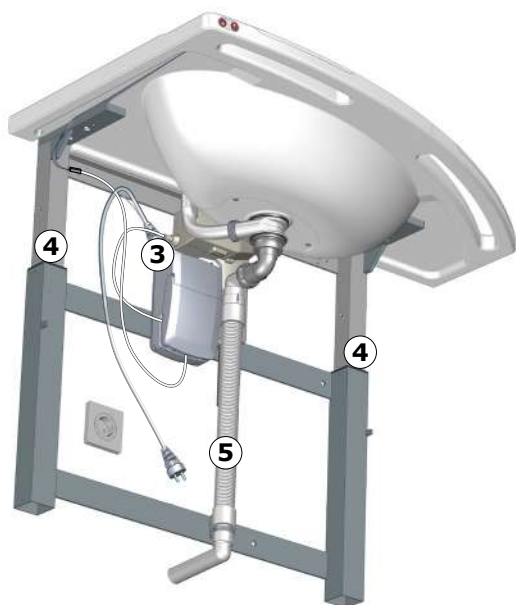Remove the screws and the lower front cover ②.

Check all electrical connections ③.

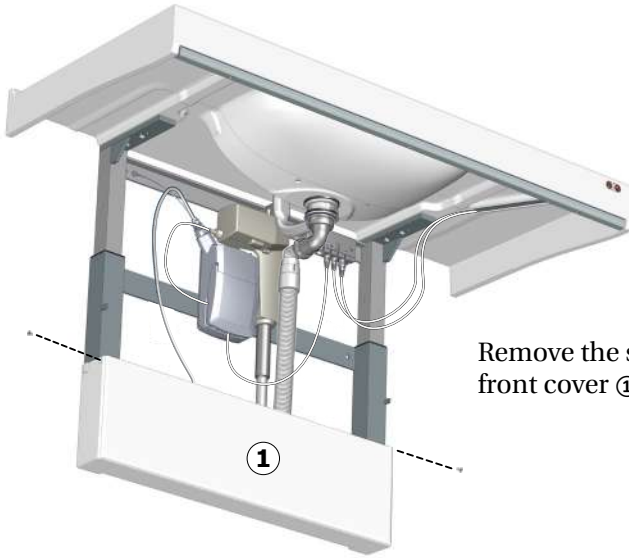
Lubricate the inner square tubes ④ and the actuator rod ⑤.

Reattach the front and the top cover.

Reconnect mains.

Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

For R4870 and 4875


Remove the screws and the lower front cover ①.

en

Check all electrical connections ②.

Lubricate the inner square tubes ③ and the actuator rod ④.

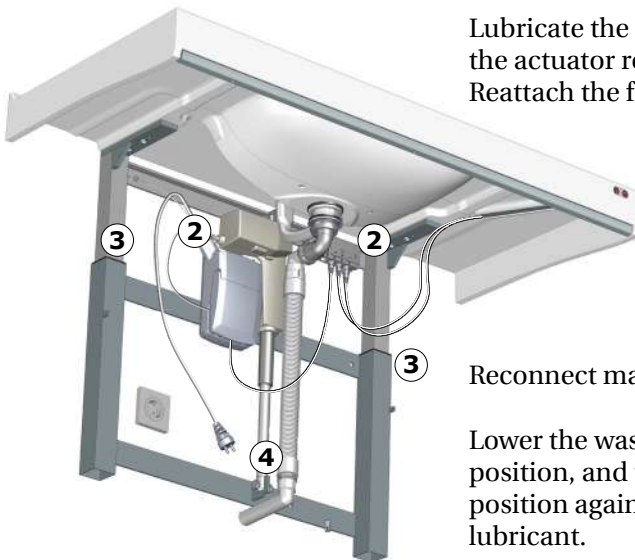Reattach the front and the top cover.


Reconnect mains.

Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.


For R482219, R482719, R485019, R485219, R485519, R485719

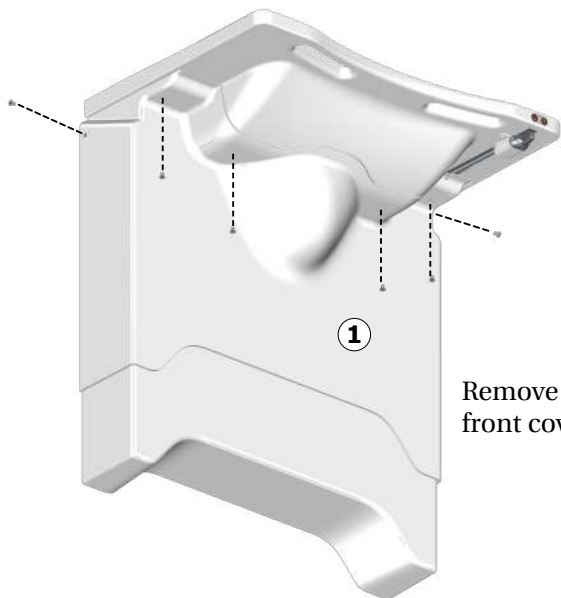Take the wash basin to its highest position.


Remove the screws and the upper front cover ①.

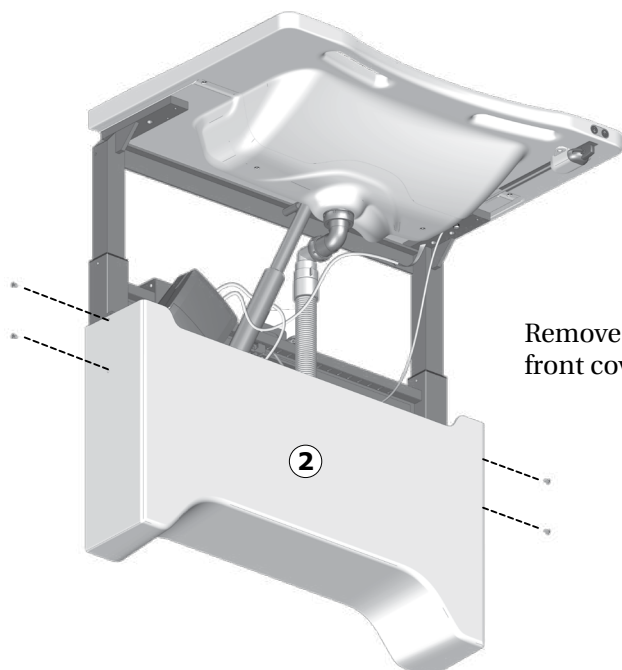en

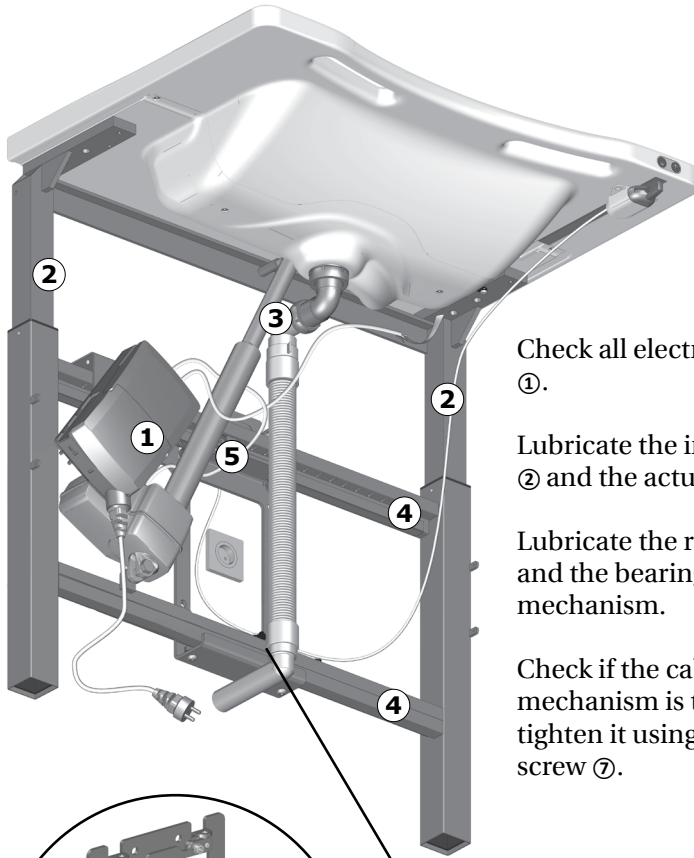Remove the screws and the lower front cover ②.



Check all electrical connections ①.

Lubricate the inner square tubes ② and the actuator rod ③.

Lubricate the rails ④, the lock ⑤ and the bearings ⑥ for the sliding mechanism.

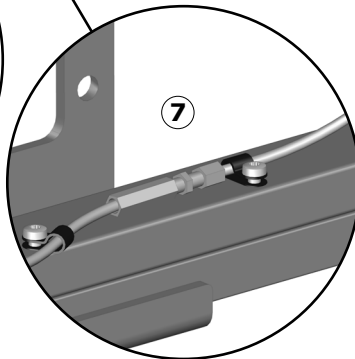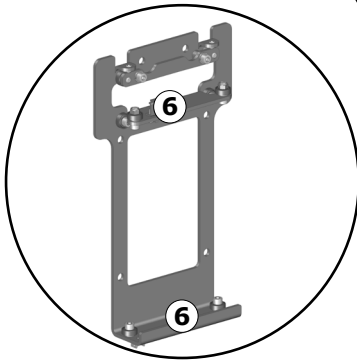Check if the cable for the locking mechanism is too loose. If so, tighten it using the adjustment screw ⑦.

Reattach the front and the top cover, and reconnect mains.
Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.
Also slide the wash basin lifter to its extreme left and right positions.

⑩ IP class

⑪ Electric Current

⑫ UDI carrier GS1-128 barcode

⑬ Product weight

⑭ Batch code

⑮ UK Conformity Assessed

These products are CE-marked in accordance with relevant requirements in:

- ⑯
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
 - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at pressalit.com

⑰ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

⑱ Product name (if any)

⑲ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑳ Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock.

㉑ Duty cycle

㉒ Consult the manual for important cautionary information such as warnings and precautions.

㉓ Consult manual for use

㉔ Name and address of manufacturer

㉕ Class II Product, double insulated

Trouble-shooting

Symptom	Cause	Action
Operating buttons does not work.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under “Maintenance”.
The product cannot be moved sideways.	Hoses or cables may be stuck.	Remove the cover(s) and check if anything is jammed.
	The locking mechanism may be stuck.	Check the lock and the cable as described under “Maintenance”.
Safety system does not work.	Faulty switches.	Check whether one or more of the safety system contacts is stuck, and if so, try to untangle it. If the problem cannot be fixed, replace the contact(s). The safety system will only work when current can be detected. Remember to reset as described under “Resetting the wash basin lifter”.

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Electromagnetic compatibility

The device is suitable for use at home, at daycare centres, at day centres for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and RF shielded rooms with medical systems for magnetic resonance imaging with high intensity of electromagnetic disturbances.

The device does not have any function related to basic safety or essential performance. In the event of performance loss or degradation caused by electromagnetic disturbances, the user's safety remains uncompromised.

Cable	Length
Power cable	3,2 m (126")
Hand control spiral cable	Approx. 5 m (197") extended

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Electromagnetic immunity

Electrostatic discharge	
	The product must not be affected by electrical discharge between a person and the product in such a way as to initiate actions or render components defective.
Test standard	Immunity test level
EN 61000-4-2	+/- 8 kV contact +/- 2, 4, 8, 15 kV air


Radiated RF electromagnetic field	
	The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.
Test standard	Immunity test level
EN 61000-4-3	15 V/m 80 MHz to 1000 MHz 15 V/m 1,01 GHz to 6 GHz 80 % AM 1 kHz

en

Immunity to proximity fields from RF wireless communication equipment

	<p>The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.</p>
<p>Test standard</p>	<p>Immunity test level</p>
<p>EN 61000-4-3*</p>	<p>385 MHz, 27V/m, 18 Hz PM (50 % duty cycle square wave) 450 MHz, 28V/m, FM +/- 5 kHz deviation, 1 kHz sine 710 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 745 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 780 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 810 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 870 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 930 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 1720 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 1845 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 1970 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 2450 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 5240 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 5500 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 5785 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)</p>

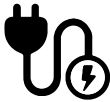Electrical fast transient/burst

	<p>The product must not be affected by rapid pulse signals through the cord. This could cause the components to become defective, or the product could initiate actions.</p>
<p>Test standard</p>	<p>Immunity test level</p>
<p>EN 61000-4-4</p>	<p>+/- 1 kV , +/- 2 kV 100 kHz repetition frequency</p>

Surge immunity

The product must not be affected to initiate actions or to render components defective as a result of lightning.

Test standard**Immunity test level**

EN 61000-4-5

Line-to-line: +/- 0.5 kV, +/- 1 kV
Line-to-ground: +/- 0.5 kV, +/- 1 kV, +/- 2 kV (N/A)

Immunity to conducted disturbances induced by RF fields

The wires must not be affected by other RF fields so as to initiate actions or render components defective.

Test standard**Immunity test level**

EN 61000-4-6

3 V (6 V in ISM bands and amateur radio bands)
0,15 to 80 MHz, 80 % AM at 1 kHz


Power frequency magnetic field immunity

The product must not be affected to initiate actions or to render components defective as a result of magnetic fields.

Test standard

EN 61000-4-8

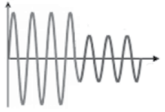30 A/m, 60 Hz

Voltage dips	
	The product must not initiate or limit actions as a result of a dip in the power supply.
Test standard	Immunity test level
EN 61000-4-11	0 % U_T ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % U_T ; 1 cycle at 0° 70 % U_T ; 25 cycles at 0° Single phase at 0°

Voltage interruptions	
	The product must not initiate or limit actions as a result of a short-term power supply stop.
Test standard	Immunity test level
EN 61000-4-11	0 % U_T ; 300 cycle

en

Electromagnetic emission

Conducted RF emission	
	The product must not send too much noise into the grid. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1

Radiated RF emission	
	The product must not emit RF signals in the room. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1

Harmonic current emissions	
	The product must not unintentionally draw power from the grid.
Test standard	Emission test class and group
IEC 61000-3-2	Class A


Voltage changes, voltage fluctuations and flicker	
	The product must not affect other products when it is connected to the grid. For example, these could be lights that could start flashing and thus bother people in the room.
Test standard	Emission test class and group
EN 61000-3-3	Tested according to the standard.

en

Technical data

Materials	Powder laquered stainless steel. Plastics (PVC, Polystyrene, ABS).
IP class	The electrical parts have an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.
Measured sound pressure level	48 dB

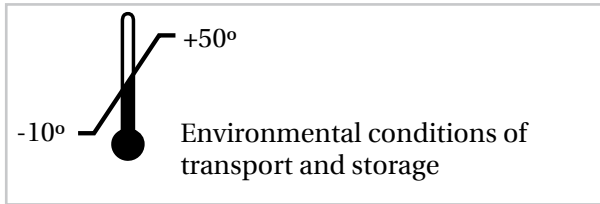Storage and transportation

- The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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