


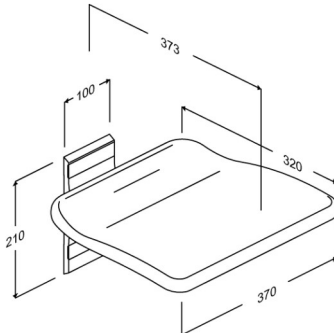

Product Item number	PLUS folding seat, fixed height R5510112
Colour Product description	VVS:777531012 NRF:7501002 RSK:8565072 EAN:5708590321332 HMI: 130159 NBS:N13:448 FDA:INN/D135865 Anthracite grey  The seat is suitable for situations where additional seating is required and space is limited, - e.g. in kitchens, lifts and corridors or on the landings. The folding seat is not suitable for use in the shower area. PLUS folding seat. For fixed mounting. Materials: Aluminium, thermoset plastic, plastic. Excl. mounting kit.
Adjustability Product capacity	Fixed mounting Maximum user mass: 435 kg. When the seat is loaded with 435 kg, each wall fastener is loaded with 199 kg. Please note that a user weight of 435 kg may require a reinforced wall for secure installation. A user weight of up to 262 kg can be installed on a standard wall, depending on the wall type, provided that our installation instructions are followed. When the seat is loaded with 262 kg, each wall fastener is loaded with 120 kg. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
Materials Ergonomy Features	The folding seat is made out of thermoset plastic, and the mounting and suspension bracket are made from 10 my anodised aluminium. The end caps are made from ABS plastic. Ergonomically shaped seat, which offers good and comfortable support. The seat can be folded away when not in use, - or to free up space. Stays up of itself when folded away.



PRESSALIT®

Product	PLUS folding seat, fixed height
Item number	R5510112
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.



Product	PLUS folding seat, fixed height
Item number	R5510112

Installation in different wall types

	Concrete >B25 V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. To be ordered separately. 
	Solid Brick >= Mz12 V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. To be ordered separately. 
	Solid Sand Lime Brick >= Hlz12 V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. To be ordered separately. 
	Aerated Concrete P4 V8654 - Mounting kit, 4 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately. 



Product Item number	PLUS folding seat, fixed height R5510112
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.