

# PRESSALIT.

Product	SCT 2000 shower change table, coated mesh cover, electrically height adjustable		
Item number	R8581218000		
	VVS:777543720		
	EAN:5708590376103		
	HMI: 140137		
	NBS:N13:40,375		
	1180		
Colour	White		
	910		
Product description	Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixe shower and changing tables of high quality. They are designed to provide a platform for showering and changing adults and children with reduced mobi These tables can be used in schools, institutions, hospitals and private hom are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for carer as well as a high level of comfort and safety for the user.		
	SCT 2000 shower change table, powered, height adjustable 700 mm; from 30 1000 mm, with wired hand control Foldable. Incl. safety rail and water collection tray with integrated outlet. Outer measurements 910 mm x 1800 mm. Excl. mounting kit.		
Adjustability	Height adjustable		
Adjustment height	700 mm		
Product capacity	Maximum user weight (MUM): 200 kg. Safe working load (SWL): 200 kg. When the table is loaded with 200 kg, each mounting bolt supports 26 kg This product has been tested statically with a load of 500 kg as the IEC 60601-1 standard requires testing with safety factor 2.5.		
Speed upwards	Speed up mm: 19 mm/s Speed down mm: 21 mm/s		
Noise level	50 dB		
Materials	Construction in powder-laquered stainless steel, mesh cover in PVC-coated polyest (for extra durability), water collection tray in PVC-U and carriage housing in polystyrene.		



## PRESSALIT.

Product	SCT 2000 shower change table, coated mesh cover, electrically height adjustable		
Item number	R8581218000		
Ergonomy	The 2000 series shower and changing tables has a low start position, to make transfers to the bench easier. The washable, perforated mesh cover lets the water pass through to the water collection tray underneath.  The flexibility of the shower and changing table provides for optimal working conditions for the carer as well as a high level of comfort and safety for the user.		
Features	A water collection tray with integrated water outlet prevents water from splashing on carers or the floor. The flexible hose attached to the centre mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. The integrated gas cylinders enable the table to be folded out of the way when not in use to maximise floor space in front of the unit.  Electric with wired hand control – the red buttons clearly indicate "Up" and "Down". The wired hand control comes with a holder for fitting to the wall.  The stretcher of the table is very easy to clean, and the stretcher cover can be unzipped for washing or easy replacement.		
Mains connection (V)	230		
Actuator (motor)	LA40 Careline (Linak), 4000 N, IPX6		
Control box	CA63 Careline, (Linak), IPX6		
Wired remote control	HB30 (Linak), IP66		
Power consumption (maximum load)	Power consumption by maximum load, watt: 420 W Power consumption by standby: 0,5 W		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.		



# PRESSALIT.®

Product	SCT 2000 shower change table, coated mesh cover, electrically height adjustable		
ltem number	R8581218000		
Cleaning	SCT 1000 & 2000 The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.  The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.  DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREVention Wipes Lingettes OxiGenesis from ENVIROX		
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro		
	The product must be wiped down with a soft rag wrung out in clear water immedia after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.		



### PRESSALIT®

Product

SCT 2000 shower change table, coated mesh cover, electrically height adjustable

Item number

R8581218000

#### Installation in different wall types

Concrete >B25  V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12.  To be ordered separately.	
Solid Brick >= Mz12  V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12.  To be ordered separately.	
Solid Sand Lime Brick >= HIz12  V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12.  To be ordered separately.	
Aerated Concrete P4  V8658 - Mounting kit, 6 coach screws Ø10 x 70 - plugs GB14.  To be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood  V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs  SX12. NB! Plugs are not needed.  To be ordered separately.	



#### PRESSALIT®

Product Item number	SCT 2000 shower change table, coated mesh cover, electrically height adjustable R8581218000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood  Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)  Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.
	Drywall (2 x 12,5 mm) water-resistant  Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com