R9612714, R9612717  
R9612719, R9612721  
R9632714, R9632717  
R9632719, R9632721  
R9652714, R9652717  
R9652719, R9652721  
R9662714, R9662717  
R9662719, R9662721

### MCT 3

|                                                                       |           |
|-----------------------------------------------------------------------|-----------|
| <b>Change trolley – Operation and maintenance manual</b>              | <b>en</b> |
| <b>Mobilt skifteleje – Brugs- og vedligeholdelsesvejledning</b>       | <b>dk</b> |
| <b>Mobile Ankleidliege – Gebrauchs- und Pflegeanleitung</b>           | <b>de</b> |
| <b>Table d'habillage mobil – Manuel d'utilisation et d'entretien</b>  | <b>fr</b> |
| <b>Mobiel aankleedtafel – Gebruiks- en onderhoudshandleiding</b>      | <b>nl</b> |
| <b>Mobil vårdbräda – Bruks- och underhållsanvisning</b>               | <b>sv</b> |
| <b>Mobilt stellebord – Bruks- og vedlikeholdsanvisning</b>            | <b>no</b> |
| <b>Camilla para cambio – Instrucciones de uso y mantenimiento</b>     | <b>es</b> |
| <b>Barella per cambio di indumenti - Manuale d'uso e manutenzione</b> | <b>it</b> |
| <b>Vaatteiden vaihtovaunu – käyttö- ja huolto-opas</b>                | <b>fi</b> |
| <b>Wózek toaletowy – Instrukcja obsługi i konserwacji</b>             | <b>pl</b> |

|                        |     |
|------------------------|-----|
| English . . . . .      | 10  |
| Dansk . . . . .        | 31  |
| Deutsch . . . . .      | 52  |
| Français . . . . .     | 73  |
| Nederlands . . . . .   | 94  |
| Svenska . . . . .      | 115 |
| Norsk . . . . .        | 136 |
| Español . . . . .      | 157 |
| Italiano . . . . .     | 178 |
| Suomi . . . . .        | 199 |
| Język polski . . . . . | 220 |

Pressalit's Environmental and Quality Management Systems are certified by LRQA

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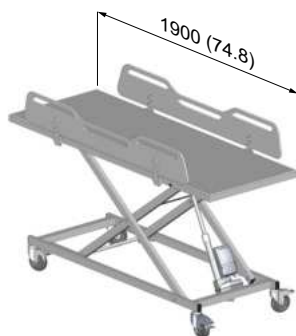
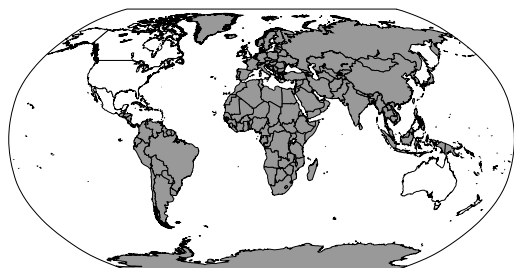
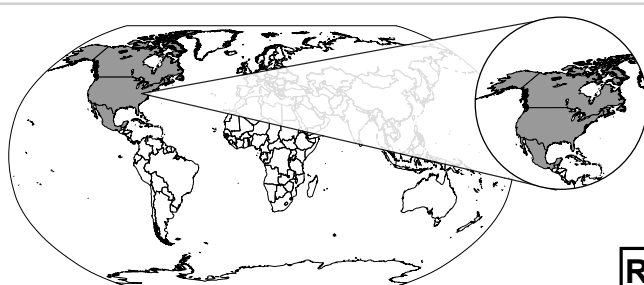
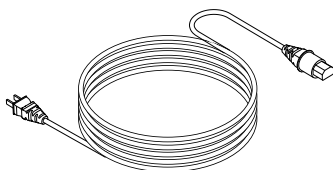
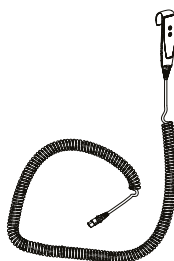
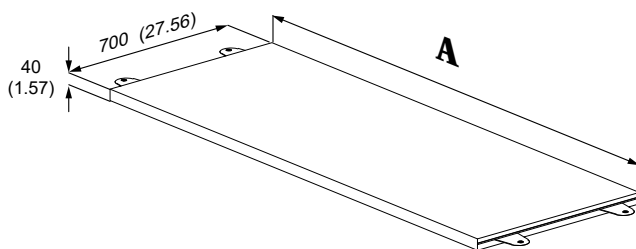
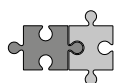
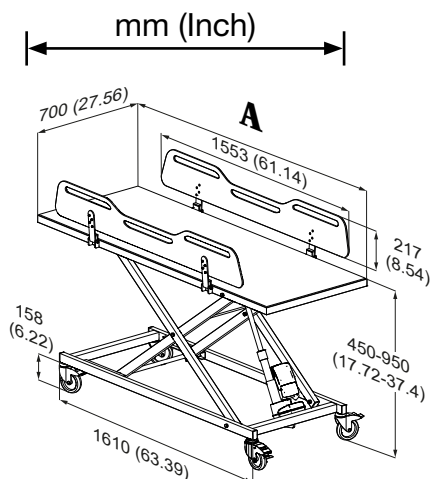
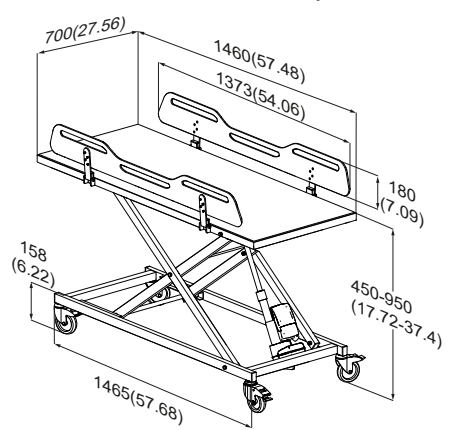
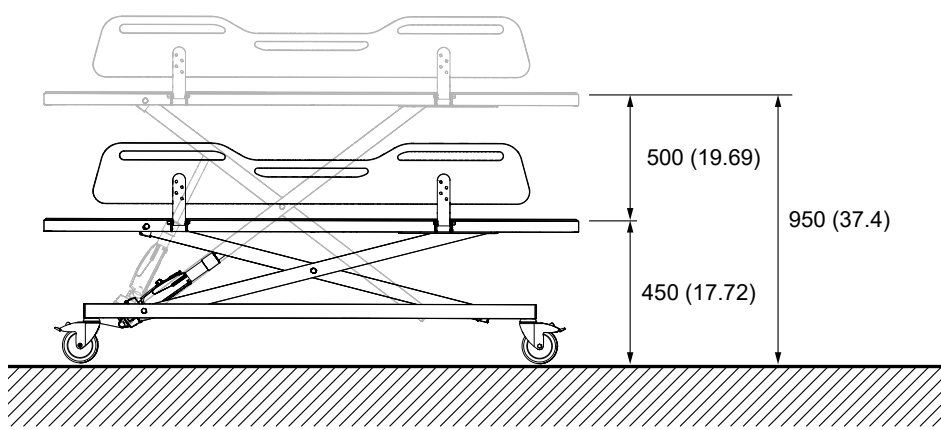
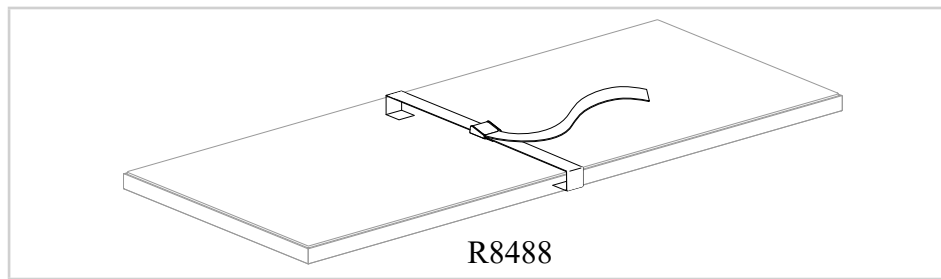
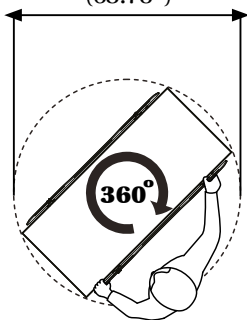
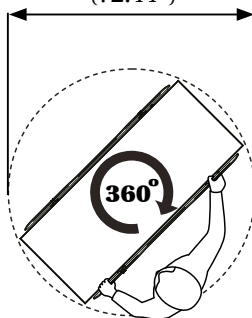
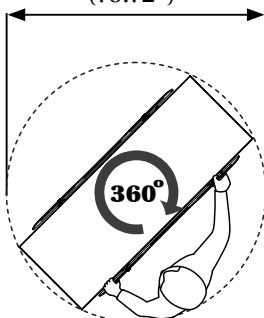
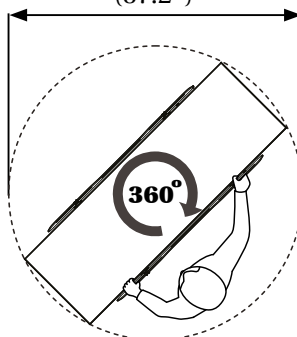
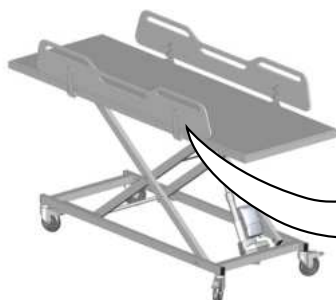
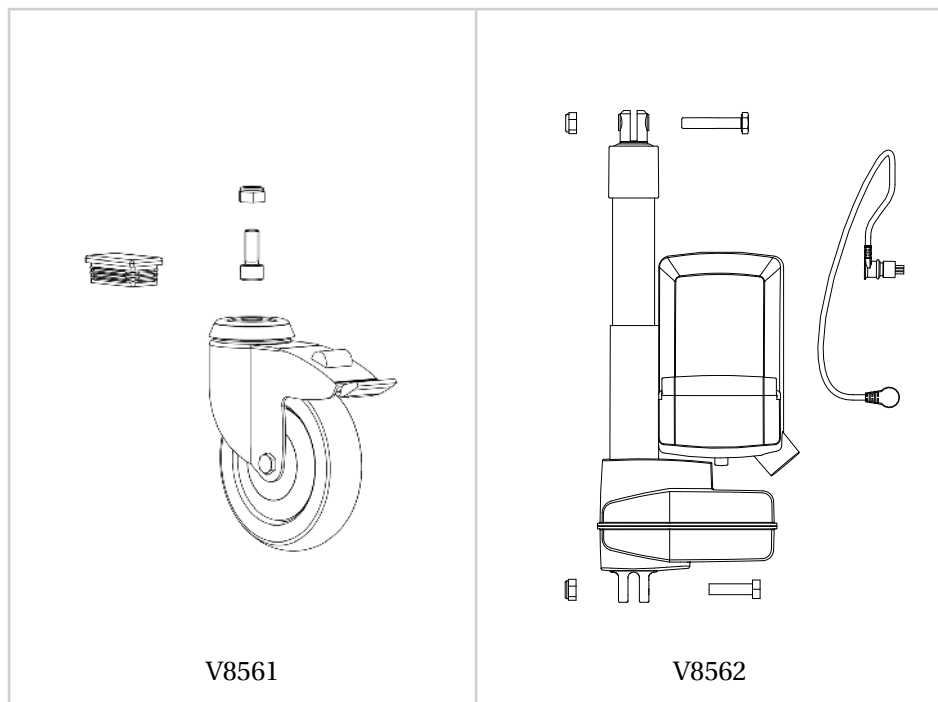
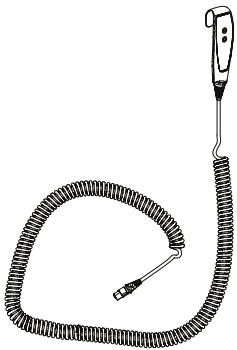
# Content

|                                     |    |
|-------------------------------------|----|
| Symbols used in this manual         | 11 |
| General safety                      | 12 |
| Installation safety                 | 13 |
| Safety distances                    | 13 |
| Connection to main supply           | 14 |
| Intended use                        | 15 |
| Intended operator profile           | 15 |
| Intended user profile               | 15 |
| Operating environment               | 15 |
| Contraindications for use           | 16 |
| Biocompatibility                    | 16 |
| Flammability                        | 16 |
| Operating instructions              | 17 |
| Safety brakes                       | 17 |
| Safety rails                        | 18 |
| Height adjustment                   | 19 |
| Moving                              | 20 |
| Accessories                         | 21 |
| Mattress                            | 21 |
| Body belt                           | 21 |
| Cleaning                            | 22 |
| Disinfection                        | 22 |
| Maintenance                         | 23 |
| Product label                       | 24 |
| Troubleshooting                     | 26 |
| Electromagnetic compatibility       | 27 |
| Electromagnetic immunity / emission | 27 |
| Storage and transportation          | 29 |
| Technical data                      | 30 |
| Disposal and recycling              | 30 |

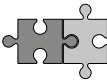
## Operational signage

A Quick-guide is included with the product. It can also be downloaded from [pressalit.com](http://pressalit.com).

# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

|                                                                                                                                                                                                                                                                                                    |                                                                                                                              |                                                                                                                              |                                                                                                                                    |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------|
| <div><b>WARNING</b></div> <p><b>WARNING</b> indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>                                                       |                                                                                                                              |                                                                                                                              |                                                                                                                                    |
| <div><b>CAUTION</b></div> <p><b>CAUTION</b> indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p> |                                                                                                                              |                                                                                                                              |                                                                                                                                    |
| <div><b>IMPORTANT</b></div> <p><b>IMPORTANT</b> indicates important information about handling and use of the product.</p>                                                                                                                                                                         |                                                                                                                              |                                                                                                                              |                                                                                                                                    |
| <div><b>NOTICE</b></div> <p><b>NOTICE</b> is used to address practices not related to personal safety.</p>                                                                                                                                                                                         |                                                                                                                              |                                                                                                                              |                                                                                                                                    |
| <div><br/>Hand injury warning</div>                                                                                                                                                                               | <div><br/>Head injury warning</div>         | <div><br/>Genitalia injury warning</div>    | <div><br/>Feet injury warning</div>               |
| <div><br/>Electricity / voltage warning</div>                                                                                                                                                                    | <div><br/>Power specifications</div>       | <div><br/>Compatible products</div>        | <div><br/>Spare parts</div>                      |
| <div><br/>Product weight</div>                                                                                                                                                                                  | <div><br/>Max. user weight</div>          | <div><br/>Article number / versions</div> | <div><br/>Do not leave patient unattended</div> |
| <div><br/>Foot placement warning</div>                                                                                                                                                                          | <div><br/>Children and pets warning</div> |                                                                                                                              |                                                                                                                                    |

# General safety

Read and understand the manual completely before use. Comply with this manual to avoid accidents involving serious personal injury and the product becoming damaged or destroyed. Save and keep the manual close to the unit.

## **WARNING**

- To reduce the risk of electric shock always unplug the unit from the electrical outlet before cleaning or servicing.
- Never use or handle this product in other ways than specified in this manual or interconnect this equipment with other equipment not described in this manual, as this can put your safety at risk and cause damage to the product.
- Never allow children, invalids, or disabled persons to operate this product unless under supervision. The product is not intended for play.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.
- If you suspect that any individual parts may be damaged, contact your Pressalit dealer or service provider. Do not perform repairs, disassembly, or assembly operations, readjustments or modifications to this product by yourself. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.
- Do not use any spare parts, extensions (e.g. extension cord) or accessories other than those specified for the product.
- Keep the cords away from heated surfaces.
- Never drop or insert any object into any openings in the product.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- This product is intended for indoor use only.
- The integrity of the external protective earthing system must be verified. See "Connection to mains supply".

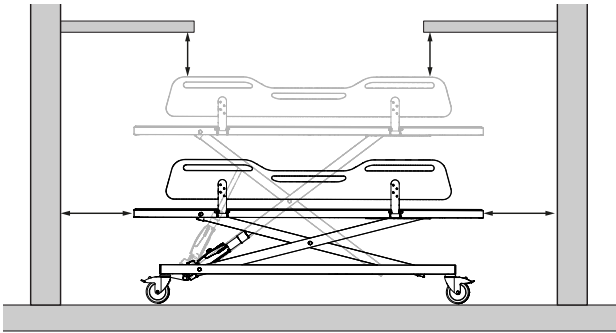
## **NOTICE**

- Pressalit cannot be held liable if the product is used in any way which differs from that described in this manual.
- We reserve the right to amend this manual without prior notice.

# Installation safety

## Safety distances

Make sure that the unit is at safe distance from surrounding objects. Check the distances before any use.

|                                                                                     |                                                                                     |                                                                                   |                          |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|--------------------------|
| ISO 17966:2016                                                                      |                                                                                     |  |                          |
|    |    | Max 120/60 mm<br>Max 4.72/2.36"                                                   | Min 300 mm<br>Min 11.81" |
|   |   | Max 8/4 mm<br>Max 0.31/0.16"                                                      | Min 25 mm<br>Min 0.98"   |
|  |  | Max 8 mm<br>Max 0.31"                                                             | Min 75 mm<br>Min 2.95"   |
|  |  | Max 35/25 mm<br>Max 1.38/0.98"                                                    | Min 120 mm<br>Min 4.72"  |

 **WARNING**

If the unit moves within the safety distances, you must keep a close eye on especially surrounding people and objects to avoid crushing hazards.

## Connection to main supply

### **WARNING**

- Connect the product to a mains supply in compliance with IEC 60601-1 and IEC 60364-7, as well as any locally applicable standards.
- Ensure that the voltage of the local mains supply to be connected is compliant with that stated on the label on the product.
- To ensure protection by electrical separation, the product must be supplied by a mains supply circuit with incorporated a residual current protective device (RCD) with a rated residual operating current not exceeding 30 mA in compliance with IEC 60601-1 and IEC 60364-7, as well as any locally applicable standards.
- Ensure that location of the mains plug is easy to access and use if separation from mains is needed.
- External cables (mains plug) need to be fixed in such a way that users cannot stumble and injure themselves.

## Intended use

This product is intended for changing a person who is unable to stand or sit. The person is lying horizontally on the change trolley.

## Intended operator profile

Only personnel who are trained in healthcare and/or in changing someone under their care should use this product.

## Intended user profile

The product can be used by all – adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 441 lbs.**

## Operating environment

The change trolley is intended to be used at room temperature in public changing places, schools, private homes, day/residential/nursing facilities and hospitals under normal working conditions.

The change trolley may be used in dry rooms and not as a shower table in showers. If the relative humidity is above 70%, the expected life of the unit will be significantly reduced. The change trolley must not be exposed to chlorine-containing fumes.

When stored outside operating temperature wait minimum 3 hours at 20°C before use. This allow the unit to reach operating conditions.

| Environmental factor   | Operating range                                              |
|------------------------|--------------------------------------------------------------|
| Temperature            | 5°C to 40°C ( 41°F to 104°F)                                 |
| Relative humidity      | 20% to 70% at 30°C (86°F)                                    |
| Atmospheric pressure   | 700 to 1060 hPa                                              |
| Meters above sea level | Max. 3000 m / 9,800 feet                                     |
| Duty cycle             | 2 minutes of continuous use, followed by an 18-minute pause. |

en

## CAUTION

The product surface may become hot or cold if exposed to external sources of heat or cold, such as sunlight or frost.

## NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities. A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

### Contraindications for use

If the product is being used for patients with damaged skin, wounds, or other similar conditions, it is recommended to use a towel as a barrier to prevent open wounds from coming into direct contact with the substrate.

### Biocompatibility

The biocompatibility of the materials that come in contact with the user's skin during the intended use has been evaluated according to the procedures outlined in DS/EN ISO 10993-1:2020, which involves assessing the materials for harmful substances. Based on this evaluation, the materials are considered suitable for their intended use.

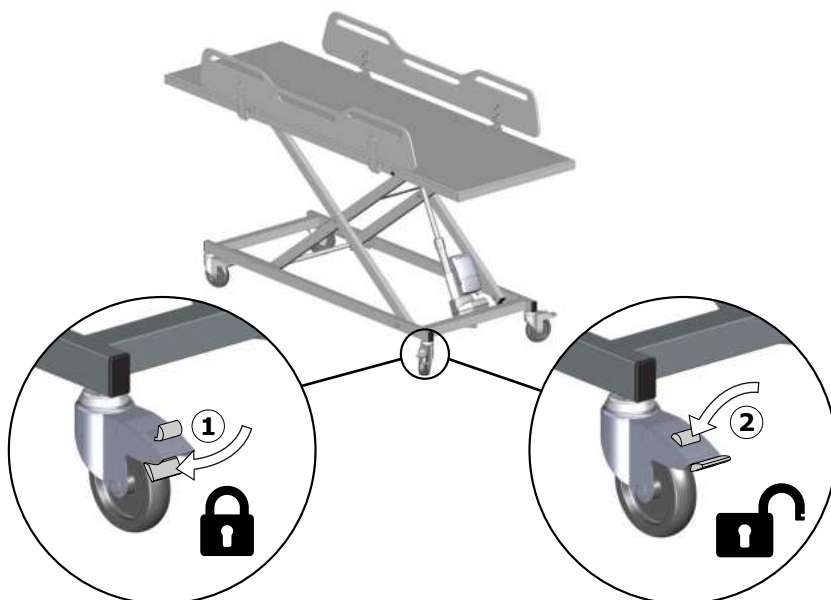
### Flammability

The product materials have undergone flammability testing in accordance with ISO 8191-1:1987 and ISO 8191-2:1988.

# Operating instructions

## Safety brakes

The change trolley is equipped with brakes on all four wheels to prevent accidental movement.

**Lock the brakes①:** Step on the lock pedal on all four wheels.

**Unlock the brakes②:** Step on the release button on all four wheels.

**Always use your foot to lock/unlock the wheels. Wear closed-toe shoes and avoid using your hands. Be mindful of the brakes to avoid tripping or bumping into them.**

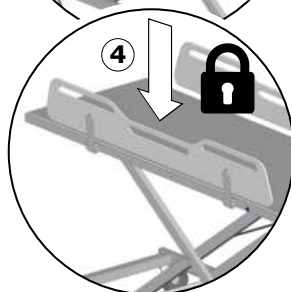
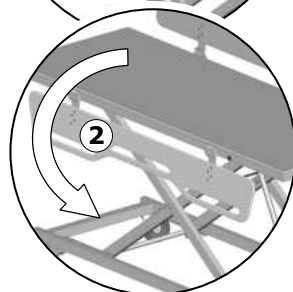
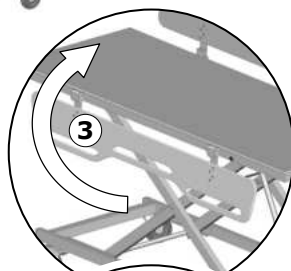
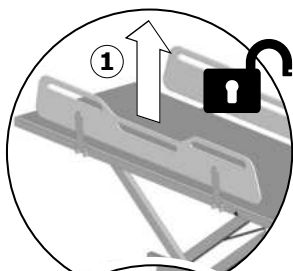
Ensure all four wheels are locked before transferring a patient to/from the mobile change trolley.

**Never leave a patient unattended on the trolley.**

## Safety rails

The safety rails can be folded up and down individually.

**Fold down:** Lift the safety rail upwards to unlock it①, and swing it down②.

**Fold up:** Swing the safety rail up③ and push it down to lock it in place④.


### CAUTION

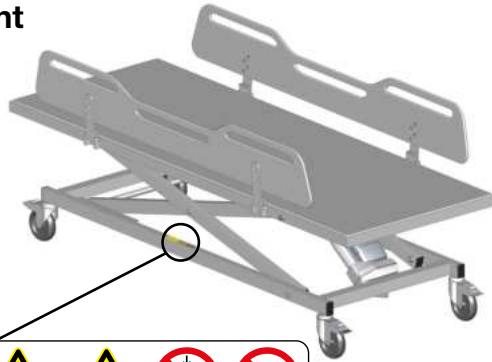To protect the patient, it is recommended that the side rails are upright and locked when using the mobile change trolley.

**Be careful when adjusting the side rails to avoid pinching hands, feet, or other body parts.**

# Height adjustment

200 kg / 440 lbs

## WARNING

The change trolley can be adjusted 500 mm / 19.69" up and down. The maximum adjustable height is 950 mm (37.4"). The height adjustment is controlled by the wired remote control unit.

Press and hold the up button to raise the unit. When the button is released, the motion stops.

Press and hold the down button to lower the unit. When the button is released, the motion stops.

en

**Do not place your foot on the bearing frame and be especially cautious of the risk of pinching hands, feet, or objects when adjusting the height of the unit.**

## WARNING

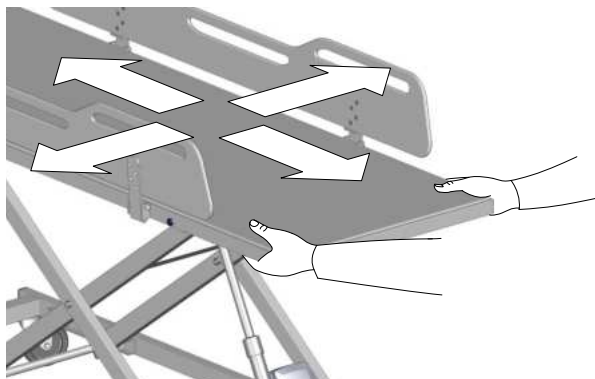
Never let children or pets be near the unit during height adjustment.

**Ensure that the remote control cord is not wrapped around the patient's neck, as it poses a strangulation risk. When not in use, place the remote control on the side rail.**

## Moving


To move the trolley, unlock all four wheels. Grab the lying surface itself, not the safety rails, and proceed with caution while moving the unit.

To prevent the trolley from tipping over, avoid using it on floors with lowered drains, holes, or significant slopes.


**The mobile change trolley is connected to a power outlet. Move it carefully to avoid damaging the cord or installation. Always inspect the cord and plug before use, and replace the cord if damaged to prevent electric shock.**

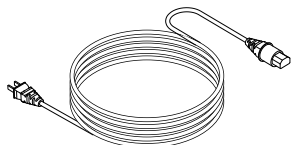Take care to avoid suffocation from the power cord or tripping over the power cord. Ensure the cord does not wrap around the patient's feet. During use, the cord should lie flat on the floor. When moving the unit, place the cord on the trolley to prevent damage.


## Accessories

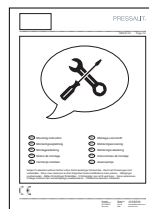A mounting instruction is included with the accessories and can also be downloaded from [pressalit.com](http://pressalit.com).


### Mattress

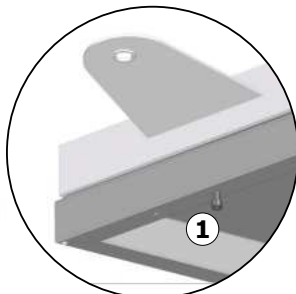A mattress can be added to the change trolley for increased comfort. The mattress is reversible and can be used on both sides.


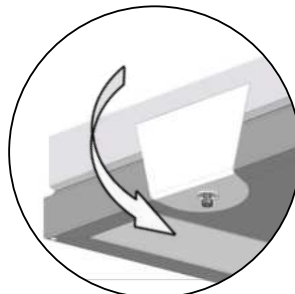
To mount the mattress, loosen the locking screw① by about one centimeter.

Pull the mattress flap down over the screw. Repeat this with all four flaps.

### Body belt

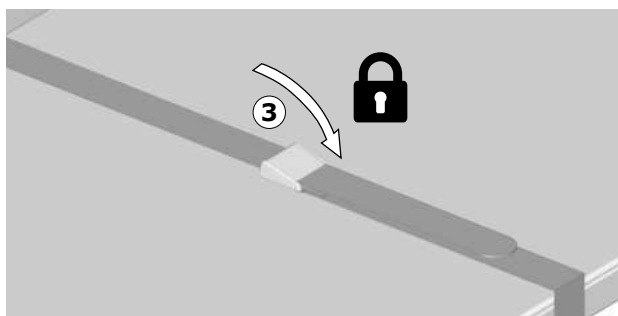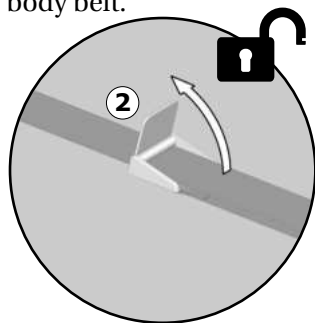To provide the user with additional support and comfort, you can install a body belt.

Pull the handle on the buckle up② to release the lock. Then press the handle down again to fasten the body belt③.

The body belt does not provide safety and should never replace supervision.

**Ensure the belt is securely fastened and does not come near the patient's neck, as it poses a strangulation risk.**

## IMPORTANT

To maintain hygiene, it is recommended to check for any visible damages or signs of wear and tear on the mattress regularly. If any damages are found, the mattress should be replaced immediately to prevent any risk of contamination or infection.

## Cleaning

Clean the product regularly and between uses to maintain general hygiene and functionality.

To clean the product, use a damp cloth that has been wrung out in a mild soap mixture, such as universal cleaning solution. Ensure that all surfaces has been wiped over and cleaned. Pay special attention to the user contact surfaces. Body belt and mattress can be detached for easier cleaning. Bodybelt can when removed from product be rinsed in clean water for easier cleaning. After cleaning, it is essential to wipe the product with a dry, soft cloth. This step is crucial to prevent the materials from discoloration. Additionally, it helps prevent any potential degradation of the surface paint.

A list of suitable cleaning agents/disinfectants tested by Pressalit can be found in the technical datasheet or on [pressalit.com](http://pressalit.com) under the specific product.

## CAUTION

- Always read the supplied instructions of the cleaning agent and comply with these instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.
- Do not perform any type of cleaning or disinfection of the change trolley while the user is lying on it.

## IMPORTANT

- Do not rinse or spray water directly on the product.
- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acid or base.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

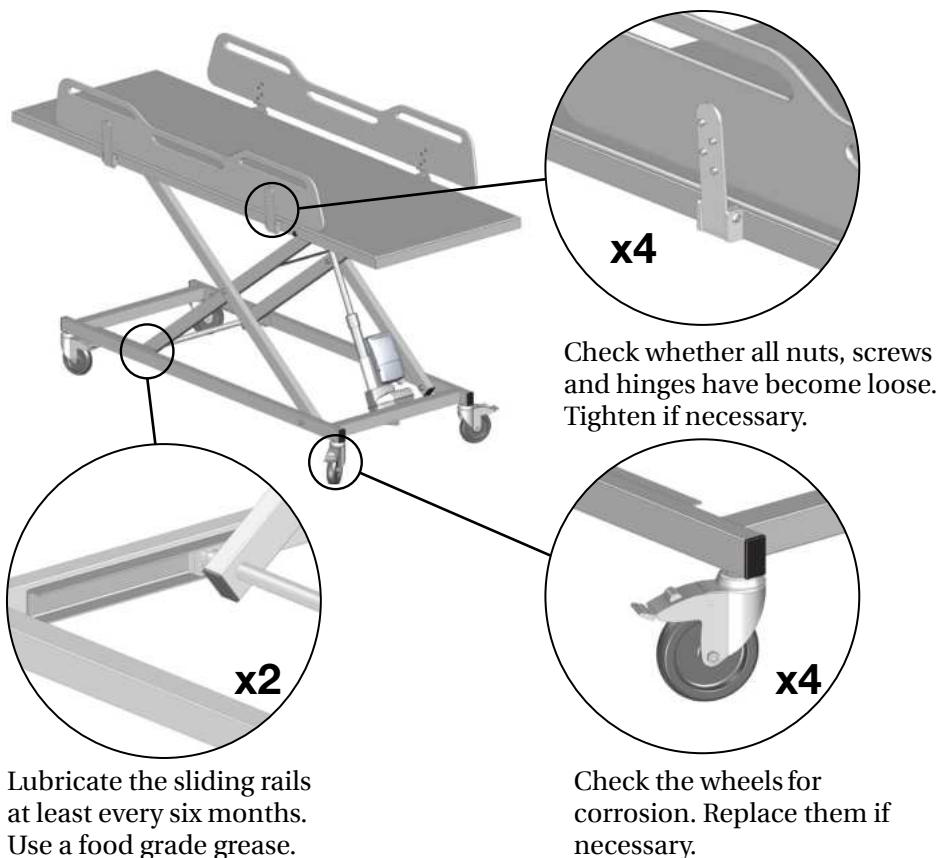
## Disinfection

When using the change trolley for multiple patients, it is necessary to both clean and disinfect the trolley between uses on different patients to prevent the risk of cross infection. Disinfection and cleaning is also required after storage or maintenance.

# Maintenance

Maintenance inspection should be carried out once a year. The inspection should be recorded in a logbook describing what has been checked and replaced. Spare parts should always be obtained from your supplier or Pressalit in order to ensure a continuously safe product and the guarantee. Contact your supplier or Pressalit for purchase of spare parts and drawings.


Perform a visual inspection of steel parts for penetrating rust.

## NOTICE

If the product is used in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.
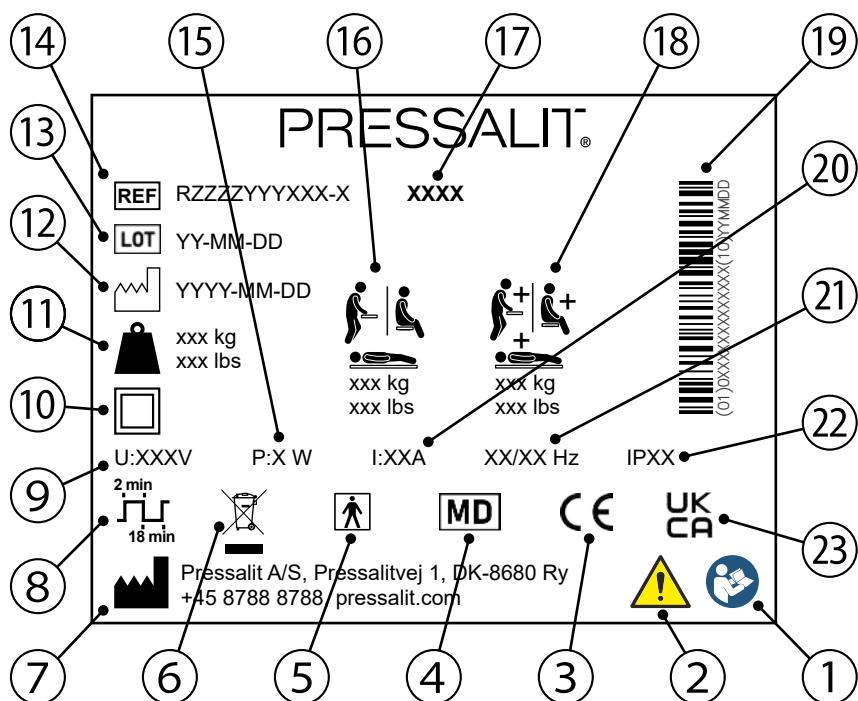
# Product label

- 1 Read the manual before use
- 2 Consult the manual for important cautionary information such as warnings and precautions
- 3 These products are CE-marked in accordance with relevant requirements in applicable legislation.  
A declaration of conformity can be found at [pressalit.com](http://pressalit.com)
- 4 Medical device
- 5 Type BF applied part. Part of the product in normal use necessarily comes into physical contact with the patient to perform its function  
The product complies with requirements of IEC 60601-1 to provide protection against electric shock
- 6 Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

|   |                                                                                                                                                        |
|---|--------------------------------------------------------------------------------------------------------------------------------------------------------|
| ⑦ | Name and address of manufacturer                                                                                                                       |
| ⑧ | Duty cycle                                                                                                                                             |
| ⑨ | Mains voltage                                                                                                                                          |
| ⑩ | Class II Product, double insulated                                                                                                                     |
| ⑪ | Product weight                                                                                                                                         |
| ⑫ | Production date                                                                                                                                        |
| ⑬ | Batch code                                                                                                                                             |
| ⑭ | Article number                                                                                                                                         |
| ⑮ | Maximum power consumption                                                                                                                              |
| ⑯ | Maximum user weight                                                                                                                                    |
| ⑰ | Model name                                                                                                                                             |
| ⑱ | Max load                                                                                                                                               |
| ⑲ | GS1-128 barcode                                                                                                                                        |
| ⑳ | Electric Current                                                                                                                                       |
| ㉑ | Frequency                                                                                                                                              |
| ㉒ | IP class                                                                                                                                               |
| ㉓ | UK Conformity Assessed. UK responsible representative: Pressalit Ltd., 100, Longwater Avenue, Green Park, Reading, Berkshire, RG2 6GP, United Kingdom. |

# Troubleshooting

| Symptom                                           | Cause                               | Action                                                   |
|---------------------------------------------------|-------------------------------------|----------------------------------------------------------|
| No power.                                         | The power plug has been pulled out. | Plug the power plug back into the socket.                |
| The trolley has difficulty moving up and/or down. | The lifting unit lacks lubrication. | Lubricate the sliding rails.<br>See <b>Maintenance</b> . |
|                                                   | The unit may be stuck.              | Remove any obstacles.                                    |

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the warranty. Expected service life of the change trolley is 10 years. Expected service life of all spare parts are 10 years, but depending on the amount of use, the service life can be reduced.

# Electromagnetic compatibility

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

## Electromagnetic immunity / emission

en

| <b>Electrostatic discharge</b>                                 |                                                                                     |
|----------------------------------------------------------------|-------------------------------------------------------------------------------------|
| Test standard                                                  | Immunity test level                                                                 |
| EN 61000-4-2                                                   | +/- 8 kV contact, +/- 2, 4, 8, 15 kV air                                            |
| <b>Radiated RF electromagnetic field</b>                       |                                                                                     |
| Test standard                                                  | Immunity test level                                                                 |
| EN 61000-4-3                                                   | 10 V/m 80 MHz to 1 GHz, 10 V/m 1,01 GHz to 2,7 GHz<br>80 % AM 1 kHz                 |
| <b>Electrical fast transient/burst</b>                         |                                                                                     |
| Test standard                                                  | Immunity test level                                                                 |
| EN 61000-4-4                                                   | +/- 1 kV , +/- 2 kV, 100 kHz repetition frequency                                   |
| <b>Surge immunity</b>                                          |                                                                                     |
| Test standard                                                  | Immunity test level                                                                 |
| EN 61000-4-5                                                   | Line-to-line: +/- 0.5 kV, +/-1 kV<br>Line-to-ground: +/- 0.5 kV, +/-1 kV, +/- 2 kV  |
| <b>Immunity to conducted disturbances induced by RF fields</b> |                                                                                     |
| Test standard                                                  | Immunity test level                                                                 |
| EN 61000-4-6                                                   | 3 V (6 V in ISM bands and amateur radio bands)<br>0,15 to 80 MHz , 80 % AM at 1 kHz |

## Immunity to proximity fields from RF wireless communication equipment

| Test standard | Immunity test level                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
|---------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| EN 61000-4-3  | 385 MHz, 27 V/m, 18 Hz PM (50 % duty cycle square wave)<br>450 MHz, 28 V/m, FM +/- 5 kHz deviation, 1 kHz sine<br>710 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)<br>745 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)<br>780 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)<br>810 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave)<br>870 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave)<br>930 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave)<br>1720 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave)<br>1845 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave)<br>1970 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave)<br>2450 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave)<br>5240 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)<br>5500 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)<br>5785 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave) |

## Voltage dips

| Test standard | Immunity test level                                                                                                                                                              |
|---------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| EN 61000-4-11 | 0 % U <sub>T</sub> ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, 0 % U <sub>T</sub> ; 1 cycle at 0°, 70 % U <sub>T</sub> ; 25 cycles at 0°<br>Single phase at 0° |

## Voltage interruptions

| Test standard | Immunity test level            |
|---------------|--------------------------------|
| EN 61000-4-11 | 0 % U <sub>T</sub> ; 300 cycle |

## Conducted / Radiated RF emission

| Test standard | Emission test class and group |
|---------------|-------------------------------|
| EN 55011      | Class B, group 1              |

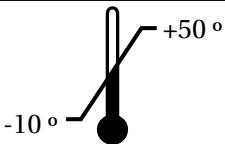
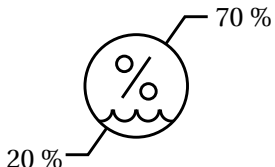
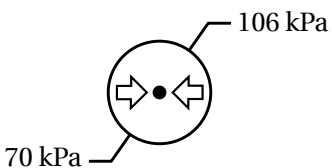
## Harmonic current emissions

| Test standard | Emission test class and group |
|---------------|-------------------------------|
| IEC 61000-3-2 | Class A                       |

## Voltage changes, voltage fluctuations and flicker

| Test standard | Emission test class and group     |
|---------------|-----------------------------------|
| EN 61000-3-3  | Tested according to the standard. |

## Storage and transportation

|                                                                                   |                                                                                                                                        |
|-----------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------|
|  | The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit. |
|  | The relative humidity for transportation and storage of the unit must be between 20 and 70% (RH).                                      |
|  | The air pressure for transportation and storage of the unit must be between 70 and 106 kPa.                                            |

### IMPORTANT

- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

en

# Technical data

|                  |                                                                                                                                                                                                |                 |
|------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------|
| Max. user weight | 200 kg / 441 lbs                                                                                                                                                                               |                 |
| Max load         | 205 kg / 452 lbs                                                                                                                                                                               |                 |
| Materials        | Powder-lacquered steel<br>Plastics<br>Compact Density Fiberboard (CDF)                                                                                                                         |                 |
| Weight           | R9612714, R9632714,<br>R9652714, R9662714                                                                                                                                                      | 72 kg / 159 lbs |
|                  | R9612717, R9632717,<br>R9652717, R9662717                                                                                                                                                      | 74 kg / 163 lbs |
|                  | R9612719, R9632719,<br>R9652719, R9662719                                                                                                                                                      | 76 kg / 168 lbs |
|                  | R9612721, R9632721,<br>R9652721, R9662721                                                                                                                                                      | 78 kg / 172 lbs |
| IP class         | The product has an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction. |                 |
| Noise level      | ≤ 50 dB (A)                                                                                                                                                                                    |                 |

## Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected. When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion. The following table provides information on the recycling and handling of the product. If in doubt contact your local authorities to determine the proper method of disposal.

| Recyclable components | Material                                                                                                                                                                                                                             |
|-----------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Steel                 | Steel can be recycled by being melted down and included as a secondary material in the production of new steel.                                                                                                                      |
| Plastics              | Use plastic sorting and recycling systems where they are locally available. If such systems are not available, incineration of plastic waste can be an alternative, as it can generate energy that can be used for power production. |
| Electronics           | The electronic components should be collected separately and returned to the designated recycling service.                                                                                                                           |

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