




<p>Product</p> <p>Item number</p>	<p>MCT 2 change trolley, electrically height adjustable</p> <p>R9412717000</p>
<p>Colour</p> <p>Product description</p>	<p>EAN:5708590373164</p>  <p>White</p> <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>MCT 2 change trolley, 820 x 1790 mm, two fold-down sides. Electrically height adjustable (battery operated) with wired remote control. 400 mm lifting range, from 550 to 950 mm. Equipped with central lock system, wire basket for convenient storage, bumpers on all four corners as well as battery and charger. Lifting columns and wired remote control IPX6, battery and charger IPX5. Mattress to be ordered separately.</p>
<p>Adjustability</p> <p>Adjustment height</p> <p>Product capacity</p> <p>Speed upwards</p>	<p>Height adjustable</p> <p>400 mm</p> <p>Maximum user weight (MUM): 200 kg. Maximum load (SWL): 250 kg. (user weight + water + accessories). <i>This product has been tested statically with a load of 625 kg as the IEC 60601-1 standard requires testing with safety factor 2.5.</i></p> <p>Speed up mm: 19 mm/s Speed down mm: 19 mm/s</p>
<p>Materials</p>	<p>Aluminium, plastics, compact laminate and powder-lacquered stainless steel.</p>

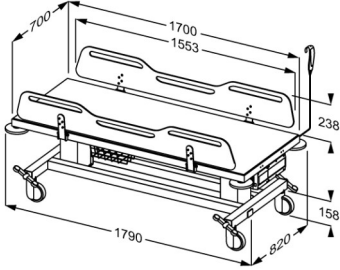

Product	MCT 2 change trolley, electrically height adjustable
Item number	R9412717000
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.
Cleaning	<p>MSCT 1, MCT 2, MSCT 4</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p>
Declaration of conformity	<p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>CE UK MDR-MASK-RoHS This product is in compliance with all relevant requirements of:</p> <ul style="list-style-type: none">- Regulation (EU) 2017/745 on medical devices- Council Directive 2006/42/EC on machinery- Council Directive 2011/65/EC on RoHS. For the UK:- Medical Device Regulation (MDR) 2002 on medical devices.- Supply of Machinery (Safety) Regulations 2008- Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012.



PRESSALIT®

Product	MCT 2 change trolley, electrically height adjustable
Item number	R9412717000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®