


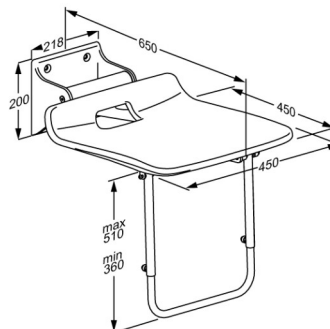

Product Item number	VALUE IV folding shower seat with supporting leg, fixed height R1622000112
Colour	VVS:777525412 NRF:6314098 RSK:8565070 EAN:5708590377728 HMI: 124486 DB: 2510921 FDA:INN/D135865 Anthracite grey
Product description	VALUE IV folding shower seat and backrest create a simple solution for users who need to sit down while showering. VALUE IV shower seat. Fixed height. The seat is foldable. Materials: Diecast aluminium, Rilsan-coated stainless steel and vacuum-moulded polyethylene (PE). Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.
Adjustability Product capacity	Fixed mounting Maximum user mass: 135 kg. When the seat is loaded with 135 kg, each wall fastener is loaded with 125 kg. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
Materials Ergonomy Features	The wall profile is made of diecast aluminium, and the frame and the leg is made of Rilsan-coated stainless steel. Rilsan provides good protection against corrosion and is comfortable to touch and easy to clean. The seat is made of vacuum-moulded polyethylene (PE). The anti-slip pattern of the seat ensures drainage of water in both the horizontal and the vertical position. The seat is slightly curved towards the middle, which helps the user stay in the middle of the seat. The folding shower seat is designed for installation in the shower area and can be easily folded up and down as required. The seat and backrest can be easily removed for cleaning.





PRESSALIT®

Product	VALUE IV folding shower seat with supporting leg, fixed height
Item number	R1622000112
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean) NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.



PRESSALIT®

Product	VALUE IV folding shower seat with supporting leg, fixed height
Item number	R1622000112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®

9. september 2026