


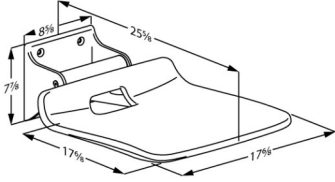

|   |  |
|---|--|
| <p><b>Product</b><br/><b>Item number</b></p>            | <p>VALUE IV shower seat, fixed height<br/>R1620000112</p>  |
| <p><b>Color</b><br/><br/><b>Product description</b></p> | <p>VVS:777525312<br/>NRF:6314097<br/>RSK:8162781<br/>EAN:5708590377711<br/>HMI: 124483<br/>DB: 2296038<br/>Anthracite gray</p>  <p>VALUE IV folding shower seat and backrest create a simple solution for users who need to sit down while showering.</p> <p>VALUE IV shower seat. Fixed height. The seat is foldable.<br/>Materials: Diecast aluminium, Rilsan-coated stainless steel and vacuum-moulded polyethylene (PE).<br/>Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p> |
| <p><b>Adjustability</b><br/><b>Product capacity</b></p> | <p>Fixed mounting</p> <p>Maximum user mass: 298 lbs.<br/>When the seat is loaded with 298 lbs, each wall fastener is loaded with 276 lbs.</p> <p><i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i></p>  |
| <p><b>Materials</b></p>                                 | <p>The wall profile is made of diecast aluminium, and the frame is made of Rilsan-coated stainless steel. Rilsan provides good protection against corrosion and is comfortable to touch and easy to clean. The seat is made of vacuum-moulded polyethylene (PE).</p>   |
| <p><b>Warranty</b></p>                                  | <p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>   |



|                                  |  |
|----------------------------------|--|
| <b>Product</b>                   | VALUE IV shower seat, fixed height   |
| <b>Item number</b>               | R1620000112  |
| <b>Cleaning</b>                  | <p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningsprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%<br/>WipeClean UniversalCleaning<br/>Chlorine Disinfection Wipes<br/>Klorilli (Kilto Clean)</p> <p><b>NON-DISINFECTANTS:</b><br/>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> |
| <b>Declaration of conformity</b> | <p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>  |



PRESSALIT®

|                    |                                    |
|--------------------|------------------------------------|
| <b>Product</b>     | VALUE IV shower seat, fixed height |
| <b>Item number</b> | R1620000112                        |

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

PRESSALIT®