


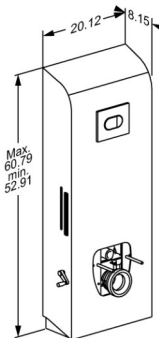

PRESSALIT®

Product Item number	SELECT TL3 toilet lifter, manual, for wall outlet R9023112
Color	VVS:617989012 NRF:6191183 EAN:5708590372822 HMI: 138237 FDA: IKX/D319729 Anthracite gray
Product description	 SELECT toilet lifts are available in several models and colors for both wall outlet and floor outlet. They are available in electrically or manually height-adjustable versions, with or without side profiles and for standard toilet bowls and shower toilets. All developed with a focus on hygiene and easy cleaning. SELECT TL3 toilet lifter, for wall outlet. Incl. cUPC-approved cistern with dual flushing system preset at 0.8/1.6 gallons (can be changed to 0.5/1.1 gallons). Manually height adjustable 8" with crank handle. Toilet pan, toilet seat and fold down grab bars to be ordered separately. Materials: Stainless steel and PMMA/ABS. Excl. wall mounting kit.
Adjustability Adjustment height Product capacity	Height adjustable 7,88 inch Maximum user weight: 440 lbs. Maximum load: 518 lbs. (user weight + accessories). When the toilet lifter is loaded with 518 lbs, each wall fastener is loaded with 143 lbs. <i>The toilet lifter has been tested statically with 1320 lbs, which is a requirement according to the applied assistive products standards. Please note that most toilets compatible with this lifter have been tested up to 880 lbs.</i>
Materials	The toilet lifter is constructed of stainless steel, the cover fairing is made of PMMA/ABS plastics.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT®

Product	SELECT TL3 toilet lifter, manual, for wall outlet
Item number	R9023112
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.



PRESSALIT®

Product	SELECT TL3 toilet lifter, manual, for wall outlet
Item number	R9023112

Mounting in different wall types

	Concrete >B25 V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	Solid Brick >= Mz12 V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	Aerated Concrete P4 V8655 - Mounting kit, 4 lag bolts Ø10 x 70, plugs GB14, with grey covers. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with white covers. NB! Plugs are not needed. To be ordered separately.	



Product Item number	SELECT TL3 toilet lifter, manual, for wall outlet R9023112		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.		