

PRESSALIT.

Product	PLUS grab bar combination for shower, 39.4" x 42.9"
Item number	RT144000
	VVS:777569700
	NRF:6311235
	RSK:8180962
	EAN:5708590247649
	HMI: 42068
	NBS:N13:436
	FDA: ILS/D135866
Color	White
	ways and they provide the final touch to furnishing a functional and flexible bathrood A reliable solution with many different options, which can easily lay the grounds for increased independence. PLUS grab bar combination for shower, A: 39.4" x B: 42.9". Compatible with RT600 shower holder. Material: Powder-coated aluminium and plastic. Excl. mounting kit.
Product capacity	Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.
Materials	Core material: Aluminium Surface material: Polyester lacquer Cover cap: Cycoloy plastic Strap: Nylon (PA)
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT.

Product Item number	PLUS grab bar combination for shower, 39.4" x 42.9" RT144000
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Millizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to
Declaration of conformity	seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water. This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.



PRESSALIT.

Product	
Item number	er

PLUS grab bar combination for shower, 39.4" x 42.9"

RT144000

Mounting in different wall types

a. V · ~ · ((· · · · · ·)	
.C	
D . C . C . C . C . C	
D. O. D. O. O. O.	
. α· Þ : Δ : Δ · ω	

Concrete >B25

V8617 - Mounting kit, 12 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.






Solid Brick >= Mz12

V8617 - Mounting kit, 12 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.



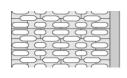

Solid Sand Lime Brick >= HIz12

V8617 - Mounting kit, 12 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.






Aerated Concrete P4

V8667 - Mounting kit, 12 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.



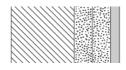

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8. NB! Plugs are not needed.

To be ordered separately.





PRESSALIT®

Product Item number	PLUS grab bar combination for shower, 39.4" x 42.9" RT144000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

pressalit.com