R8406
R8407

SCT 1000

Shower change table – Operation and maintenance manual	en
Bruseleje – Brugs- og vedligeholdelsesvejledning	dk
Duschliege – Gebrauchs- und Pflegeanleitung	de
Table de douche et de change – Manuel d'utilisation et d'entretien	fr
Douchebrancards – Gebruiks- en onderhoudshandleiding	nl
Duschbrits – Bruks- och underhållsanvisning	sv
Dusjstøllebord – Bruks- og vedlikeholdsveiledning	no
Suihkupaarit - Käyttö- ja huolto-ohjeet	fi
Camilla de ducha - Instrucciones de uso y mantenimiento	es
Stół prysznicowy – Instrukcja obsługi i konserwacji	pl
洗澡床 - 操作和维护手册	中文

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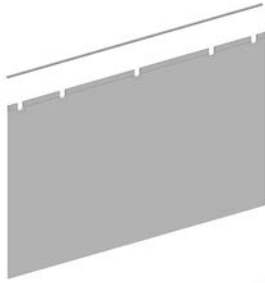
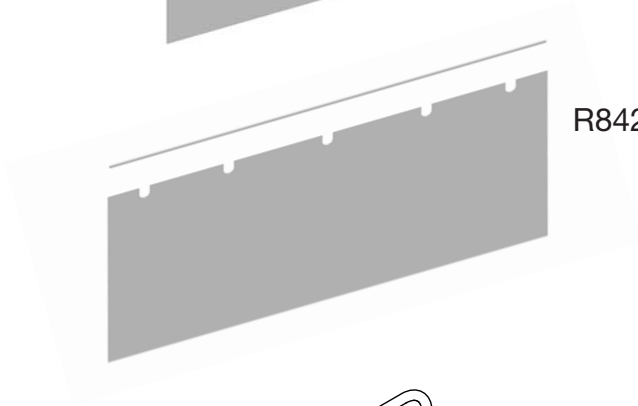
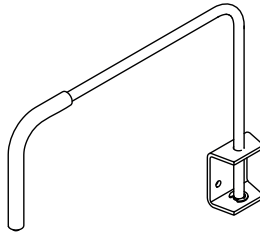
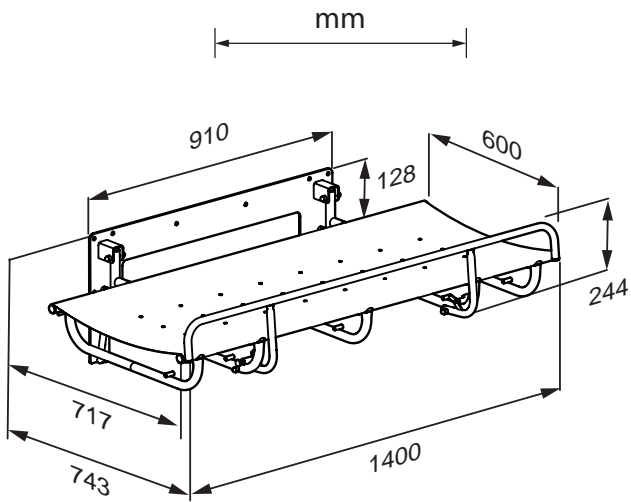
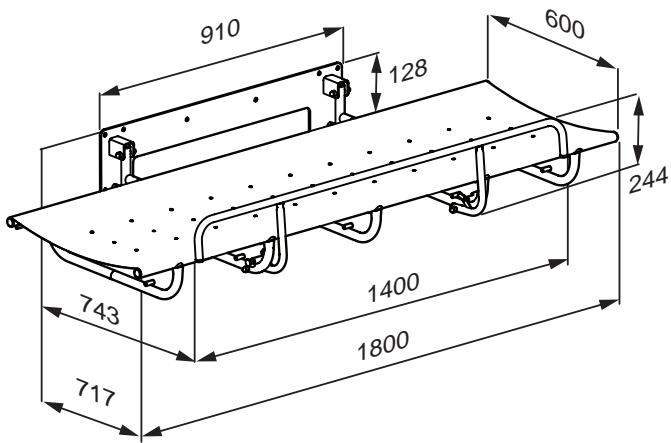
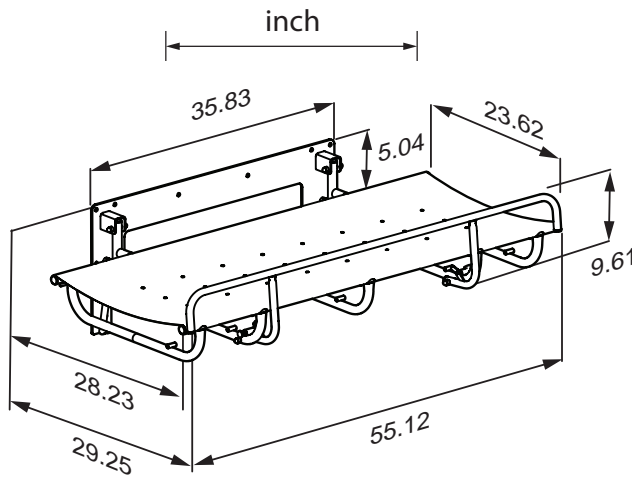
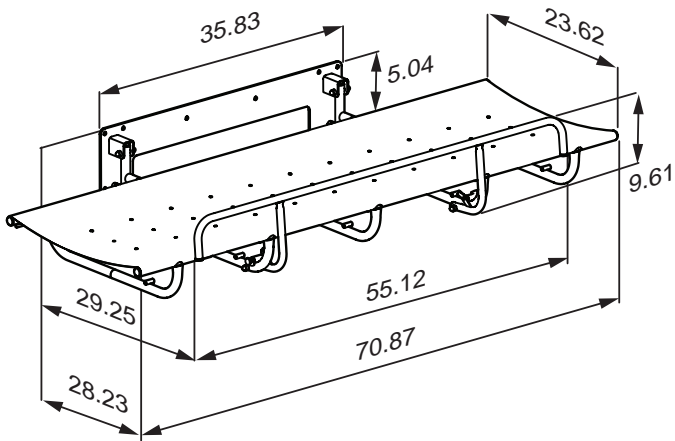
R8407

Content

Mounting instruction	7
Operational signage	7
Symbols used in this manual	8
General safety	9
Intended use	10
Intended operator profile	10
Intended user profile	10
Operating environment	10
Contraindications for use	11
Biocompatibility	11
Operating instructions	12
Safety rail	13
Cleaning	14
Disinfection	14
Maintenance	15
Maintenance inspection	15
Product label	16
Trouble-shooting	17
Technical data	17
Disposal and recycling	18

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage is included with the product. It can also be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

en

 <p>Hand injury warning</p>	 <p>Do not leave patient unattended</p>	 <p>Compatible products</p>
 <p>Article number/ versions</p>	 <p>Product weight</p>	 <p>Max. user weight</p>

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

This product is intended for bathing a person who is unable to stand up or sit down. The person is lying horizontally on the shower table.

Intended operator profile

The product is to be used by a person trained in health care and/or trained in washing a person in their care.

Intended user profile

The product can be used by all – adults, children, young, old, whose weight does not exceed the **maximum load of 150 kg / 330 lbs.**

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature, in private homes, nursing homes and hospitals under normal working conditions. The product must not be exposed to chlorine containin fumes.

The product surface may become hot or cold if exposed to external sources of heat or cold, such as sunlight or frost.

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

Contraindications for use

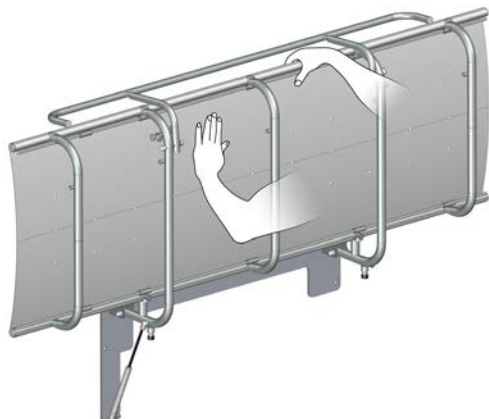
Use of the unit is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the unit only after consulting with the attending physician.

Biocompatibility

The biocompatibility of the materials that come in contact with the user's skin during the intended use has been evaluated according to the procedures outlined in DS/EN ISO 10993-1:2020, which involves assessing the materials for harmful substances. Based on this evaluation, the materials are considered suitable for their intended use.

Operating instructions

Support the shower table using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinder makes it easy to fold the shower table up against the wall after use.

Take care to avoid injury to hands when in the vicinity of closing mechanical parts.

CAUTION

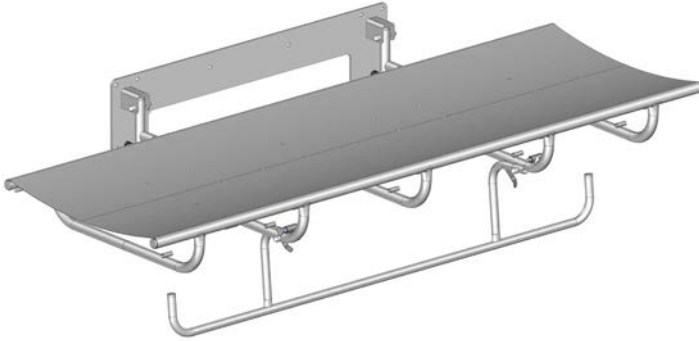
Always be careful when moving a person to and from the shower table. It is not recommended to move a wet patient.


CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

Safety rail

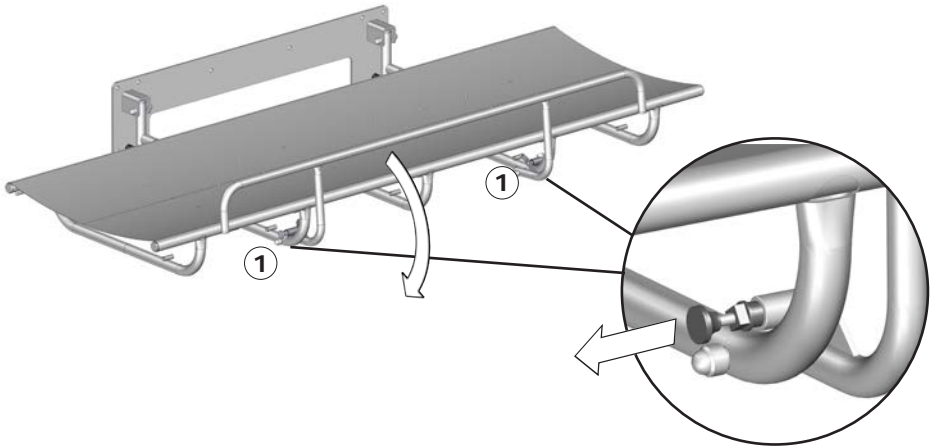To pull the safety rail up: Swing the rail upwards from under the shower table. The safety rail will automatically lock into position.

To fold the safety rail away: Release the two locks ① simultaneously and swing the safety rail down under the shower table.

Do not leave the patient unattended.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

Disinfection

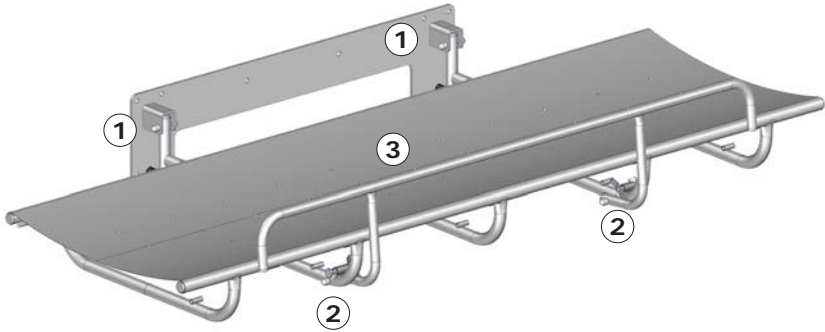
When using the shower table for multiple patients, it is necessary to both clean and disinfect the shower table between uses on different patients to prevent the risk of cross infection. A list of suitable disinfectants can be found at pressalit.com.

Maintenance

Expected service life of the shower table is 10 years, provided that maintenance is carried out as described below:

It is required that maintenance inspection is done one a year.


Maintenance inspection

en

- Check the hinges ① on the shower table and tighten if necessary.
- Check the hinges ② on the safety rail and tighten if necessary.
- Check the canvas ③ for damage and replace if necessary.

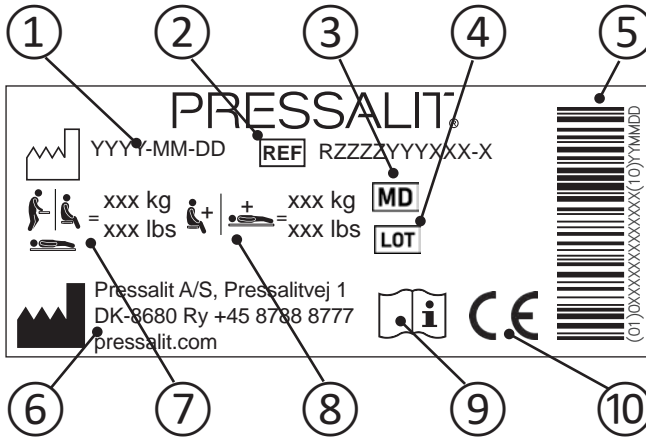Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load


⑨ Consult manual for use

These products are CE-marked in accordance with relevant requirements in:

- ⑩
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
- A declaration of conformity can be found at pressalit.com

Trouble-shooting

Symptom	Cause	Action
The shower table does not stay in upright position.	The gas cylinders are defective.	Contact Pressalit or your local dealer.
The shower table is not level.	Adjustment screws too high or too low.	Adjust the adjustment screws ①.

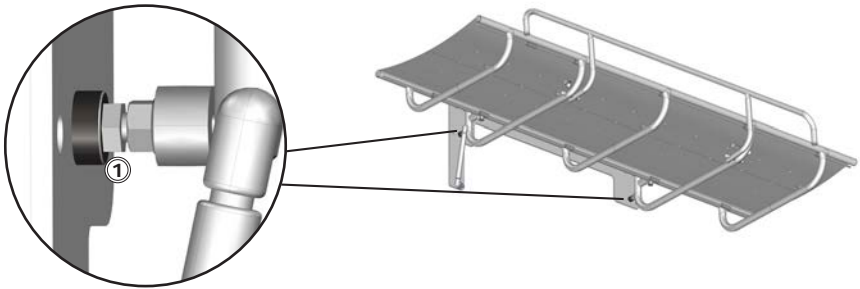en

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Technical data

Materials	Powder laquered stainless steel Plastics: PVC (canvas)
-----------	--

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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