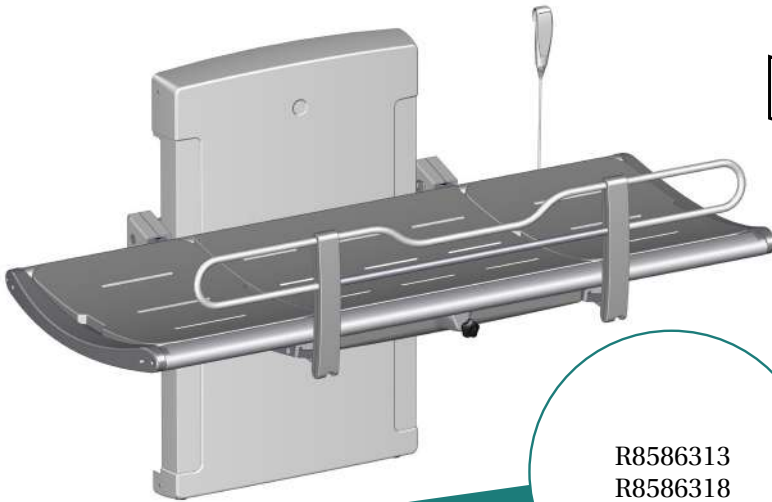
R8586313
R8586318

SCT 3500

Pressure-sensitive stop function

en

*SCT 3500 is part of the SCT 3000 product family.
This leaflet (220351) supplements the Operation and Maintenance Manual (V5325). The technical specifications stated in this leaflet override the corresponding specifications in the main manual.*

Please note the following regarding the item numbers:

The item numbers shown on this leaflet are:

- **R8586313**
- **R8586318**

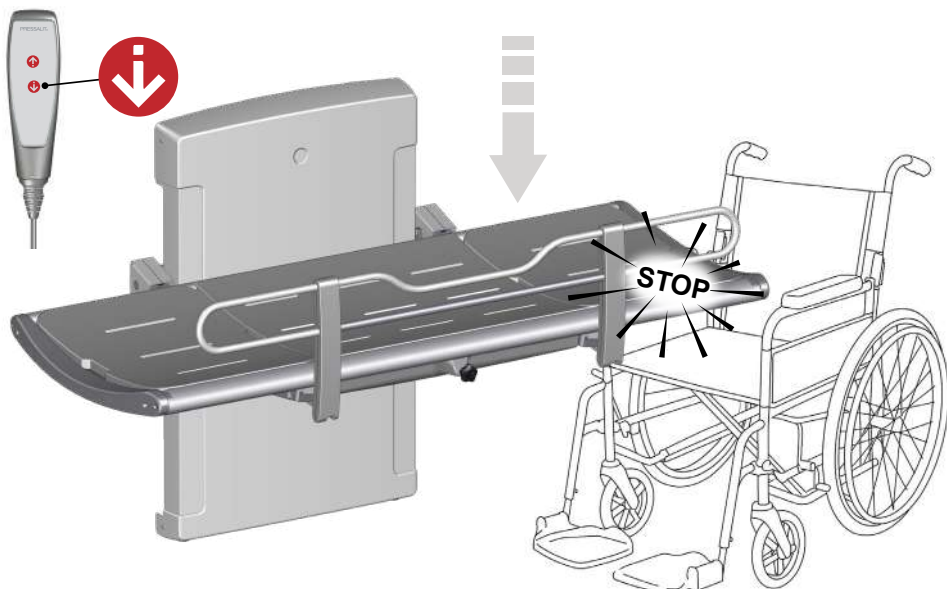
In the main Operation and Maintenance Manual, these correspond to the following item numbers:

- **R8582313**
- **R8582318**


Use the item numbers from the main manual when searching for product information, spare parts, or when navigating the documentation.

Pressure-sensitive stop function

This SCT 3500 shower change table is equipped with a built-in pressure-sensitive stop function. If objects are overlooked beneath the table, the feature provides an additional stop. When sufficient counterforce is detected during lowering, the downward movement stops automatically.

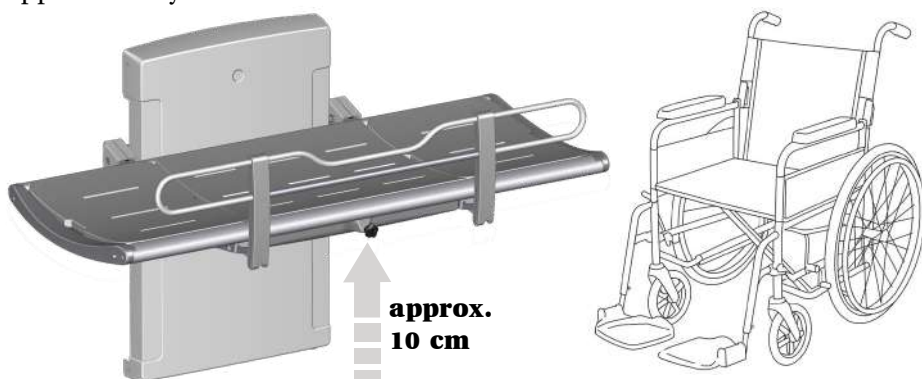
After the stop function is activated, the bench automatically rises by approximately 10 cm.

Refer to the product's Operation and Maintenance Manual for warnings and for general instructions on height adjustment.

Required counterforce

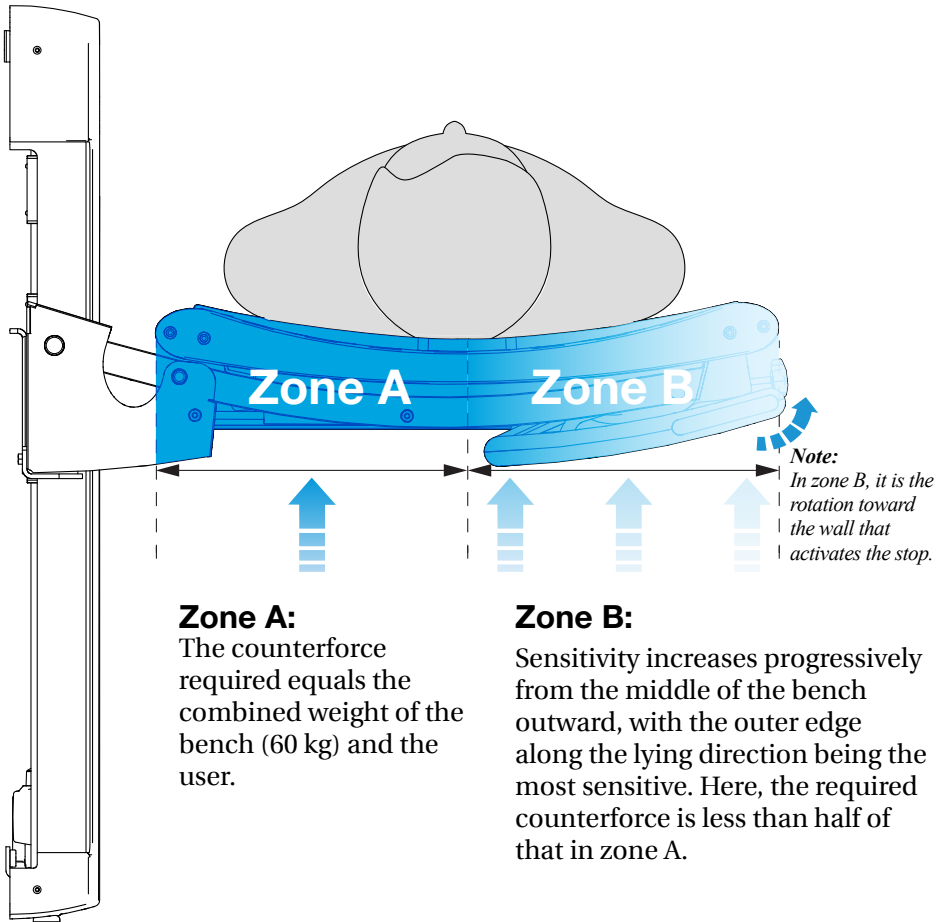The amount of counterforce required to activate the pressure-sensitive stop depends on the combined load of the bench and the user, as well as on the position of the obstruction beneath the bench.

CAUTION

- This function provides only an additional stop if objects were not noticed.
- The pressure-sensitive stop must not be used as a general stop control.
- Pressure applied before activation can damage objects.
- The function responds only to applied pressure; soft or flexible items may compress without activating the stop.

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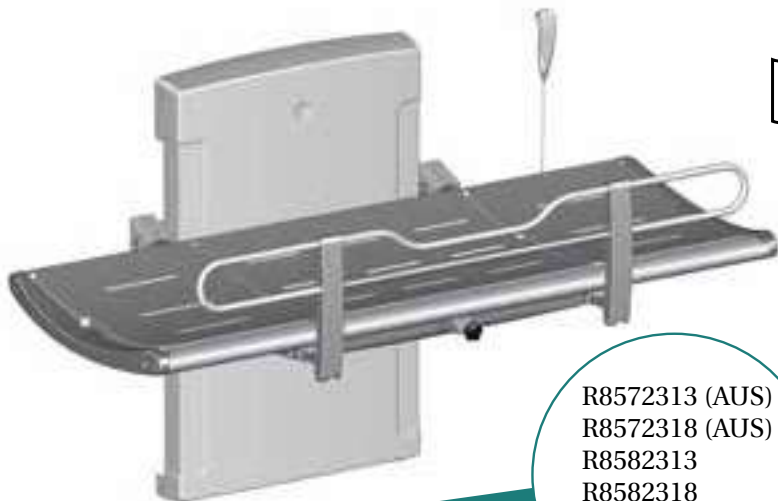
Pressalit's Environmental and Quality Management Systems are certified by LRQA

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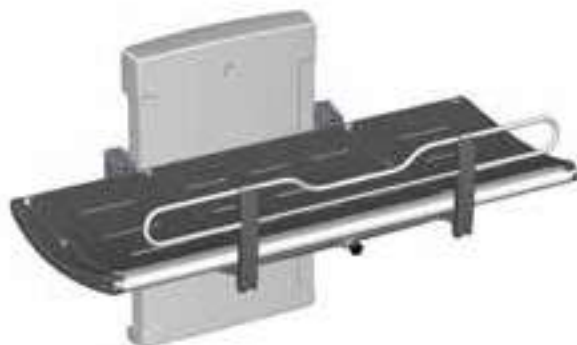
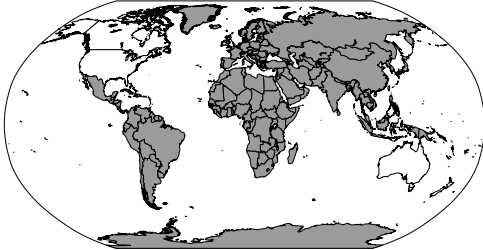
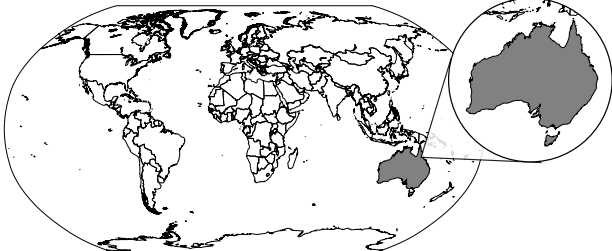
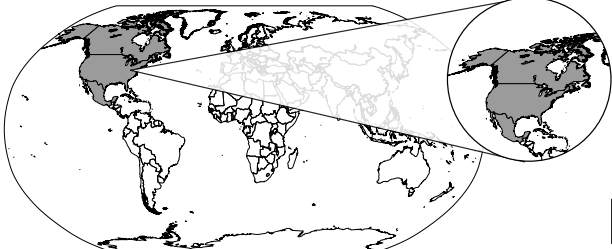
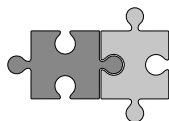
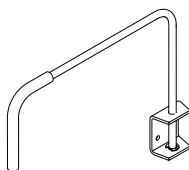
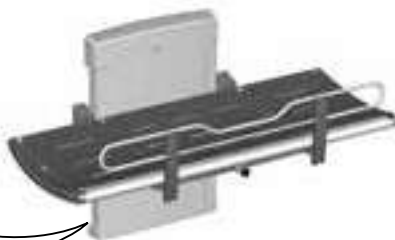
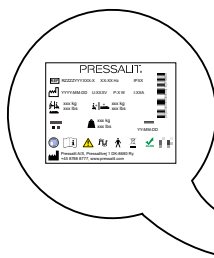
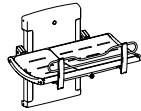
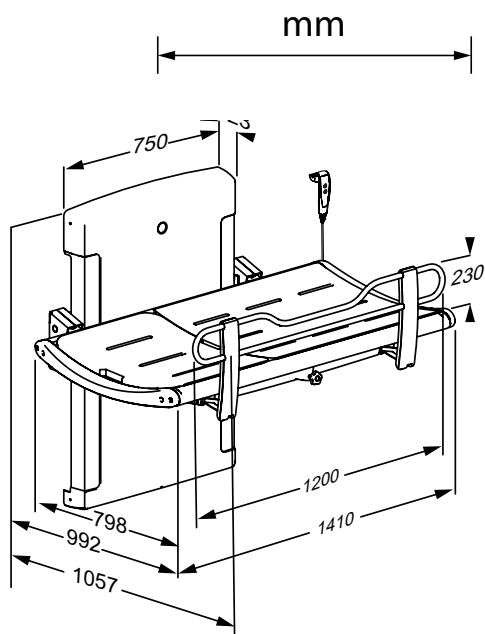
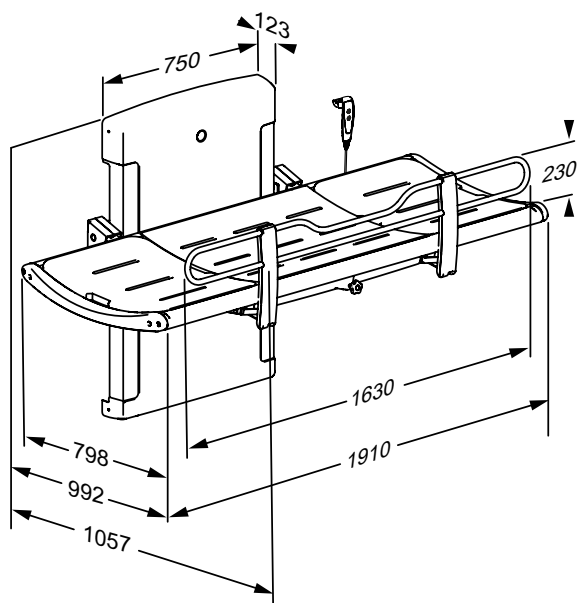
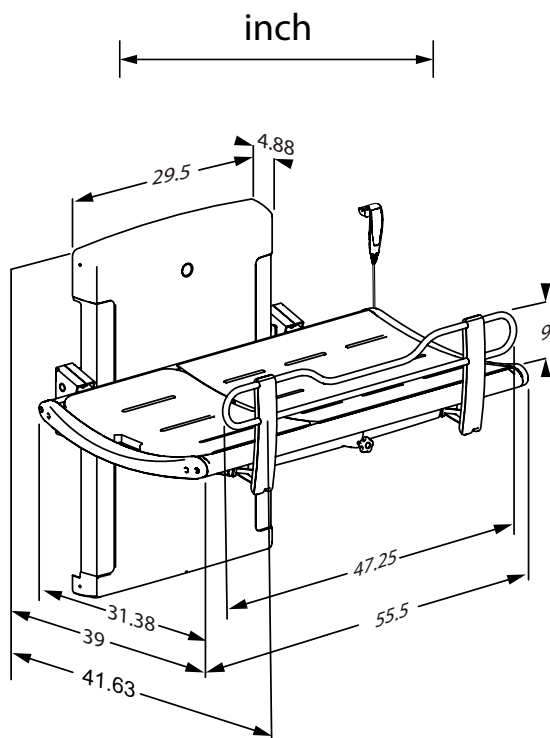
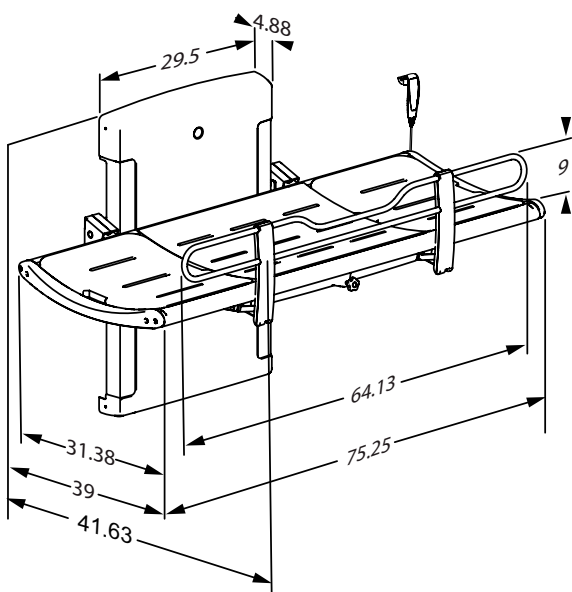
R8572318 - 8582318 - R8592318

Content

Mounting instruction	10
Operational signage	10
Symbols used in this manual	11
General safety	12
Intended use	13
Intended operator profile	13
Intended user profile	13
Operating environment	13
Contraindications for use	13
Biocompatibility	14
Operating instructions	14
Height adjustment	16
Head support	17
Cleaning	18
To remove limescale	18
Disinfection	19
Maintenance	20
Maintenance inspection	20
Product label	22
Resetting	24
Trouble-shooting	25
Electromagnetic compatibility	26
Electromagnetic immunity	27
Electromagnetic emission	30
Storage and transportation	31
Technical data	32
Disposal and recycling	33

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

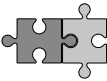
A Quick-guide is included with the product. It can also be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

 WARNING	WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.		
 CAUTION	CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.		
IMPORTANT	IMPORTANT indicates important information about handling and use of the product.		
NOTICE	NOTICE is used to address practices not related to personal safety.		
 Hand injury warning	 Head injury warning	 Electricity / voltage warning	 Other specific warning
 Compatible products	 Spare parts	 Power specifications	 EMC related symbols
 Product weight	 Max. user weight	 Article number / versions	 Do not leave patient unattended

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

This product is intended for bathing a person who is unable to stand up or sit down. The person is lying horizontally on the shower table.

Intended operator profile

The product is to be used by a person trained in health care and/or trained in washing a person in their care.

Intended user profile

The product can be used by all - adults, children, young, old, whose weight does not exceed the **maximum load of 227 kg / 500 lbs.**

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature, in private homes, nursing homes and hospitals under normal working conditions.

Environmental factor	Operating range
Temperature	5°C to 40°C (41°F to 104°F)
Relative humidity	20% to 80% @ 30°C (86°F)
Atmospheric pressure	700 to 1060 hPa
Meters above sea level	Max. 3000 m / 9 800 feet
Duty cycle	2 minutes of continuous use, followed by an 18-minute pause.

Contraindications for use

Use of the unit is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions. Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the unit only after consulting with the attending physician.

NOTICE

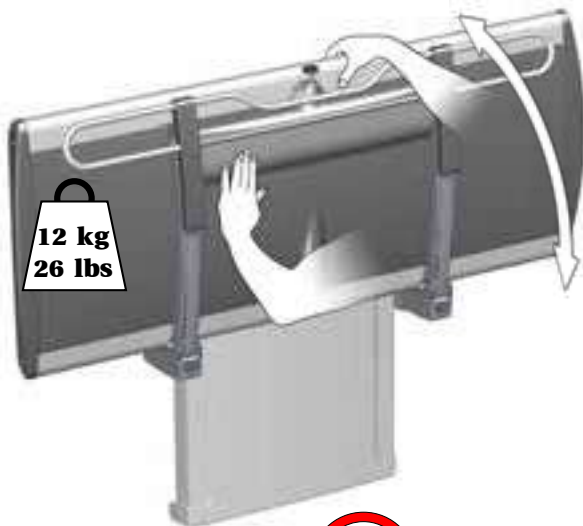
Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

Biocompatibility

The materials in contact with the skin of the user during the intended use have been evaluated for their content of harmful substances according to the procedures in BS EN ISO 10993-1:2020 Biological evaluation of medical devices – Part 1 and are therefore considered suitable for this purpose.

Operating instructions

Support the shower table using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinders make it easy to fold the shower table up against the wall after use.

Do not leave the patient unattended.

CAUTION

Always be careful when moving a person to and from the shower table.

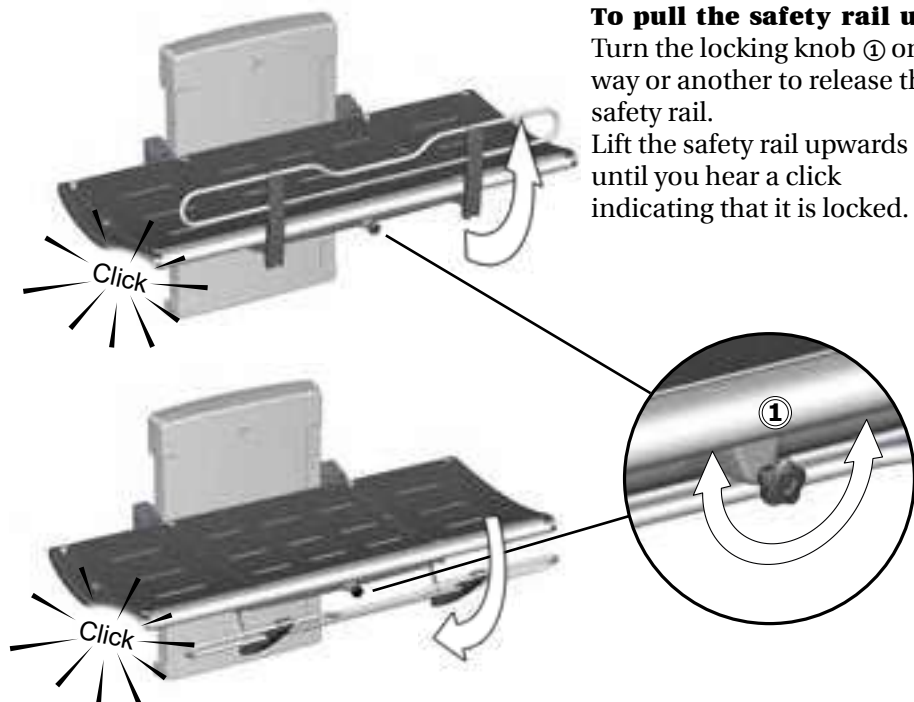
Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

en

To fold the safety rail away:

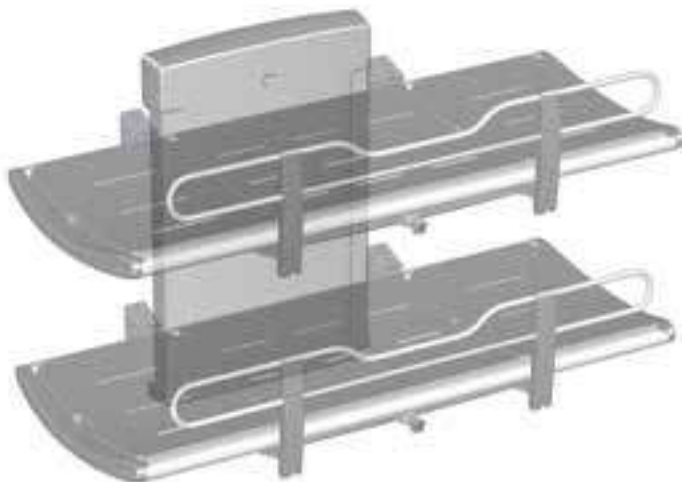
Turn the locking knob ① one way or another to release the safety rail.
Push the safety rail back to the underside of the table until you hear a click indicating that it is locked.

NOTICE

If the locks do not release when turning the release knob, gently lift the rail, to take the weight off the locking mechanism. This will ensure the release of the locks.

Height adjustment

When the table is installed at the correct height (recommended installation height is 868 mm above the floor), it can be adjusted between 300 and 1000 mm above the floor.

The height adjustment is controlled by the wired remote control unit.

Press and hold the up button to raise the table, and press and hold the down button to lower the table.

Make sure that there are no objects under the shower table when adjusting the height.

Head support

The end pad can be used as a head support for increased comfort. Use the cut-out grip ① to lift the pad. Fold the metal bracket ② down to hold the pad in the upright position.

en

The end pads are interchangeable and can be placed at either end of the table.

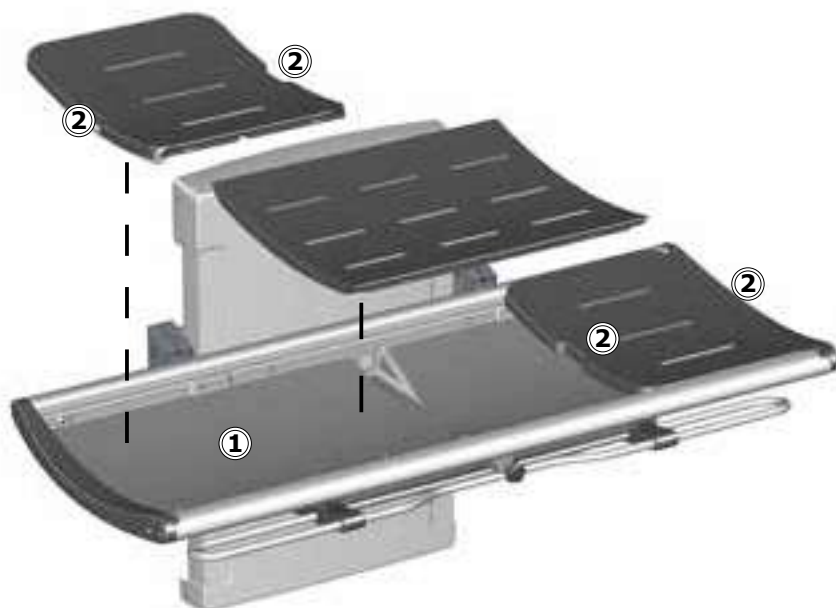
CAUTION

Never let a person lie on the shower table without the pads in place. The tray under the pads is for water collection only. It is not to be used as a bathtub. Risk of user injury and damage to product.

Cleaning

Clean the shower table after each use. Use ordinary soapy water. Rinse off with fresh water and dry with a soft cloth.

Do not leave the pads soaked in water. Remove the pads and dry the water collection tray ①. Use the cut-out grips ② when lifting the pads.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on an area that is not visible when the table is in use.

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

Disinfection

When using the shower table for multiple patients, it is necessary to both clean and disinfect the shower table between uses on different patients to prevent the risk of cross infection. A list of suitable disinfectants can be found at pressalit.com.

NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty of the nursing bench will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.

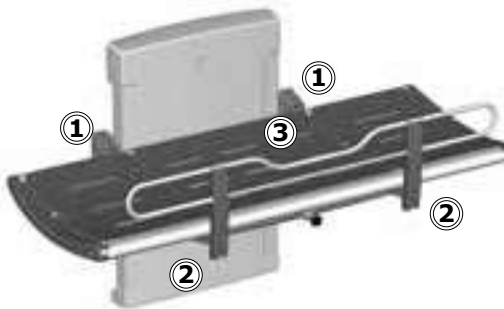
Maintenance

Expected service life of the shower table is 10 years for institutional use, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that maintenance inspection is done once a year.


Maintenance inspection

- Check the hinges ① on the table and the safety rail ②. Tighten if necessary.
- Check the pads ③ for damage and replace if necessary.

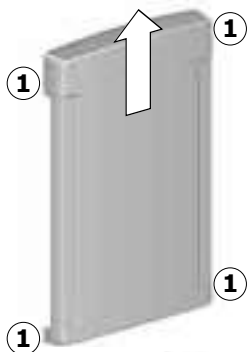- Check the drain pipe connections ④, and check and clean the water trap (if fitted).


Disconnect mains before removing the covers.

- Remove the upper and lower retaining screws ① on the lift unit cover and carefully remove the cover.

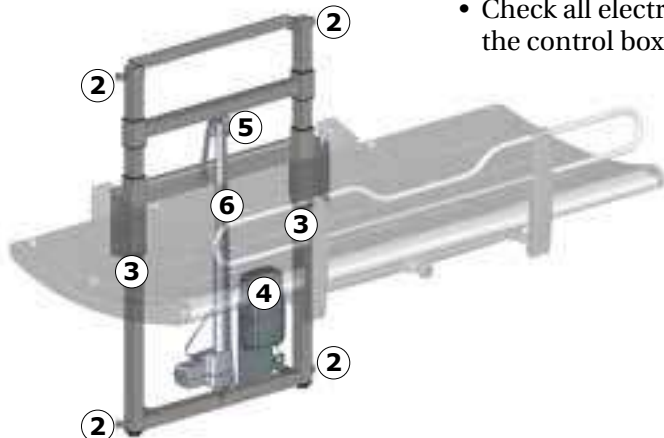
- Check the wall brackets ② and tighten if necessary.
- Oil moving parts, especially the sliding brackets ③.
- Check all electrical connections to the control box ④.

- Check the chain wheel ⑤ for wear.
- Visually inspect the chain ⑥ and other steel parts for penetrating rust.
- Oil the chain ⑥.
- Put the lift unit cover back on and tighten the retaining screws.
- Reconnect the power supply.

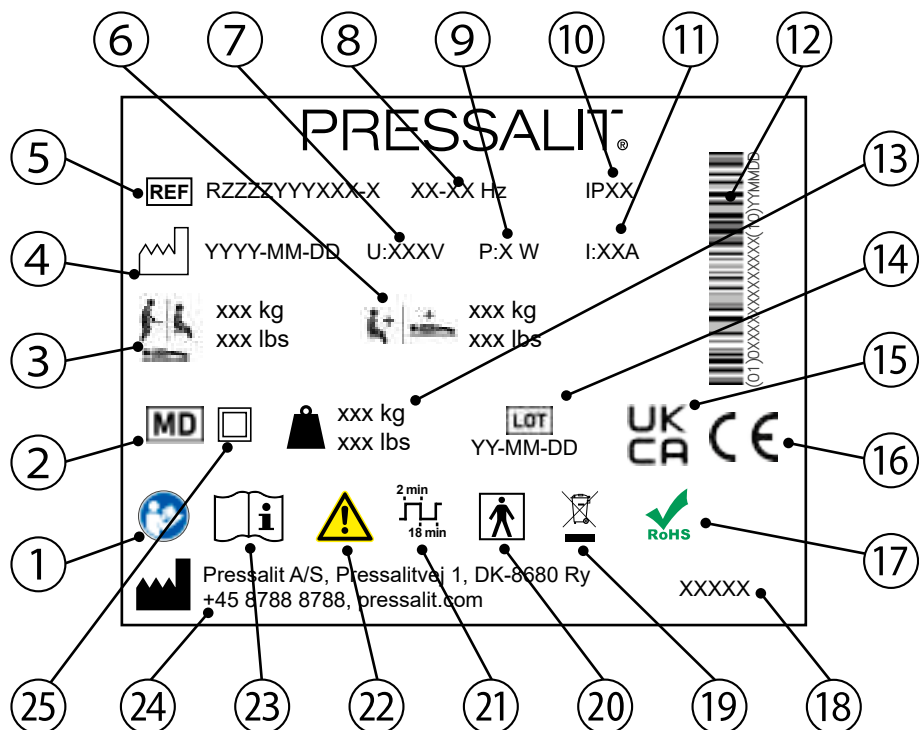Product label

- ① Consult manual for use
- ② Medical device
- ③ Maximum user weight
- ④ Date of manufacture
- ⑤ Article number
- ⑥ Max load
- ⑦ Mains voltage
- ⑧ Frequency
- ⑨ Maximum power consumption

⑩	IP class
⑪	Electric Current
⑫	UDI carrier GS1-128 barcode
⑬	Product weight
⑭	Batch code
⑮	UK Conformity Assessed
These products are CE-marked in accordance with relevant requirements in: • Regulation (EU) 2017/745 (5 April 2017) on medical devices. • Council Directive 2006/42/EC (of 17 May 2006) on machinery. • Council Directive 2011/65/EC (of 8 June 2011) on RoHS. A declaration of conformity can be found at pressalit.com	
⑰	This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
⑱	Product name (if any)
⑲	Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
⑳	Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function The product complies with requirements of IEC 60601-1 to provide protection against electric shock.
㉑	Duty cycle
㉒	Consult the manual for important cautionary information such as warnings and precautions.
㉓	Consult manual for use
㉔	Name and address of manufacturer
㉕	Class II Product, double insulated

Resetting

If the height adjustment does not work, and the control box beeps and/or flashes, the unit must be reset.


Press and hold the up button and the down button simultaneously. The control box emits a beep sound. When the beep stops, release both buttons.

Take the unit to its lowest position.
The unit is now reset.

Trouble-shooting

Symptom	Cause	Action
The shower table does not stay in upright position.	One or more gas cylinders are defective.	Contact Pressalit A/S or your local dealer.
The product will not work, even though the remote control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product does not work, and the control box beeps and/or flashes.	The product needs to be reset.	See "Resetting".
The water does not run out of the water collection tray.	Adjustment screws too high or too low.	Adjust the adjustment screws ①.
The shower table runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

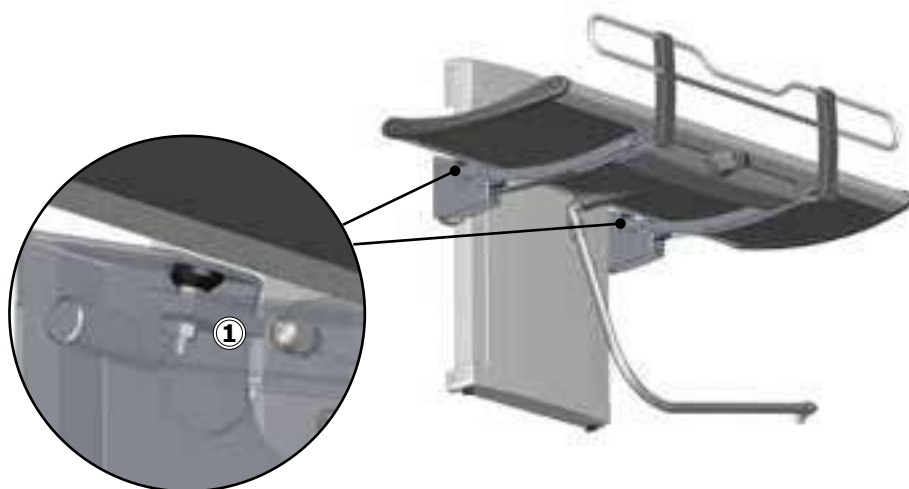Contact Pressalit A/S or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Electromagnetic compatibility

The device is suitable for use at home, at daycare centres, at day centres for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and RF shielded rooms with medical systems for magnetic resonance imaging with high intensity of electromagnetic disturbances.

The device does not have any function related to basic safety or essential performance. In case of loss or degradation of performance due to electromagnetic disturbances, the user still can get on and off the product.

Cable	Length
Power cable	3,2 m (126")
Hand control spiral cable	Approx. 5 m (197") extended

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Electromagnetic immunity

Electrostatic discharge	
	The product must not be affected by electrical discharge between a person and the product in such a way as to initiate actions or render components defective.
Test standard	Immunity test level
EN 61000-4-2	+/- 8 kV contact +/- 2, 4, 8, 15 kV air


Radiated RF electromagnetic field	
	The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.
Test standard	Immunity test level
EN 61000-4-3	10 V/m 80 MHz to 2,7 MHz 80 % AM 1 kHz

en

Immunity to proximity fields from RF wireless communication equipment

	The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.
Test standard	Immunity test level
EN 61000-4-3*	385 MHz, 27 V/m, 18 Hz PM (50 % duty cycle square wave) 450 MHz, 28 V/m, FM +/- 5 kHz deviation, 1 kHz sine 710 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave) 745 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave) 780 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave) 810 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave) 870 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave) 930 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave) 1720 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave) 1845 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave) 1970 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave) 2450 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave) 5240 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave) 5500 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave) 5785 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)

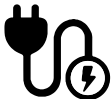Electrical fast transient/burst

	The product must not be affected by rapid pulse signals through the cord. This could cause the components to become defective, or the product could initiate actions.
Test standard	Immunity test level
EN 61000-4-4	Signal line: +/- 1 kV, AC power line: +/- 2 kV 100 kHz repetition frequency

Surge immunity	
	The product must not be affected to initiate actions or to render components defective as a result of lightning.
Test standard	Immunity test level
EN 61000-4-5	Line-to-line: +/- 0.5 kV, +/- 1 kV Line-to-ground: +/- 0.5 kV, +/- 1 kV, +/- 2 kV

Immunity to conducted disturbances induced by RF fields	
	The wires must not be affected by other RF fields so as to initiate actions or render components defective.
Test standard	Immunity test level
EN 61000-4-6	3 V (6 V in ISM bands and amateur radio bands) 0,15 to 80 MHz , 80 % AM at 1 kHz

en

Voltage dips	
	The product must not initiate or limit actions as a result of a dip in the power supply.
Test standard	Immunity test level
EN 61000-4-11	0 % U_T ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, 0 % U_T ; 1 cycle at 0°, 70 % U_T ; 25/30 cycles at 0° Single phase at 0°

Voltage interruptions	
	The product must not initiate or limit actions as a result of a short-term power supply stop.
Test standard	Immunity test level
EN 61000-4-11	0 % U_T ; 300 cycle. 250/300 cycles

Electromagnetic emission




Conducted RF emission	
	The product must not send too much noise into the grid. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1

Radiated RF emission	
	The product must not emit RF signals in the room. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1

Harmonic current emissions	
	The product must not unintentionally draw power from the grid.
Test standard	Emission test class and group
IEC 61000-3-2	Class A

Voltage changes, voltage fluctuations and flicker	
	The product must not affect other products when it is connected to the grid. For example, these could be lights that could start flashing and thus bother people in the room.
Test standard	Emission test class and group
EN 61000-3-3	Tested according to the standard.

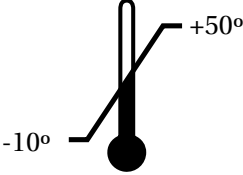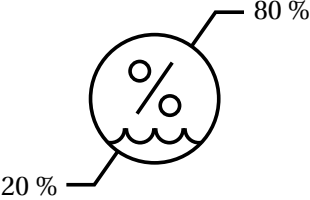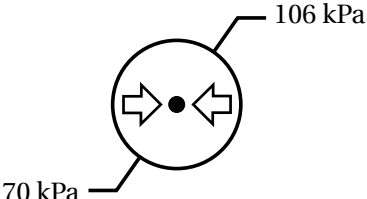Storage and transportation

	The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.
	The relative humidity for transportation and storage of the unit must be between 20 and 80% (RH).
	The air pressure for transportation and storage of the unit must be between 70 and 106 kPa.

en

- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Technical data

Max load	227 kg / 500 lbs	
Materials	Powder laquered stainless steel Aluminum Plastics (PVC, Polystyrene)	
Weight	R8572313 R8582313 R8592313	77,60 kg / 171 lbs
	R8572318 R8582318 R8592318	82,35 kg / 182 lbs
IP class	The electrical parts have an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.	

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
PVC and polystyrene	Hard PVC is granulated and re-melted. Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production.
Polyurethane	Like other plastics, polyurethane can be recycled in two primary ways: mechanical recycling, in which the material is reused in its polymer form, and chemical recycling that takes the material back to its various chemical constituents.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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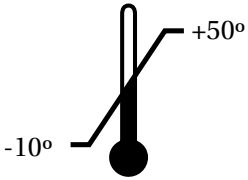
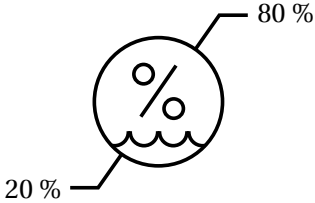
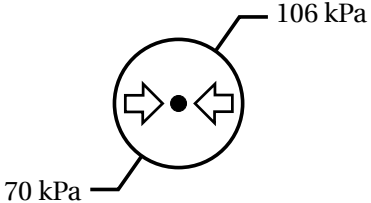
Opbevaring og transport

 A thermometer icon with a vertical scale. A line points from the top of the scale to the text '+50°'. Another line points from the bottom of the scale to the text '-10°'.	Temperaturen for transport og opbevaring af enheden skal være mellem -10 og 50 grader Celsius.
 A circular icon containing a percentage symbol (%). A wavy line is at the bottom of the circle. A line points from the top of the circle to the text '80 %'. Another line points from the bottom of the circle to the text '20 %'.	Den relative luftfugtighed for transport og opbevaring af enheden skal være mellem 20 og 80% (RH).
 A circular icon containing a central dot and two arrows pointing towards it from the left and right sides. A line points from the top of the circle to the text '106 kPa'. Another line points from the bottom of the circle to the text '70 kPa'.	Lufttrykket for transport og opbevaring af enheden skal være mellem 70 og 106 kPa.

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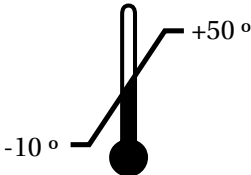
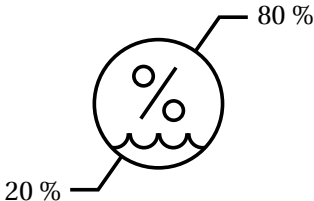
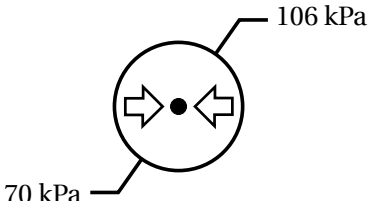
Stockage et transport

 A thermometer icon with a vertical scale. A line points from the top of the scale to the text '+50 °' and another line points from the bottom of the scale to the text '-10 °'.	La température de stockage et de transport de l'unité doit être comprise entre -10 et 50 degrés Celsius.
 A circular icon containing a percentage symbol (%). A wavy line is at the bottom of the circle. A line points from the top of the circle to the text '80 %' and another line points from the bottom of the circle to the text '20 %'.	L'humidité relative pour le stockage et le transport de l'unité doit être comprise entre 20 et 80 % (HR).
 A circular icon containing a central dot and two arrows pointing towards it from the left and right sides. A line points from the top of the circle to the text '106 kPa' and another line points from the bottom of the circle to the text '70 kPa'.	La pression de l'air pour le stockage et le transport de l'unité doit être comprise entre 70 et 106 kPa.

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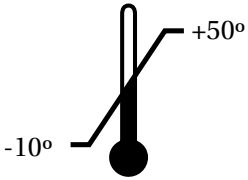
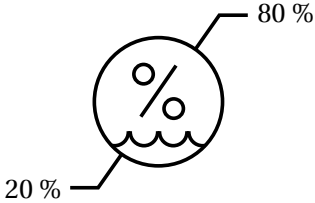
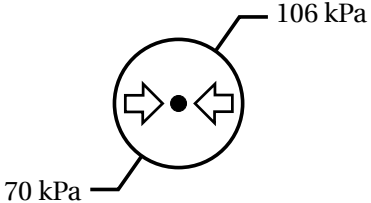
 A thermometer icon with a vertical scale. A line points from the top of the scale to the text '+50°' and another line points from the bottom of the scale to the text '-10°'.	Temperaturen for transport og oppbevaring av enheten må være mellom -10 og 50 grader Celsius.
 A circular icon containing a percentage sign (%). A wavy line at the bottom of the circle represents water. A line points from the top of the circle to the text '80 %' and another line points from the bottom of the circle to the text '20 %'.	Den relative fuktigheten for transport og oppbevaring av enheten må være mellom 20 og 80 % (RF).
 A circular icon containing a central dot and two arrows pointing towards it from the left and right sides. A line points from the top of the circle to the text '106 kPa' and another line points from the bottom of the circle to the text '70 kPa'.	Lufttrykket for transport og oppbevaring av enheten må være mellom 70 og 106 kPa.

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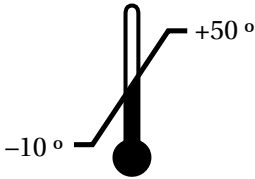
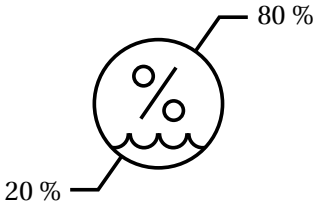
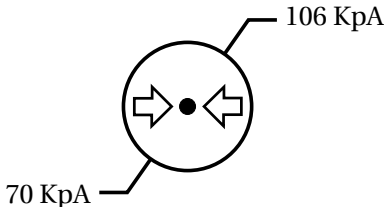
 A thermometer icon with a vertical scale. A line points from the top of the scale to the text '+50 °' and another line points from the bottom of the scale to the text '-10 °'.	La temperatura para el transporte y el almacenamiento de la unidad debe estar comprendida entre -10 y 50 grados.
 A circle containing a percentage sign (%) and wavy lines representing humidity. A line points from the top of the circle to the text '80 %' and another line points from the bottom of the circle to the text '20 %'.	La humedad relativa para el transporte y el almacenamiento de la unidad debe estar comprendida entre 20 y 80 % (HR).
 A circle containing a central dot and two arrows pointing towards it from opposite sides, representing pressure. A line points from the top of the circle to the text '106 KpA' and another line points from the bottom of the circle to the text '70 KpA'.	La presión de aire para el transporte y el almacenamiento de la unidad debe estar comprendida entre 70 y 106 KpA.

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