

# PRESSALIT.

Product	SELECT wash basin bracket, wall-mounted with floor support, electrically height adjustable with wired hand control	
	R4951112	
Item number	R4951112	
	VVS:654005212	
	NRF:6132544	
	EAN:5708590322445	
	HMI: 82619	
	NBS:N13:30,335	
Colour	Anthracite grey	
Product description	SELECT wash basin lifter are equipped with a front cover plate, which conceals the plumbing installations (flexible feed and waste system to be bought separately).	
	SELECT wash basin lifter, floor standing. Electrically height adjustable 300 mm with wired hand control. Recommendation: flexible feed/waste system R2065 (UK market R2075). Incl. fixing bolts for wash basin (Recommended with MATRIX SMALL and MATRIX CURVE/NEW CURVE wash basins). Materials: Anodized aluminium and polystyrene (PS). Excl. mounting kit.	
Adjustability	Height adjustable	
Adjustment height	300 mm	
Product capacity	Maximum load: 150 kg. Suitable for wash basins weighing no more than 22 kg. When the wash basin is loaded with 150 kg, each wall fastener will be carrying 18 kg. Wash basin brackets need to be able to carry the weight of the wash basin, as well as support the weight of the person who is leaning against the wash basin or who is using the wash basin to rise from seated position.  Neither wash basin nor wash basin bracket are intended to carry the full body weight of a person, and the test methods applicable to wash basin brackets reflect this: ISO 17966 prescribes a static test on the front edge of the wash basin at 75% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 150 kg and the wash basin weighs 22 kg, it is – according to ISO 17966 – the equivalent of a user weighing 171 kg leaning on the front edge of the wash basin.	
Speed upwards	Speed up mm: 21 mm/s Speed down mm: 21 mm/s	
Noise level	45 dB	



# PRESSALIT.

Product	SELECT wash basin bracket, wall-mounted with floor support, electrically height adjustable with wired hand control	
Item number	R4951112	
Materials	Anodized aluminium. The front plates are made of polystyrene (PS). Top plate in compact laminate. Flammability-tested (ISO 8191-1 and ISO 8191-2). Test report available.	
Ergonomy	The lifter's stringent and pure design as well as smooth surfaces are easy to clean and attractive to look at. They also help ensure good hygiene in the bathroom. The exclusive aluminium side profi les are light, durable and elegant to look at.	
Features	The wash basin is raised or lowered using the wired hand control or the lever control at the end of the wash basin. The lever control can be positioned on either side of the wash basin and also serves as a towel holder. The lever control is strong enough to be used as an extra support.  The wash basin can be adjusted 300 mm – from 670 to 970 mm above the floor.	
Mains connection (V)	230	
Actuator (motor)	LA31 Careline, (Linak), 1500N, IPX4	
Control box	CB9 Careline, (Linak), IPX6	
Wired remote control	HB3 (Linak), IP66	
Power consumption (maximum load)	Power consumption by maximum load, watt: 75 W Power consumption by standby: 1,5 W	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



## PRESSALIT®

Product	SELECT wash basin bracket, wall-mounted with floor support, electrically height adjustable with wired hand control		
Item number	R4951112		
Cleaning	PLUS & SELECT wash basin brackets The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.		
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.		
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%		
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro		
	The product must be wiped down with a soft rag wrung out in clear water immedia after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.		
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the p must be wiped over with a damp cloth wrung out in clear water.		
Declaration of conformity	CE UK MDR-MASK-RoHS This product is in compliance with all relevant requirements of:  - Regulation (EU) 2017/745 on medical devices		

- Council Directive 2006/42/EC on machinery

Equipment Regulations 2012.

- Supply of Machinery (Safety) Regulations 2008

Council Directive 2011/65/EC on RoHS. For the UK:
Medical Device Regulation (MDR) 2002 on medical devices.

- Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic



### PRESSALIT®

Product Item number SELECT wash basin bracket, wall-mounted with floor support, electrically height adjustable with wired hand control

R4951112

#### Installation in different wall types

Concrete >B25  V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers.  To be ordered separately.	
Solid Brick >= Mz12  V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers.  To be ordered separately.	
Solid Sand Lime Brick >= HIz12  V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers.  To be ordered separately.	
Aerated Concrete P4  V8655 - Mounting kit, 4 coach screws Ø10 x 70, plugs GB14, with grey screw covers. To be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood  V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with white screw covers. NB! Plugs are not needed.  To be ordered separately.	



### PRESSALIT®

Product Item number	SELECT wash basin bracket, wall-mounted with floor support, electrically height adjustable with wired hand control R4951112
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood  Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)  Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.
	Drywall (2 x 12,5 mm) water-resistant  Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT.