R8212, R8222
R8232, R8242
R8312, R8322
R8332, R8342

SELECT TL1 & TL2 toilet lifter – Operation and maintenance manual	en
SELECT TL1 & TL2 toiletløfter – Brugs- og vedligeholdelsesvejledning	dk
SELECT TL1 & TL2 WC-Lifter – Gebrauchs- und Pflegeanleitung	de
SELECT TL1 & TL2 WC réglable en hauteur – Manuel d'utilisation et d'entretien	fr
SELECT TL1 & TL2 hoog-laag-toiletsysteem – Gebruiks- en onderhoudshandleiding	nl
SELECT TL1 & TL2 Upphängning till toalett – Bruks- och underhållsanvisning	sv
SELECT TL1 & TL2 Oppheng til toalett – Bruks- og vedlikeholdsveiledning	no
SELECT TL1 & TL2 Soporte de inodoro - Instrucciones de uso y mantenimiento	es
SELECT TL1 i TL2 Podnośnik toalet – Instrukcja obsługi i konserwacji	pl
SELECT TL1 & TL2 坐便器升降装置 — 操作和维护手册	中文
SELECT TL1 & TL2 رافعة كرسي المراض – دليل التشغيل والصيانة	العربية

English	9
Dansk	22
Deutsch	35
Français	48
Nederlands	61
Svenska	74
Norsk	87
Español	100
Polski	113
中文	126
عربي	139

Pressalit's Environmental and Quality Management Systems are certified by LRQA

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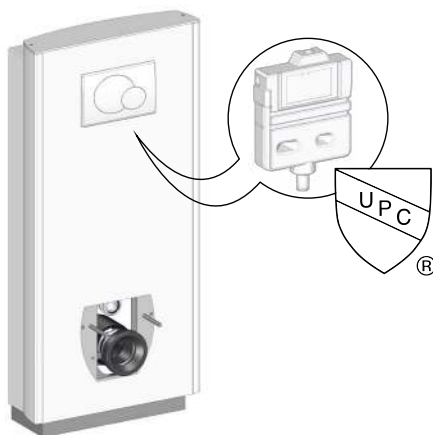
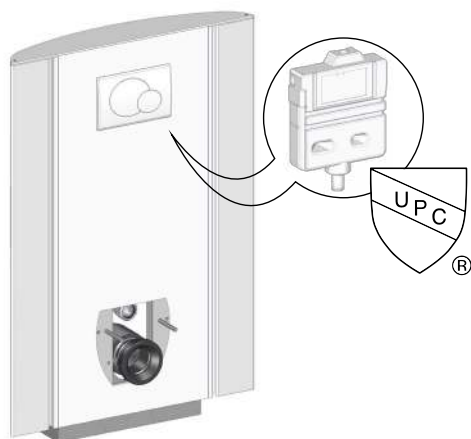
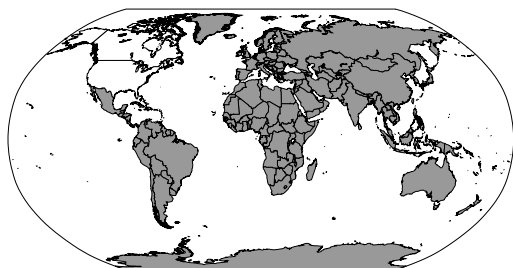
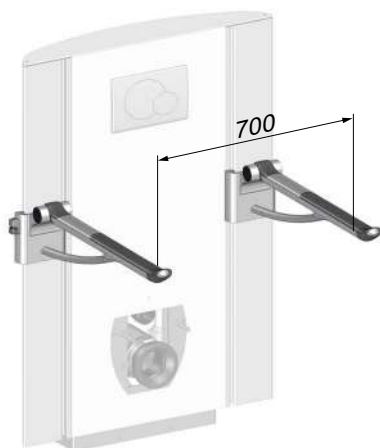
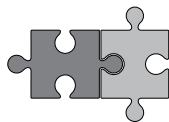
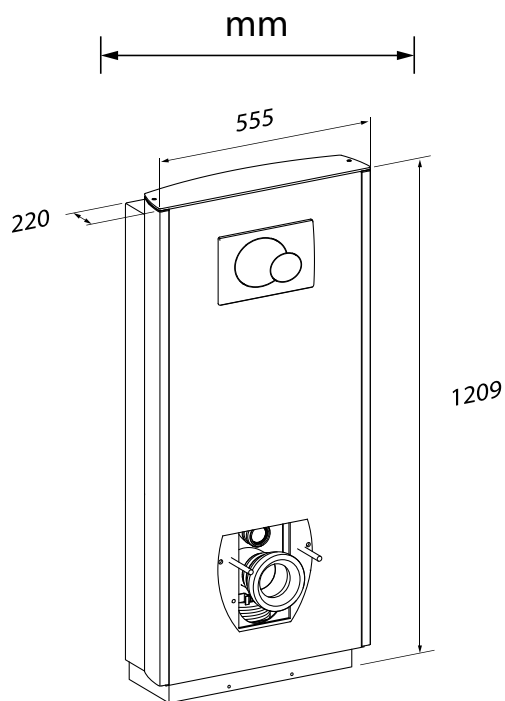
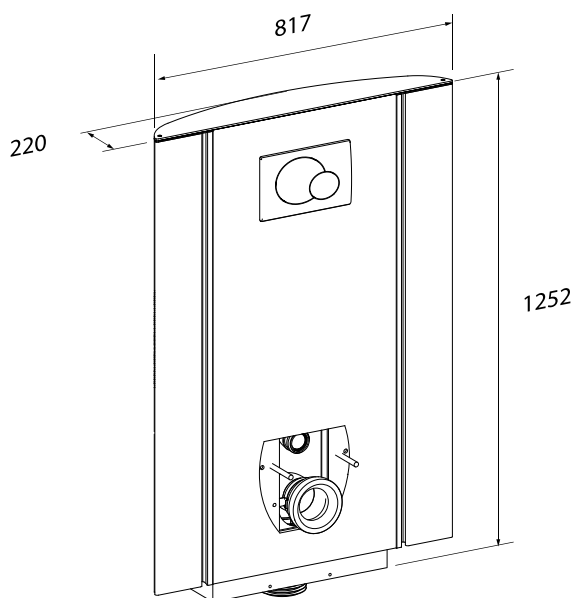
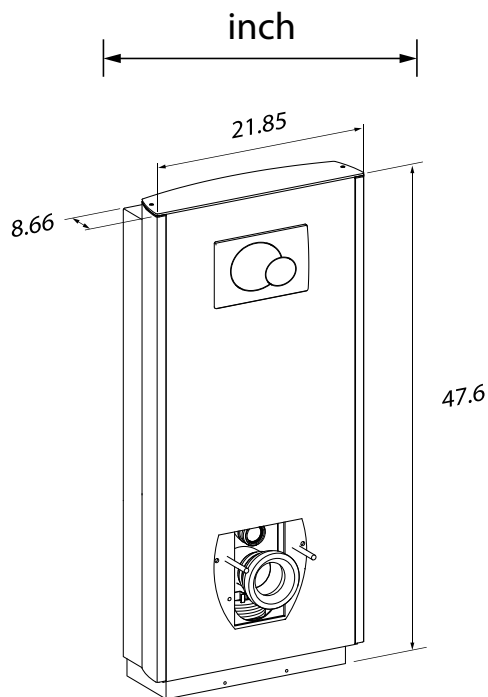
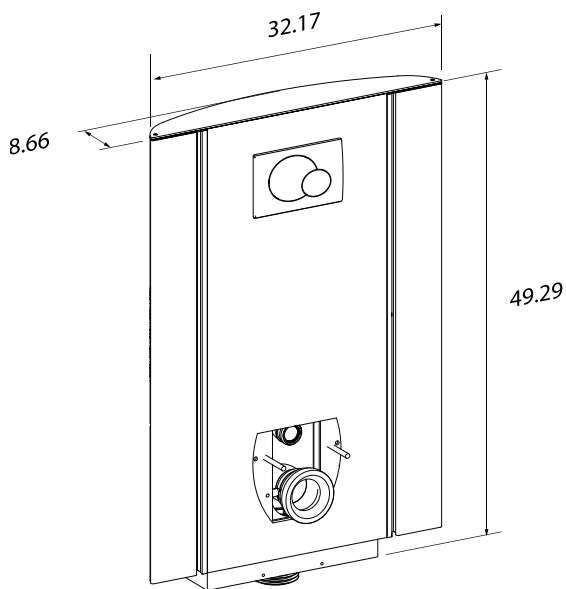
R8312, R8322, R8332, R8342

Content

Mounting instruction	9
Symbols used in this manual	10
General safety	11
Product label	12
Intended use	14
Intended operator profile	14
Operating environment	14
Operating instructions	15
Support arms (optional)	16
Cleaning	17
To remove limescale	17
Maintenance	18
Maintenance inspection	18
Technical data	20
Trouble-shooting	21
Disposal and recycling	21

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

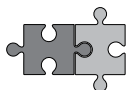
Head injury warning

Feet injury warning

Electricity or
potentially dangerous
voltage

Compatible products

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

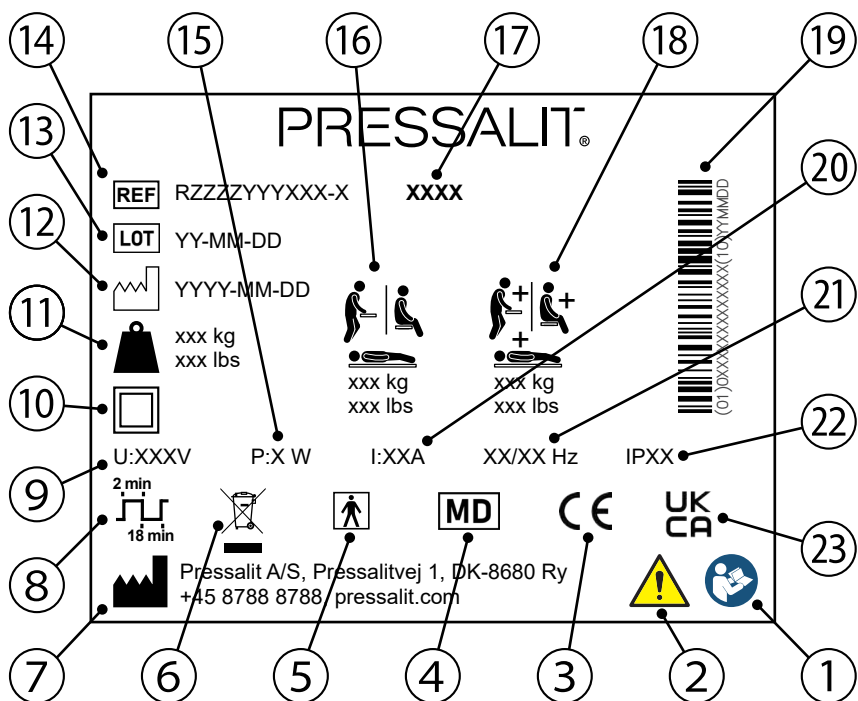
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

en

- 1 Read the manual before use
- 2 Consult the manual for important cautionary information such as warnings and precautions
- 3 These products are CE-marked in accordance with relevant requirements in applicable legislation.
A declaration of conformity can be found at pressalit.com
- 4 Medical device
- 5 Type BF applied part. Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock
- 6 Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑦	Name and address of manufacturer
⑧	Duty cycle
⑨	Mains voltage
⑩	Class II Product, double insulated
⑪	Product weight
⑫	Production date
⑬	Batch code
⑭	Article number
⑮	Maximum power consumption
⑯	Maximum user weight
⑰	Model name
⑱	Max load
⑲	GS1-128 barcode
⑳	Electric Current
㉑	Frequency
㉒	IP class
㉓	UK Conformity Assessed. UK responsible representative: Pressalit Ltd., 100, Longwater Avenue, Green Park, Reading, Berkshire, RG2 6GP, United Kingdom.

Product label placement ①.

Intended use

This product is intended for optimum ergonomics at a toilet, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

Operating instructions

The toilet lifter can be adjusted 400 mm / 15¾" up and down.

Max user weight: 200 kg / 440 lbs

Max load: 235 kg / 518 lbs

The height adjustment is controlled by the wired remote control unit.

Press and hold the Up button to raise the unit, and press and hold the Down button to lower the unit. When the button is released, the motion stops.

Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

Never place objects underneath the toilet.

Support arms (optional)

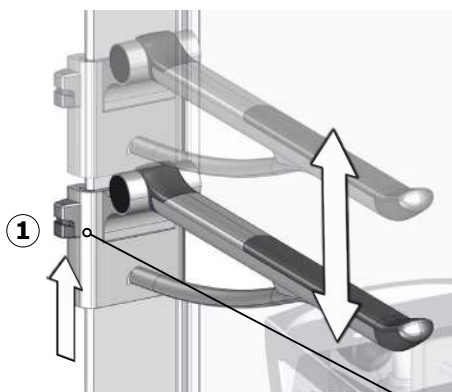
The toilet lifters model R8312, R8322, R8332 and R8342 can be equipped with support arms.

The support arms can be folded up and down. Fold them up when not in use.

The support arms can also be adjusted 240 mm / 8 $\frac{1}{16}$ inches in height.


en

To adjust upwards, pull the support arm upwards.

To adjust the support arm downwards, pinch the locking button ① and pull the support arm a little bit upwards while holding the button. Then move the support arm downwards. Release the locking button to secure the support arm in place.

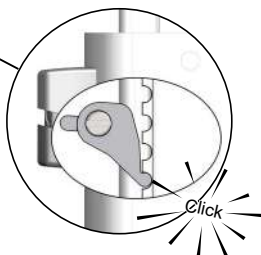Make sure that the locking button pin is engaged in the notch.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used, provided that the disinfectant is suitable for this product and that the disinfectant manufacturer's safety instructions are followed.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the product is in use.

Maintenance

Expected service life of the toilet lifter is 10 years, provided that maintenance is carried out once a year as described below.

Maintenance inspection

Take the toilet to its highest position.

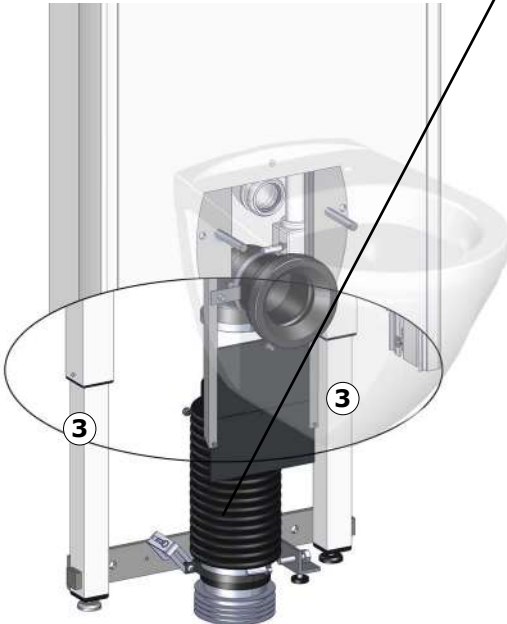
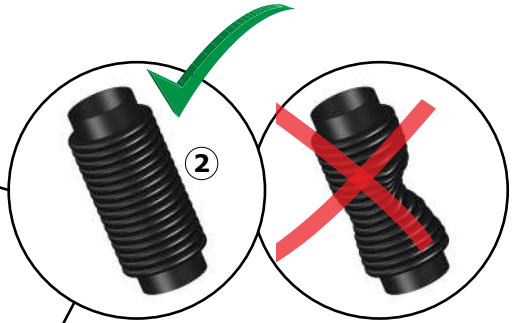
**Disconnect mains
before removing
the cover.**

Remove the lower cover plate ①
(can be clicked off).

Reach up under the front cover to unplug the mains plug ① from the control box inside the cabinet.

Ensure that the flexible outlet hose ② can move freely when the toilet is height adjusted. The flexible outlet must be symmetrical when stretched out.

Lubricate the inner square tubes ③ on all 4 sides as far as can be reached.

Reconnect the mains plug.

Take the toilet to its lowest position, and then run it to the top position again to distribute the lubricant.

Reattach the lower cover plate.

Technical data

Max user weight	200 kg / 440 lbs	
Max load	235 kg / 518 lbs	
Product weight	R8212, R8222 R8232, R8242	54,7 kg/ 120,59 lbs
	R8312, R8322 R8332, R8342	60,7 kg / 133,82 lbs

Trouble-shooting

Symptom	Cause	Action
Operating buttons don't work.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected.
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

en

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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