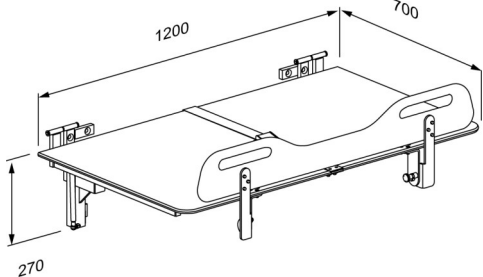




<p>Product Item number</p>	<p>CT 5000 change table, fixed height R8455512000</p>
<p>Colour Product description</p>	<p>VVS: 777540820 EAN: 5708590376585 HMI: 138135 FDA: INT/D124320</p> <p>White</p>  <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>CT 5000 change table, lying surface in compact laminate, 700 x 1200 mm , white. With safety rail and body belt, foldable, fixed height, wall-mounted.</p>
<p>Adjustability Adjustment height Product capacity</p>	<p>Fixed mounting 0 mm</p> <p>Maximum user weight (MUM): 150 kg. Safe working load (SWL): 150 kg. When the table is loaded with 150 kg, each mounting bolt supports 52 kg</p> <p><i>This product has been tested statically with a load of 225 kg as the ISO 17966 standard requires testing with safety factor 1.5.</i></p>
<p>Materials Ergonomy</p>	<p>Powder-laquered stainless steel, lying surface Compact Density Fiberboards (CDF).</p> <p>The ergonomically designed safety rail has a practical lower central section, which provides better access and work environment for the carer while transferring, changing and dressing the user. Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.</p>

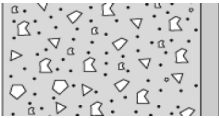
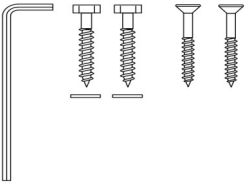

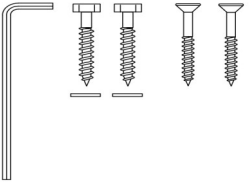
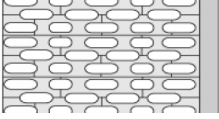
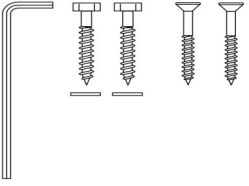
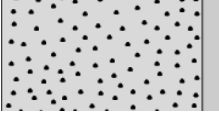
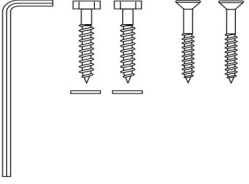
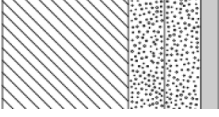
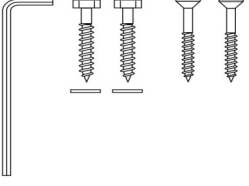


Product	CT 5000 change table, fixed height
Item number	R8455512000
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date.
Cleaning	<p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes</p> <p>OxiGenesis from ENVIROX</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>

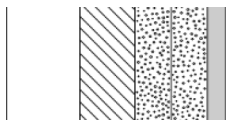

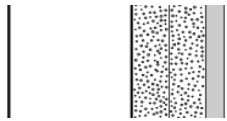

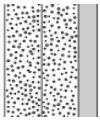



Product	CT 5000 change table, fixed height
Item number	R8455512000

Installation in different wall types

	Concrete >B25 V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	
	Solid Brick >= Mz12 V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	
	Solid Sand Lime Brick >= Hlz12 V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	
	Aerated Concrete P4 V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8597 - Mounting kit, 2 coach screws Ø10 x 70 and 2 countersunk coach screws Ø10 x 80	



Product Item number	CT 5000 change table, fixed height R8455512000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.