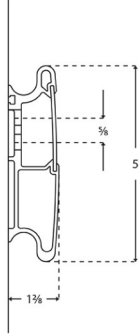




<p>Product</p> <p>Item number</p>	<p>PLUS wall track, 47.2"</p> <p>R9812112</p>
<p>Color</p> <p>Product description</p>	<p>VVS:615841112</p> <p>NRF:7501015</p> <p>EAN:5708590305776</p> <p>HMI: 71741</p> <p>Anthracite gray</p>  <p>PLUS wall tracks are used for mounting of support arms, shower seats and sink brackets, and allows sideways adjustment of these products. See below a selection of products for fitting to the wall track.</p> <p>PLUS wall track, 47.2", for horizontal mounting. The cover plate hides bolts, wires and simplifies cleaning. Materials: Aluminium and polystyrene. Excl. mounting kit.</p>
<p>Adjustability</p> <p>Product capacity</p>	<p>Fixed mounting</p> <p>Maximum load: Up to 2200 lbs. depending on the wall type.</p>
<p>Materials</p> <p>Ergonomy</p> <p>Features</p>	<p>The wall track is primarily made of 10-micrometer anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.</p> <p>The track distributes the weight of the individual bathroom product, for example a sink bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions.</p> <p>The horizontal track makes it possible to shift the product along the wall to generate extra space as needed. The front cover plate conceals mounting bolts and any cables.</p>
<p>Warranty</p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>

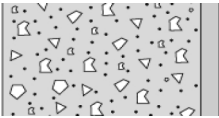
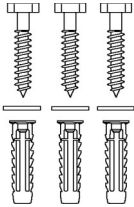

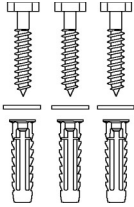
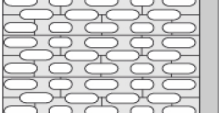
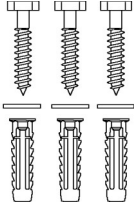
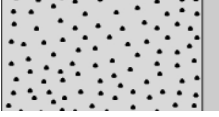
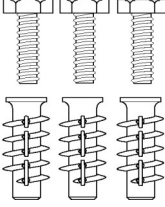
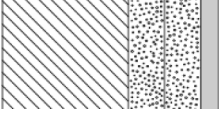
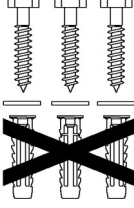


Product Item number	PLUS wall track, 47.2" R9812112
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
Declaration of conformity	<p>This product fulfils all relevant requirements in European Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an accessory for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical devices to be used in accordance with their intended purposes.</p>



Product	PLUS wall track, 47.2"
Item number	R9812112

Mounting in different wall types

	Concrete >B25 V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Brick >= Mz12 V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Aerated Concrete P4 V8670 - Mounting kit, 3 bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8620 - Mounting kit, 6 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



PRESSALIT®

Product	PLUS wall track, 47.2"
Item number	R9812112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®