R1100, R1101
R1102, R1109
R1110, R1111
R1119, R1161

| | |
|--|-----------|
| VALUE support arm – Operation and maintenance manual | en |
| VALUE toiletstøtte – Brugs- og vedligeholdelsesvejledning | dk |
| VALUE Stützklappgriff – Gebrauchs- und Pflegeanleitung | de |
| VALUE barre de relèvement – Manuel d'utilisation et d'entretien | fr |
| VALUE toiletsteun – Gebruiks- en onderhoudshandleiding | nl |
| VALUE toalettarmstöd – Bruks- och underhållsanvisning | sv |
| VALUE toalettstøtte – Bruks- og vedlikeholdsveiledning | no |
| VALUE apoyabrazos - Instrucciones de uso y mantenimiento | es |
| VALUE maniglione - Manuale d'uso e manutenzione | it |
| VALUE podpora – Instrukcja obsługi i konserwacji | pl |
| Opěrné rameno VALUE – Návod k obsluze a údržbě | cs |
| VALUE 臂撑 - 操作和维护手册 | 中文 |

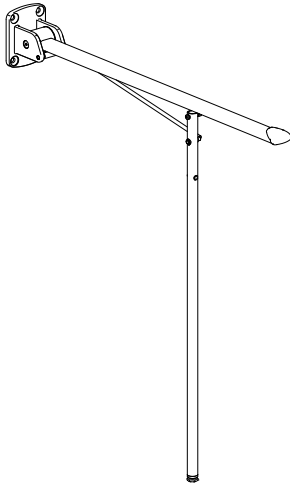
| | |
|------------|-----|
| English | 10 |
| Dansk | 21 |
| Deutsch | 32 |
| Français | 43 |
| Nederlands | 54 |
| Svenska | 65 |
| Norsk | 76 |
| Español | 87 |
| Italiano | 98 |
| Polski | 109 |
| Čeština | 120 |
| 中文 | 131 |

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


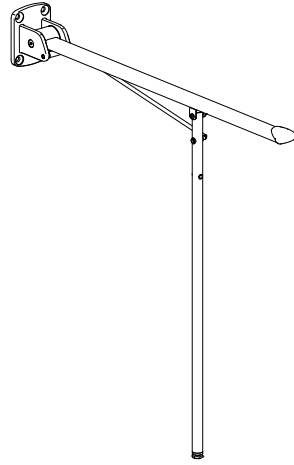
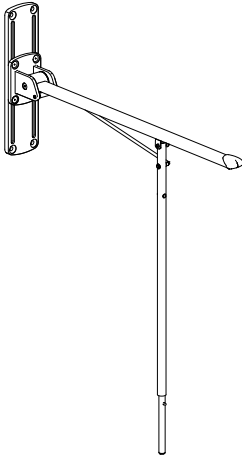
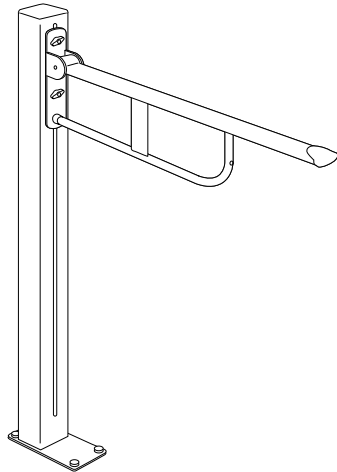
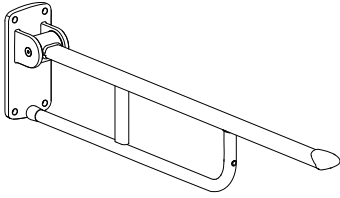
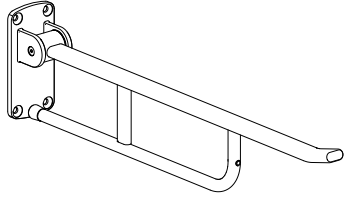
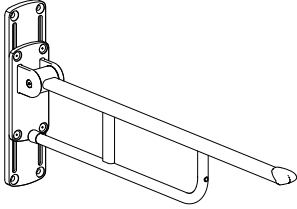
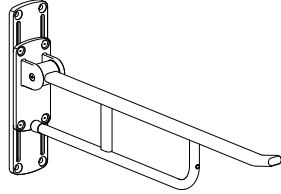
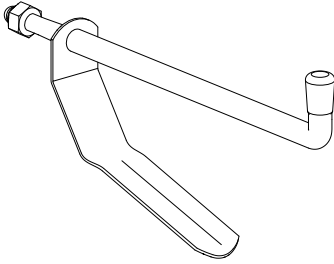
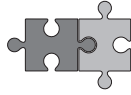
R1151

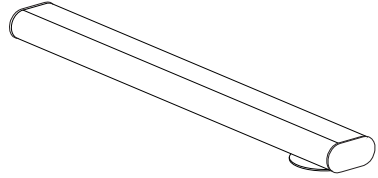
R1152

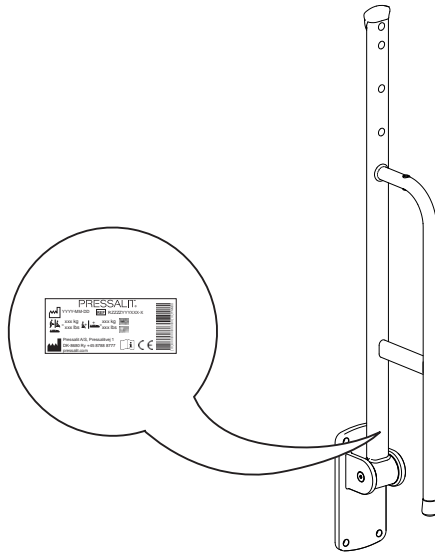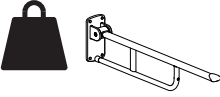
| |  |  |
|-------|---|---|
| R1100 | 2,62 kg (5,77 lbs) | 120 kg (264lbs) |
| R1101 | 2,97 kg (6,54 lbs) | 135 kg (297 lbs) |
| R1102 | 2,88 kg (6,34 lbs) | 135 kg (297 lbs) |
| R1109 | 2,62 kg (5,77 lbs) | 120 kg (264lbs) |
| R1110 | 3,43 kg (7,56lbs) | 120 kg (264lbs) |
| R1111 | 3,88 kg (8,55lbs) | 135 kg (297 lbs) |
| R1119 | 3,43 kg (7,56lbs) | 120 kg (264lbs) |
| R1161 | 10,12 kg (22,31) | 120 kg (264lbs) |

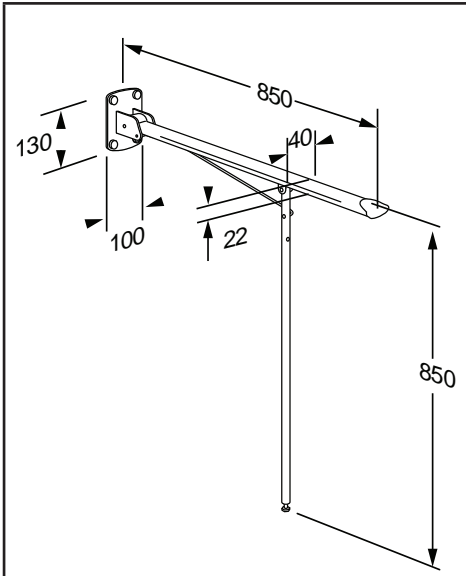
mm


R1101

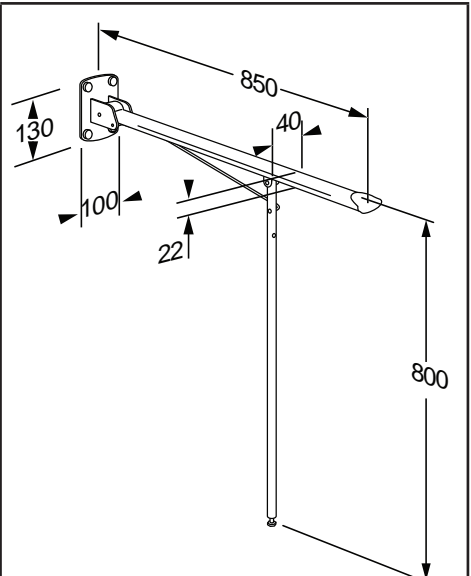R1102

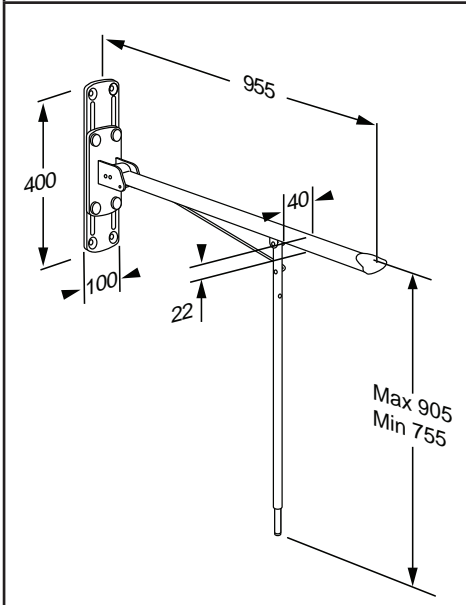
R1111

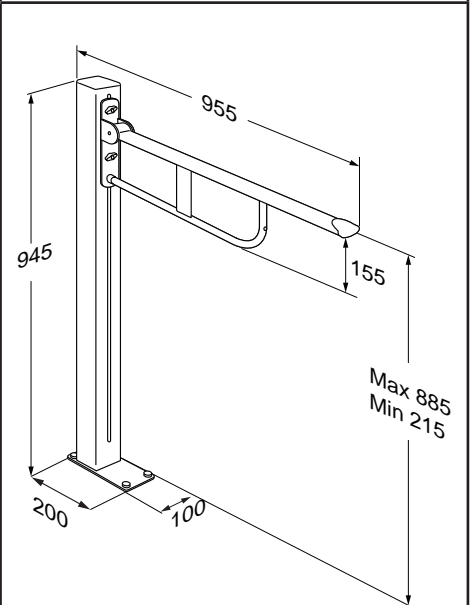
R1161

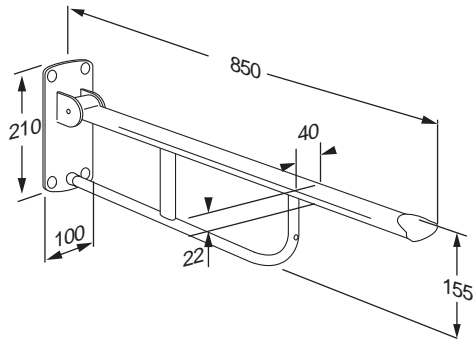
R1100


R1109


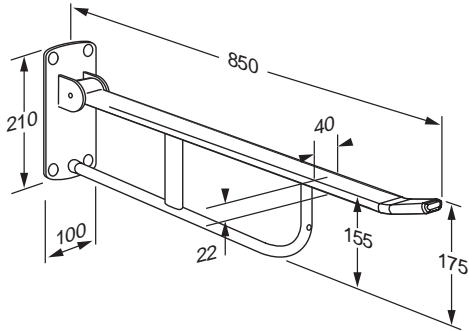
R1110

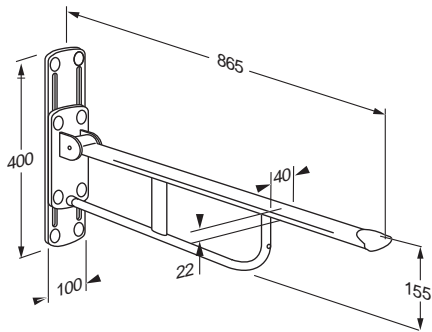
R1119

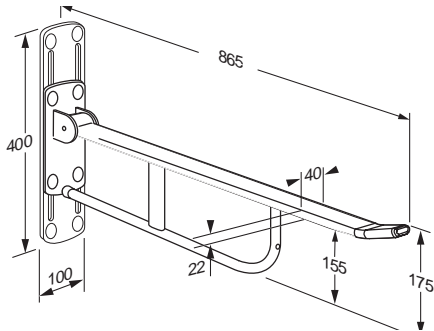
inch

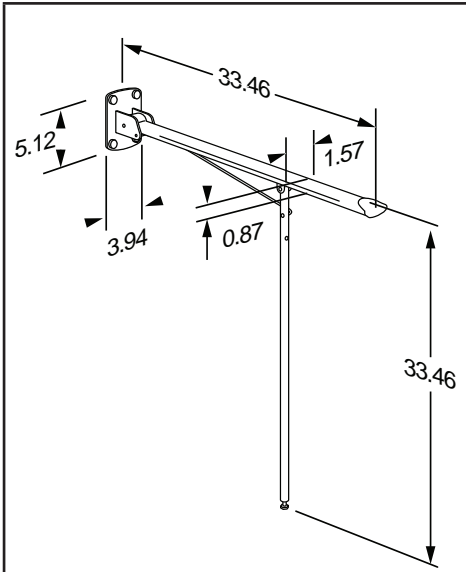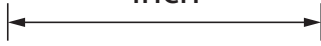
R1101

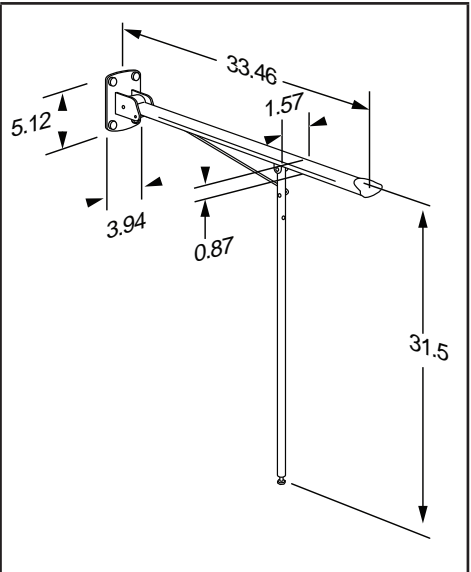R1102

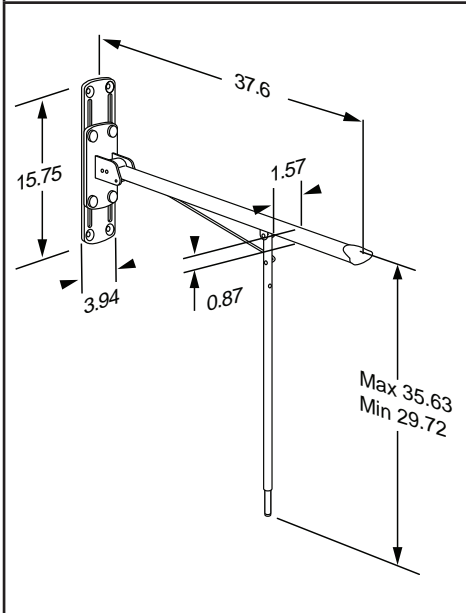
R1111

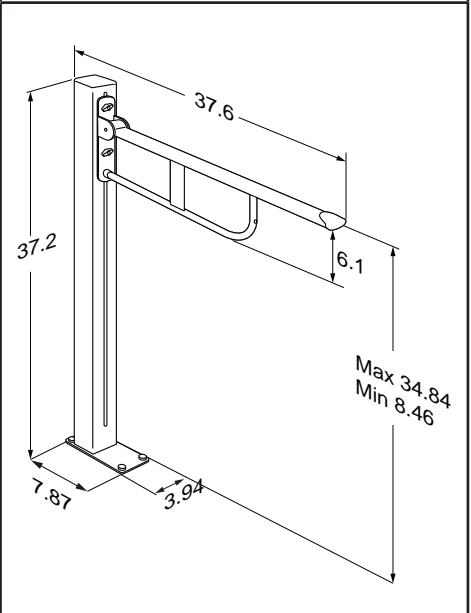
R1161

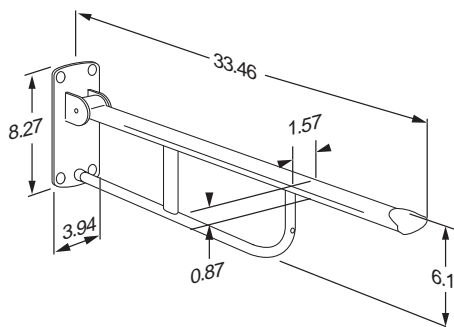
R1100


R1109

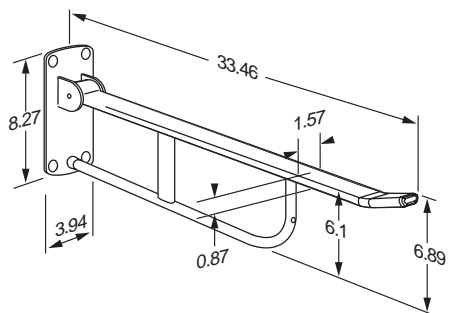R1110

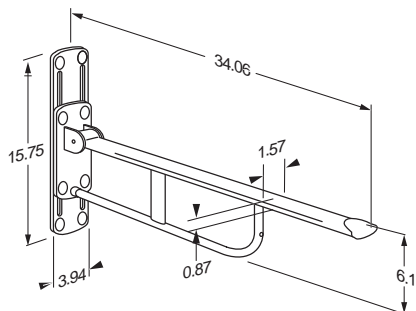R1119

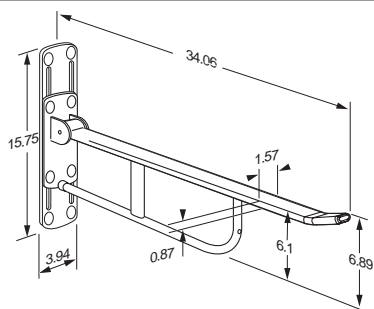Content

| | |
|--------------------------------------|----|
| Mounting instruction | 11 |
| Operational signage | 11 |
| Symbols used in this manual | 12 |
| General safety | 13 |
| Intended use | 14 |
| Intended user profile | 14 |
| Operating environment | 14 |
| Biocompatibility | 14 |
| Operating instructions | 16 |
| All support arms | 16 |
| Adjustment | 16 |
| Height adjustable support arms | 17 |
| Support arms with legs | 18 |
| Cleaning | 19 |
| Disinfection | 19 |
| Maintenance | 19 |
| Product label | 20 |
| Trouble-shooting | 21 |
| Disposal and recycling | 21 |

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage can be downloaded from pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Article number/
versions

Compatible products

Product weight

Max. user weight

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

The support arm is used as support for a person when he or she sits down or gets up from a toilet.

Intended user profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room, in private homes, nursing homes and hospitals. For institutional use, the expected life time is 10 years.

Life cycle test performed according to ISO 17966:2016.

NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 5 years to 1 year.

Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.

Biocompatibility

The materials in contact with the skin of the user during the intended use have been evaluated for their content of harmful substances according to the procedures in BS EN ISO 10993-1:2020 Biological evaluation of medical devices – Part 1 and are therefore considered suitable for this purpose.

CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

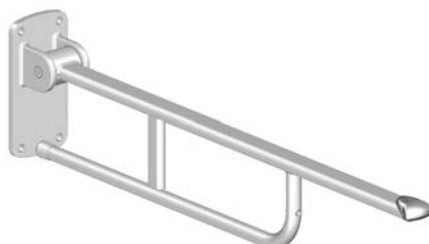
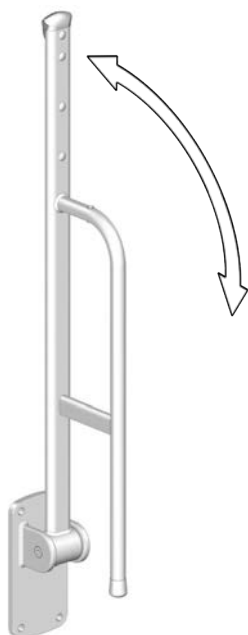
NOTICE

Any serious incident that has occurred in relation to the product should be reported to Pressalit and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

Operating instructions

All support arms

Fold the support arm fully down. Fold it back up when not in use. You can hear a click when the support arm is correctly folded up.

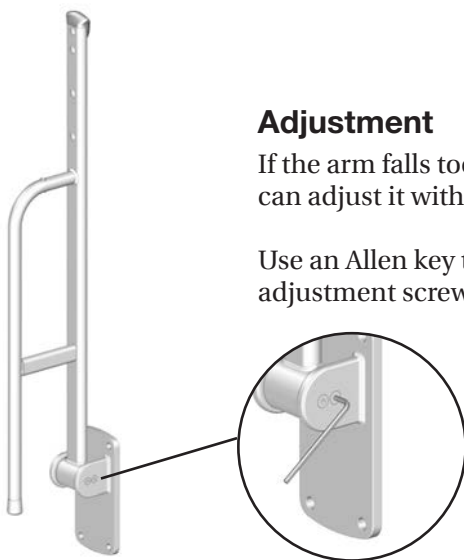
Take care to avoid injury to hands when in the vicinity of closing moveable parts.


Adjustment

If the arm falls too easily or is difficult to fold up, you can adjust it with the two screws.



Use an Allen key to either tighten or loosen the two adjustment screws.


Height adjustable support arms

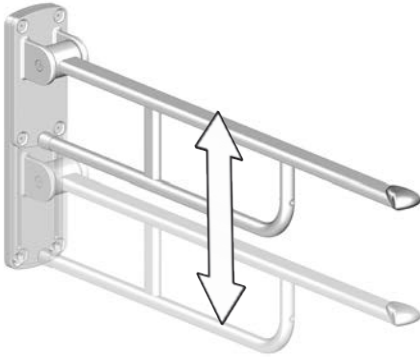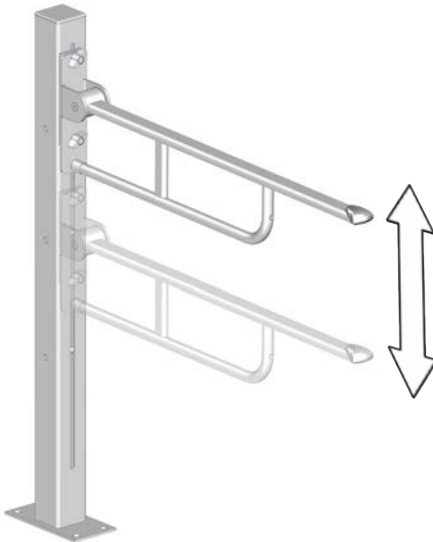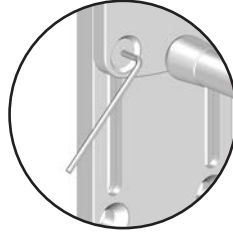Support arms R1111, R1110 and R1119: Use an Allen key to loosen the four screws, adjust the height of the support arm and tighten the four screws again.

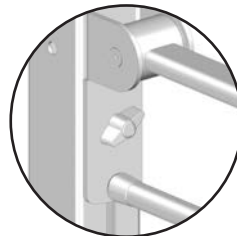Support arm R1161: Loosen the two thumb screws, adjust the height of the support arm, and then tighten the two thumb screws again.


en


Support arms with legs

The support arms R1101, R1102, and R1111 are equipped with legs for added support.

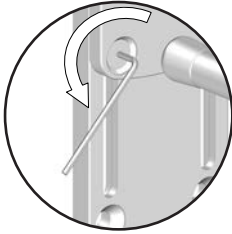
Support arm R1111: The support arm R1111 can be adjusted in height and is therefore also equipped with a height-adjustable leg.

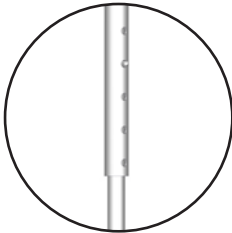
Use an Allen key to loosen the four screws on the baseplate.

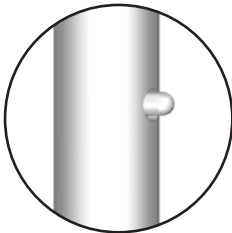Push the leg's release pin in and adjust the leg accordingly, either up or down to the desired height.

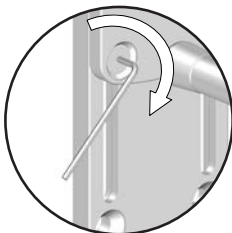Ensure that the release pin pops out of its hole again.

Adjust the support arms baseplate so the arm sits level.

Finally, tighten the four screws on the baseplate again.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. Daily cleaning is recommended.

To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

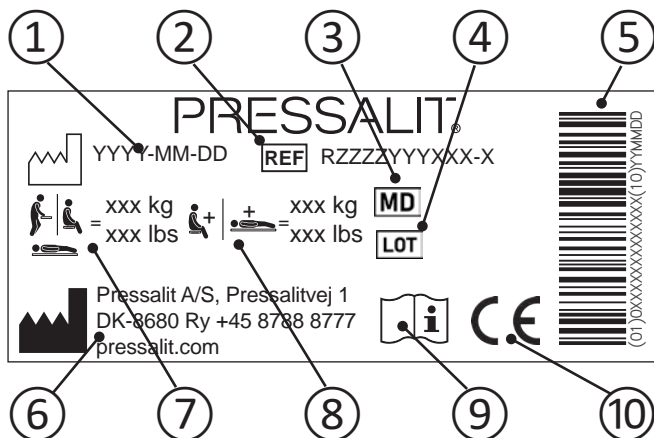
Disinfection

When using support the arms for multiple patients, it is necessary to clean and disinfect them between uses to prevent the risk of cross infection.

Maintenance

Regularly inspect the product monthly for loose screws and signs of rust on the screws. Replace any mounting screws that show rust. If other parts besides the mounting screws exhibit rust, please contact Pressalit for a replacement.

Product label

- ① Production date
- ② Article number
- ③ Medical device
- ④ Batch code
- ⑤ GS1-128 barcode
- ⑥ Name and address of manufacturer
- ⑦ Maximum user weight
- ⑧ Max load
- ⑨ Consult manual for use

⑩

These products are CE-marked in accordance with relevant requirements in regulation (EU) 2017/745 (5 April 2017) on medical devices.

A declaration of conformity can be found at pressalit.com.

Trouble-shooting

| Symptom | Cause | Action |
|--|--------------------------------|---|
| The support arm cannot be folded up or down. | Something is caught or jammed. | Check if anything is caught or jammed, and remove it. |

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

en

| | |
|------------------|--|
| Steel | Steel can be recycled by re-melting and incorporating it as a secondary material in the production of new steel. |
| Polyamid (nylon) | Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production. |

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