


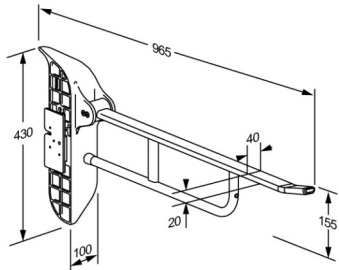

<p><b>Product</b> <b>Item number</b></p>	<p>VALUE II support arm, 850 mm R1181000</p>
<p><b>Colour</b>  <b>Product description</b></p>	<p>VVS:615892100 NRF:6191059 EAN:5708590373010 HMI: 138203</p> <p>White</p>  <p>VALUE and VALUE II support arms with the ALBA wing mounting profile are specifically designed for use with SELECT TL3 and TL4 toilet lifters. The height adjustment of the toilet lifter is done only with the control panel on the support arm.</p> <p>VALUE II support arm, 850 mm, foldable, height adjustable 150 mm without the use of tools (right hand operated). Provides firm support when sitting, standing, or transferring to and from a toilet. Materials: Rilsan-coated steel and diecast aluminium. For fitting to TL3/TL4 electrical/manual toilet lifter, ALBA wing mounting profile included. For mounting on user's left side.</p>
<p><b>Adjustability</b> <b>Adjustment height</b> <b>Product capacity</b></p>	<p>Height adjustable 150 mm Maximum load: 120 kg</p>
<p><b>Materials</b>  <b>Ergonomy</b></p>	<p>The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The mounting profile is made from diecast aluminium.</p> <p>VALUE II support arm with ALBA wing, 850 mm, white. Height adjustable, right hand operated, for fitting to TL3/TL4 (left)</p>
<p><b>Warranty</b></p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



<b>Product</b>	VALUE II support arm, 850 mm
<b>Item number</b>	R1181000
<b>Cleaning</b>	<p>VALUE support arms The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean)</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
<b>Declaration of conformity</b>	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.



# PRESSALIT®

<b>Product</b>	VALUE II support arm, 850 mm
<b>Item number</b>	R1181000

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

PRESSALIT®