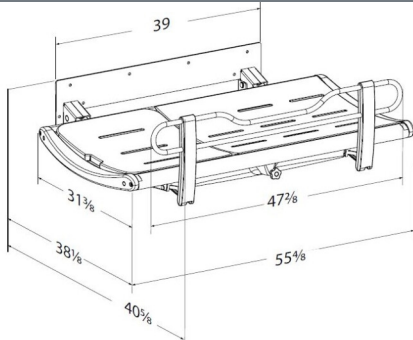




PRESSALIT®

Product Item number	SCT 3000 shower change table, fixed height R8435313301
Color	VVS: 777543143 EAN: 5708590374024 HMI: 130168 FDA: INT/D124320 Graphite gray (RAL 7024)
Product description	 Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user. The SCT 3000 series has a stylish and award-winning design focused on comfort and ease of use. SCT 3000 shower change table, fixed height. Foldable. Incl. safety rail, flexible hose (Ø1.26") and water collection tray with integrated outlet. Surface length 51.2", outer measurements 40.6" x 55.5". Excl. mounting kit.
Adjustability Product capacity	Fixed mounting Maximum user weight (MUM): 441 lbs. Maximum load (SWL): 463 lbs. When the table is loaded with 441 lbs, each mounting bolt supports 132 lbs. <i>This product has been tested statically with a load of 694 lbs, as the ISO 17966 standard requires testing with a safety factor 1.5.</i>
Materials	Powder-lacquered stainless steel, aluminium and plastics (PVC, Polystyrene). Pads in polyurethane foam, are tested according to the DIN EN 1021-1 standard (flame resistance).



Product	SCT 3000 shower change table, fixed height
Item number	R8435313301
Ergonomy	• The ergonomically designed safety rails have a practical lower central section, which provides better access and work environment for the carer while transferring, showering, drying, changing and dressing the user. • One hand operated safety rails allows the carer to maintain eye contact as well as physical contact with the user during the whole process. • Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.
Features	• A water collection tray with integrated water outlet prevents water from splashing on caregiver or the floor. The flexible hose (Ø1.26") attached to the center mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. • Adjustable neck support which can be moved to either end of the 51.2" changing tables. • The slightly curved shape makes the table comfortable and safe to lie on •The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.



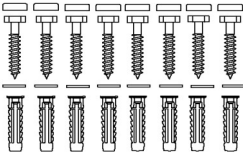
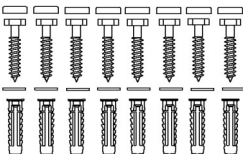
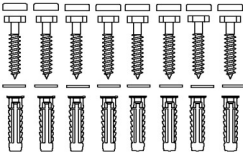
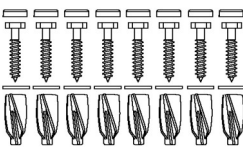
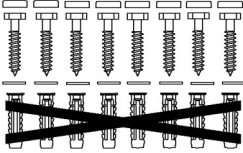
PRESSALIT®

Product	SCT 3000 shower change table, fixed height
Item number	R8435313301
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes OxiGenesis from ENVIROX NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.



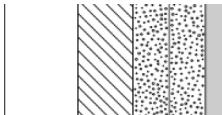
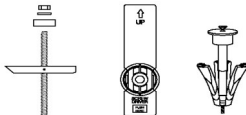
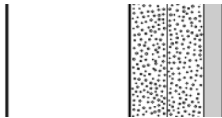
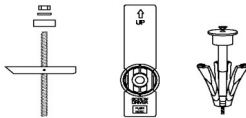
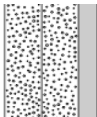
Product	SCT 3000 shower change table, fixed height
Item number	R8435313301

Mounting in different wall types

	Concrete >B25 V8613 - Mounting kit, 8 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	Solid Brick >= Mz12 V8613 - Mounting kit, 8 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8613 - Mounting kit, 8 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
	Aerated Concrete P4 V8663 - Mounting kit, 8 lag bolts Ø10 x 70, plugs GB14, with grey covers. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8613 - Mounting kit, 8 lag bolts Ø10 x 70, plugs SX12, with grey covers. NB! Plugs are not needed. To be ordered separately.	



PRESSALIT®

Product Item number	SCT 3000 shower change table, fixed height R8435313301
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®