




PRESSALIT®

Product Item number	PLUS wall track, 3300 mm R9833112	
Colour	VVS:615841133 NRF:7501066 EAN:5708590305912 HMI: 71749 Anthracite grey	
Product description	<p>PLUS wall tracks are used for mounting of support arms, shower seats and wash basin brackets, and allows sideways adjustment of these products. See below a selection of products for fitting to the wall track.</p> <p>PLUS wall track, 3300 mm, for horizontal mounting. The cover plate hides bolts and wires and facilitates cleaning. Materials: Aluminium and polystyrene. See below a selection of products for fitting to the wall track. Excl. mounting kit.</p>	
Adjustability Product capacity	Fixed mounting Maximum load: Up to 1000 kg depending on the wall type.	
Materials Ergonomy Features	<p>The wall track is primarily made of 10-micrometre anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.</p> <p>The track distributes the weight of the individual bathroom product, for example a wash basin bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions.</p> <p>The horizontal track makes it possible to shift the product along the wall to constantly optimize the available space in the bathroom. The front cover plate conceals mounting bolts and any cables.</p>	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	

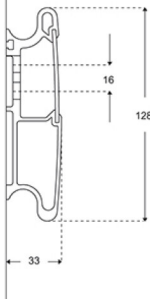

PRESSALIT®

Product	PLUS wall track, 3300 mm
Item number	R9833112
Cleaning	<p>PLUS wall tracks</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS:</p> <p>Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p>NON-DISINFECTANTS:</p> <p>Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS:</p> <p>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
Declaration of conformity	<p>MDR-ACC This product fulfils all relevant requirements in Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an accessory for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical devices to be used in accordance with their intended purposes.</p>



PRESSALIT®

Product	PLUS wall track, 3300 mm
Item number	R9833112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®