




PRESSALIT®

Product	VALUE II support arm, 850 mm, with control panel
Item number	R1183000
Colour	<p>VVS:615892300 NRF:6191061 EAN:5708590373034 HMI: 138210 FDA: ILS/D135866 White</p>
Product description	 <p>VALUE and VALUE II support arms with the ALBA wing mounting profile are specifically designed for use with SELECT TL3 and TL4 toilet lifters. The height adjustment of the toilet lifter is done only with the control panel on the support arm.</p> <p>VALUE II support arm, incl. control panel for TL3/TL4 toilet lifter, 850 mm, foldable, height adjustable 150 mm without the use of tools (left hand operated). Provides firm support when sitting, standing, or transferring to and from a toilet. Materials: Rilsan-coated steel and diecast aluminium. For fitting to TL3/TL4 electrical toilet lifter, ALBA wing mounting profile included. For mounting on user's right side.</p>
Adjustability	Height adjustable
Adjustment height	150 mm
Product capacity	Maximum load: 120 kg
Materials	The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The mounting profile is made from diecast aluminium.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

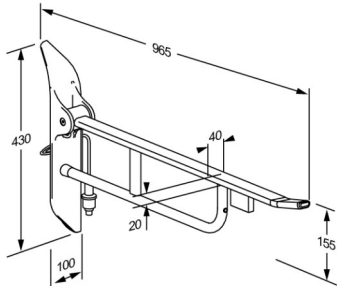

Product	VALUE II support arm, 850 mm, with control panel
Item number	R1183000
Cleaning	<p>VALUE support arms</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS:</p> <p>Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean)</p> <p>NON-DISINFECTANTS:</p> <p>Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
WEEE	<p>Please enquire of your local Pressalit subsidiary or distributor (in countries where there is no subsidiary) that will arrange for recycling of electronic waste in accordance with the WEEE directive.</p>



PRESSALIT®

Product	VALUE II support arm, 850 mm, with control panel
Item number	R1183000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®