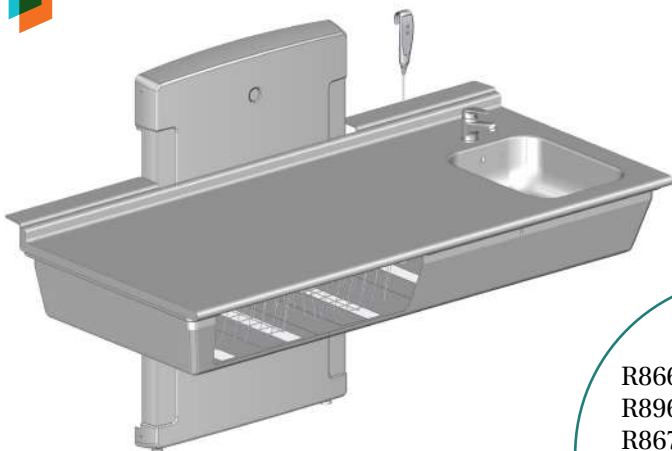
R8661 - R8961 - R8662
 R8962 - R8663 - R8963
 R8670 - R8671 - R8971
 R8672 - R8972 - R8673
 R8682 - R8683 - R8684
 R8685

Day care changing table – Operation and maintenance manual	en
Puslebord – Brugs- og vedligeholdelsesvejledning	dk
Wickeltische – Gebrauchs- und Pflegeanleitung	de
Tables à langer – Manuel d'utilisation et d'entretien	fr
Aankleedtafels – Gebruiks- en onderhoudshandleiding	nl
Skötbord – Bruks- och underhållsanvisning	sv
Stellebord – Bruks- og vedlikeholdsveiledning	no
Hoitopöytä päivähoitoon – käyttö- ja huolto-opas	fi
Cambiador - Instrucciones de uso y mantenimiento	es
Przewijak – Instrukcja obsługi i konserwacji	pl
换洗床 - 操作和维护手册	中文

English	12
Dansk	26
Deutsch	40
Français	54
Nederlands	68
Svenska	82
Norsk	96
Suomi	110
Español	124
Polski	138
中文	152


Pressalit's Environmental and Quality Management Systems are certified by LRQA

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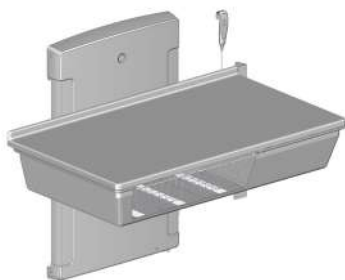
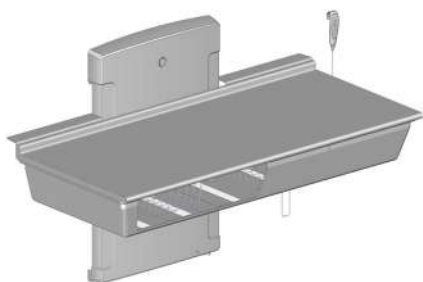
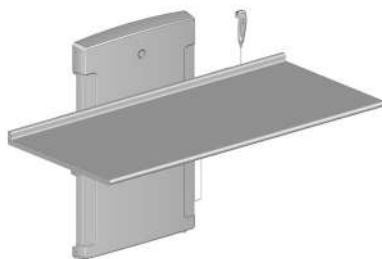
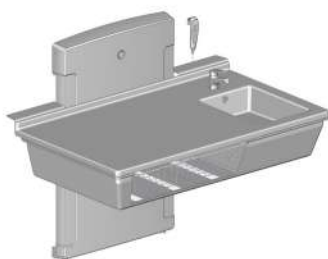
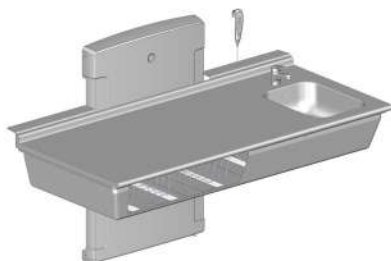
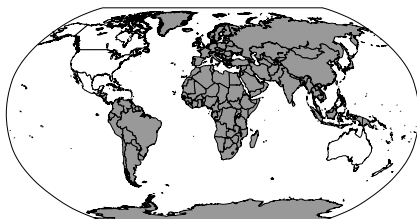
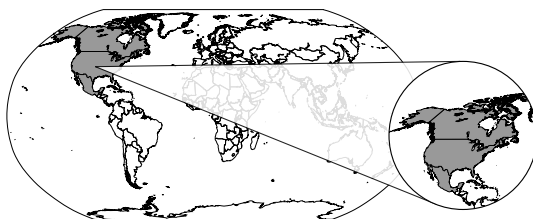
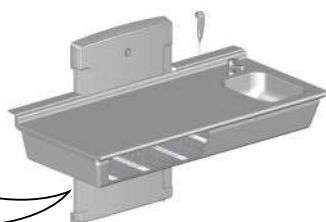
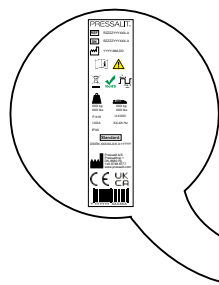
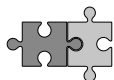
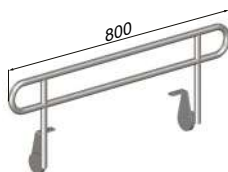
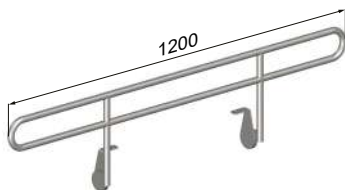
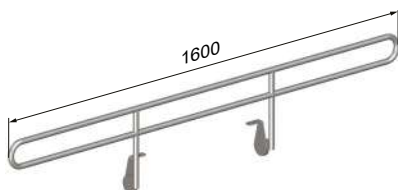
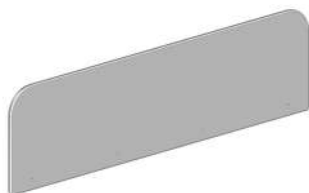
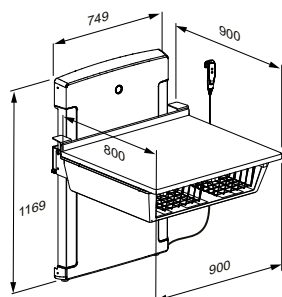
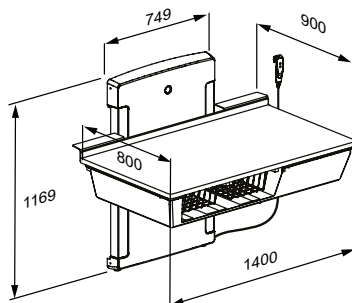
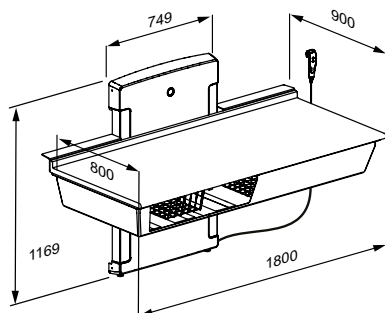
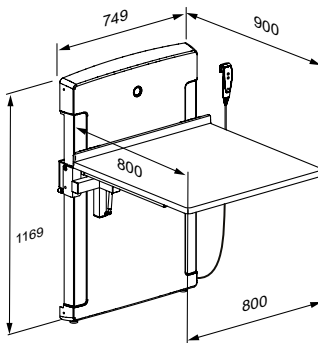
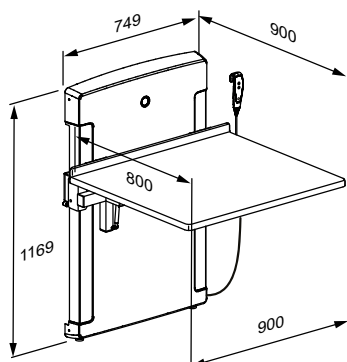
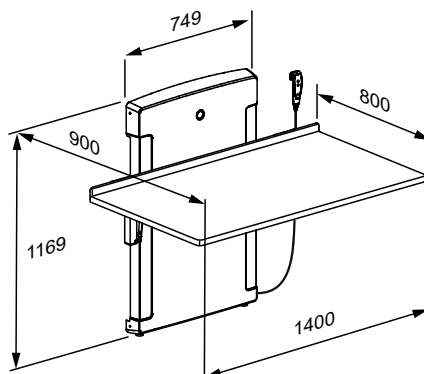
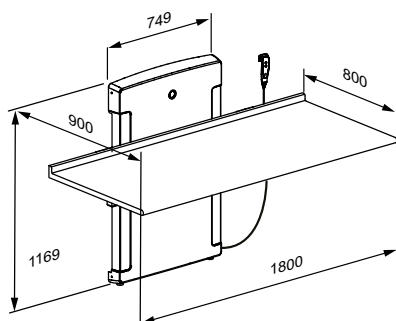
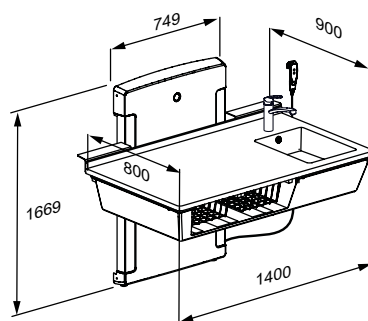
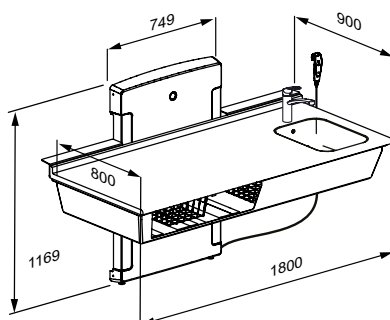
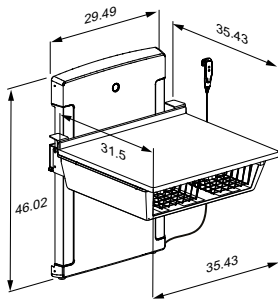
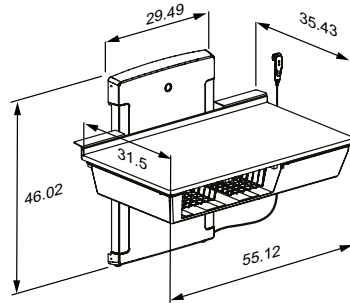
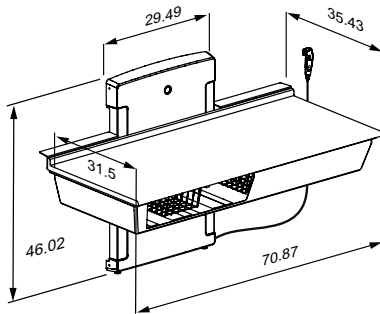
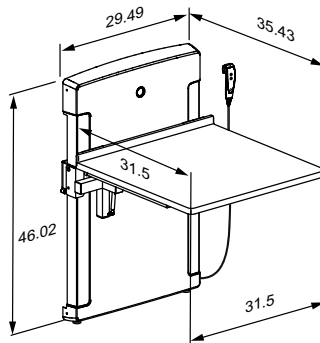
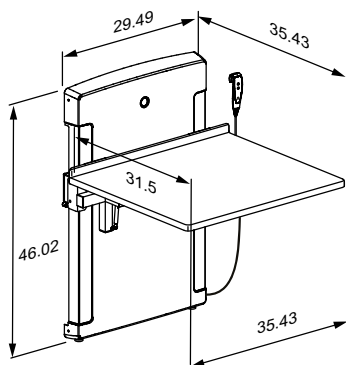
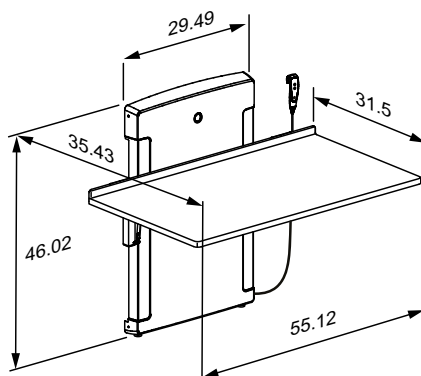
R8673

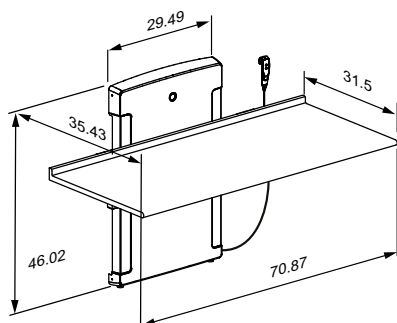Content

Mounting instruction	12
Operational signage	12
Symbols used in this manual	13
General safety	14
Safety distances	15
Intended use	16
Intended operator profile	16
Operating environment	16
Operating instructions	17
The models with safety tray	18
Resetting	18
Cleaning	19
Maintenance	20
Maintenance inspection	20
Product label	22
Trouble-shooting	24
Storage and transportation	25
Disposal and recycling	25

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage can be downloaded from pressalit.com.

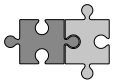
IMPORTANT

RETAIN THIS MANUAL FOR FUTURE REFERENCE. READ CAREFULLY.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING		WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.	
CAUTION		CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.	
IMPORTANT		IMPORTANT indicates important information about handling and use of the product.	
NOTICE		NOTICE is used to address practices not related to personal safety.	
 Head injury warning	 Hand injury warning	 Feet injury warning	 Electricity / voltage warning
 Strong heat warning	 Do not leave child unattended	 Compatible products	 Spare parts
 Power specifications	 Product weight	 Max. user weight	 Article number / versions

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

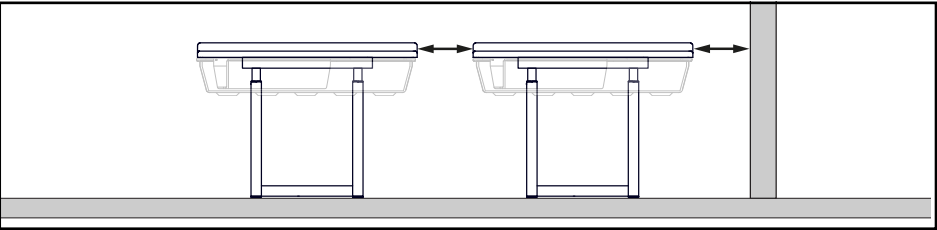
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.


Safety distances

Make sure that the changing table is at safe distance from surrounding objects. Check the distances before any use.

	EN 12221-1:2008	Pressalit recommends
	Max 65 mm (2.56") Min 223 mm (8.78")	Max 60 mm (2.36"") Min 300 mm (11.81")
	Max 7 mm (0.28") Min 12 mm (0.47")	Max 4 mm (0.16") Min 25 mm (0.98")
	Max 25 mm (0.98") Min 45 mm (1.77")	Max 8 mm (0.31") Min 120 mm (4.72"")

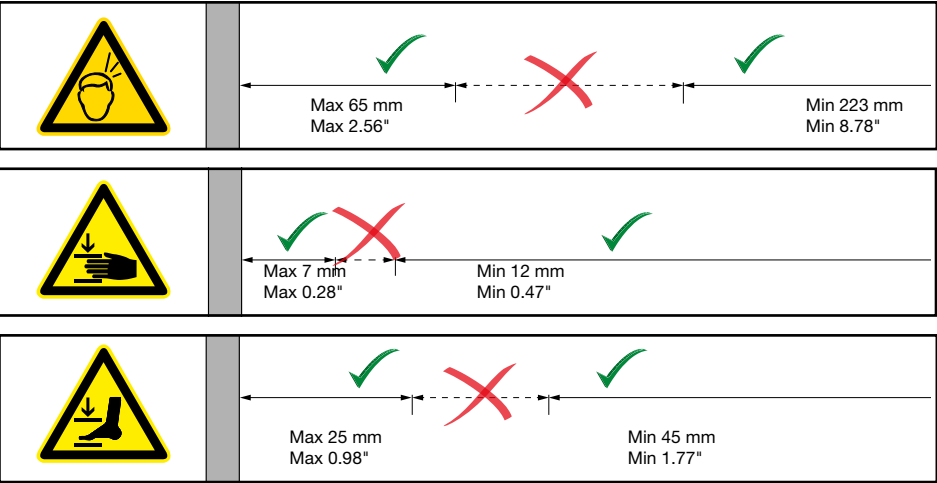en

Intended use

This product is intended for optimum ergonomics when caring for children.

The child's weight must not exceed 75 kg / 165 lbs.

Intended operator profile

The product shall only be operated by adults.

Operating environment

The product is intended to be used indoor at room temperature under normal working conditions. For institutional use, the expected life time is 8 years.

Life cycle test performed according to ISO 17966:2016.

This product is tested according to DS/EN 12221-1 and 12221-2.

en

Do not leave the child unattended.

Be aware of the risk of open fires and other sources of strong heat, such as electric bar fires, gas fires, etc. in the near vicinity of the changing table.

NOTICE

Always place the child on a changing table mattress and not directly on the table.

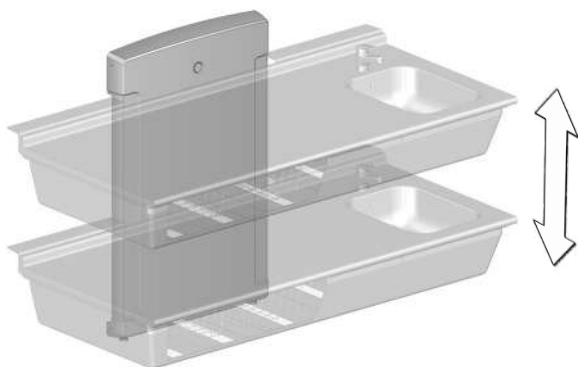
CAUTION

Never allow children to operate this product without supervision.

Operating instructions

The changing table can be adjusted from 300 to 1000 mm / 12 to 39 inches up and down. **Max load is 75 kg / 165 lbs.**

The height adjustment is controlled by the wired remote control unit.

Press and hold the up button to raise the unit.

When the button is released, the motion stops.

Press and hold the down button to lower the unit. When the button is released, the motion stops.

To avoid the risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.

Take care to avoid injury to hands, head and feet when in the vicinity of moving mechanical parts.

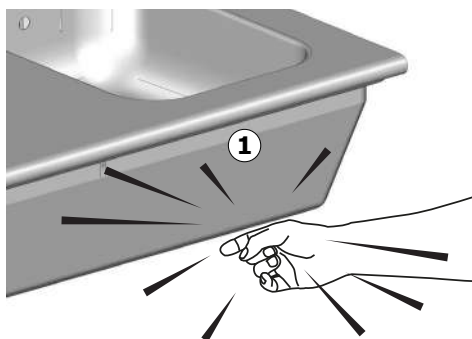
Never place objects underneath the changing table.

The models with safety tray

All lowstart changing tables, except foldable models, are equipped with a safety tray.

If the safety tray ① comes into contact with an object when the table is being lowered, the motor stops immediately and the table will rise 3-4 cm / 1 inch to reduce the risk of pinching/compression injuries under the table. The table might need resetting before it can be used again. See „Resetting“.

Resetting

①

Make sure that the safety tray is not activated. Lower the changing table all the way down to its starting position using the down button.

②

Release the down button, and after 3-4 sec. press it in again until you hear a slight click after approx. 10 sec.

③

Release the down button and then raise the changing table to its highest position. You will hear another click.

④

Lower the changing table a couple of centimetres/approximately one inch.

⑤

Then raise it again to its highest position. You will hear another click. The changing table is now reset.

The changing table should also be reset if there has been an interruption of the power supply.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

Maintenance

Expected service life of the changing table is 8 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

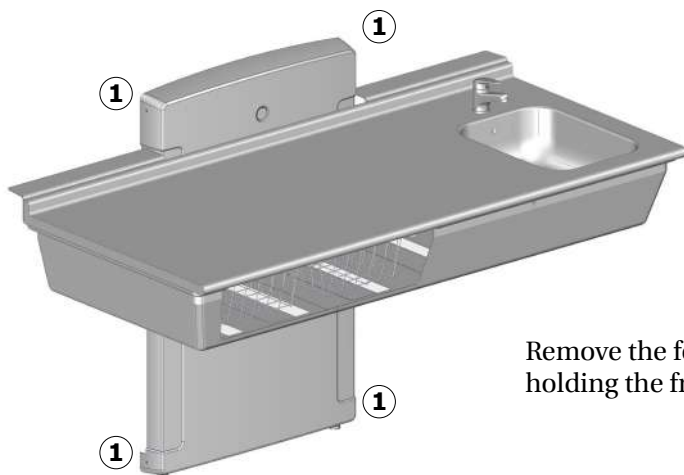
It is recommended that service inspection is done once a year.

Maintenance inspection

Take the changing table to its highest position.


Disconnect mains before removing the covers.

Remove the four screws ① holding the front cover.

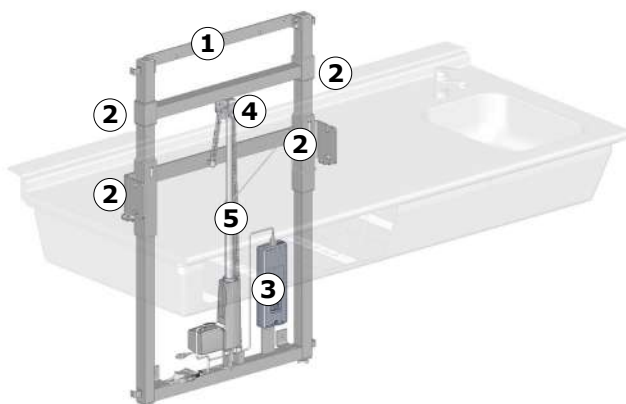
Lift off the top cover.

en

- Check the wall brackets ① and tighten the screws if necessary.
- Oil moving parts, especially the sliding brackets ②.
- Check all electrical connections to the control box ③.
- Check the chain wheel ④ for wear.
- Oil the chain ⑤.
- Put the unit cover back again and tighten the retaining screws. Reconnect mains.

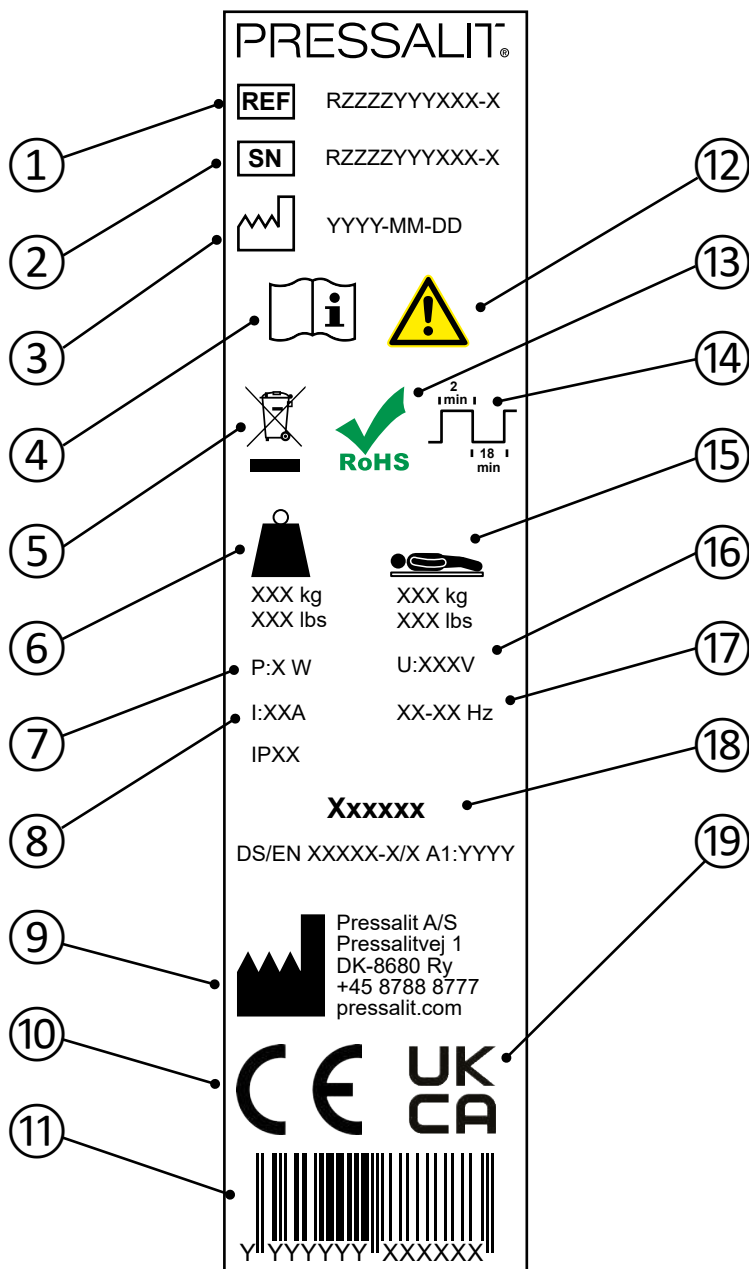Product label

①	Article number
②	Serial number
③	Production date
④	Consult manual for use
⑤	Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
⑥	Product weight
⑦	Maximum power consumption
⑧	Electric current
⑨	Name and address of manufacturer
⑩	These products are CE-marked in accordance with relevant requirements in: • Council Directive 2006/42/EC (of 17 May 2006) on machinery. • Council Directive 2011/65/EC (of 8 June 2011) on RoHS.
⑪	EAN 13 barcode
⑫	Consult the manual for important cautionary information such as warnings and precautions.
⑬	This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
⑭	Duty cycle
⑮	Maximum user weight
⑯	Mains voltage
⑰	Frequency
⑱	Standard, which product is manufactured in accordance with (if any). These products are UKCA-marked in accordance with relevant requirements in: • Supply of Machinery (Safety) Regulations 2008. • The restriction of the use of certain hazardous substances in electrical and electronic equipment regulations 2012. A declaration of conformity can be found at pressalit.com.

Trouble-shooting

Symptom	Cause	Action
The product will not work, even though the control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".
The baskets are unable to close/open.	The basket may have been twisted unintentionally.	Try to straighten the basket.
The product is jammed.	Interruption of the power supply.	Reset the product. See "Resetting".

Contact Pressalit A/S or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Storage and transportation

- The temperature for transportation and storage of the unit must be between 5 and 40° Celsius (41 and 104° F).
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

en

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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