




Product Item number	SCT 2000 shower change table, coated mesh cover, electrically height adjustable with wall-mounted safety stop and backup battery R8591218000D
Color Product description	EAN:5708590400860 FDA: INQ/D124319 White  <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>IAPMO-certified. SCT 2000 shower change table with wall-mounted safety stop and backup battery, powered, height adjustable 27.6"; from 11.8" - 39.4", with wired hand control Foldable. Incl. safety rail and water collection tray with integrated outlet. Outer measurements 35.8" x 70.9". Excl. mounting kit.</p>
Adjustability Adjustment height Product capacity Speed up Noise level	Height adjustable 27,58 inch Maximum user weight (MUM): 441 lbs. Maximum load (SWL): 441 lbs. When the table is loaded with 441 lbs, each mounting bolt supports 57 lbs. <i>This product has been tested statically with a load of 1102 lbs as the IEC 60601-1 standard requires testing with a safety factor 2.5.</i> Speed up: 0,75 inch/s Speed down: 0,83 inch/s 50 dB

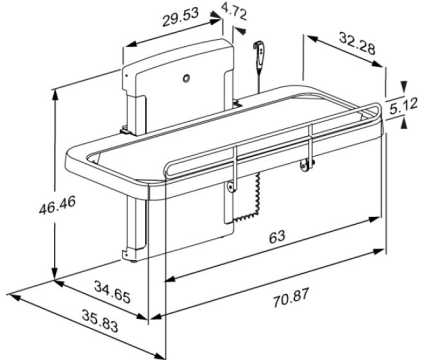

<p>Product</p> <p>Item number</p>	<p>SCT 2000 shower change table, coated mesh cover, electrically height adjustable with wall-mounted safety stop and backup battery</p> <p>R8591218000D</p>
<p>Materials</p> <p>Ergonomy</p> <p>Features</p> <p>Mains connection (V)</p> <p>Actuator (motor)</p> <p>Control box</p> <p>Wired remote control</p> <p>Power consumption (maximum load)</p>	<p>Construction in powder-laquered stainless steel, mesh cover in PVC-coated polyester (for extra durability), water collection tray in PVC-U and carriage housing in polystyrene.</p> <p>The 2000 series changing table has a low start position to make transfers to the table easier. The washable, perforated mesh cover lets the water pass through to the water collection tray underneath.</p> <p>The flexibility of the changing table provides for optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <ul style="list-style-type: none"> • Wall-mounted safety stop The wall mounted safety stop is an optional breaker system which shuts down the operation of the changing table when activated. When the turn knob is deactivated, you may operate the table as normal. The safety stop can be mounted externally and does not require changes in the wall structure. • Backup battery The backup battery is an optional factory mounted feature which allows for continued operation in the event of a power failure. The backup battery allows for app. 15 cycles (up/down) depending on the load. The battery automatically charges when the changing table is connected to power. • A water collection tray with integrated water outlet prevents water from splashing on caregiver or the floor. The flexible hose attached to the center mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. • The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space in front of the unit. • Electric with wired hand control – the red buttons clearly indicate “Up” and “Down”. The wired hand control comes with a holder for fitting to the wall. • The stretcher of the table is very easy to clean, and the stretcher cover can be unzipped for washing or easy replacement. <p>120</p> <p>LA40 Careline (Linak), 4000 N, IPX6</p> <p>CA63 Careline, (Linak), IPX6</p> <p>HB30 (Linak), IP66</p> <p>Power consumption by maximum load: 420 W Power consumption by standby: 2,5 W</p>




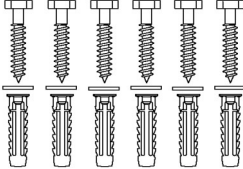
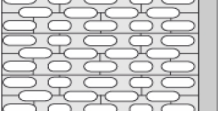
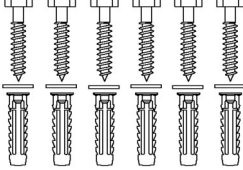
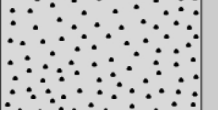
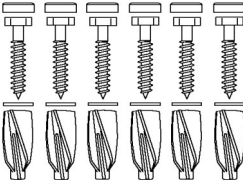
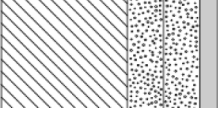
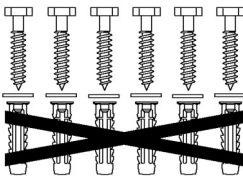


Product	SCT 2000 shower change table, coated mesh cover, electrically height adjustable with wall-mounted safety stop and backup battery
Item number	R8591218000D
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes OxiGenesis from ENVIROX</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
WEEE	Please enquire of your local Pressalit subsidiary or distributor (in countries where there is no subsidiary) that will arrange for recycling of electronic waste in accordance with the WEEE directive.









Product	SCT 2000 shower change table, coated mesh cover, electrically height adjustable with wall-mounted safety stop and backup battery
Item number	R8591218000D

Mounting in different wall types

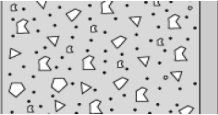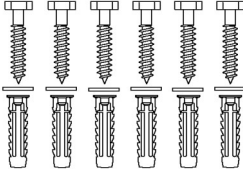	Concrete >B25 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.	
	Solid Brick >= Mz12 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.	
	Aerated Concrete P4 V8658 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs GB14. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. NB! Plugs are not needed. To be ordered separately.	

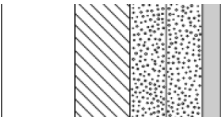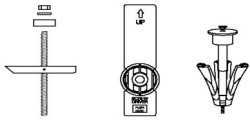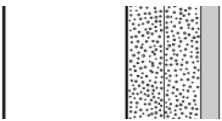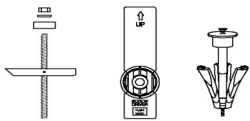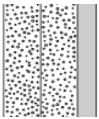

Product Item number	SCT 2000 shower change table, coated mesh cover, electrically height adjustable with wall-mounted safety stop and backup battery R8591218000D
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

It is the responsibility of the installer to ensure that suitable wall fasteners are used.