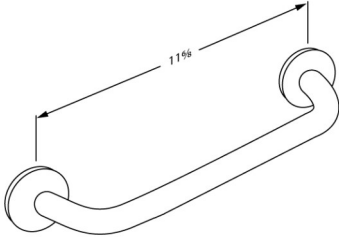




<b>Product</b> <b>Item number</b>	PLUS grab bar 11.8" RT100108
<b>Color</b>	VVS:777565108 NRF:6317162 EAN:5708590153940 HMI: 19081 NBS:N13:436 FDA:ILS/D135866 Dark blue (RAL 5011) 
<b>Product description</b>	<p>PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS grab bar, diameter 1.3", 11.8", for fixed mounting. Material: Powder-coated aluminium and plastic. Excl. mounting kit.</p>
<b>Product capacity</b>	Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. <i>Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i>
<b>Materials</b>	Basic material: Aluminium Surface material: Polyester lacquer Cover cap: Cycology plastic
<b>Ergonomy</b>	The 1.3" diameter enables the rail to be held comfortably and firmly. The projection of 2" from the wall makes it easy to take hold of the rail.
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

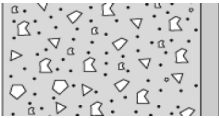
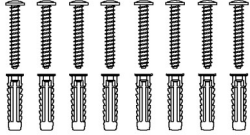

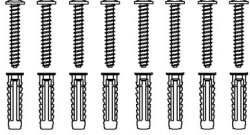
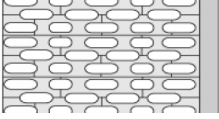
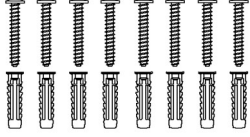
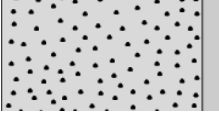
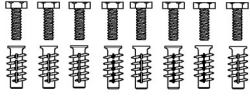
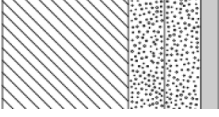
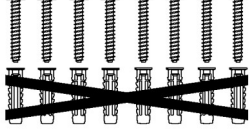


<b>Product</b>	PLUS grab bar 11.8"
<b>Item number</b>	RT100108
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>

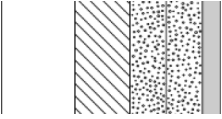
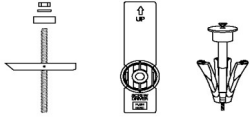

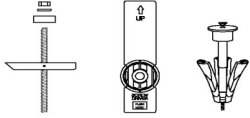
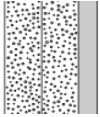


<b>Product</b>	PLUS grab bar 11.8"
<b>Item number</b>	RT100108

## Mounting in different wall types

	<b>Concrete &gt;B25</b> V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	<b>Solid Brick &gt;= Mz12</b> V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	<b>Solid Sand Lime Brick &gt;= Hlz12</b> V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	<b>Aerated Concrete P4</b> V8666 - Mounting kit, 8 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b> V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately.	



<b>Product</b> <b>Item number</b>	<b>PLUS grab bar 11.8"</b> <b>RT100108</b>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b></p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or Wingt anchors or similar.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b></p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or Wingt anchors or similar.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant</b></p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>	