R1620
R1622

VALUE IV

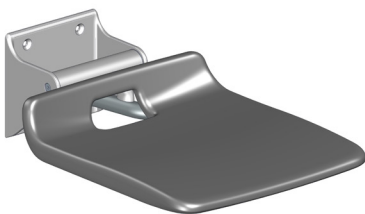
Shower seat – Operation and maintenance manual	en
Brusesæde – Brugs- og vedligeholdelsesvejledning	dk
Duschsitz – Gebrauchs- und Pflegeanleitung	de
Siège de douche – Manuel d'utilisation et d'entretien	fr
Douchezitting – Gebruiks- en onderhoudshandleiding	nl
Duschsits – Bruks- och underhållsanvisning	sv
Dusjsete – Bruks- og vedlikeholdsveiledning	no
Asiento de ducha - Instrucciones de uso y mantenimiento	es
Sedile per doccia - Manuale d'uso e manutenzione	it
Siedzisko prysznicowe – instrukcja użytkowania i konserwacji	pl
浴椅 - 操作和维护手册	中文

English	7
English	16
Deutsch	25
Français	34
Nederlands	43
Svenska	52
Norsk	61
Español	70
Italiano	79
Język polski	88
中文	97

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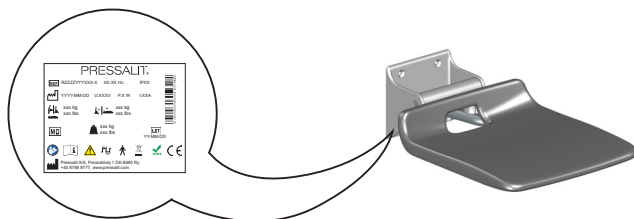
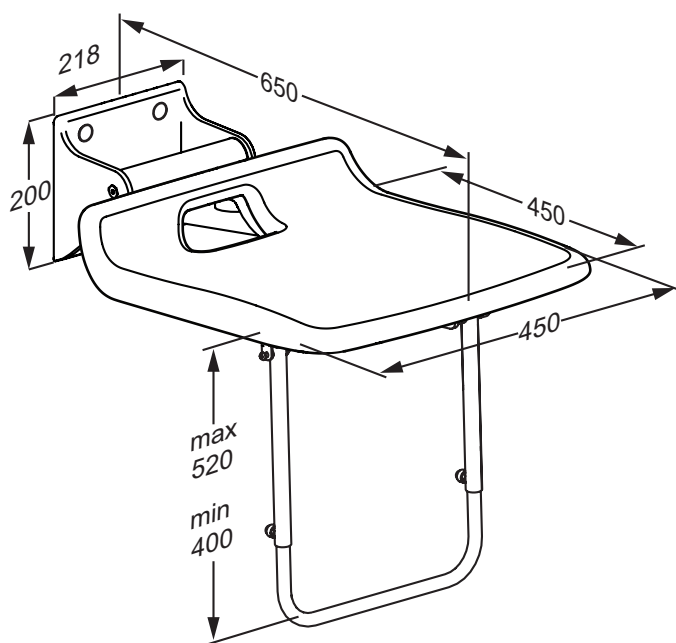
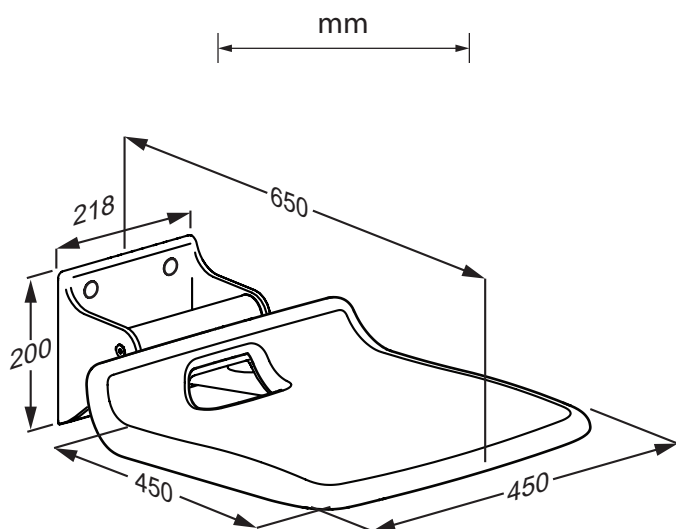
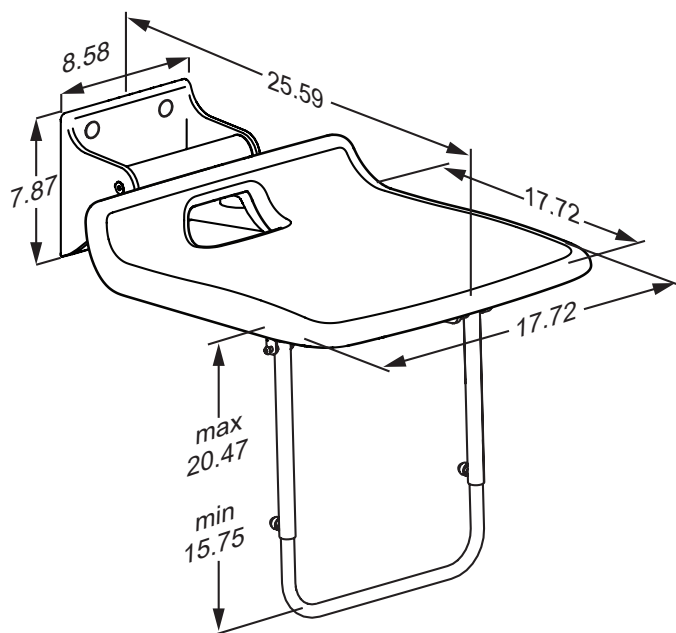
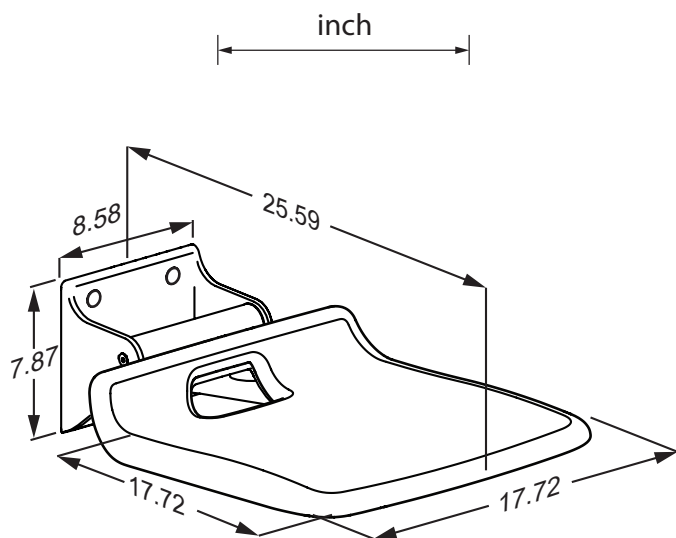
Content

Mounting instruction	7
Symbols used in this manual.....	8
General safety.....	9
Intended use	10
Intended user profile.....	10
Contraindications for use.....	10
Operating environment	10
Operating instructions, R1620	11
Folding the seat	11
Operating instructions, R1622	11
Folding the seat	11
Adjustment, R1622.....	12
Cleaning.....	13
Product label.....	14
Trouble-shooting.....	15
Disposal and recycling	15

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

en

Hand injury warning

Compatible products

Product weight

Max. user weight

Article number/
versions

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

The shower seat is used to sit on during showering.

Intended user profile

The shower seat is well suited for persons whose weight does not exceed the **maximum load of 135 kg / 297 lbs.**

Contraindications for use

People who are not capable of keeping their balance in a seated position are not recommended to use this product.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

NOTICE

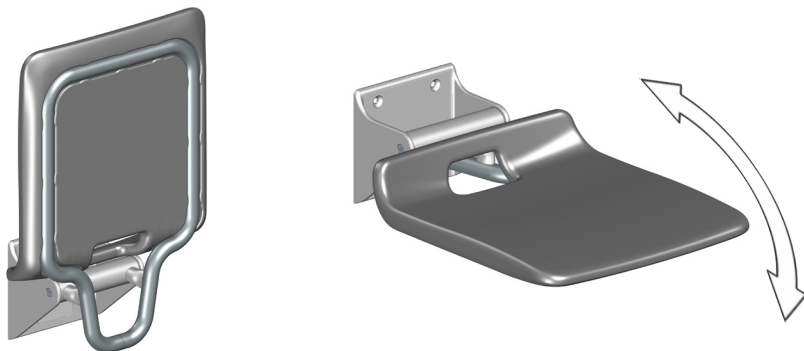
Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year.

Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.

Operating instructions, R1620

Folding the seat

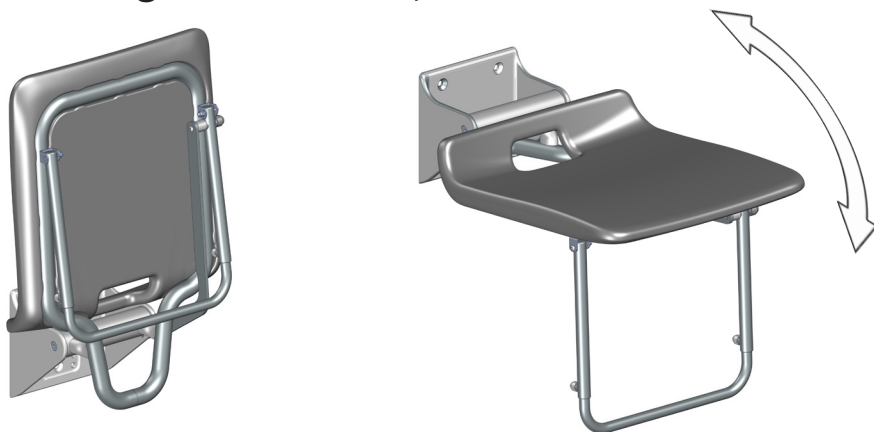
Fold the shower seat fully down. Fold it back up when not in use. You can hear a click when the shower seat is correctly folded up.

Take care to avoid injury to hands when in the vicinity of closing moveable parts.

en


Operating instructions, R1622

Folding the seat

Fold the shower seat fully down. The support will fold down automatically. Fold the seat back up when not in use. You can hear a click when the shower seat is correctly folded up.

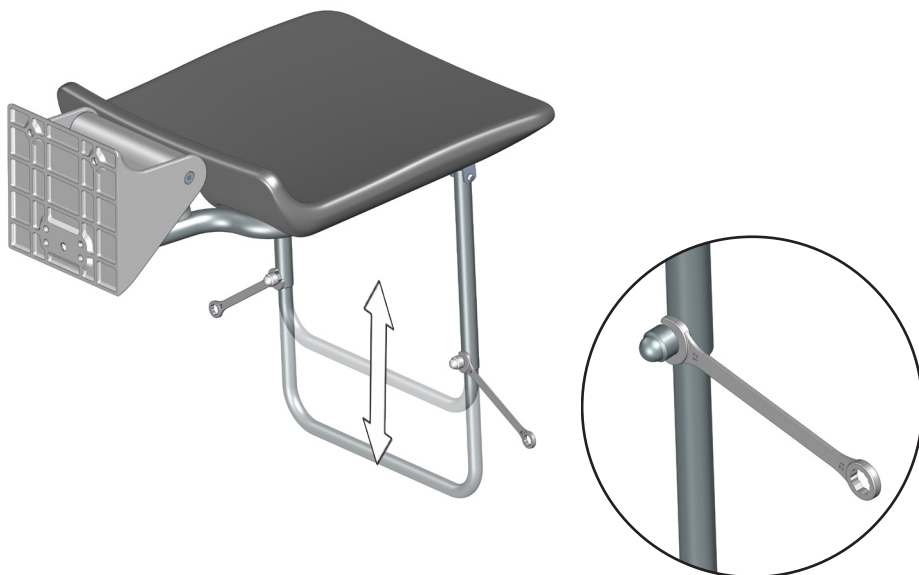
Adjustment, R1622


en

Maintenance

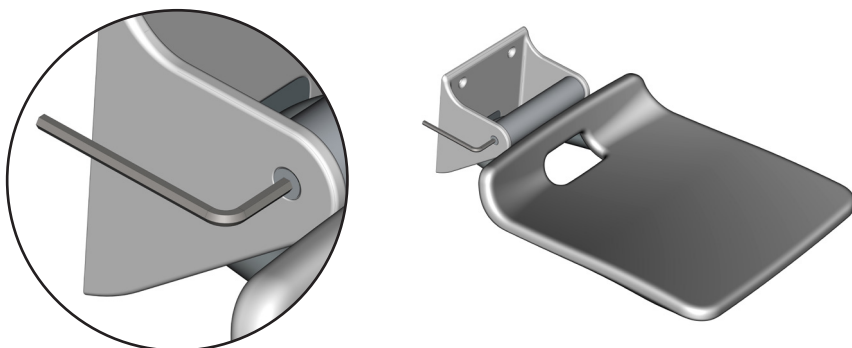It is recommended on a regular basis to adjust the screws to the swivel so that the seat is neither too loose nor too tight when folding up and down.

Cleaning

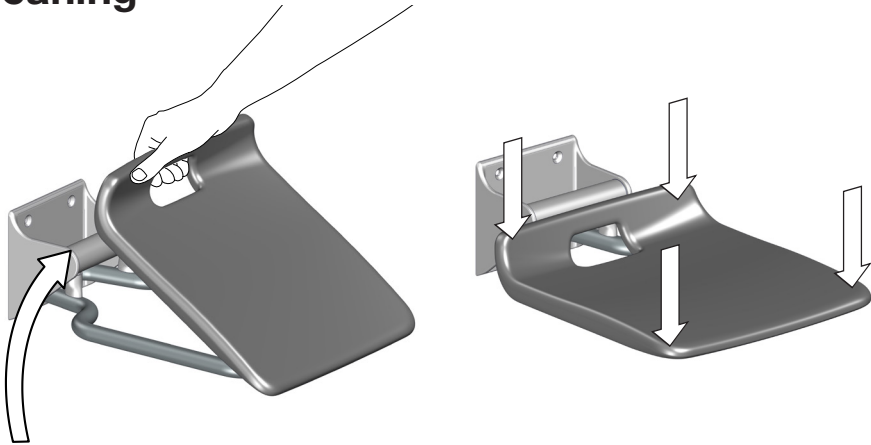The plastic part of the seat can be removed for cleaning. Grab the handle at the back and carefully pull the seat off the frame. After cleaning, put the seat back on to the frame by pressing down on the four corners.

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

en

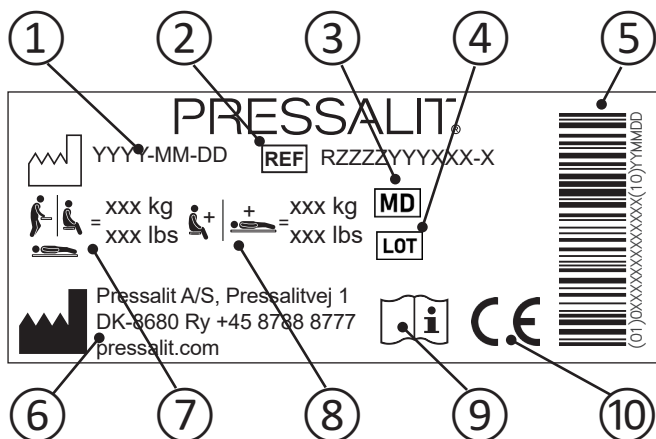
CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

Product label

- ① Production date
- ② Article number
- ③ Medical device
- ④ Batch code
- ⑤ GS1-128 barcode
- ⑥ Name and address of manufacturer
- ⑦ Maximum user weight
- ⑧ Max load
- ⑨ Consult manual for use

- ⑩ These products are CE-marked in accordance with relevant requirements in regulation (EU) 2017/745 (5 April 2017) on medical devices.
A declaration of conformity can be found at pressalit.com.

Trouble-shooting

Symptom	Cause	Action
The seat doesn't stay in the upright position.	The screws to the swivel are too loose	Tighten the screws as described under “Maintenance”.

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
HDPE	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production.

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