- Ⓔ Replacement instructions
- Ⓓ Udskiftningsvejledning
- Ⓓ Austauschanleitung
- Ⓕ Instructions de remplacement

- Ⓓ Vervangingsinstructies
- Ⓓ Utbytesanvisning
- Ⓓ Utskiftingsveiledning
- Ⓔ Instrucciones de sustitución

Subject to change without notice. Ret til ændringer forbeholdes. Änderungen vorbehalten. Nous nous réservons le droit de modifier sans préavis. Wijzigingen voorbehouden. Rätt till ändringar förbehålls. Endringer forbeholdes. Sujeto a cambios sin previo aviso.

EN Replacement Instructions

Make sure to read and follow these replacement instructions carefully. For further information, visit pressalit.com.

**WARNING**

The following precautions must be observed to avoid personal injury or damage to the product:

- Use only genuine Pressalit spare parts.
- If you suspect that one or more parts are damaged, contact your Pressalit dealer or service provider. Do not remove, attempt to repair, or install any parts yourself other than those included in this kit, unless instructed to do so by Pressalit or its representatives.
- Do not make any modifications to this equipment without the explicit permission of Pressalit.
- These replacement instructions must be followed, and the spare part must be installed correctly by a qualified professional with the appropriate authorisation, to prevent personal injury or damage to property.
- Disconnect and secure relevant supplies (e.g. water supply / pressure) before beginning work.
- Secure or relieve moving or loaded components before loosening fasteners to prevent unexpected movement during replacement.
- Ensure that the product cannot be accessed by unauthorised persons or children during the replacement.
- Pressalit will not be held liable for injuries to persons or damage to property resulting from incorrect installation.

Assembly and installation:

- Install the spare part in accordance with the illustrations in these replacement instructions.
- External cables must be secured in such a way that they do not cause a trip hazard.
- Check that all fasteners are tight and that the product functions as described in the user manual before signing off.

Only relevant for products with electrical components:

Check whether the product has a plug or is hardwired.

Hardwired: For permanently connected products (without a plug), the product must be connected to the mains supply and to a protective earth connection by a qualified electrician in accordance with IEC 60601-1, IEC 60364-7, and any locally applicable standards.

Plug connection: For products with a plug, replacement of mechanical spare parts may be carried out without authorisation.

**WARNING**

- Permanent installations must always be performed by a qualified electrician in accordance with applicable regulations and standards.
- Always disconnect the power supply before beginning work, and ensure that plugs and cables remain under control during the replacement.
- External cables must be secured so they cannot come loose or cause injury.

Special precautions must be taken when using electromedical equipment.

Electromagnetic compatibility (EMC): It is highly unlikely that the user will experience problems with the product due to insufficient electromagnetic compatibility. However, electromagnetic immunity is always relative, and the standards are based on defined usage environments. Portable and mobile RF communication equipment may affect electromedical equipment.

- When the product is connected to the local mains supply, it is recommended to use electromagnetic shielding and take appropriate measures to avoid conducted emissions.
- Ensure that the product is installed in a location with easy access and sufficient space to adjust the product.

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