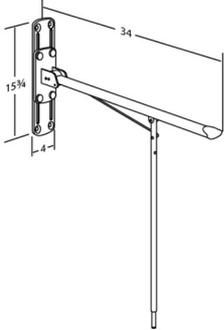




<p>Product Item number</p>	<p>VALUE fold down grab bar, height adjustable R1111000</p>
<p>Color Product description</p>	<p>VVS:615722410 NRF:6191644 RSK:7972413 EAN:5708590176437 HMI: 25414 NBS:N13:436 FDA:ILS/D135866 White</p>  <p>Fold down grab bar with leg, height adjustable 6". Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p> <p>Fold down grab bar with leg, height adjustable 6". Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p>
<p>Adjustability Adjustment height Product capacity</p>	<p>Height adjustable 5,91 inch Maximum load: 300 lbs. When the fold down grab bar is loaded with 300 lbs, each mounting bolt is loaded with 69 lbs.</p>
<p>Materials Ergonomy</p>	<p>Basic material: Steel Surface material: Rilsan End cap: Nylon (PA)</p> <p>Provides firm support when you are sitting, standing, or transferring to and from a toilet.</p> <p>Good, broad support surface for the hand and arm. The rounded form gives a firm and comfortable grip.</p>

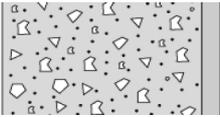
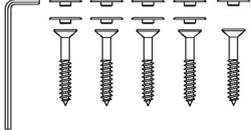
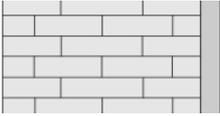
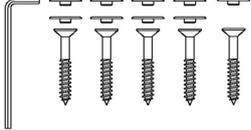
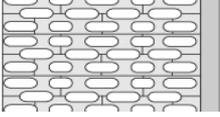
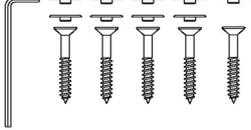
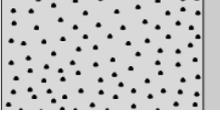
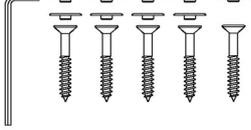
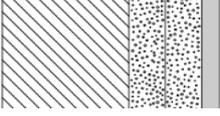
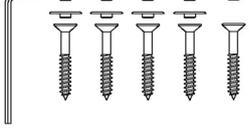


Product	VALUE fold down grab bar, height adjustable
Item number	R1111000
Features	<p>The grab bar can fold up when not in use or to make more space.</p> <p>With leg for extra support.</p> <p>Height adjustable 6".</p>
Warranty	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kilto Clean)</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>

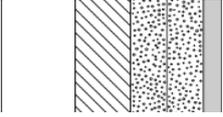
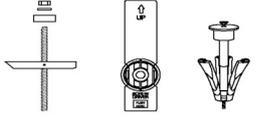
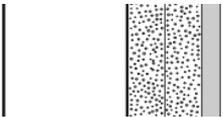
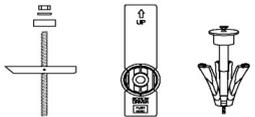
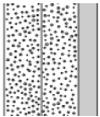


Product	VALUE fold down grab bar, height adjustable
Item number	R1111000

Mounting in different wall types

	Concrete >B25 V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Solid Brick >= Mz12 V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Aerated Concrete P4 V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	



Product Item number	VALUE fold down grab bar, height adjustable R1111000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

It is the responsibility of the installer to ensure that suitable wall fasteners are used.