




PRESSALIT®

Product Item number	PLUS folding seat, fixed height R5510000
Color Product description	VVS:777531000 NRF:7501001 RSK:7973939 EAN:5708590306339 HMI: 130159 NBS:N13:448 FDA:INN/D135865 White  The seat is suitable for situations where additional seating is required and space is limited, - e.g. in kitchens, lifts and hallways. The folding seat is not suitable for use in the shower area. PLUS folding seat. For fixed mounting. Materials: Aluminium, thermoset plastic, plastic. Excl. mounting kit.
Adjustability Product capacity	Fixed mounting Maximum user mass: 960 lbs. When the seat is loaded with 960 lbs, each wall fastener is loaded with 436 lbs. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
Materials Ergonomy Features	The folding seat is made out of thermoset plastic, and the mounting and suspension bracket are made from 3/8" anodised aluminium. The end caps are made from ABS plastic. Ergonomically shaped seat, which offers good and comfortable support. The seat can be folded away when not in use, - or to free up space. Stays up by itself when folded away.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

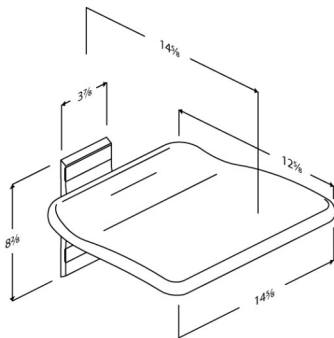

Product	PLUS folding seat, fixed height
Item number	R5510000
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.



PRESSALIT®

Product	PLUS folding seat, fixed height
Item number	R5510000

Mounting in different wall types

	Concrete >B25 V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately. 
	Solid Brick >= Mz12 V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately. 
	Solid Sand Lime Brick >= Hlz12 V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately. 
	Aerated Concrete P4 V8654 - Mounting kit, 4 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately. 



PRESSALIT®

Product Item number	PLUS folding seat, fixed height R5510000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.   
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.   
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

pressalit.com

Pressalit A/S
 Pressalitvej 1
 DK-8680 Ry
 Tel.: +45 8788 8788
 Fax: +45 8788 8789
 E-mail: pressalit@pressalit.com

PRESSALIT®