R8573114  
R8573118  
R8583114  
R8583118  
R8593114  
R8593118

## SCT 1000

**Shower change table – Operation and maintenance manual** en

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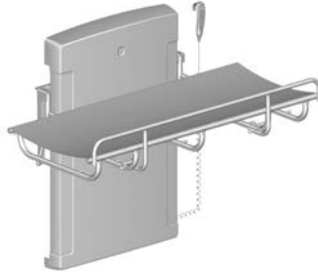
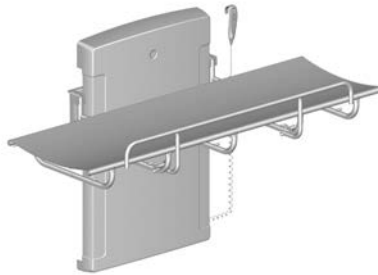
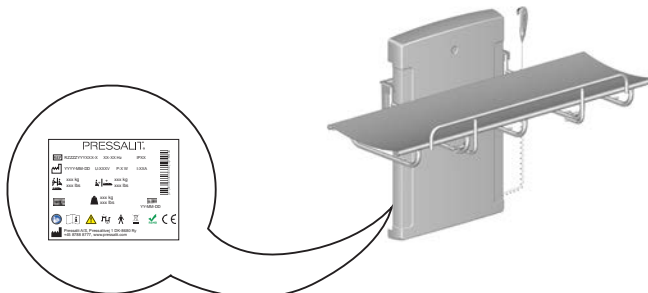
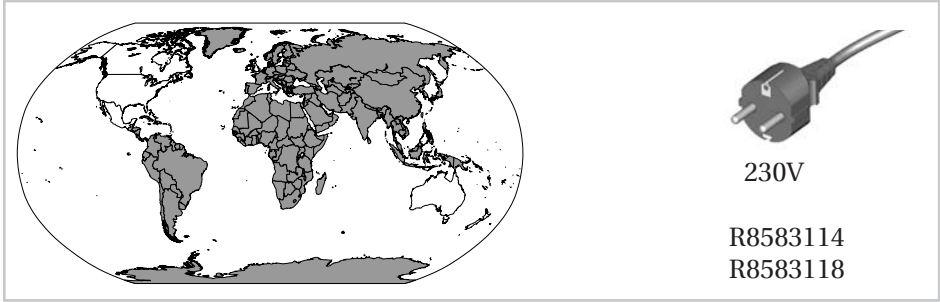
230V

R8583114  
R8583118

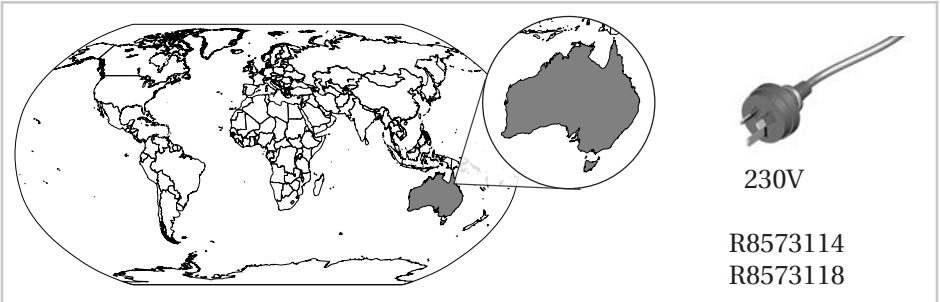
This block features a world map on the left with several regions shaded in grey, including parts of South America, Africa, Europe, and Asia. To the right is a black power plug with two prongs. Below the plug are the voltage specification '230V' and two model numbers, 'R8583114' and 'R8583118'.

120V

R8593114  
R8593118


This block features a world map on the left with North America highlighted in grey. A circular inset on the right provides a magnified view of the North American continent. To the right is a black power plug with two prongs. Below the plug are the voltage specification '120V' and two model numbers, 'R8593114' and 'R8593118'.


230V

R8573114  
R8573118

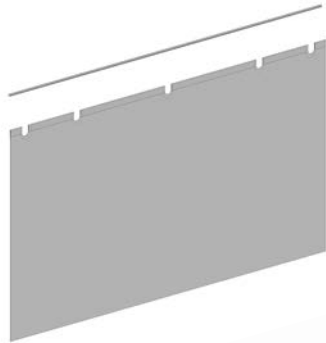
This block features a world map on the left with Australia and parts of Southeast Asia highlighted in grey. A circular inset on the right provides a magnified view of Australia. To the right is a black power plug with two prongs. Below the plug are the voltage specification '230V' and two model numbers, 'R8573114' and 'R8573118'.

R8420

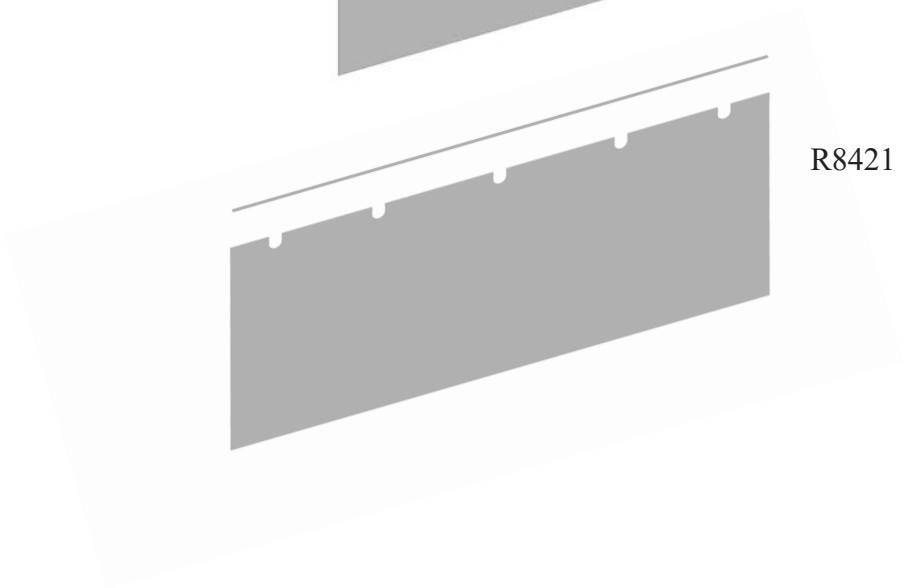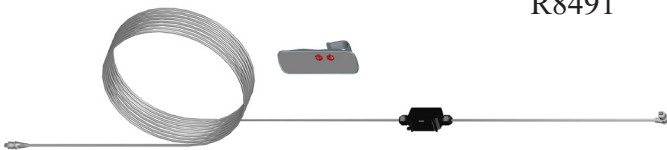R8421


R8491


	 	 
R8573114 R8583114 R8593114	44.46 kg, 98.01 lbs	200 kg / 440 lbs
R8573118 R8583118 R8593118	45.92 kg, 101.23 lbs	


	<b>Max P</b>	<b>U</b>	<b>Max I</b>
R8573114 R8573118 R8583114 R8583118	70 W	230V	0,3A
R8593114 R8593118	70 W	120V	0.6A

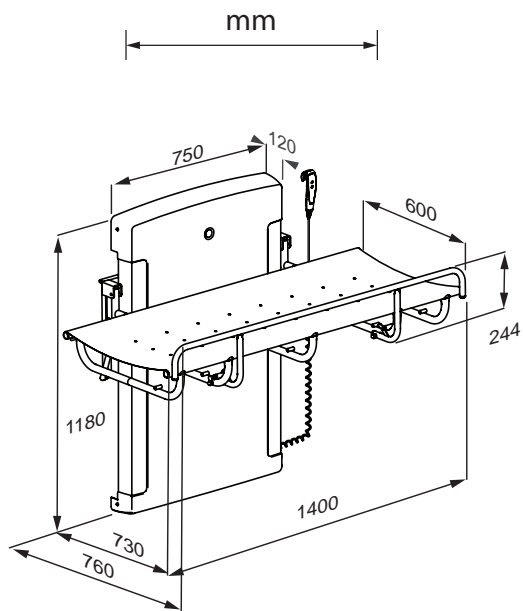
R8583114, R8593114, R8573114

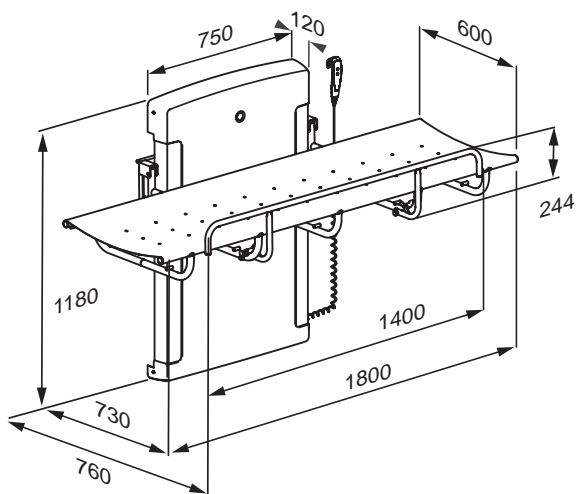R8583118, R8593118, R8573118

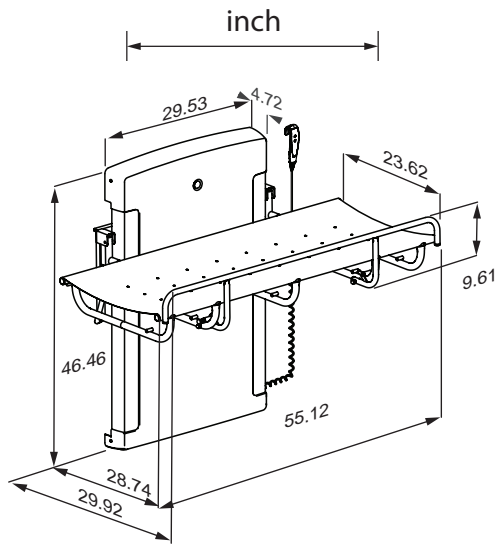R8583114, R8593114, R8573114

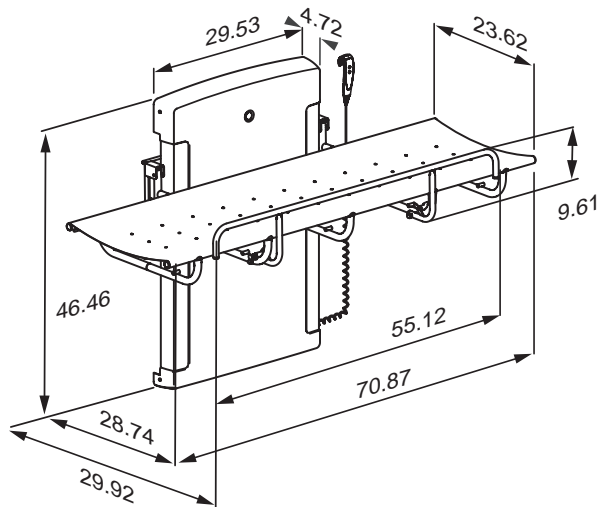R8583118, R8593118, R8573118

# Content

Mounting instruction .....	9
Operational signage .....	9
Symbols used in this manual .....	10
General safety .....	11
Intended use .....	12
Intended operator profile .....	12
Intended user profile .....	12
Operating environment .....	12
Contraindications for use .....	12
Biocompatibility .....	12
Operating instructions .....	13
Safety rail .....	14
Height adjustment .....	15
Cleaning .....	16
Disinfection .....	16
Maintenance .....	17
Maintenance inspection .....	17
Resetting .....	19
Trouble-shooting .....	19
Electromagnetic compatibility .....	20
Electromagnetic immunity .....	21
Electromagnetic emission .....	24
Product label .....	25
Technical data .....	27
Disposal and recycling .....	27

## Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from [www.pressalit.com](http://www.pressalit.com).

## Operational signage

An operational signage is included with the product. It can also be downloaded from [www.pressalit.com](http://www.pressalit.com).

# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

**WARNING** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION** indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

**IMPORTANT** indicates important information about handling and use of the product.

**NOTICE** is used to address practices not related to personal safety.

en

			
Hand injury warning	Head injury warning	Electricity / voltage warning	Do not leave patient unattended
			
Compatible products	Power specifications	Spare parts	Article number / versions
			
Product weight	Max. user weight		

# General safety

## **WARNING**

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

## **CAUTION**

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

## **NOTICE**

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

## Intended use

This product is intended for bathing a person who is unable to stand up or sit down. The person is lying horizontally on the shower table.

## Intended operator profile

The product is to be used by a person trained in health care and/or trained in washing a person in their care.

## Intended user profile

The product can be used by all - adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 440 lbs.**

## Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature, in private homes, nursing homes and hospitals under normal working conditions.

en

## Contraindications for use

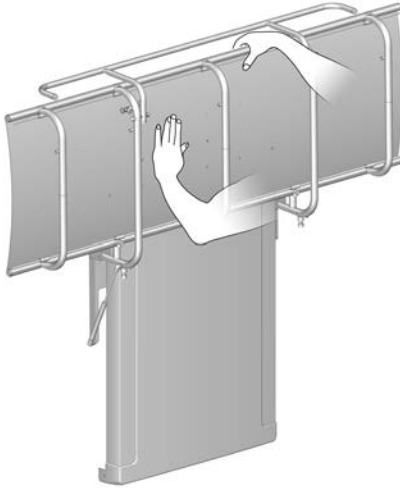
Use of the unit is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the unit only after consulting with the attending physician.

## Biocompatibility

The materials in contact with the skin of the user during the intended use have been evaluated for their content of harmful substances according to the procedures in BS EN ISO 10993-1:2020 Biological evaluation of medical devices – Part 1 and are therefore considered suitable for this purpose.

# Operating instructions

Support the shower table using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinder makes it easy to fold the shower table up against the wall after use.

**Take care to avoid injury to hands when in the vicinity of closing mechanical parts.**

**CAUTION**

Always be careful when moving a person to and from the shower table.

**CAUTION**

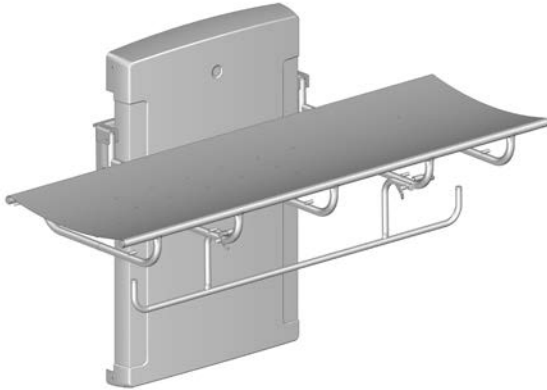
The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

**NOTICE**


Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

en

## Safety rail

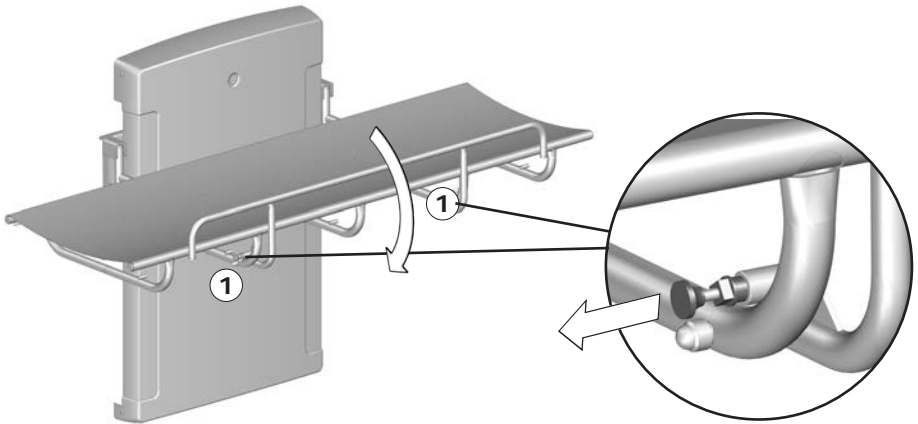**To pull the safety rail up:** Swing the rail upwards from under the shower table. The safety rail will automatically lock into position.


**To fold the safety rail away:** Release the two locks ① simultaneously and swing the safety rail down under the shower table.

**Do not leave the patient unattended.**
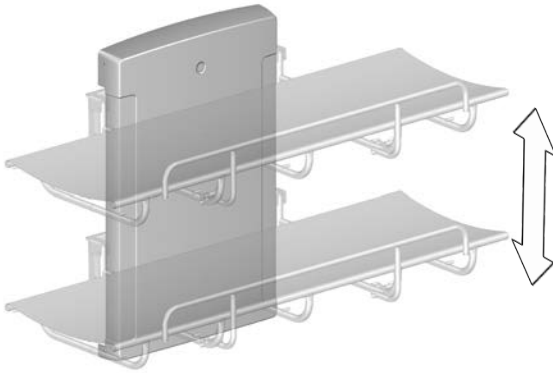
## Height adjustment

When the shower table is installed at the recommended height, it can be adjusted between 480 and 880 mm (18.9" and 34.6") above the floor.

The height adjustment is controlled by the wired remote control unit.

Press and hold the up button to raise the unit. When the button is released, the motion stops.

Press and hold the down button to lower the unit. When the button is released, the motion stops.

**To avoid risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.**

**Take care to avoid injury to hands and head when in the vicinity of moving mechanical parts.**

**CAUTION**

Never place objects underneath the shower table.

## Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit [Pressalit.com](http://Pressalit.com) and refer to the product's item number. Pressalit recommends cleaning the product after installation.

### CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

### IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

## Disinfection

When using the shower table for multiple patients, it is necessary to both clean and disinfect the shower table between uses on different patients to prevent the risk of cross infection. A list of suitable disinfectants can be found at [pressalit.com](http://pressalit.com).

# Maintenance

Expected service life of the shower table is 10 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the shower table to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

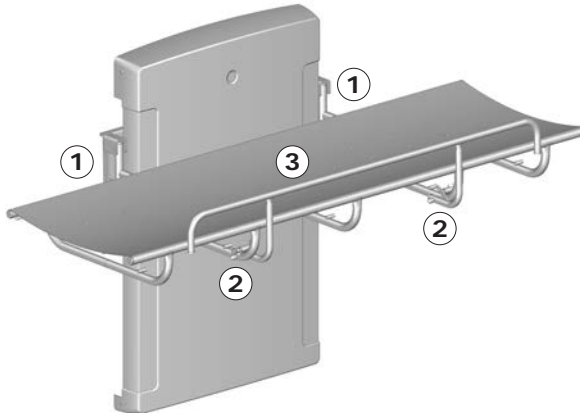
It is recommended that maintenance inspection is done once a year.

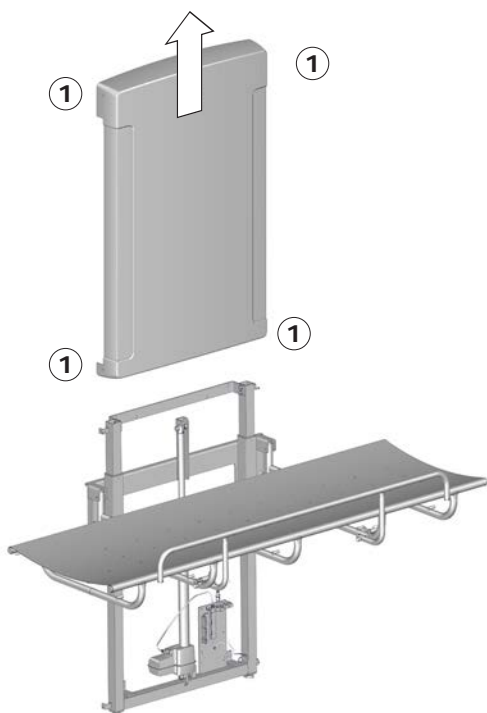
**Disconnect mains before removing the covers.**


## Maintenance inspection

- Check the hinges ① on the shower table and the safety rail ② . Tighten if necessary.
- Check the canvas ③ for damage and replace if necessary.

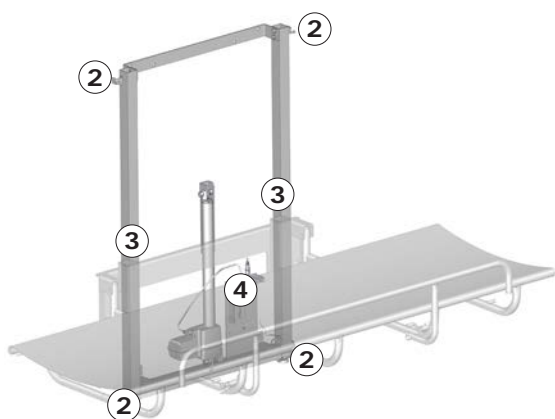- Remove the upper and lower retaining screws ① on the cover and remove it carefully.

- Check the wall brackets ② and tighten if necessary.
- Oil moving parts, especially the sliding brackets ③.
- Check all electrical connections to the control box ④.
- Put the unit cover back again and tighten the retaining screws.
- Reconnect mains.

# Resetting

If the height adjustment does not work, and the control box beeps and/or flashes, the unit must be reset.

Press and hold the up button and the down button simultaneously. The control box emits a beep sound. When the beep stops, release both buttons.

Take the unit to its lowest position. The unit is now reset.

# Trouble-shooting

Symptom	Cause	Action
The shower table does not stay in upright position.	The gas cylinder is defective.	Contact Pressalit or your local dealer.
The product will not work, even though the remote control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit or your local dealer.
The product does not work, and the control box beeps and/or flashes.	The product needs to be reset.	See "Resetting".
The shower table runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

# Electromagnetic compatibility

The device is suitable for use at home, at daycare centres, at day centres for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and RF shielded rooms with medical systems for magnetic resonance imaging with high intensity of electromagnetic disturbances.

The device does not have any function related to basic safety or essential performance. In case of loss or degradation of performance due to electromagnetic disturbances, the user can still get on and off the product.

<b>Cable</b>	<b>Length</b>
Power cable	3,2 m (126")
Hand control spiral cable	Approx. 5 m (197") extended

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

# Electromagnetic immunity

<b>Electrostatic discharge</b>	
	<p>The product must not be affected by electrical discharge between a person and the product in such a way as to initiate actions or render components defective.</p>
<p><b>Test standard</b></p>	<p><b>Immunity test level</b></p>
<p>EN 61000-4-2</p>	<p>+/- 8 kV contact +/- 2, 4, 8, 15 kV air</p>

<b>Radiated RF electromagnetic field</b>	
	<p>The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.</p>
<p><b>Test standard</b></p>	<p><b>Immunity test level</b></p>
<p>EN 61000-4-3</p>	<p>15 V/m 80 MHz to 1000 MHz (only on the front) 15 V/m 1.01 GHz to 3 GHz 80 % AM 1 kHz</p>

en

**Immunity to proximity fields from RF wireless communication equipment**

	<p>The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.</p>
<p><b>Test standard</b></p>	<p><b>Immunity test level</b></p>
<p>EN 61000-4-3*</p>	<p>385 MHz, 27V/m, 18 Hz PM (50 % duty cycle square wave)            450 MHz, 28V/m, FM +/- 5 kHz deviation, 1 kHz sine            710 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            745 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            780 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            810 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave)            870 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave)            930 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave)            1720 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave)            1845 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave)            1970 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave)            2450 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave)            5240 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            5500 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            5785 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)</p>

en

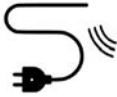
**Electrical fast transient/burst**

	<p>The product must not be affected by rapid pulse signals through the cord. This could cause the components to become defective, or the product could initiate actions.</p>
<p><b>Test standard</b></p>	<p><b>Immunity test level</b></p>
<p>EN 61000-4-4</p>	<p>+/- 1 kV , +/- 2 kV            100 kHz repetition frequency</p>

### Surge immunity


	The product must not be affected to initiate actions or to render components defective as a result of lightning.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-5	Line-to-line: +/- 0.5 kV, +/-1 kV Line-to-ground: +/- 0.5 kV, +/-1 kV, +/- 2 kV

### Immunity to conducted disturbances induced by RF fields

	The wires must not be affected by other RF fields so as to initiate actions or render components defective.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-6	3 V (6 V in ISM bands and amateur radio bands) 0,15 to 80 MHz , 80 % AM at 1 kHz

en

### Voltage dips

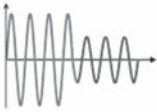	The product must not initiate or limit actions as a result of a dip in the power supply.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-11	0 % $U_T$ ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, 0 % $U_T$ ; 1 cycle at 0°, 70 % $U_T$ ; 25 cycles at 0° Single phase at 0°

### Voltage interruptions

	The product must not initiate or limit actions as a result of a short-term power supply stop.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-11	0 % $U_T$ ; 300 cycle

## Electromagnetic emission

<b>Conducted RF emission</b>	
	The product must not send too much noise into the grid. This could lead to a disruption of other products.
<b>Test standard</b>	<b>Emission test class and group</b>
EN 55011	Class B, group 1

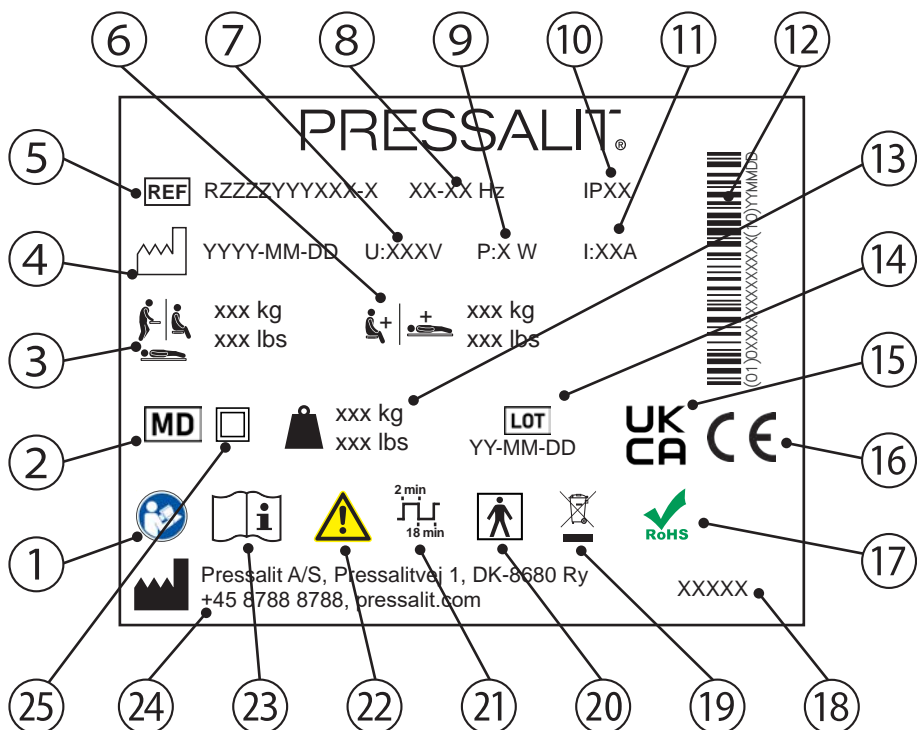
<b>Radiated RF emission</b>	
	The product must not emit RF signals in the room. This could lead to a disruption of other products.
<b>Test standard</b>	<b>Emission test class and group</b>
EN 55011	Class B, group 1

<b>Harmonic current emissions</b>	
	The product must not unintentionally draw power from the grid.
<b>Test standard</b>	<b>Emission test class and group</b>
IEC 61000-3-2	Class A

<b>Voltage changes, voltage fluctuations and flicker</b>	
	The product must not affect other products when it is connected to the grid. For example, these could be lights that could start flashing and thus bother people in the room.
<b>Test standard</b>	<b>Emission test class and group</b>
EN 61000-3-3	Tested according to the standard.

# Product label

en

- ① Consult manual for use

---

- ② Medical device

---

- ③ Maximum user weight

---

- ④ Production date

---

- ⑤ Article number

---

- ⑥ Max load

---

- ⑦ Mains voltage

---

- ⑧ Frequency

---

- ⑨ Maximum power consumption

⑩ IP class

---

⑪ Electric Current

---

⑫ GS1-128 barcode

---

⑬ Product weight

---

⑭ Batch code

---

⑮ UK Conformity Assessed

---

These products are CE-marked in accordance with relevant requirements in:

- ⑯
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
  - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
  - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at [pressalit.com](http://pressalit.com)

---

⑰ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

---

⑱ Product name (if any)

---

⑲ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

---

⑳ Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function  
The product complies with requirements of IEC 60601-1 to provide protection against electric shock

---

㉑ Duty cycle

---

㉒ Consult the manual for important cautionary information such as warnings and precautions

---

㉓ Consult manual for use

---

㉔ Name and address of manufacturer

---

㉕ Class II Product, double insulated

---

# Technical data

Materials	Powder laquered stainless steel Plastics: PVC (canvas) Polystyrene (cover)
IP class	The product has an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.

## Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

<b>Recyclable components</b>	<b>Material</b>
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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