


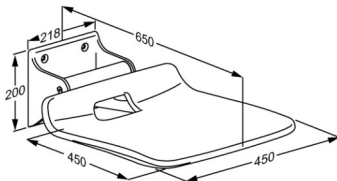

Product Item number	VALUE IV shower seat, fixed height R1620000112
Colour	VVS:777525312 NRF:6314097 RSK:8565069 EAN:5708590377711 HMI: 124483 DB: 2296038 FDA:INN/D135865 Anthracite grey
Product description	 VALUE IV folding shower seat and backrest create a simple solution for users who need to sit down while showering. VALUE IV shower seat. Fixed height. The seat is foldable. Materials: Diecast aluminium, Rilsan-coated stainless steel and vacuum-moulded polyethylene (PE). Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.
Adjustability Product capacity	Fixed mounting Maximum user mass: 135 kg. When the seat is loaded with 135 kg, each wall fastener is loaded with 125 kg. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
Materials	The wall profile is made of diecast aluminium, and the frame is made of Rilsan-coated stainless steel. Rilsan provides good protection against corrosion and is comfortable to touch and easy to clean. The seat is made of vacuum-moulded polyethylene (PE).
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



Product	VALUE IV shower seat, fixed height
Item number	R1620000112
Cleaning	The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean) NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.