

PRESSALIT.

Product	VALUE support arm, height adjustable		
Item number	R1111000		
	VVS:615722410		
	NRF:6191644		
	RSK:7972413		
	EAN:5708590176437		
	HMI: 25414		
	NBS:N13:436		
	FDA: ILS/D135866		
Colour	White		
Product description	Support arm with leg, height adjustable 150 mm. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.		
	Support arm with leg, height adjustable 150 mm. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.		
Adjustability	Height adjustable		
Adjustment height	150 mm		
Product capacity	Maximum load: 135 kg. When the support arm is loaded with 135 kg, each wall fastener is loaded with 29 kg		
Materials	The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA).		
Ergonomy	Provides firm support when you are sitting, standing, or transferring to and from a toilet.		
	Good, broad support surface for the hand and arm.		



PRESSALIT.

Product	VALUE support arm, height adjustable	
Item number	R1111000	
Features	The support arm can fold up when not in use or to make more space.	
	With leg for extra support.	
	Height adjustable 150 mm.	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	
Cleaning	VALUE and art arms	
	VALUE support arms The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.	
	For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.	
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.	
	DISINFECTANTS:	
	Clinimax Difficil-S Erisan OXY+	
	Sprit (husholdningssprit)	
	Suma Bac D10 Virkon	
	Rodalon Desinfektionsspray	
	Lysoformin Spezial	
	CaviCide Sufanios Premium	
	Tvättsprit 70%	
	WipeClean UniversalCleaning	
	Chrolorine Disinfection Wipes Klorilli (Kiilto Clean)	
	NON-DISINFECTANTS:	
	Suma Star free D1	
	Taski 100 Taski 200	
	Milizid Citro	
	The product must be wiped down with a soft rag wrung out in clear water immedia after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.	



PRESSALIT.

Product

VALUE support arm, height adjustable

Item number

R1111000

Installation in different wall types




Concrete >B25

V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.



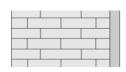


Solid Brick >= Mz12

V8592 - 5 countersunk screws \emptyset 10 x 80, with white screw covers are included.

Plugs to be ordered separately.



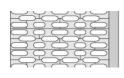

Solid Sand Lime Brick >= Hlz12

V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.






Aerated Concrete P4

V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.



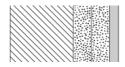

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.





PRESSALIT®

Product Item number	VALUE support arm, height adjustable R1111000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>!</u>

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com