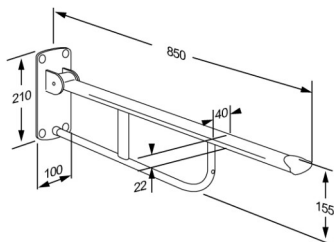




|   |   |
|---|---|
| <b>Product</b><br><b>Item number</b>            | VALUE support arm, fixed height<br>R1100000   |
| <b>Colour</b>                                   | VVS:615722300<br>NRF:6191642<br>RSK:7972410<br>EAN:5708590176406<br>HMI: 25411<br>NBS:N13:436<br>DB: 2076622<br>FDA:ILS/D135866<br>White  |
| <b>Product description</b>                      | <div>  </div> <p>VALUE support arm, fixed, foldable, projection 850 mm.<br/>         The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends.<br/>         Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p> <p>VALUE support arm, fixed, foldable, projection 850 mm.<br/>         The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends.<br/>         Materials: Rilsan-coated steel, pressure-cast aluminium, plastic.<br/>         Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p> |
| <b>Adjustability</b><br><b>Product capacity</b> | Fixed mounting<br>Maximum load: 120 kg.<br>When the support arm is loaded with 120 kg, each wall fastener is loaded with 242 kg.  |
| <b>Materials</b><br><b>Ergonomy</b>             | <p>The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA).</p> <p>Provides firm support when you are sitting, standing, or transferring to and from a toilet.</p> <p>Good, broad support surface for the hand and arm.<br/>         The rounded form gives a firm and comfortable grip.</p>  |



# PRESSALIT®

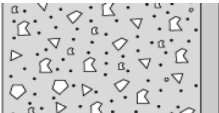
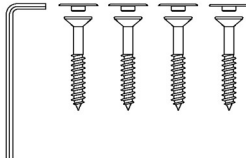

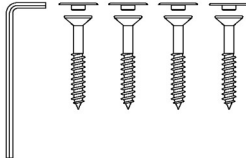
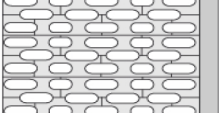
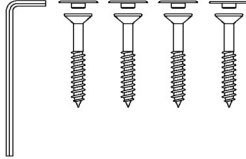
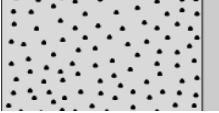
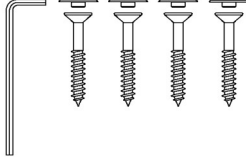
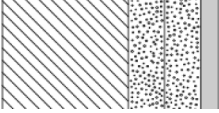
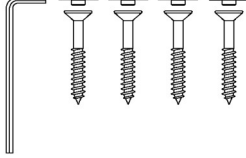
|                                  |  |
|----------------------------------|--|
| <b>Product</b>                   | VALUE support arm, fixed height  |
| <b>Item number</b>               | R1100000   |
| <b>Features</b>                  | The support arm can fold up when not in use or to make more space.   |
| <b>Warranty</b>                  | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |
| <b>Cleaning</b>                  | <p>VALUE support arms.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b></p> <p>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%<br/>WipeClean UniversalCleaning<br/>Chlorine Disinfection Wipes<br/>Klorilli (Kiilto Clean)</p> <p><b>NON-DISINFECTANTS:</b></p> <p>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> |
| <b>Declaration of conformity</b> | CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.   |



PRESSALIT®

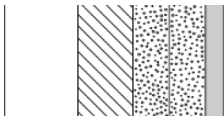
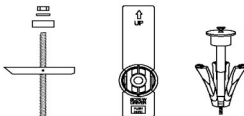
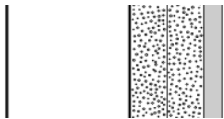
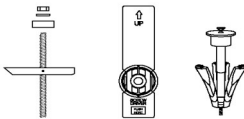
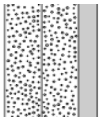

|                    |                                 |
|--------------------|---------------------------------|
| <b>Product</b>     | VALUE support arm, fixed height |
| <b>Item number</b> | R1100000                        |

## Installation in different wall types

|   |  |   |
|---|--|---|
|    | <p><b>Concrete &gt;B25</b></p> <p>V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included.<br/>Plugs to be ordered separately.</p>   |    |
|   | <p><b>Solid Brick &gt;= Mz12</b></p> <p>V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included.<br/>Plugs to be ordered separately.</p>                                       |   |
|  | <p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included.<br/>Plugs to be ordered separately.</p>                            |  |
|  | <p><b>Aerated Concrete P4</b></p> <p>V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included.<br/>Plugs to be ordered separately.</p>  |  |
|  | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included.<br/>Plugs to be ordered separately.</p> |  |



# PRESSALIT®

| Product<br>Item number  | VALUE support arm, fixed height<br>R1100000   |
|---|---|
|    | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b></p> <p>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</p>        |
|    | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b></p> <p>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</p>  |
|  | <p><b>Drywall (2 x 12,5 mm) water-resistant</b></p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>    |

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.

[pressalit.com](http://pressalit.com)