




# PRESSALIT®

<b>Product</b> <b>Item number</b>	VALUE fold down grab bar, fixed height R1100000
<b>Color</b>	VVS:615722300 NRF:6191642 RSK:7972410 EAN:5708590176406 HMI: 25411 NBS:N13:436 DB: 2076622 FDA:ILS/D135866 White
<b>Product description</b>	<p>           VALUE fold down grab bar, fixed, foldable, length 33.5".            The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends.            Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.         </p> <p>           VALUE fold down grab bar, fixed, foldable, length 33.5".            The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends.            Materials: Rilsan-coated steel, pressure-cast aluminium, plastic.            Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.         </p>
<b>Adjustability</b>	Fixed mounting
<b>Product capacity</b>	Maximum load: 265 lbs. When the fold down grab bar is loaded with 265 lbs, each mounting bolt is loaded with 534 lbs.
<b>Materials</b>	Basic material: Steel Surface material: Rilsan End cap: Nylon (PA)

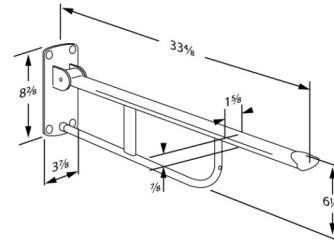



<b>Product</b>	VALUE fold down grab bar, fixed height
<b>Item number</b>	R1100000
<b>Ergonomy</b>	<p>Provides firm support when you are sitting, standing, or transferring to and from a toilet.</p> <p>Good, broad support surface for the hand and arm. The rounded form gives a firm and comfortable grip.</p>
<b>Features</b>	<p>The grab bar can fold up when not in use or to make more space.</p>
<b>Warranty</b>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean)</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>







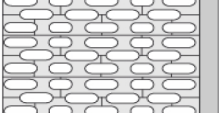
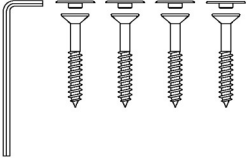
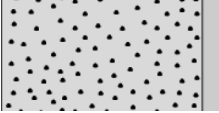
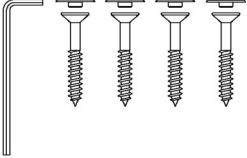
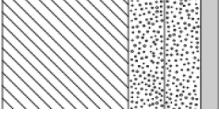
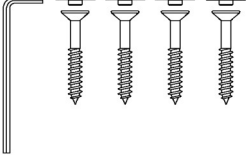
# PRESSALIT<sup>®</sup>

<b>Product</b>	VALUE fold down grab bar, fixed height
<b>Item number</b>	R1100000
<b>Declaration of conformity</b>	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.




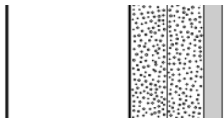
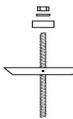

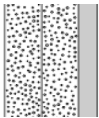



<b>Product</b>	VALUE fold down grab bar, fixed height
<b>Item number</b>	R1100000
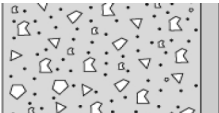
## Mounting in different wall types

	<p><b>Concrete &gt;B25</b></p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p><b>Aerated Concrete P4</b></p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	

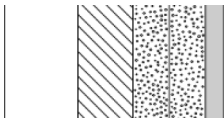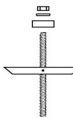

<b>Product</b> <b>Item number</b>	VALUE fold down grab bar, fixed height R1100000		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b> Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.		

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.