

Product Item number	VALUE support arm with leg, fixed height R1101000		
	VVS:615722400		
	NRF:6191641		
	EAN:5708590176413		
	HMI: 25412		
	NBS:N13:436		
	DB: 2076621		
	FDA: ILS/D135866		
Colour	White		
Product description	 VALUE support arm with leg, fixed, foldable, projection 850 mm, height 850 mm. The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately. VALUE support arm with leg, fixed, foldable, projection 850 mm, height 850 mm. The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Materials: Rilsan-coated steel, pressure-cast aluminium, plastic. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately. 		
Adjustability	Fixed mounting		
Product capacity	Maximum load: 135 kg. When the support arm is loaded with 135 kg, each wall fastener is loaded with 81 k		
Materials	The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. T end caps are manufactured from nylon (PA).		
Ergonomy	Provides firm support when you are sitting, standing, or transferring to and from a toilet.		
	Good, broad support surface for the hand and arm. The rounded form gives a firm and comfortable grip.		



Product	VALUE support arm with leg, fixed height		
Item number	R1101000		
Features	The support arm can fold up when not in use or to make more space.		
	With leg for extra support.		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		
Cleaning	 VALUE support arms. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. 		
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Klorilli NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200		
	The product must be wiped down with a soft rag wrung out in clear water immediat after cleaning, as some materials may otherwise be discolored and stained, and th surface coating may deteriorate.		
Declaration of conformity	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.		



Product

Item number

VALUE support arm with leg, fixed height R1101000

Installation in different wall types

Concrete >B25 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
Solid Brick >= Mz12 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
Solid Sand Lime Brick >= HIz12 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
Aerated Concrete P4 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	



Product Item number	VALUE support arm with leg, fixed height R1101000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>^!</u>

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.

pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

