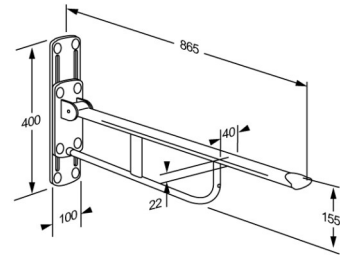




# PRESSALIT®

<b>Product</b>	VALUE support arm, height adjustable
<b>Item number</b>	R1110000
<b>Colour</b>	<p>VVS:615722405  NRF:6191643  RSK:7972412  EAN:5708590176420  HMI: 25413  NBS:N13:436  FDA:ILS/D135866</p> <p>White</p>
<b>Product description</b>	<p>VALUE support arm, wall-mounted, foldable, height adjustable 150 mm, projection 865 mm.  The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends.  Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p> <p>VALUE support arm, wall-mounted, foldable, height adjustable 150 mm, projection 865 mm.  The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends.  Materials: Rilsan-coated steel, diecast aluminium, plastic.  Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p>
<b>Adjustability</b>	Height adjustable
<b>Adjustment height</b>	150 mm
<b>Product capacity</b>	<p>Maximum load: 120 kg.  When the support arm is loaded with 120 kg, each wall fastener is loaded with 86 kg.</p>
<b>Materials</b>	<p>The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA).</p>





# PRESSALIT®

<b>Product</b>	VALUE support arm, height adjustable
<b>Item number</b>	R1110000
<b>Ergonomy</b>	<p>Provides firm support when you are sitting, standing, or transferring to and from a toilet.</p> <p>Good, broad support surface for the hand and arm. The rounded form gives a firm and comfortable grip.</p>
<b>Features</b>	<p>The support arm can fold up when not in use or to make more space.</p> <p>Height adjustable 150 mm.</p>
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



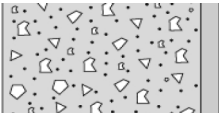
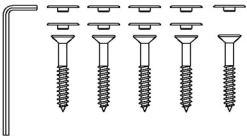

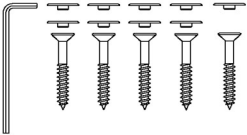
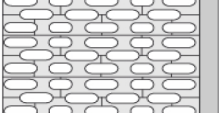
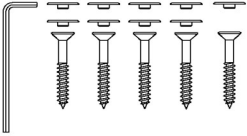
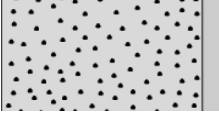
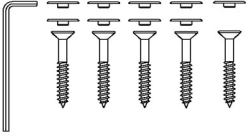
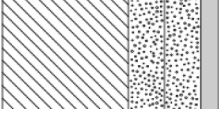
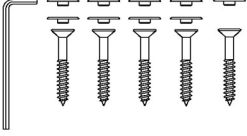
<b>Product</b>	VALUE support arm, height adjustable
<b>Item number</b>	R1110000
<b>Cleaning</b>	<p>VALUE support arms.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b></p> <p>Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättssprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean)</p> <p><b>NON-DISINFECTANTS:</b></p> <p>Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
<b>Declaration of conformity</b>	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.



PRESSALIT®

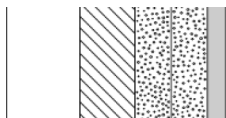
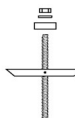

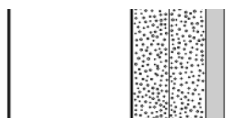
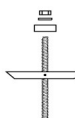

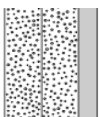

<b>Product</b>	VALUE support arm, height adjustable
<b>Item number</b>	R1110000

## Installation in different wall types

	<p><b>Concrete &gt;B25</b></p> <p>V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p> 
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p> 
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p> 
	<p><b>Aerated Concrete P4</b></p> <p>V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p> 
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p> 



# PRESSALIT®

Product Item number	VALUE support arm, height adjustable R1110000		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b> Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.		
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.		

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

[pressalit.com](http://pressalit.com)