

PRESSALIT.

Product	VALUE fold down grab bar, height adjustable		
Item number	R1111000		
	VVS:615722410		
	NRF:6191644		
	RSK:7972413		
	EAN:5708590176437		
	HMI: 25414		
	NBS:N13:436		
	FDA:ILS/D135866		
Color	White		
Product description	Fold down grab bar with leg, height adjustable 6". Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.		
	Fold down grab bar with leg, height adjustable 6". Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.		
Adjustability	Height adjustable		
Adjustment height	5,91 inch		
Product capacity	Maximum load: 300 lbs. When the fold down grab bar is loaded with 300 lbs, each mounting bolt is loaded with 69 lbs.		
Materials	Basic material: Steel		
	Surface material: Rilsan		
	End cap: Nylon (PA)		
Ergonomy	Provides firm support when you are sitting, standing, or transferring to and from a toilet.		
	Good, broad support surface for the hand and arm. The rounded form gives a firm and comfortable grip.		



PRESSALIT.®

Product	VALUE fold down grab bar, height adjustable R1111000		
Features	The grab bar can fold up when not in use or to make more space.		
i catales			
	With leg for extra support.		
	Height adjustable 6".		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.		
	For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.		
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.		
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+		
	Sprit (husholdningssprit) Suma Bac D10		
	Virkon Rodalon Desinfektionsspray		
	Lysoformin Spezial CaviCide		
	Sufanios Premium Tvättsprit 70%		
	WipeClean UniversalCleaning Chrolorine Disinfection Wipes Klorilli (Kiilto Clean)		
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200		
	Milizid Citro		
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.		
Declaration of conformity	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.		



PRESSALIT.

Product			
Item number			

VALUE fold down grab bar, height adjustable

R1111000


Mounting in different wall types

Concrete >B25

V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.



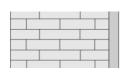


Solid Brick >= Mz12

V8592 - 5 countersunk bolts \emptyset 10 x 80, with white screw covers are included.

Plugs to be ordered separately.



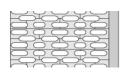

Solid Sand Lime Brick >= Hlz12

V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.






Aerated Concrete P4

V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.



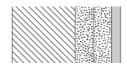

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8592 - 5 countersunk bolts Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.





PRESSALIT®

Product Item number	VALUE fold down grab bar, height adjustable R1111000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>^!</u>

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com