





**Toilet seat Ergosit with cover, 50 mm raised
With stabilising buffers and special hinges adapted to raised toilet
seat.**

<p>Item number</p> <p>Colour</p> <p>Description</p>	<p>R20000 VVS:615691000 NRF:6190519 EAN:5708590094779 HMI: 12501 NBS:N13:300</p> <p>Toilet seat, Ergosit, with stabilising buffers, w/cover, raised 50 mm With raised cross bar hinge</p> <p>White</p> <p>Toilet seat, Ergosit, with stabilising buffers, w/cover, raised 50 mm With raised cross bar hinge</p> 	
<p>Materials</p> <p>Product capacity</p>	<p>Seat/cover: Thermoset plastic (urea)</p> <p>Strap: Stainless steel V2A</p> <p>Raiser: Nylon (PA)</p> <p>Special buffer: ABS-plastic</p> <p>This product has been tested statically up to 375 kg. As the European standard for technical aids for disabled, BS/EN 12182:2012, requires a static test with safety factor 1.5, the nominal maximum load for this product is therefore 250 kg.</p>	
<p>Weight</p> <p>Gross weight</p> <p>Weight per crate</p> <p>Qty per crate</p> <p>Crates per pallet</p> <p>Qty per pallet</p>	<p>3,5 kg</p> <p>3,8 kg</p> <p>0 kg</p> <p>0</p> <p>0</p> <p>0</p>	

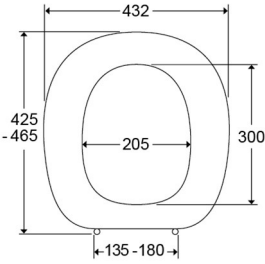



PRESSALIT®

**Toilet seat Ergosit with cover, 50 mm raised
With stabilising buffers and special hinges adapted to raised toilet
seat.**

Cleaning

Use a mild soap solution to clean the toilet seat. Please take care that the seat and hinges are not left damp. For disinfecting the seat, e.g. ethanol 85% can be used. Make sure that the product is wiped down with a soft rag wrung out in clear water and dried off with a soft, dry cloth immediately after disinfection. *Avoid contact of the seat and hinges with abrasive, corrosive or chlorine-based cleaners, as these may cause damage or lead to flash rust. Therefore, when cleaning the bowl, make sure that the seat and cover are kept in an upright position until all cleaning agent has been flushed away.*

Warranty

Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



www.pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®