

PRESSALIT.

Product	PLUS grab bar 11.8"			
Item number	RT100035			
	VVS:777565135			
	NRF:6317161			
	EAN:5708590145822			
	HMI: 19081			
	NBS:N13:436			
Color	Red (RAL 3003)			
Product description	PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom A reliable solution with many different options, which can easily lay the grounds for increased independence. PLUS grab bar, diameter 1.3", 11.8", for fixed mounting. Material: Powder-coated aluminium and plastic. Excl. mounting kit.			
Product capacity	Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.			
Materials	Basic material: Aluminium			
	Surface material: Polyester lacquer			
	Cover cap: Cycoloy plastic			
Ergonomy	The 1.3" diameter enables the rail to be held comfortably and firmly. The projection of 2" from the wall makes it easy to take hold of the rail.			
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.			



PRESSALIT.

Product	PLUS grab bar 11.8"			
Item number	RT100035			
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.			
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%			
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the			
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.			
Declaration of conformity	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.			



PRESSALIT.

Product	

PLUS grab bar 11.8"

Item number

RT100035


Mounting in different wall types

α.	. V. • ,	2·[]		0
1.0	₹;	α.	7 .1	3:
D.	R.	3:0	. ∀.	
10	$\Box \Diamond$	D. <	?	7 ⋅
	a . D	:2:		0

Concrete >B25


V8616 - Mounting kit, 8 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.




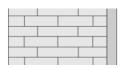

Solid Brick >= Mz12

V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.

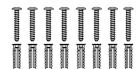


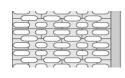

Solid Sand Lime Brick >= Hlz12

V8616 - Mounting kit, 8 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.




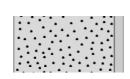

Aerated Concrete P4

V8666 - Mounting kit, 8 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.




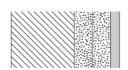

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8616 - Mounting kit, 8 bolts \emptyset 5 x 50 - plugs SX8. NB! Plugs are not needed.

To be ordered separately.

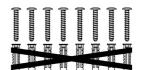



PRESSALIT®

Product Item number	PLUS grab bar 11.8" RT100035	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>^</u>

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com