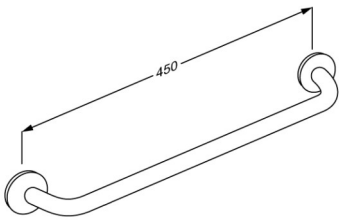




|                                      |  |
|--------------------------------------|--|
| <b>Product</b><br><b>Item number</b> | PLUS handrail 450 mm<br>RT101035   |
| <b>Colour</b>                        | VVS:777565235<br>NRF:6317163<br>EAN:5708590145891<br>HMI: 19082<br>NBS:N13:436<br>Red (RAL 3003)   |
| <b>Product description</b>           |  <p>PLUS handrails are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS handrail, diameter 33 mm, length 450 mm, for fixed mounting.<br/> The projection of 50 mm from the wall makes it easy to take hold of the rail.<br/> Inkl. wall rosette, which covers the fixings.<br/> Material: Powder-coated aluminium and plastic<br/> Excl. mounting kit.</p> |
| <b>Product capacity</b>              | Maximum load: 235 kg.<br>When the handrail is loaded with 235 kg, each wall fastener is loaded with 56 kg.<br><i>Handrails are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i>  |
| <b>Materials</b>                     | Basic material: Aluminium<br><br>Surface material: Polyester lacquer   |
| <b>Ergonomy</b>                      | Cover cap: Cycloplastic<br>The 33 mm diameter enables the rail to be held comfortably and firmly. The projection of 50 mm from the wall makes it easy to take hold of the rail.  |
| <b>Warranty</b>                      | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |



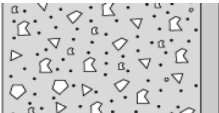
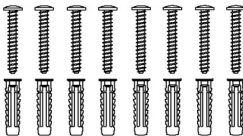

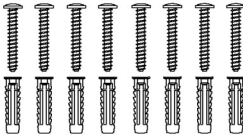
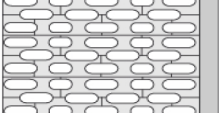
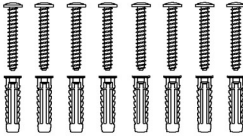
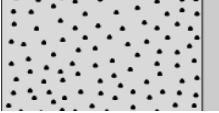
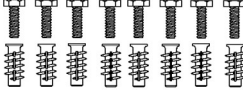
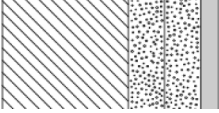
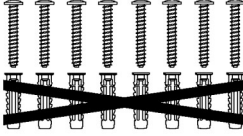
|                                  |   |
|----------------------------------|---|
| <b>Product</b>                   | PLUS handrail 450 mm  |
| <b>Item number</b>               | RT101035  |
| <b>Cleaning</b>                  | <p>Handrails ALU.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b></p> <p>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b></p> <p>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b></p> <p>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> |
| <b>Declaration of conformity</b> | <p>CE UK MDR This product is in compliance with all relevant requirements of:</p> <p>- Regulation (EU) 2017/745 on medical devices.</p> <p><b>For the UK:</b> - Medical Device Regulation (MDR) 2002 on medical devices.</p>  |



PRESSALIT®

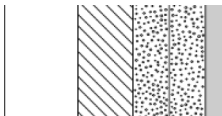
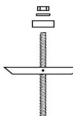

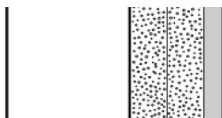
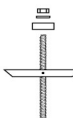

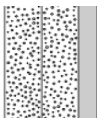

|                    |                      |
|--------------------|----------------------|
| <b>Product</b>     | PLUS handrail 450 mm |
| <b>Item number</b> | RT101035             |

## Installation in different wall types

|   |   |   |
|---|---|---|
|    | <p><b>Concrete &gt;B25</b></p> <p>V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p>   |    |
|   | <p><b>Solid Brick &gt;= Mz12</b></p> <p>V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p>   |   |
|  | <p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p>                                    |  |
|  | <p><b>Aerated Concrete P4</b></p> <p>V8666 - Mounting kit, 8 bolts M5 x 50 - plugs FTP K 6. To be ordered separately.</p>   |  |
|  | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately.</p> |  |



# PRESSALIT®

|   |  |   |   |
|---|--|---|---|
| <b>Product</b><br><b>Item number</b>  | PLUS handrail 450 mm<br>RT101035   |   |   |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b><br>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.       |    |  |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b><br>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. |    |  |
|  | <b>Drywall (2 x 12,5 mm) water-resistant</b><br>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.   |  |   |

[pressalit.com](http://pressalit.com)