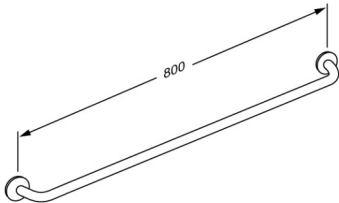




# PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS handrail 800 mm RT103035
<b>Colour</b>	VVS:777565435 NRF:6317167 EAN:5708590145938 HMI: 19084 NBS:N13:436 Red (RAL 3003)
<b>Product description</b>	 <p>PLUS handrails are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS handrail, diameter 33 mm, length 800 mm, for fixed mounting.  The projection of 50 mm from the wall makes it easy to take hold of the rail.  Inkl. wall rosette, which covers the fixings.  Materials: Powder-coated aluminium and plastic  Excl. mounting kit.</p>
<b>Product capacity</b>	Maximum load: 235 kg. When the handrail is loaded with 235 kg, each wall fastener is loaded with 56 kg. <i>Handrails are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i>
<b>Materials</b>	Basic material: Aluminium  Surface material: Polyester lacquer  Cover cap: Cicoloy plastic
<b>Ergonomy</b>	The 33 mm diameter enables the rail to be held comfortably and firmly. The projection of 50 mm from the wall makes it easy to take hold of the rail.
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



# PRESSALIT®

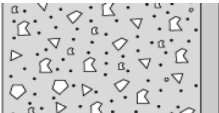
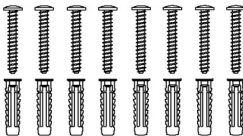

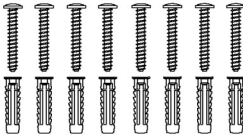
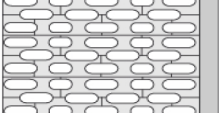
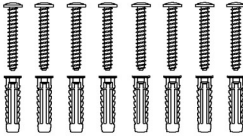
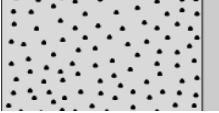
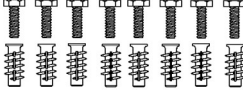
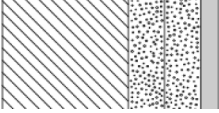
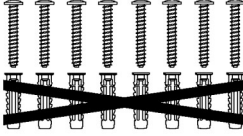
<b>Product</b>	PLUS handrail 800 mm
<b>Item number</b>	RT103035
<b>Cleaning</b>	<p>Handrails ALU.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b></p> <p>Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b></p> <p>Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b></p> <p>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
<b>Declaration of conformity</b>	<p>CE UK MDR This product is in compliance with all relevant requirements of:</p> <p>- Regulation (EU) 2017/745 on medical devices.</p> <p><b>For the UK:</b> - Medical Device Regulation (MDR) 2002 on medical devices.</p>



PRESSALIT®

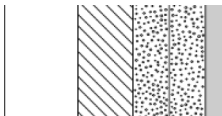
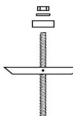

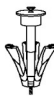
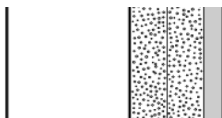
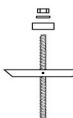

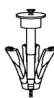
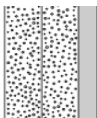

<b>Product</b>	PLUS handrail 800 mm
<b>Item number</b>	RT103035

## Installation in different wall types

	<p><b>Concrete &gt;B25</b></p> <p>V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p>	
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p>	
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p>	
	<p><b>Aerated Concrete P4</b></p> <p>V8666 - Mounting kit, 8 bolts M5 x 50 - plugs FTP K 6. To be ordered separately.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately.</p>	



# PRESSALIT®

Product Item number	PLUS handrail 800 mm RT103035		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	  	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b> Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	  	
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.		

pressalit.com

Pressalit A/S  
 Pressalitvej 1  
 DK-8680 Ry  
 Tel.: +45 8788 8788  
 Fax: +45 8788 8789  
 E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®