

| Product             | PLUS 90° angle grab bar 15.8" x 18.7"  |  |
|---------------------|--|--|
| Item number         | RT114108   |  |
|                     | VVS:777567208  |  |
|                     | NRF:6317173  |  |
|                     | EAN:5708590154008  |  |
|                     | HMI: 25431   |  |
| )                   | NBS:N13:436  |  |
| Color               | Dark blue (RAL 5011)   |  |
|                     |  |  |
| Product description | PLUS grab bars are high-quality products and they can be combined in countless<br>ways and they provide the final touch to furnishing a functional and flexible bathroom<br>A reliable solution with many different options, which can easily lay the grounds for<br>increased independence. |  |
|                     | PLUS grab bar, angle 90° - right, diameter 1.3", A:15.8" x B:18.7".<br>Material: Powder-coated aluminium and plastic.<br>Excl. mounting kit.   |  |
| Product capacity    | Maximum load: 440 lbs.<br>When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs.<br>Grab bars are tested with a 1.5 safety factor in accordance with the requirements in<br>ISO 17966:2016.  |  |
| Materials           | Core material: Aluminium<br>Surface material: Polyester lacquer<br>Cover cap: Cycoloy plastic<br>Strap: Nylon (PA)   |  |
| Ergonomy            | The 1.3" diameter enables the rail to be held comfortably and firmly. The projection of 2" from the wall makes it easy to take hold of the rail.   |  |
| Warranty            | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |  |



| Product                      | PLUS 90° angle grab bar 15.8" x 18.7"<br>RT114108   |  |  |
|------------------------------|---|--|--|
| ltem number                  |   |  |  |
| Cleaning                     | This product should be cleaned with hot water or a mild cleaning solution.<br>Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.  |  |  |
|                              | The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.  |  |  |
|                              | DISINFECTANTS:<br>Clinimax Difficil-S<br>Erisan OXY+<br>Sprit (husholdningssprit)<br>Suma Bac D10<br>Virkon<br>Rodalon Desinfektionsspray<br>Lysoformin Spezial<br>CaviCide<br>Sufanios Premium<br>Tvättsprit 70%   |  |  |
|                              | NON-DISINFECTANTS:<br>Suma Star free D1<br>Taski 100<br>Taski 200<br>Milizid Citro  |  |  |
|                              | The product must be wiped down with a soft rag wrung out in clear water immediat after cleaning, as some materials may otherwise be discolored and stained, and th surface coating may deteriorate.   |  |  |
|                              | DECALCIFICATION OF ALUMINUM PARTS:<br>Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wa<br>tracks and other aluminum parts using a solution of three parts of 30% acetic acid<br>seven parts tap water. When the aluminum parts have been washed down with this<br>solution, the parts should be wiped over with a damp cloth wrung out in plain water |  |  |
| Declaration of<br>conformity | This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.  |  |  |



Product

Item number

PLUS 90° angle grab bar 15.8" x 18.7" RT114108

### Mounting in different wall types

| <b>Concrete &gt;B25</b><br>V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for<br>brick wall. Must be ordered separately.  | (maana kasa)<br>(maana kasa)<br>(maana kasa)<br>(maana kasa)<br>(maana kasa)<br>(maana kasa)<br>(maana kasa)<br>(maana kasa)<br>(maana kasa)<br>(maana kasa) |
|--|--|
| <b>Solid Brick &gt;= Mz12</b><br>V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for<br>brick wall. Must be ordered separately.  | (annum kanna)<br>(annum kanna)<br>(annum kanna)<br>(annum kanna)<br>(annum kanna)<br>(annum kanna)<br>(annum kanna)<br>(annum kanna)<br>(annum kanna)        |
| Solid Sand Lime Brick >= HIz12<br>V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for<br>brick wall. Must be ordered separately.   | (names kand)<br>(names kand)<br>(names kand)<br>(names kand)<br>(names kand)<br>(names kand)<br>(names kand)<br>(names kand)<br>(names kand)                 |
| <b>Aerated Concrete P4</b><br>V8667 - Mounting kit, 12 bolts M5 x 50 - plugs FTP K 6, for<br>aerated concrete. To be ordered separately.   |  |
| <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b><br>V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8. NB!<br>Plugs are not needed.<br>To be ordered separately. |  |



| Product<br>Item number | PLUS 90° angle grab bar 15.8" x 18.7"<br>RT114108   |  |
|------------------------|---|--|
|                        | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with</b><br><b>22 mm plywood</b><br>Wall fasteners (12 pcs) must be bought separately, for<br>drywall it is possible to use e.g. SnapTight or WingIt<br>anchors or similar. |  |
|                        | Drywall (2 x 12,5 mm) water-resistant, re-enforced with<br>steel plates (.08 inch)<br>Wall fasteners (12 pcs) must be bought separately, for<br>drywall it is possible to use e.g. SnapTight or WingIt<br>anchors or similar.     |  |
|                        | <b>Drywall (2 x 12,5 mm) water-resistant</b><br>Mounting in this wall type is not recommended - risk of<br>fatal injury to persons and walls.   |  |

#### pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

