



# PRESSALIT®

|                                      |  |
|--------------------------------------|--|
| <b>Product</b><br><b>Item number</b> | PLUS grab bar, corner 30" x 17.9"<br>RT128035  |
| <b>Color</b>                         | VVS:777568435<br>NRF:6317185<br>EAN:5708590146386<br>HMI: 25427<br>NBS:N13:436<br>Red (RAL 3003)   |
| <b>Product description</b>           | <p>PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS grab bar, 1.3", reversible right/left, A:30" x B:17.9".<br/>Material: Powder-coated aluminium and plastic.<br/>Excl. mounting kit.</p> |
| <b>Product capacity</b>              | Maximum load: 440 lbs.<br>When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs.<br><i>Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i>  |
| <b>Materials</b>                     | Core material: Aluminium<br>Surface material: Polyester lacquer<br>Cover cap: Cycloy plastic<br>Strap: Nylon (PA)  |
| <b>Ergonomy</b>                      | The 1.3" diameter enables the rail to be held comfortably and firmly. The projection of 2" from the wall makes it easy to take hold of the rail.   |
| <b>Warranty</b>                      | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |



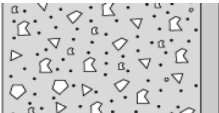



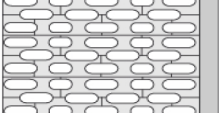

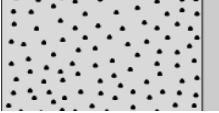

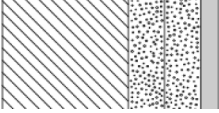
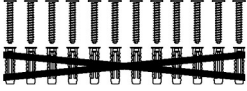
|                                  |  |
|----------------------------------|--|
| <b>Product</b>                   | PLUS grab bar, corner 30" x 17.9"  |
| <b>Item number</b>               | RT128035   |
| <b>Cleaning</b>                  | <p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b><br/>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b><br/>Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p> |
| <b>Declaration of conformity</b> | <p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>  |



PRESSALIT®

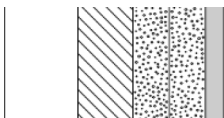
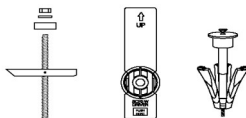
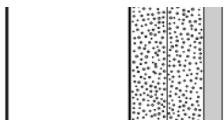
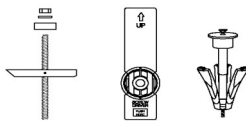
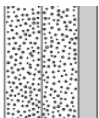

|                    |                                   |
|--------------------|-----------------------------------|
| <b>Product</b>     | PLUS grab bar, corner 30" x 17.9" |
| <b>Item number</b> | RT128035                          |

## Mounting in different wall types

|   |  |
|---|--|
|    | <b>Concrete &gt;B25</b><br>V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.    |
|   | <b>Solid Brick &gt;= Mz12</b><br>V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.    |
|  | <b>Solid Sand Lime Brick &gt;= Hlz12</b><br>V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.   |
|  | <b>Aerated Concrete P4</b><br>V8667 - Mounting kit, 12 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.   |
|  | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b><br>V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8. NB!<br>Plugs are not needed.<br>To be ordered separately.  |



# PRESSALIT®

|   |  |
|---|--|
| <b>Product</b><br><b>Item number</b>  | PLUS grab bar, corner 30" x 17.9"<br>RT128035  |
|    | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b></p> <p>Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>            |
|    | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b></p> <p>Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>  |
|  | <p><b>Drywall (2 x 12,5 mm) water-resistant</b></p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>   |

pressalit.com

Pressalit A/S  
 Pressalitvej 1  
 DK-8680 Ry  
 Tel.: +45 8788 8788  
 Fax: +45 8788 8789  
 E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®