



<b>Product</b> <b>Item number</b>	PLUS grab bar, corner with vertical grip 30" x 17.9" x 42.9" RT138000
<b>Color</b>	VVS:777568700 NRF:6311233 RSK:8980588 EAN:5708590146492 HMI: 25429 NBS:N13:436 White
<b>Product description</b>	<p>PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS grab bar combination, 1.3", reversible right/left, A:30" x B:17.9" x C: 42.9". Material: Powder-coated aluminium and plastic. Excl. mounting kit.</p>
<b>Product capacity</b>	Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. <i>Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i>
<b>Materials</b>	Core material: Aluminium Surface material: Polyester lacquer Cover cap: Cyclopy plastic Strap: Nylon (PA)
<b>Ergonomy</b>	The 1.3" diameter enables the rail to be held comfortably and firmly. The projection of 2" from the wall makes it easy to take hold of the rail.
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



# PRESSALIT®





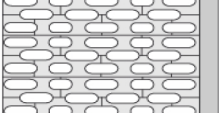

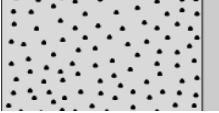
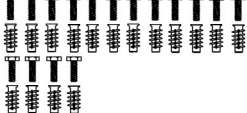
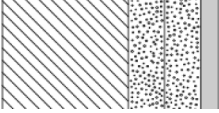
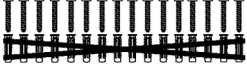
<b>Product</b>	PLUS grab bar, corner with vertical grip 30" x 17.9" x 42.9"
<b>Item number</b>	RT138000
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>Declaration of conformity</b>	<p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>



PRESSALIT®


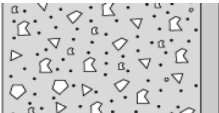





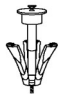
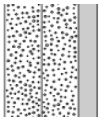

<b>Product</b>	PLUS grab bar, corner with vertical grip 30" x 17.9" x 42.9"
<b>Item number</b>	RT138000

## Mounting in different wall types

	<p><b>Concrete &gt;B25</b></p> <p>V8618 - Mounting kit, 16 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p> 
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8618 - Mounting kit, 16 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p> 
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8618 - Mounting kit, 16 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.</p> 
	<p><b>Aerated Concrete P4</b></p> <p>V8668 - Mounting kit, 16 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.</p> 
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8618 - Mounting kit, 16 bolts Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately.</p> 

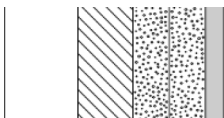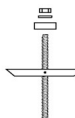

# PRESSALIT®

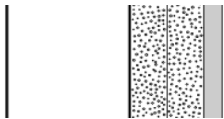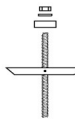Product Item number	PLUS grab bar, corner with vertical grip 30" x 17.9" x 42.9" RT138000		
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (16 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	  	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b> Wall fasteners (16 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	  	
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.		

[pressalit.com](http://pressalit.com)

Pressalit A/S  
 Pressalitvej 1  
 DK-8680 Ry  
 Tel.: +45 8788 8788  
 Fax: +45 8788 8789  
 E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®