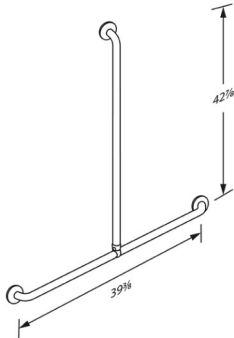




PRESSALIT®

Product Item number	PLUS grab bar combination for shower, 39.4" x 42.9" RT144108
Color	VVS:777569708 NRF:6317193 EAN:5708590247663 HMI: 42068 NBS:N13:436 Dark blue (RAL 5011)
Product description	 <p>PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS grab bar combination for shower, A: 39.4" x B: 42.9". Compatible with RT600 shower holder. Material: Powder-coated aluminium and plastic. Excl. mounting kit.</p>
Product capacity	Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. <i>Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i>
Materials	Core material: Aluminium Surface material: Polyester lacquer Cover cap: Cyclooy plastic Strap: Nylon (PA)
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT®

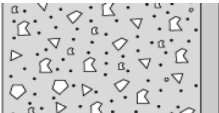



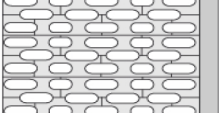

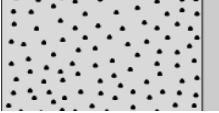

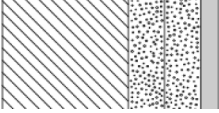
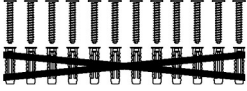
Product	PLUS grab bar combination for shower, 39.4" x 42.9"
Item number	RT144108
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
Declaration of conformity	<p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>



PRESSALIT®

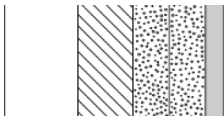
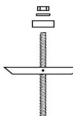

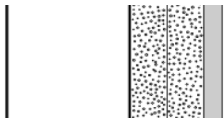
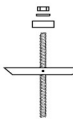

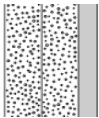

Product	PLUS grab bar combination for shower, 39.4" x 42.9"
Item number	RT144108

Mounting in different wall types

	Concrete >B25 V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately. 
	Solid Brick >= Mz12 V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately. 
	Solid Sand Lime Brick >= Hlz12 V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately. 
	Aerated Concrete P4 V8667 - Mounting kit, 12 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately. 



PRESSALIT®

Product Item number	PLUS grab bar combination for shower, 39.4" x 42.9" RT144108		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.		
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.		

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®