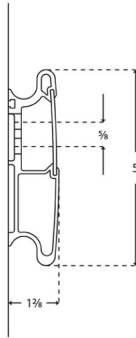




<b>Product</b> <b>Item number</b>	PLUS wall track, 23.6" R9806112
<b>Color</b>  <b>Product description</b>	<p>VVS:615841106  NRF:7501013  EAN:5708590305738  HMI: 71739</p> <p>Anthracite gray</p>  <p>PLUS wall tracks are used for mounting of support arms, shower seats and sink brackets, and allows sideways adjustment of these products.  See below a selection of products for fitting to the wall track.</p> <p>PLUS wall track, 23.6", for horizontal mounting.  The cover plate hides bolts, wires and simplifies cleaning.  Materials: Aluminium and polystyrene.  Excl. mounting kit.</p>
<b>Adjustability</b> <b>Product capacity</b>	<p>Fixed mounting</p> <p>Maximum load: Up to 2200 lbs. depending on the wall type.</p>
<b>Materials</b> <b>Ergonomy</b> <b>Features</b>	<p>The wall track is primarily made of 10-micrometer anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.</p> <p>The track distributes the weight of the individual bathroom product, for example a sink bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions.</p> <p>The horizontal track makes it possible to shift the product along the wall to generate extra space as needed. The front cover plate conceals mounting bolts and any cables.</p>
<b>Warranty</b>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



# PRESSALIT®

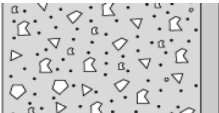
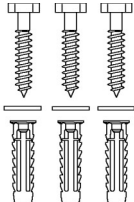

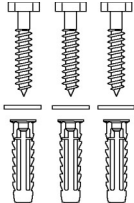
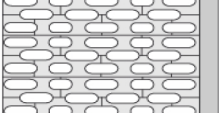
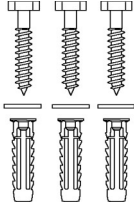
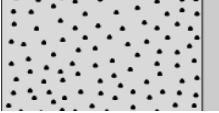
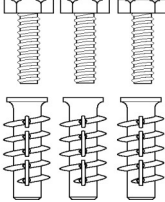
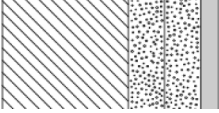
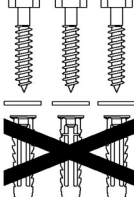
<b>Product</b>	PLUS wall track, 23.6"
<b>Item number</b>	R9806112
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>Declaration of conformity</b>	<p>This product fulfils all relevant requirements in European Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an <b>accessory</b> for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical devices to be used in accordance with their intended purposes.</p>



PRESSALIT®

<b>Product</b>	PLUS wall track, 23.6"
<b>Item number</b>	R9806112

## Mounting in different wall types

	<b>Concrete &gt;B25</b> V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Solid Brick &gt;= Mz12</b> V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Solid Sand Lime Brick &gt;= Hlz12</b> V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Aerated Concrete P4</b> V8670 - Mounting kit, 3 bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b> V8620 - Mounting kit, 6 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



# PRESSALIT®

<b>Product</b>	PLUS wall track, 23.6"
<b>Item number</b>	R9806112

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

## PRESSALIT®