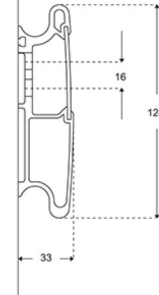




|                            |  |
|----------------------------|--|
| <b>Product</b>             | PLUS wall track, 1800 mm   |
| <b>Item number</b>         | R9818112   |
| <b>Colour</b>              | Anthracite grey  |
| <b>Product description</b> | <p>VVS:615841118<br/>NRF:7501017<br/>EAN:5708590305813<br/>HMI: 71743</p> <p>PLUS wall tracks are used for mounting of support arms, shower seats and wash basin brackets, and allows sideways adjustment of these products. See below a selection of products for fitting to the wall track.</p> <p>PLUS wall track, 1800 mm, for horizontal mounting.<br/>The cover plate hides bolts and wires and facilitates cleaning.<br/>Materials: Aluminium and polystyrene.<br/>See below a selection of products for fitting to the wall track.<br/>Excl. mounting kit.</p> |
| <b>Adjustability</b>       | Fixed mounting   |
| <b>Product capacity</b>    | Maximum load: Up to 1000 kg depending on the wall type.  |
| <b>Materials</b>           | The wall track is primarily made of 10-micrometre anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.  |
| <b>Ergonomy</b>            | The track distributes the weight of the individual bathroom product, for example a wash basin bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions.  |
| <b>Features</b>            | The horizontal track makes it possible to shift the product along the wall to constantly optimize the available space in the bathroom. The front cover plate conceals mounting bolts and any cables.   |
| <b>Warranty</b>            | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |



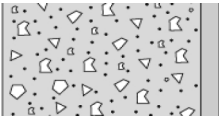
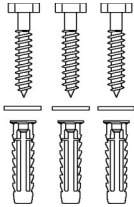

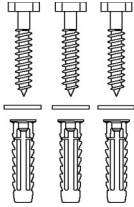
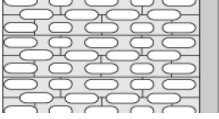
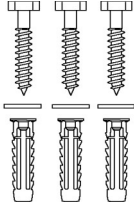
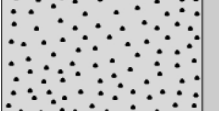
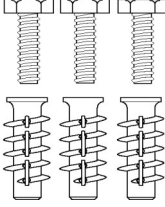
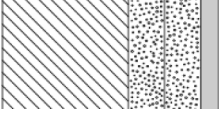
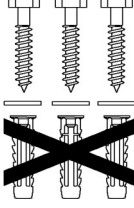


|                                  |   |
|----------------------------------|---|
| <b>Product</b>                   | PLUS wall track, 1800 mm  |
| <b>Item number</b>               | R9818112  |
| <b>Cleaning</b>                  | <p>PLUS wall tracks.<br/>The product is to be cleaned with hot water or a mild soapy cleaning agent.<br/>Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningsprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b><br/>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b><br/>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> |
| <b>Declaration of conformity</b> | <p>MDR-ACC This product fulfils all relevant requirements in Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an <b>accessory</b> for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical devices to be used in accordance with their intended purposes.</p>  |



|                    |                          |
|--------------------|--------------------------|
| <b>Product</b>     | PLUS wall track, 1800 mm |
| <b>Item number</b> | R9818112                 |

## Installation in different wall types

|   |  |   |
|---|--|---|
|    | <b>Concrete &gt;B25</b><br>V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10.<br>To be ordered separately.  |    |
|   | <b>Solid Brick &gt;= Mz12</b><br>V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10.<br>To be ordered separately.  |   |
|  | <b>Solid Sand Lime Brick &gt;= Hlz12</b><br>V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10.<br>To be ordered separately.   |  |
|  | <b>Aerated Concrete P4</b><br>V8670 - Mounting kit, 3 bolts M8 x 35, plugs FTP M 8.<br>To be ordered separately.   |  |
|  | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b><br>V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10.<br>NB! Plugs are not needed.<br>To be ordered separately. |  |



# PRESSALIT®

|                    |                          |
|--------------------|--------------------------|
| <b>Product</b>     | PLUS wall track, 1800 mm |
| <b>Item number</b> | R9818112                 |

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®