


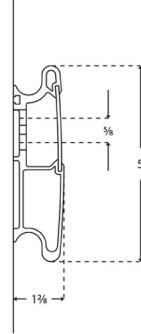

PRESSALIT®

Product Item number	PLUS wall track 153.3" R9839112	
Color	VVS:615841139 NRF:7501068 EAN:5708590305950 HMI: 71751 Anthracite gray	
Product description	PLUS wall tracks are used for mounting of support arms, shower seats and sink brackets, and allows sideways adjustment of these products. See below a selection of products for fitting to the wall track. PLUS wall track, 153.5", for horizontal mounting. The cover plate hides bolts, wires and simplifies cleaning. Materials: Aluminium and polystyrene. Excl. mounting kit.	
Adjustability	Fixed mounting	
Ergonomy Features	The track distributes the weight of the individual bathroom product, for example a sink bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions. The horizontal track makes it possible to shift the product along the wall to generate extra space as needed. The front cover plate conceals mounting bolts and any cables.	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



PRESSALIT®

Product	PLUS wall track 153.3"
Item number	R9839112
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: CaviCide The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.
Declaration of conformity	This product fulfils all relevant requirements in European Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an accessory for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical devices to be used in accordance with their intended purposes.

pressalit.com

PRESSALIT®