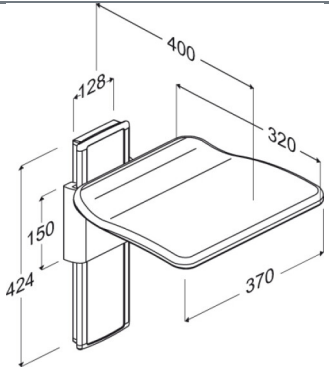




<p><b>Product</b></p> <p><b>Item number</b></p>	<p>PLUS folding seat, manually height adjustable</p> <p>R5520112000</p>
<p><b>Colour</b></p> <p><b>Product description</b></p>	<p>VVS:777531200          NRF:7501003          RSK:7973927          EAN:5708590306360          HMI: 130158          NBS:N13:448          FDA:INN/D135865</p> <p>White</p>  <p>PLUS folding seat. Manually height adjustable by 240 mm. Anthracite cover plate for vertical wall track. The seat is foldable. Excl. mounting kit. The folding seat is not suitable for use in the shower area.</p> <p>PLUS folding seat. Manually height adjustable by 240 mm. Anthracite cover plate for vertical wall track. Materials: Aluminium, thermoset plastic, plastic. Excl. mounting kit.</p>
<p><b>Adjustability</b></p> <p><b>Adjustment height</b></p> <p><b>Product capacity</b></p>	<p>Height adjustable</p> <p>240 mm</p> <p>Maximum user mass: 265 kg.          When the seat is loaded with 265 kg, each wall fastener is loaded with 23 kg.</p> <p><i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i></p>
<p><b>Materials</b></p> <p><b>Ergonomy</b></p> <p><b>Features</b></p>	<p>The folding seat is made out of thermoset plastic, and the mounting and suspension bracket are made from 10 my anodised aluminium. The end caps are made from ABS plastic.</p> <p>Ergonomically shaped seat, which offers good and comfortable support. The seat is suitable for situations where additional seating is required and space is limited, - e.g. in kitchens, lifts and corridors or on the landings.</p> <p>The seat can be folded away when not in use, - or to free up space. Stays up of itself when folded away.</p>

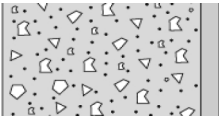
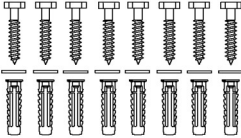

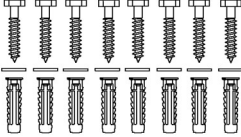
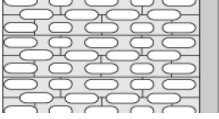
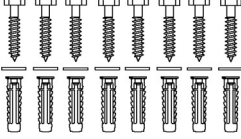
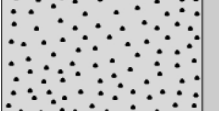
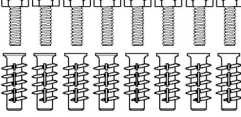
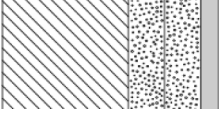
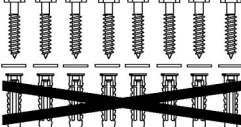


<b>Product</b>	PLUS folding seat, manually height adjustable
<b>Item number</b>	R5520112000
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
<b>Cleaning</b>	<p>Folding seat. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
<b>Declaration of conformity</b>	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.

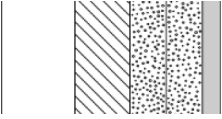
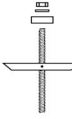

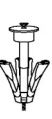
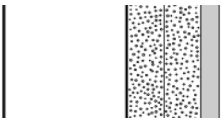
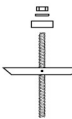


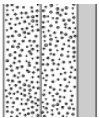


<b>Product</b>	PLUS folding seat, manually height adjustable
<b>Item number</b>	R5520112000

## Installation in different wall types

	<b>Concrete &gt;B25</b> V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Solid Brick &gt;= Mz12</b> V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Solid Sand Lime Brick &gt;= Hlz12</b> V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	<b>Aerated Concrete P4</b> V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b> V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



<b>Product</b> <b>Item number</b>	PLUS folding seat, manually height adjustable R5520112000
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.   
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b> Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.   
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 