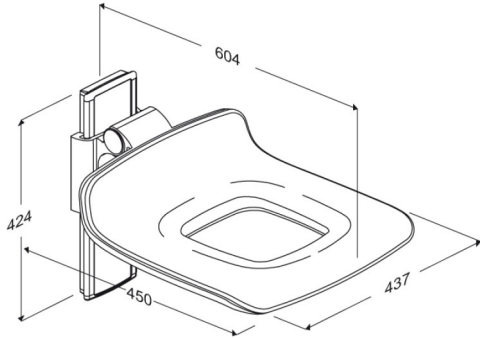




Product Item number	PLUS shower seat 450 with aperture, manually height adjustable R7411112000
Colour	VVS:777535200 NRF:6311158 EAN:5708590311333 HMI: 71804 NBS:N13:448 FDA:INN/D135865 White
Product description	 PLUS shower seat 450 with aperture. Manually height adjustable by 220 mm. The seat is foldable. Anthracite cover plate for vertical wall track. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. Excl. mounting kit. PLUS shower seat 450 with aperture. Manually height adjustable by 220 mm. The seat is foldable. Anthracite cover plate for vertical wall track. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. Excl. mounting kit.
Adjustability Adjustment height Product capacity	Height adjustable 220 mm Maximum user mass: 200 kg. When the seat is loaded with 200 kg, each wall fastener is loaded with 47 kg. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
Materials Ergonomy Features	The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with an in-mould coated surface that is protected using a clear PU lacquer. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS. Ergonomically designed seat - contact surface are 450 x 437 mm. Soft and slip-resistant material that has a warm feel. The seat can fold up when not in use or to maximise floor space.



PRESSALIT®

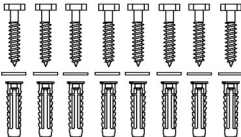
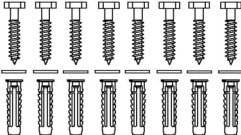
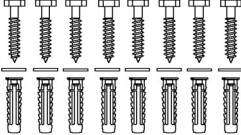
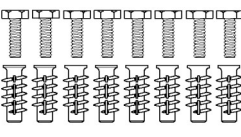
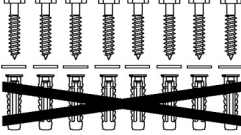
Product	PLUS shower seat 450 with aperture, manually height adjustable
Item number	R7411112000
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	PLUS shower seat. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH) NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa) Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water. The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.



PRESSALIT®

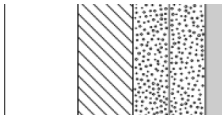
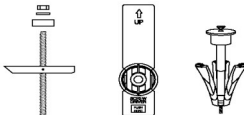
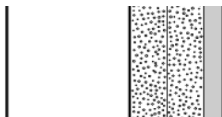
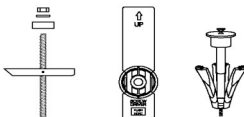
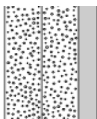
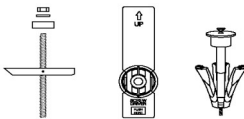
Product	PLUS shower seat 450 with aperture, manually height adjustable
Item number	R7411112000

Installation in different wall types

	Concrete >B25 V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Brick >= Mz12 V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Aerated Concrete P4 V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



PRESSALIT®

Product Item number	PLUS shower seat 450 with aperture, manually height adjustable R7411112000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 
	Drywall (2 x 12,5 mm) water-resistant Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 

pressalit.com

Pressalit A/S
 Pressalitvej 1
 DK-8680 Ry
 Tel.: +45 8788 8788
 Fax: +45 8788 8789
 E-mail: pressalit@pressalit.com

PRESSALIT®